

# Fluid Fighter<sup>®</sup> | Suction Strip

## Instructions for Use

### Pre-Surgery

1. Remove the Fluid Fighter<sup>®</sup> Suction Strip from its packaging and place it on the operating room floor with the label sticker facing up.
2. Position the suction strip with the suction tubing facing closest to the suction source.
3. Ensure the suction strip is placed flat on the floor.
4. Connect the suction strip's pre-attached tubing to the suction source.
5. Power on the vacuum source.

### Post-Surgery

1. Following the procedure, run the vacuum source for a few minutes to allow residual fluid in the suction strip to be vacuumed into the suction source.
2. Power off the suction source and disconnect the tubing from the suction source.
3. Dispose of the suction strip per your facility's protocol and place it in an appropriate waste receptacle.