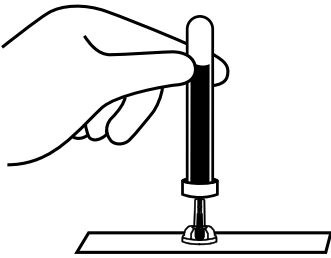
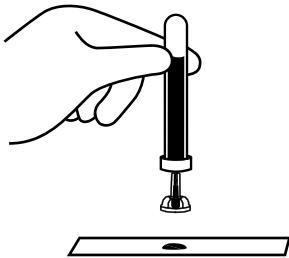
1.

2.

3.

Instructions
English

Intended Use:

Smearsafe™ is intended to provide a safe means for obtaining a drop of blood from a closed collection tube without requiring the user to open the tube. It is designed to produce a drop of blood suitable for a manual smear. It is a single-use device and is compatible with Vacutainer® and Vacuette® tubes.

Smearsafe is clear, which allows you to see the blood drop as it is dispensed. Smearsafe is intended for use by laboratory professionals.

Hold the specimen tube upright. Point the cannula of the Smearsafe device so that it is parallel to the specimen tube. Insert the Smearsafe device into the specimen tube septum until the cannula is no longer visible.

Invert the specimen tube and position Smearsafe on the middle of a glass slide or other target surface. Press the tube gently downward until a blood drop forms at the tip of the Smearsafe device.

Release the pressure once the drop touches the glass and lift the tube straight up from the slide. NOTE: The size of the drop can be controlled based on the speed and pressure applied to the tube. Discard the Smearsafe device in a sharps medical waste container after use. Smearsafe may be discarded while still in the tube or removed from the tube and discarded separately. If discarded separately, hold the specimen tube cap in place while pulling Smearsafe out.

CAUTION: Wear gloves when using Smearsafe. Use care when handling Smearsafe as the cannula can cause injury or tear gloves if handled inappropriately. Do not insert Smearsafe when the specimen tube is upside-down. For specimen tubes that have been previously opened and recapped, be sure the tube cap is fully seated on the tube to avoid excess blood being dispensed onto the target surface. Any serious incident that has occurred in relation to the device shall be reported to Typenex Medical.

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Intended use: Smearsafe™ is intended to provide a safe means for obtaining a drop of blood from a closed collection tube without requiring the user to open the tube. It is designed to produce a drop of blood suitable for a manual smear. It is a single-use device and is compatible with Vacutainer® and Vacuette® tubes.	Hold the specimen tube upright. Point the cannula of the Smearsafe device so that it is parallel to the specimen tube. Insert the Smearsafe device into the specimen tube septum until the cannula is no longer visible.	Invert the specimen tube and position Smearsafe on the middle of a glass slide or other target surface. Press the tube gently downwards until a blood drop forms at the tip of the Smearsafe device.	Release the pressure once the drop touches the glass and lift the tube straight up from the slide. NOTE: The size of the drop can be controlled based on the speed and pressure applied to the tube. Discard the Smearsafe device in a sharps medical waste container after use. Smearsafe may be discarded while still in the tube or removed from the tube and discarded separately. If discarded separately, hold the specimen tube cap in place while pulling Smearsafe out.	CAUTION: Wear gloves when using Smearsafe. Use care when handling Smearsafe as the cannula can cause injury or tear gloves if handled inappropriately. Do not insert Smearsafe when the specimen tube is upside down. For specimen tubes that have been previously opened and recapped, make sure that the tube cap is fully seated on the tube to avoid excess blood being dispensed onto the target surface. Any serious incident that has occurred in relation to the device shall be reported to Typenex Medical.
--	--	--	--	---

Smearsafe is clear, which allows you to see the blood drop as it is dispensed. Smearsafe is intended for use by laboratory professionals.				
---	--	--	--	--

Instructions
English IE

Intended use: Smearsafe™ is intended to provide a safe means for obtaining a drop of blood from a closed collection tube without requiring the user to open the tube. It is designed to produce a drop of blood suitable for a manual smear. It is a single-use device and is compatible with Vacutainer® and Vacuette® tubes.	Hold the specimen tube upright. Point the cannula of the Smearsafe device so that it is parallel to the specimen tube. Insert the Smearsafe device into the specimen tube septum until the cannula is no longer visible.	Invert the specimen tube and position Smearsafe on the middle of a glass slide or other target surface. Press the tube gently downwards until a blood drop forms at the tip of the Smearsafe device.	Release the pressure once the drop touches the glass and lift the tube straight up from the slide. NOTE: The size of the drop can be controlled based on the speed and pressure applied to the tube. Discard the Smearsafe device in a sharps medical waste container after use. Smearsafe may be discarded while still in the tube or removed from the tube and discarded separately. If discarded separately, hold the specimen tube cap in place while pulling Smearsafe out.	CAUTION: Wear gloves when using Smearsafe. Use care when handling Smearsafe as the cannula can cause injury or tear gloves if handled inappropriately. Do not insert Smearsafe when the specimen tube is upside down. For specimen tubes that have been previously opened and recapped, make sure that the tube cap is fully seated on the tube to avoid excess blood being dispensed onto the target surface. Any serious incident that has occurred in relation to the device shall be reported to Typenex Medical.
--	--	--	--	---

Smearsafe is clear, which allows you to see the blood drop as it is dispensed. Smearsafe is intended for use by laboratory professionals.				
---	--	--	--	--

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Typenex Medical, LLC