



REF M6010

Blood Collection Device

INDICATIONS FOR USE: The Blood Collection Device may be used for the collection of a specimen from a catheter hub or luer activated connector to Blood Collection Tube(s).

DIRECTIONS: Use aseptic technique.

1. Peel open package at arrow and remove device.
2. Insert the tip of the male luer lock into mating device. Push and turn assembly clockwise onto the catheter hub until it fits securely.
3. Center the Blood Collection Tube in the holder portion of the access device and push it in.
4. When blood flow stops, remove Blood Collection Tube from the holder. Repeat steps 3 and 4 as necessary.
5. Hold catheter port and remove the assembly by rotating it counter clockwise.
6. Discard immediately after use into a biohazard Sharps container approved for this disposal. DO NOT disassemble and reuse Blood Collection Device.

WARNINGS AND CAUTIONS:

- Handle all biological samples and blood collection sharps in accordance with your facility's policies and procedures.
- Discard all blood collection sharps in biohazard containers approved for their disposal.
- Obtain appropriate medical treatment in the event of any exposure to biological samples (e.g. through puncture injury) since samples may transmit infectious diseases.
- Allow natural vacuum of tube to draw specimen into tube. Filling is complete when vacuum no longer continues to draw.

Sterile EO

Made in China

For single use only



IMPORTANT
DO NOT
RE-STERILIZE



DEHP
NOT MADE
WITH DEHP



NOT MADE
WITH NATURAL
RUBBER LATEX

⚠ WARNING: THIS PRODUCT CAN EXPOSE YOU TO CHEMICALS INCLUDING DIISONONYL PHTHALATE (DINP), WHICH IS KNOWN TO THE STATE OF CALIFORNIA TO CAUSE CANCER AND BIRTH DEFECTS OR OTHER REPRODUCTIVE HARM. FOR MORE INFORMATION GO TO www.P65Warnings.ca.gov.