



**REF** M6320

## Blood Collection Luer Adapter

**INDICATIONS FOR USE:** The Blood Collection Luer Adapter is intended to be used with a Blood Collection Tube Holder for the collection of a specimen from a patient to Blood Collection Tube(s) or Blood Culture Bottle(s).

**DIRECTIONS:** Use aseptic technique.

1. Twist blue and clear Luer Adapter Caps simultaneously in opposite directions to open.
2. While holding the blue Male Luer Slip Cap, pull off clear Needle Cap.
3. Screw Luer Adapter onto Blood Collection Tube Holder.
4. Remove blue Male Luer Slip Cap by pulling away from the holder.
5. Assembled device is ready for use.

### **WARNINGS AND CAUTIONS:**

- Handle all biological samples and blood collection sharps in accordance with your facility's policies and procedures.
- Discard all blood collection sharps in biohazard containers approved for their disposal.
- Obtain appropriate medical treatment in the event of any exposure to biological samples (e.g. through puncture injury) since samples may transmit infectious diseases.
- Allow natural vacuum of tube to draw specimen into tube. Filling is complete when vacuum no longer continues to draw.

**Sterile** EO

**For single use only**



IMPORTANT  
DO NOT  
RE-STERILIZE



DEHP  
NOT MADE  
WITH DEHP



NOT MADE  
WITH NATURAL  
RUBBER LATEX

**R<sub>x</sub> Only**

Made in China