

Blood Collection Tube Holder

INDICATIONS FOR USE: The Blood Collection Tube Holder may be used in connection with a Blood Collection Needle, Set, or Luer Adapter for the collection of a specimen from a patient to blood collection tube(s).

DIRECTIONS: Use aseptic technique.

1. Insert the male threaded assembly of the Blood Collection Needle, Set, or Luer Adapter into the female threaded portion of the Blood Collection Tube Holder.
2. Push and turn Blood Collection Needle, Set, or Luer Adapter male assembly clockwise into the Blood Collection Tube Holder until it fits securely.
3. Follow Blood Collection Needle, Set, or Luer Adapter directions and perform venipuncture in accordance with your facility's policies and procedures.
4. Center the Blood Collection Tube in the holder portion of the Blood Collection Tube Holder and push it in.
5. When blood flow stops, remove Blood Collection Tube from the holder. Repeat steps 4 and 5 as necessary.
6. Discard immediately after use into a biohazard Sharps container approved for this disposal. DO NOT disassemble and reuse Blood Collection Tube Holder.

WARNINGS AND CAUTIONS:

- Handle all biological samples and blood collection sharps in accordance with your facility's policies and procedures.
- Discard all blood collection sharps in biohazard containers approved for their disposal.
- Obtain appropriate medical treatment in the event of any exposure to biological samples (e.g. through puncture injury) since samples may transmit infectious diseases.
- Allow natural vacuum of tube to draw specimen into tube. Filling is complete when vacuum no longer continues to draw.

Made in China

For single use only



IMPORTANT
DO NOT
RE-STERILIZE



DEHP
NOT MADE
WITH DEHP



NOT MADE
WITH NATURAL
RUBBER LATEX

⚠ WARNING: THIS PRODUCT CAN EXPOSE YOU TO CHEMICALS INCLUDING DIISONONYL PHTHALATE (DINP), WHICH IS KNOWN TO THE STATE OF CALIFORNIA TO CAUSE CANCER AND BIRTH DEFECTS OR OTHER REPRODUCTIVE HARM. FOR MORE INFORMATION GO TO www.P65Warnings.ca.gov.