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AC-N95

FITTING INSTRUCTIONS:

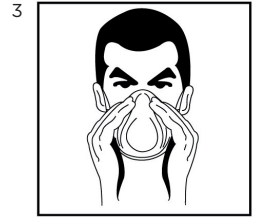
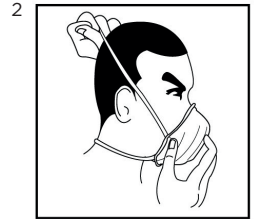
1. Hold the respirator in position over nose and mouth, with retaining straps around back of hand. Pull lower portion of strap over head and fit around neck.
2. Stretch and pull the upper strap over your head on the back of your head above your ears.
3. Manoeuvre the respirator in order to obtain a good facial seal.
4. Perform a User Seal Check prior to each wearing: cup both hands over the respirator and exhale sharply. If air leaks around the edges, reposition the respirator for a good facial seal.
5. Change the respirator immediately if breathing becomes difficult or the respirator becomes damaged or distorted, or a proper face fit cannot be maintained. Careful observance of these instructions is an important step in safe respirator use.

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AC-N95

WARNING:

1. This product does not eliminate the risk of contracting any disease or infection.
2. Failure to follow all instructions and limitations on the use of this product could reduce the effectiveness of respirator and result in sickness or death.
3. Before occupational use of this respirator, a written respiratory protection program must be implemented meeting all the local government requirements. In the United States, employers must comply with OSHA 29 CFR 1910.134 which includes medical evaluation, training, and fit testing.
4. This product does not supply oxygen. Use only in adequately ventilated areas containing sufficient oxygen to support life. Do not use this respirator when oxygen concentration is less than 19.5%.
5. Do not use when concentrations of contaminants are immediately dangerous to health or life.
6. Leave work area immediately and return to fresh air if (a) breathing becomes difficult, or (b) dizziness or other distress occurs.
7. Facial hairs or beards and certain facial characteristics may reduce the effectiveness of this respirator.
8. Never alter or modify this respirator in any way.
9. Only for single use. No maintenance necessary. Discard the used respirator after single use.
10. Keep respirators in the display box away from direct sunlight until use.

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S - Special Instructions : The model AC-N95 filtering facepiece respirator has been manufactured by San Huei United Co., Ltd. for Alpha Solway Ltd. under NIOSH TC-84A-7631.

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