EN - Single use nitrile examination glove. Powder-free. Food safe. Not Made With Natural Rubber Latex. Textured finish. Non-Sterile. Prevents Type I Allergies. Protection is limited to the hand only. If irritation occurs seek medical advice. Obsolescence:- When stored as recommended the product will not suffer any change in chemical properties for up to 5 years from date of manufacture. Service life cannot be specified and depends on application and responsibility of user to ascertain suitability of the glove for its intended use. Before usage, inspect the gloves for any defect or imperfections. Inappropriate glove use - Latitless activity and dexterity reduces with a poor-fitting glove, which can cause fatigue in the hands and fingers. Wrong size and poor fitting lead to poor hand protection. Glove donning - Choose the appropriate glove size. Check for any physical damages or contamination prior to each use of the glove. Discard if there is any damage or contaminant. Glove doffing:- Remove the gloves as soon as you see wearing or damage. When the gloves is covered in a contaminant that is not removable or presents a potential hazard it is advisable to wash left and right gloves off alternately using the glove and so that the gloves are removed without the contaminant contacting the skin. Hand Hygiene:- To maintain hand hygiene wash hands thoroughly before donning gloves and after removing gloves. Used gloves should be disposed of by incineration or as per the regulation of local authorities. To access and download Declaration of Conformity please visit gg-doc.com/skytec. The product is in compliance with the PPE regulation (EU) 2016/425. Tested according to EN ISO 21420:2020.

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EN ISO 374-1: 2016+A1:2018/ TYPE B

K P T

Degradation levels indicate the change in Puncture Resistance of the glove after exposure to the challenge chemical

EN ISO 374-5:2016

VIRUS

Chemical	Code Letter	Breakthrough Time	Level	Mean Degradation % EN ISO 374-4:2019
40% Sodium Hydroxide	K	>480 min	6	9.8
30% Hydrogen Peroxide	P	>120 min	4	6.6
37% Formaldehyde	T	>30 min	2	41.3

Performance Level	1	2	3	4	5	6
Breakthrough Time (min)	> 10	> 30	> 60	> 120	> 240	> 480

Performance Level	AQL	Inspection Levels
Level 3	< 0.65	G1
Level 2	< 1.5	G1
Level 1	< 4.0	S4

Tested in accordance to EN ISO 374-2:2019

Penetration air leak: PASS

Penetration water leak: PASS

Performance level: Level 2

Micro-organisms risk in accordance with EN ISO 374-5:2016

Protection against bacteria and fungi: PASS

Protection against Viruses: PASS

Products conforms to the requirements of: EU regulation 2016/425 of the European Parliament of the Council as Personal Protective Equipment (PPE). Type examination certificate Module B issued by: EU: SATRA Technology Europe Ltd, Bracetown Business Park, Clonee, Dublin, D15 YN2P, Ireland (Notified body No. 2777). Module D assessment carried out by: EU: SGS Fimko Oy, Takomotie 8, FI-00380 Helsinki, Finland. (Notified Body No. 0598).

Intended: Medical and PPE Cat III

STATEMENT AND CAUTION Test results are taken from palm area of the glove. Glove performance quoted is based on laboratory data and may not reflect the actual duration of protection in the workplace due to the differentiation between mixtures and pure chemicals and other factors influencing the performance such as temperature, abrasion, degradation etc. The chemical resistance has been assessed under laboratory conditions from samples taken from the palm only and relates only to the chemical tested. It can be different if the chemical is used in a mixture. When used, protective gloves may provide less resistance to the dangerous chemical due to changes in physical properties. Movements, snagging, rubbing, degradation caused by the chemical contact etc. may reduce the actual use time significantly. For corrosive chemicals, degradation can be the most important factor to consider in selection of chemical resistant gloves. The penetration resistance has been assessed under laboratory conditions and relates only to the tested specimen. There are no possible allergens within the glove that are known to cause harm to the wearer. Before usage, inspect the gloves for any defects or imperfections.

GLOBUS

Manufacture

Globus (Shetland) Ltd, T2 Trafford Point, Twining Road, Trafford Park, Manchester, M17 1SH, UK | T: +44 (0)161877 4747 | www.globusgroup.com

Globus EMEA FZE, Jafza View 1, Office 2201, Jebel Ali, P.O.Box 61195, Dubai, United Arab Emirates | T: +9714 882 9962

[EU] [REP]

Globus EMEA Ltd Authorized Representative in the European Community

Globus EMEA Ltd. 51 Dawson St. Dublin, DO2 AN25, Ireland

MADE IN SRILANKA

Globus is a registered trademark owned by Globus (Shetland) Ltd. © 2023 Globus (Shetland Ltd. All Rights Reserved | SKY-PCK-DISP-TX4524-1023_2