EN- Single use nitrile glove. Powder-Free. Not Made With Natural Rubber Latex. Finger textured finish. Non-Sterile. Prevents Type I Allergies. Protection is limited to the hand only. If irritation occurs seek medical advice. Obsolescence: - When stored as recommended the product will not suffer any change in chemical properties for up to 5 years from date of manufacture. Service life cannot be specified and depends on application and responsibility of user to ascertain suitability of the glove for its intended use. Before usage, inspect the gloves for any defect or imperfections. Inappropriate glove use: - Tactile sensitivity and dexterity reduces with a poor-fitting glove, which can cause fatigue in the hands and fingers. Wrong size and poor fitting lead to poor hand protection. Glove donning:- Choose the appropriate glove size. Check for any physical damages or contamination prior to each use of the glove. Discard if there is any damage or contamination. Glove doffing:- Remove the glove as soon as you see wearing or damage. When the glove is covered in a contaminant that is not removable or presents a potential hazard it is advisable to ease left and right gloves off alternately using the gloved hand so that the gloves are removed without the contaminant contacting the skin. Hand Hygiene:- To maintain hand hygiene wash hands thoroughly before donning gloves and after removing gloves. Used gloves should be disposed by incineration or as per the regulation of local authorities. To access and download Declaration of Conformity please visit gg-doc.com/skytec. The product is in compliance with the PPE regulation (EU) 2016/425. Tested according to EN ISO 21420:2020.

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ENISO374-1:
2016-AT2018/
TYPE B

KPT

Chemical	Code Letter	Breakthrough Time	Level	Mean Degradation % EN ISO 374-4:2019
40% Sodium Hydroxide	K	> 480min	6	-32.3
30% Hydrogen Peroxide	P	> 30 min	2	41.4
37% Formaldehyde	T	> 480min	6	21.6

Performance Level	1	2	3	4	5	6
Breakthrough Time (min)	> 10	> 30	> 60	> 120	> 240	> 480

Degradation levels indicate the change in Puncture Resistance of the glove after exposure to the challenge chemical

Performance Level	AQL	Inspection Levels	Tested in accordance to EN ISO 374-2:2019 Penetration air leak: PASS Penetration water leak: PASS Performance level: Level 2 Micro-organisms risk in accordance with EN ISO 374-5:2016 Protection against bacteria and fungi: PASS Protection against viruses: PASS
Level 3	< 0.65	G1	
Level 2	< 1.5	G1	
Level 1	< 4.0	S4	

Products conforms to the requirements of: EU regulation 2016/425 of the European Parliament of the Council as Personal Protective Equipment (PPE). Type examination certificate Module B issued by: EU: SATRA Technology Europe, Braetown Business Park, Clonee, Dublin, D15 YN2P, Ireland (Notified body No. 2777). Module D assessment carried out by: EU: SGS Fimko Oy, Takomitie 8, FI-00380 Helsinki, Finland. (Notified Body No. 0598).

Intended:

PPE Cat III

	Comply to PPE Regulation (EU) 2016/425 Cat III
---	--

STATEMENT AND CAUTION Test results are taken from palm area of the glove. Glove performance quoted is based on laboratory data and may not reflect the actual duration of protection in the workplace due to the differentiation between mixtures and pure chemicals and other factors influencing the performance such as temperature, abrasion, degradation etc. The chemical resistance has been assessed under laboratory conditions from samples taken from the palm only and relates only to the chemical tested. It can be different if the chemical is used in a mixture. When used, protective gloves may provide less resistance to the dangerous chemical due to changes in physical properties. Movements, snagging, rubbing, degradation caused by the chemical contact etc. may reduce the actual use time significantly. For corrosive chemicals, degradation can be the most important factor to consider in selection of chemical resistant gloves. The penetration resistance has been assessed under laboratory conditions and relates only to the tested specimen. There are no possible allergens within the glove that are known to cause harm to the wearer. Before usage, inspect the gloves for any defects or imperfections.

		
---	---	---

GLOBUS

Globus (Shetland) Ltd, T2 Trafford Point, Twining Road, Trafford Park, Manchester, M17 1SH, UK | T: +44 (0)161877 4747 | www.globusgroup.com

Globus EMEA FZE, Jafza View 1, Office 2201, Jebel Ali, PO Box 61195, Dubai, United Arab Emirates | T: +971 4 882 9962

 **Globus EMEA Ltd** Authorized Representative in the European Community

Globus EMEA Ltd. 51 Dawson St. Dublin, DO2 AN25, Ireland

MADE IN SRI LANKA