



BIODEGRADABLE SINGLE USE GLOVE, 100% NITRILE, POWDER FREE, 240MM LONG, 0.07MM THICK

High performance biodegredable nitrile single use examination glove. Sustainable, dual certified (Industrial and Medical) and food safe.

FEATURES

- > High quality nitrile, soft and durable
- > Palm thickness 0.07mm (3 mil), weight: 3.5g size M
- > Tested to ASTM D5526 (determine the degree and rate of anaerobic biodegradation of materials in accelerated landfill conditions). Results: 30% biodegredation after 202 days. Tested to ASTM D5511 (determine the degree and rate of anaerobic biodegradation of materials in high-solids anaerobic-digestion conditions) Results: 90% biodegredation after 490 days
- Textured finish on fingertips for improved grip and touch
- > Powder-free, latex free, non-sterile, AQL 1.5
- Chlorinated inside to offer more comfort and reduce friction when donning
- Antistatic properties due to inherent antistatic properties of nitrile
- Approved for food handling ((EU) No 10/2011, (EC) 1935/2004 and FDA 21 CFR 177.2600)
- > Dual labelling: PPE and medical
- Medical device compliant (MDD 93/42/EEC), conforms to EN455 (all parts). Compliant with ASTM D6319



SUITABLE FOR

Typical Industries


- Aerospace
- Chemical
- Electronics
- Food
- Pharmaceutical
- Manufacturing
- Oil and Gas

Suitable Applications

- Chemical Handling
- Food Handling
- Handling Oily Components
- Laboratory Analysis
- Light Assembly
- Paint Spraying
- Examination

CAT III CAT III CAT III See overleaf for explanation



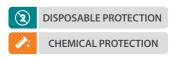



PRODUCTION INFORMATION Nitrile LINER: MATERIALS COATING: COLOUR Violet LENGTH (mm) 240 **CUFF STYLE** Beaded

ORDERING DETAILS

SIZE	CODE	PACKAGING
6/XS 7/S 8/M 9/L 10/XL	SKG00324IB SKG00324ID SKG00324IF SKG00324IH SKG00324IJ	100 singles per box 10 boxes per case

RECOMMENDATIONS FOR USE

- USE: Single use glove. Not suitable for mechanical, thermal, heat, electrical, heavy chemical protection. Do not use near moving machines if there is a risk of entanglement
- STORAGE: Store in dry conditions in the original packaging and away from
- CLEANING: To clean, wipe with a damp cloth. Note: The performance characteristics of worn and laundered gloves may differ from the results shown. Inspect the gloves to ensure no damage is present
- LIFETIME: Service life depends on the glove application and therefore cannot be specified. It is the responsibility of user to ensure the glove is suitable for its intended use

CERTIFICATION LEGENDS



PROTECTION AGAINST MICRO-ORGANISMS EN 374-5

VIRUS = Glove has passed ISO 16604: 2004 (method B)



FOOD CONTACT DIRECTIVE 2002 / 72 EC



RESISTANCE TO CHEMICAL PERMEATION - EN ISO 374:2016

CODE A	CHEMICAL Methanol	CODE J	CHEMICAL n-Heptane Sodium hydroxide 40% Sulphuric acid 96% 65% Nitric acid 99% Acetic acid 25% Ammonium hydroxide 30% Hydrogen peroxide 40% Hydrofluoric acid 37% Formaldehyde	TYPE OF GLOVES	BREAKTHROUGH TIME
B C D	Acetone Acetonitrile Dichloromethane	K L M		А	≥30 min for at least 6 chemicals
E F G	Carbon Disulfide Toluene Diethylamine	N O		В	≥30 min for at least 3 chemicals
H	Tetrahydrofurane Ethyl acetate	S T		С	≥10 min for at least 1 chemical

GLOBUS

EUROPE

www.globusgroup.com

E: sales@globus.co.uk +44 (0)161 877 4747 +44 (0)161 877 4746

MIDDLE EAST & AFRICA

www.globusgroup.com/gcc

E: gcc@globusgroup.com

T: +97148829962 F: +97148829963

Globus Group, T2 Trafford Point, Twining Road, Trafford Park, Manchester, M171SH, UNITED KINGDOM Globus EMEA FZE, Jafza One, Tower A, Office 2201, Jebel Ali, PO Box 61195, Dubai, UNITED ARAB EMIRATES Globus Americas LLC, 45 Rockefeller Plaza, Suite 2000, New York City, New York, 10111, USA

© 2023 Globus (Shetland) Ltd | SKY-DATA-TX324-1123_1

GLOBUS AMERICAS

www.globusgroup.com

E: americas@globusgroup.com

EU Type-Examination Certificate issued by SATRA Technology Europe Limited. Bracetown Business Park. Clonee. D15YN2P. Republic of Ireland. (Notified Body No. 2777)