

INFINITY[®]

MASSAGE CHAIR



INFINITY[®]

72 Start Road, Seabrook, NH 03874
603-910-5000
www.infinitymassagechairs.com

User Manual

GENESIS MAX 4D

Thank you for purchasing this product. Please read this user manual carefully before using to ensure proper installation and operation.

Please keep this USER MANUAL for further reference.

INFINITY Customer Care: 1-603-910-5000

Video Support: infinitymassagechairs.com/assembly-tutorials


CONTENTS

SAFETY PRECAUTIONS	1
NAME AND FUNCTION OF EACH COMPONENT	3
External Structure	3
Internal Structure	3
Remote Control	4
Remote Control Instructions	5
Display Screen Menu Instructions	7
FEATURES	13
APP DOWNLOAD AND INSTALLATION	14
PRELIMINARY CHECKS	15
Installation Site	15
Floor Protection	16
Grounding Instructions	16
Initial Assembly	17
Moving Methods	20
Connect to Power	20
Inspections Before Sitting	21
After Use	21
CARE AND MAINTENANCE	22
TROUBLE SHOOTING	23
TECHNICAL SPECIFICATION	24


SAFETY PRECAUTIONS

- Precautions listed below will help to ensure proper use of this appliance
- The hazards that may be caused by incorrect use are classified into "WARNING" and "CAUTION", please ensure that you read all instructions before using the massage chair.

 **WARNING** Improper operation may cause severe personal injury or substantial property damage.

 **CAUTION** Improper operation may cause personal injury or product damage.

 Prohibited

 Do not Disassemble

Not Suitable For Below Persons

We recommend that those with any of the following conditions should consult your physician before using.

- Those who are using electronic medical instruments embedded in the body, such as a cardiac pacemaker.
- Those undergoing treatment, especially those who are not feeling well.
- Those with malignant tumors or malignant abscesses. (Need to be supervised)
- Women who are pregnant.
- Those with osteoporosis or a fractured spine.
- Those with skin diseases or those with skin injuries.
- Those with a high fever.
- Close supervision is necessary when this product is used by, on, or near children, invalids, or disabled persons.
- The appliance has a heated surface. Persons insensitive to heat must be careful when using the appliance.
- This appliance can be used by children aged 8 years and above and persons with reduced physical, sensory, or mental capabilities, or lack of experience and knowledge, if they have been given supervision or instruction concerning the use of the appliance in a safe way and understand the hazards involved.
- Children shall not play with the appliance.
- Cleaning and user maintenance shall not be made by children without supervision.

Control Buttons Precautions

- Do not operate the control buttons with wet hands.
- Do not spill water or other liquids onto the control buttons to avoid fault.

Cautions Regarding the Adapters and Cord

- Always unplug this appliance from the electrical outlet immediately after using and before cleaning.
- An appliance should never be left unattended when plugged in. Unplug from outlet when not in use.
- Unplug from outlet during thunder or lightning storms, and before putting on or taking off parts.
- Check that the power voltage stated on the rating label corresponds to your local electricity supply.
- Do not use a damp hand to unplug the cord to avoid electric shock.
- Do not drag the cord, reach to the front plastic end to unplug it.
- Do not squeeze, over bend, pull hard, or twist the cord.
- Keep the cord away from heated surfaces.
- Do not carry this appliance by supply cord or use cord as a handle.
- Never operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return the appliance to a service center for examination and repair.
- To disconnect, turn all controls to the off position, then remove plug from outlet.
- The appliance is only to be used with the power supply unit provided with the appliance.
- It must only be supplied at safety extra low voltage corresponding to the marking on the appliance.
- If the supply cord is damaged, it must be replaced by the manufacturer or its service agent or a similarly qualified person in order to avoid a hazard.

Below Form shows the name and content of hazardous substances for this product

Parts Name	Restricted Substances					
	Pb	Hg	Cd	Cr	PBB	PBDE
Outer Panels	○	○	○	○	○	○
Circuit Board Assembly	×	○	○	○	○	○
Metal Work	×	○	○	○	○	○
Rubber Piece	○	○	○	○	○	○
Power Cable/PowerAdapter	×	○	○	○	○	○
Inner Signal Connecting Cable	○	○	○	○	○	○
Upholstery	○	○	○	○	○	○

This form is made under the regulation of SJ/T 11364

※ : Curcuit Board Assemblies include the spare parts which make up a whole PCB, such as resistor, capacitor, integrated circuit, connectors, etc.

○ : Indicates that the concentration of the hazardous substance in all homogeneous materials in the parts is below the relevant threshold of the GB/T26572 standard.

× : Indicates that the concentration of the hazardous substance of at least one of all homogeneous materials in the parts is above the relevant threshold of the GB/T26572 standard.

Notes on environment-friendly lifetime:

Lifetime of this electrical and electronic products, if operated as set forth in the manual, with the hazardous substance or element not to emit, neither to pollute the environment nor to cause harm to person or property, is limited to 10 years.



TECHNICAL SPECIFICATION

Model Number	: Genesis max 4D
Rated Voltage	: 110-120V~
Rated Frequency	: 50/60Hz
Rated Power	: 160W
Safety Design	: Class I
Maximum User Weight	: 300 LBS
Chair Weight	: 323.4 LBS
Rated Current	: 1.55A

SAFETY PRECAUTIONS

Best Circumstances



CAUTION

- Do not operate under blanket or pillow. Excessive heating can occur and cause fire, electric shock, or injury to persons.
- Do not use this appliance when ambient temperature is over 40 C/104 F.
- Women who are pregnant.
- Do not use this appliance in humid or dusty environment, or when any part of the body is in contact with plumbing or any similar ground.
- Never operate the appliance with the air openings blocked. Keep the air openings free of lint, hair, and the like.
- Do not use outdoors.
- Do not operate where aerosol (spray) products are being used or where oxygen is being administered.
- Please place the product on a flat level floor.
- Do not store near heat or open flame.
- Connect this unit to a properly grounded outlet only. See Grounding Instructions.

Best Practices For Safe Use



CAUTION

- Only for users who weigh below 300 lbs.
- Use this product only for its intended use as described in this manual. Do not use attachments not recommended by the manufacturer.
- Appliance should only be used by one person, do not use by two or more persons at the same time to avoid damage to the product.
- Do not use the product right after a meal, it may cause discomfort.
- Remove personal accessories from arms and empty pockets while using the massage services.
- Check the upholstery under the cushions and other spots, if any damages are present, do not use it and refer to a qualified service professional for servicing.
- If you feel any abnormal discomfort during the operation of this product, turn off the power immediately and do not use the product.
- Use of the appliance for 20 minutes each time is highly recommended
- Make sure there are no obstacles behind the chair and there is enough clearance before adjusting the backrest or legrest angle.
- Make sure there is no other excessive load pressed on the backrest or legrest besides the user.
- Make sure there are no children, pets or other objects under the backrest or legrest.
- Please be seated properly while using the product.
- Do not drag or push the chair or roll on its caster on wood floor or carpet to avoid damage




- Do not stand on or stack heavy objects on the footrest.
- Do not fall asleep while the product is operating.
- Avoid using the product under the influence of alcohol.
- Never drop or insert any object into any openings.
- If there is any smoke or unusual smell while using, stop using immediately, and unplug the power cord.




- Do not disassemble this product yourself to avoid failure.
- Do not dismantle, fix or modify the product yourself. Please contact authorized dealer or service agent.

NAME AND FUNCTION OF EACH COMPONENT

External Structure

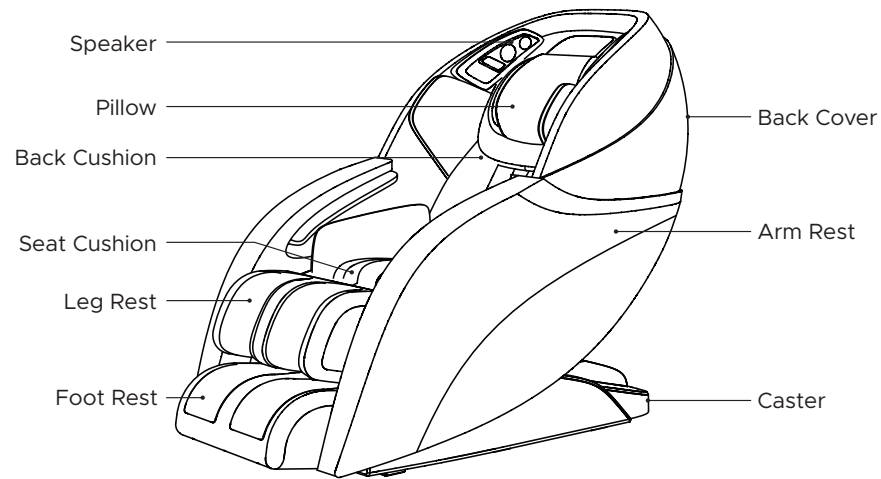

Internal Structure

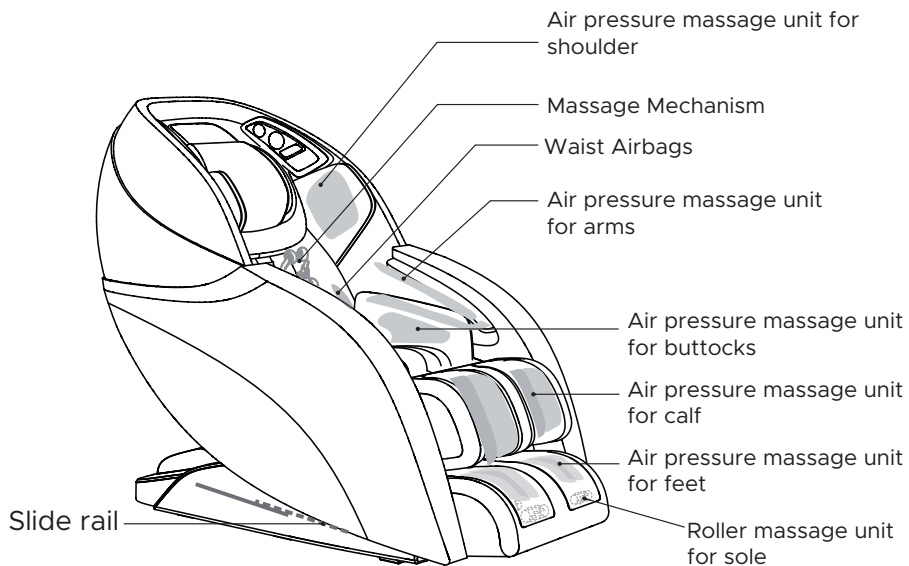

TROUBLE SHOOTING

No.	Symptoms	Possible Causes	Possible Solution
1	Noise is heard while in use	These noises are normal and arise from friction between the massage rollers and the cover	No need to take any measures
2	Massage Chair does not start	The power cord is not secured into the socket	Insert the power plug properly into the power socket
		Not switched ON	Turn on the switch found on chair side. Then push power button on remote
		The fuse is blown	Replace only with a fuse of same type and rating
3	No music comes from the speaker	The volume is turned to the lowest setting	Adjust the speaker volume from the mobile device. Make sure Bluetooth setting on chair and phone are both ON
4	Backrest or footrest can not raise or recline	Excessive load has been pressed on the backrest or legrest	Replace the load and try again
5	Airbag malfunction	The air hose is blocked	Straight the air hose and clear out the blockage
6	Nature sounds are not playing	Bluetooth is OFF	Make sure Bluetooth is set to ON

If problem remains, please contact Infinity for maintenance.

Tips: Do not attempt to repair this product yourself, our company shall not be liable for the injury or damage caused by any servicing without authorization. Thank you for your cooperation.

CARE AND MAINTENANCE

Care and Maintenance Instructions

For best results, use Infinity's Cleaning and Conditioning Kit.
(www.infinitymassagechairs.com/massage-chairs/accessories)

- To treat stains on back panel, armrest, footrest, plastic parts or air hose, spot clean with mild detergent foam, then wipe it with a dry cleaning cloth.
- To treat Stains on controller or power box, spot clean with a dry cleaning cloth.
- To treat stains on back cushion or pillows, use a water-based upholstery cleaner only. Wipe with moist cloth, then air dry.
- To clean the foot/calf unit covers, remove from the chair, hand wash with a mild detergent, then air dry.



- Unplug cord before performing care and maintenance. To avoid electrical shock or injury, do not touch the power cord with wet hands. Make sure no detergent enter into the power box while cleaning.
- Do not use benzene, thinner or other solvents on your product, it may cause fading or damage to the product.
- Do not iron the upholstery.

Storage Methods

- Keep the product free from dust. Cover the product with a dust-free cloth to prevent collection of dust if not in use for a long period of time.



- Do not leave this product exposed to direct sunlight for long periods of time. This may cause fading or damage.

Abnormal Symptoms

During daily use, please check for the following symptoms:

- Burnt smell.
- The product experiences unusual power ON/OFF when touching the power cord
- Hot power cord.
- Other abnormal symptoms




- Please stop using the product to avoid breakdowns or accidents if any above symptom is spotted
- Switch off the power button and disconnect the plug. Contact Infinity for repair.
- All other servicing except above care and maintenance should be serviced by an authorized dealer or service center.



- Pull out the power cord from the power before care and maintenance. To avoid electrical shock or injury, do not handle the power cord with wet hands. Do not try to dismantle or repair the product yourself, please send your massage chair to the authorized service center.



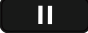
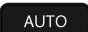


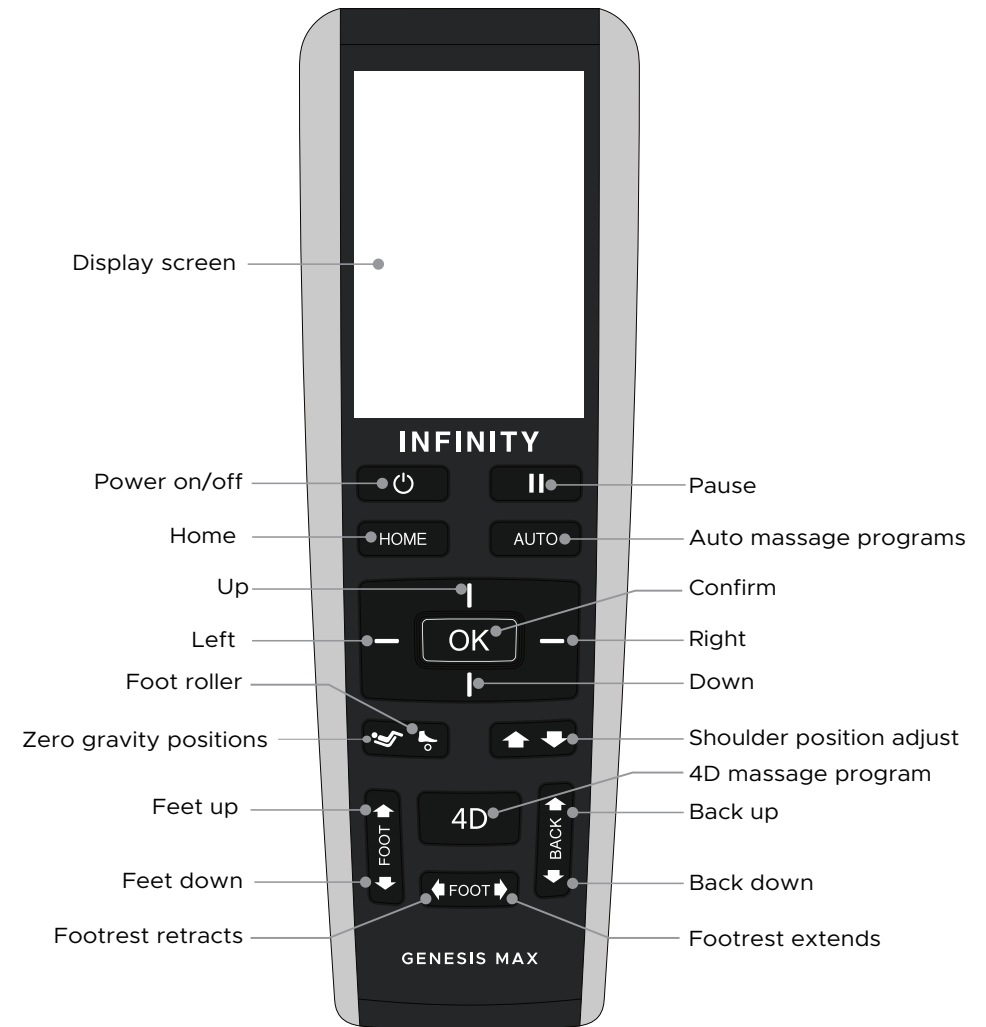

NAME AND FUNCTION OF EACH COMPONENT

Remote Control

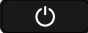

NAME AND FUNCTION OF EACH COMPONENT

Remote Control Instructions

-  **Power ON/OFF:** Press this button to power ON/OFF the controller
-Press this button while connected to power to show the Home display, the display screen lights up, use the up/down/left/right buttons to navigate the screen, press OK to select.
-Press this button again to turn off the product
-  **Home**
-Always returns you to the main Home Screen when pressed.
-  **Pause**
-Press to pause massage chair function, press this button again to continue.
-  **Auto Program**
- When massage chair is operating, press AUTO button to enter Auto Program Menu. Choose from SPORTS REFRESH, EXTENSION, REST & SLEEP, WORKING RELIEF, NECK & SHOULDER, WAIST & SPINE, DEEP SHIATSU, HEALTHY BREATH, MESSAGE EXTEND, RELAXATION, ALL AIR, DEMO.
-  **Navigation**
-Use the up/down/left/right buttons to navigate the display screen, press OK to select.
-While massage program is running, press UP/DOWN button to change the Mechanism massage speed; press LEFT/RIGHT button to change the Airbag massage intensity.
-  **Zero Gravity Position**
-Press to position the chair in Zero Gravity Position 1, press again to position the chair in Zero Gravity Position 2, press for the third time to restore to default recline position.
-  **Foot Roller**
-Press to turn on foot rollers; press again to turn off foot rollers
-  **Shoulder Position Adjust (Moves mechanism up and down)**
-Press and hold on Position UP/DOWN button to adjust the rollers to the desired position while under manual point zone mode.
-Press and hold on Position UP/DOWN button to adjust the rollers to the desired position while under the body scan procedure.

PRELIMINARY CHECKS

Inspections Before Sitting

- Use the remote control to recline the backrest to its lowest position, raise and extend the footrest to its top position, make sure there is enough clearance before commissioning.
- Return the backrest and footrest back to its default position and turn off the power switch.
- Ensure the footrest is well assembled before sitting on the massage chair.
- Ensure the massage chair is returned to its original position before sitting on it. if there is any part not in default position, turn off the massage chair to allow it to restore.



- Do not take a seat while the footrest is lifted.
- To avoid damage, do not put excessive load on the footrest.


After Use

- Always unplug the massage chair from the electrical outlet immediately after use.
- Turn off the power switch and unplug the massage chair from the electrical outlet while not in use to avoid children powering on the massage chair accidentally.
- When not in use for a long period of time, we recommend that the chair be covered the power cord be coiled and that the unit be placed in an environment free of dust and moisture.

PRELIMINARY CHECKS

Move Methods

- Turn off the power switch and unplug the appliance from the electrical outlet.
- Using two people, push down on the backrest and lift up on the front side by grabbing the moving holder (located to the front bottom of the armrest), roll the chair on its casters to the desired location (check the sketch for reference).
- Roll the chair on its casters to the desired location.
- Slowly incline the backrest to upright position.

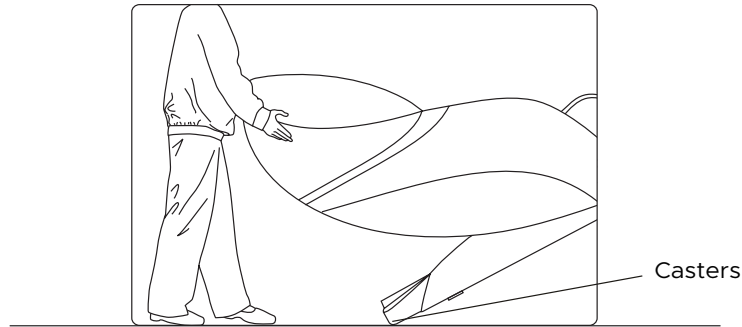

- Do not move the chair while there is a user inside.
- To carry this product, do not lift by the footrest or armrest, hold the chassis and steady the backrest (have 4 or more people to carry the product).

Connect to Power

- Connect this appliance to a 3-wire properly grounded outlet.
- Turn on the main power switch by the right hand side of the massage chair.



- Please verify that there is no damage to the supply cord before switching on the massage chair.
- Please verify that the main power switch is turned OFF before connecting to power

NAME AND FUNCTION OF EACH COMPONENT

4D

4D Massage

- While massage program is running, press this button to change the 4D protruding intensity.
- 4D strength can be see on the In-Use display screen.



Footrest Extends

- Press and hold to extend the footrest, release the button to stop. (Make sure there is no obstruction in advance)

Footrest Retracts

- Press and hold to retract the footrest, release the button to stop.



Feet Up

- Press and hold to raise the legrest, release the button to stop.



Feet Down

- Press and hold to lower the legrest, release the button to stop.



Back Up

- Press and hold to raise the backrest, release the button to stop.



Back Down

- Press and hold to lower the backrest, release the button to stop.




- The remote control will give a single short beep sounds when press the buttons.
- The remote control will give short beeps to indicate that limit position is reached when incline or recline the backrest.
- Please do not play with the backrest incline/recline button to avoid fault.

NAME AND FUNCTION OF EACH COMPONENT

Display Screen Instructions

Auto Massage Program

- Press to display the Auto Program, select among 12 Auto Programs: When massage chair is operating, press AUTO button to enter Auto Program Menu, Choose from SPORTS REFRESH, EXTENSION, REST & SLEEP, WORKING RELIEF, NECK & SHOULDER, WAIST & SPINE, DEEP SHIATSU, HEALTHY BREATH, MESSAGE EXTEND, RELAXATION, ALL AIR, and DEMO. Press OK to enter the program you desire. Selected program will show on the home screen.

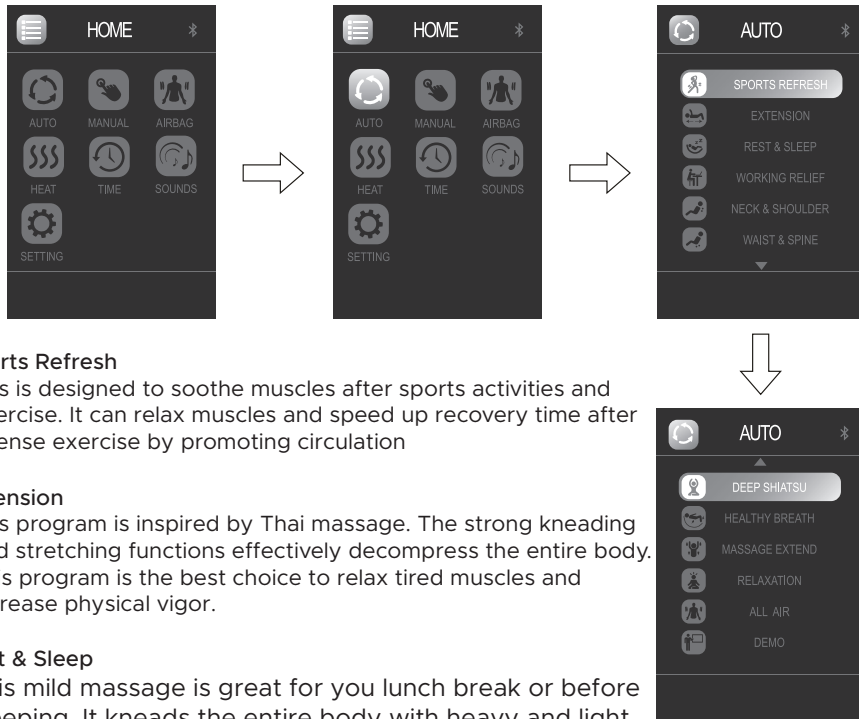

Sports Refresh

-This is designed to soothe muscles after sports activities and exercise. It can relax muscles and speed up recovery time after intense exercise by promoting circulation

Extension

-This program is inspired by Thai massage. The strong kneading and stretching functions effectively decompress the entire body. This program is the best choice to relax tired muscles and increase physical vigor.

Rest & Sleep

-This mild massage is great for you lunch break or before sleeping. It kneads the entire body with heavy and light intensity massage to relieve tired muscles gradually.

Working Relief

- This is a great program for people who work at a desk or travel a lot. It performs a strong massage to ease muscles and recuperate the body.


Neck & Shoulder

- This program relieves neck and shoulder pain, as 80% of the massage time focuses on the neck and shoulders. The curved design of the chair backrest allows the massage rollers to completely touch the shoulder and neck. This function effectively relieves fatigue in these areas.

PRELIMINARY CHECKS

Floor Protection

-We recommend to place a piece of thin carpet or soft mat underneath it to avoid damaging the floor or chair.

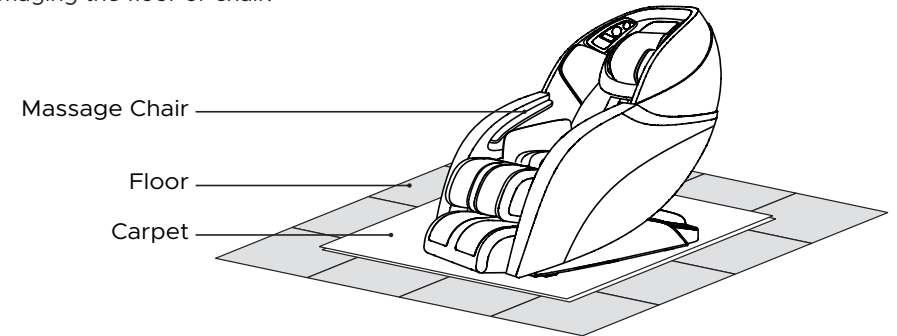

WARNING

- Do not use the massage chair in high moisture environment such as near a swimming pool or bathroom to prevent electric shock accident.
- Please keep the massage chair away from heated surfaces to prevent fire hazard or damage to the product.

Grounding Instructions

-This product must be grounded. If it should malfunction or breakdown, grounding provides a path of least resistance for electric current to reduce the risk of electric shock. This product is equipped with a cord having an equipment-grounding conductor and a grounding plug. The plug must be plugged into an appropriate outlet that is properly installed and grounded in accordance with all local codes and ordinances.



This product must be grounded, the plug must be plugged into an appropriate 3-wire grounded outlet that is properly installed and grounded in accordance with all local codes and ordinances to reduce risk of electrical shock.


PRELIMINARY CHECKS

Installation Site

Clearance space for the massage chair:

-Ensure there is an adequate clearance for the installation:It needs 4” clearance for the recline of the backrest: It needs 24” of clearance space for the extension of the footrest and sliding forward of the chair.

Keep it 3 feet away from the television, radio, or other audio and video appliance to avoid signal interference.

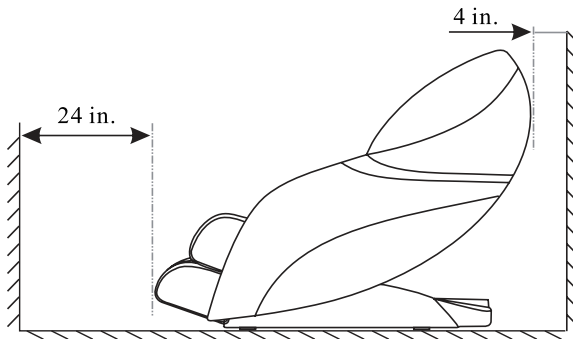

CAUTION

Keep the distance between the power socket and power switch within 60” to avoid damage of power cord while massage chair is sliding forward.

Straighten the power cord while in use to avoid tangle or damage.

When the massage chair is sliding forwards or back, please check to make sure there are no children, pets or other obstacles around to avoid accident.

NAME AND FUNCTION OF EACH COMPONENT

Waist & Spine

-To relieve lower back pain, 80% of this massage focuses on the lumbar vertebra and the rollers will conduct targeted massage to relieve tension in this area.

Healthy Breath

-Rhythmic five-minute massage to aid with deep breathing

Deep Shiatsu

-Deep shiatsu on the back for 10 minutes.

Massage Extend

-Designed for morning or midday relaxing, it helps to boost the viability of muscles.

Relaxation

-Selected neck & shoulder massage techniques to soothing pain and tension in the neck and shoulders.

All Air

-A full-body airbag-only massage.

Demo


-A quick show of massage chair's features and functions. That can be accompanied by a explanatory voice over.

-The voice can be turned off and on in SOUNDS.

Display Screen Instructions


Manual Programs

-Press Manual from Home display, press UP or DOWN button to navigate among: MODE, PART, WIDTH, SPEED, and ROLLER.

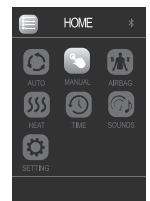

Massage Mode

-Press MODE from MANUAL display, press RIGHT button then UP or DOWN button to navigate among: Kneading, Tapping, Knocking, Shiatsu, Sync, and 4D Shiatsu massage techniques. Press “OK” to enter the setting you desire. Selected mode will show on the home screen.

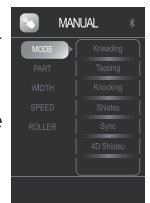

Back Mechanism Massage Part

- Press PART from MANUAL display, press RIGHT button then press UP or DOWN button to navigate among: Whole, Partial, and Point massage regions. Press “OK” to activate your desired setting.

-Whole travels the entire length of the track, Partial travels a short distance and reverses direction for focused relief, Point stops on the track for targeted relief. Point location can be fine tuned using the arrow up ▲ or arrow down ▼ buttons.



NAME AND FUNCTION OF EACH COMPONENT

Back Mechanism Massage Width


-Press WIDTH from MANUAL display, press RIGHT button then press UP or DOWN button to navigate among: Wide, Medium, and Narrow mechanism roller width. Press "OK" button to enter the setting you desire. Selected Width will show on the in-use screen. (Back Mechanism massage Width can't adjust when under Kneading, or Sync Manual modes)

Mechanism Massage Speed

-Press SPEED from MANUAL display, press RIGHT button then press UP or DOWN button to navigate among: level 1, 2, 3, 4, 5, 6, Mechanism massage speed. Press "OK" button to enter the speed level you desire. (Back Mechanism massage Width can't adjust when under Shiatsu, 3D Shiatsu Manual mode)

Foot Roller Speed

-Press ROLLER from MANUAL display, press RIGHT button then press UP or DOWN button to navigate among: foot roller speed level 1, 2, 3, or OFF (deactivate foot roller). Press "OK" button to enter the speed level you desire.

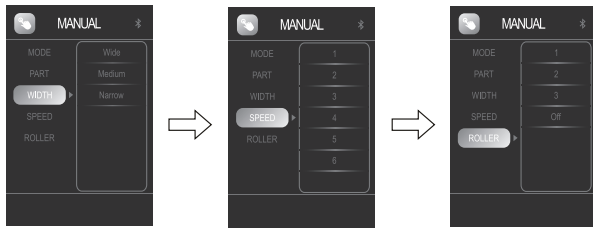


Air Pressure Massage Airbag

Part-Air Pressure Region

-Press PART from AIRBAG display, press RIGHT button then press UP or DOWN button to navigate among: Full, Arm & Shoulder, Back & Waist, Leg & Foot, Buttock air bag positions. Press "OK" button to enter the airbag position you desire.
-FULL defaults to all airbags on. To turn all airbags off, select FULL a second time so no selections are lit.

Force-Air Pressure Intensity

-Press FORCE from AIRBAG display, press RIGHT button then press UP or DOWN button to navigate among: level 1, 2, 3, 4, 5, airbag intensity, or OFF (deactivate airbag intensity function), Press "OK" button to enter the airbag setting you desire.
-Level 5 is the most intense airbag setting.

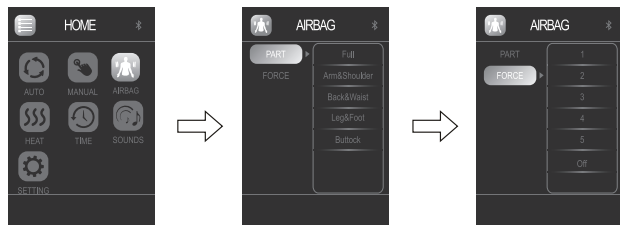

APP DOWNLOAD AND INSTALLATION

Apple iOS: (iPhone or iPad devices)

-Open the App Store . At the top right corner, search for keywords like "Infinity Massage Chair," "Genesis Max," or "Massage Chair" to find the Apple app. Click "Get" at the top right corner, to download and install the software.

-In your device settings, open Bluetooth and connect it to the massage chair.

-**Please note:** Only Apple products, compatible with Bluetooth 4.0 can connect to the chair's Bluetooth.

Android System Software (Android System V2.2 or later)

-Download the Android app from the Infinity Massage Chairs Google Play store.

-Install Genesis Max APK, then press Settings button on your Android device > then press Wireless and Web settings > Bluetooth > Scan Bluetooth device > Genesis Max > then Pair.

-Start the APK, press Settings menu > Bluetooth > and pair with the Genesis Max to link the APK.

Bluetooth Connection Method

Bluetooth-Enabled Audio Devices

-This product has Bluetooth compatibility to connect with audio devices.

-Open Bluetooth from audio device and pair with Genesis Max 4D to play music.


CAUTION

When connecting your Bluetooth-enabled device, please make sure the chair and Bluetooth setting is turned on, otherwise your device will not be able to find the chair.

FEATURES

SPACE-SAVING ZERO-WALL FUNCTION

-The Genesis Max 4D massage chair has the ability to move forward before performing a massage. This allows it to be placed closer to the wall than a standard massage chair. Keep this in mind when positioning the chair.

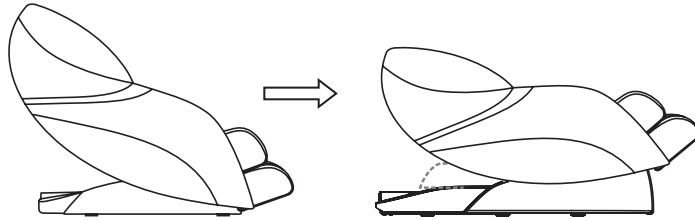

ZERO GRAVITY

-Zero gravity creates a completely relaxing naturally comfortable massage position. This design places the back, buttocks, and legs, in the same angle that astronauts experience in space. Zero gravity provides a sense of weightlessness and promotes circulation and a steady heart rate.

4D MASSAGE MECHANISM

-The new design of the 4D mechanism allows users to adjust the depth of their massage intensity.

AUTO/MANUAL FOOTREST EXTENSION

-Footrest can extend 3 inches, to accommodate different leg lengths in both Auto and Manual Modes.

Airbag Massage

-Airbags are located throughout the chair in order to massage the arms, back, waist, shoulders, buttock, legs, and feet.

FOOT MASSAGE ROLLERS

-Three massage rollers in each foot unit conduct an all-over shiatsu massage on the sole.

DIVERSIFIED MASSAGE TECHNIQUES

-Massage techniques include: knocking, kneading, tapping, kneading/knocking, combination, shiatsu, and kneading on the leg, which imitates a professional massage therapist.

BLUETOOTH AUDIO

-The massage chair includes Bluetooth compatibility. Easily play music by connecting your Android or iOS device.


CALF MASSAGE

-Airbags surround each side of the lower legs while massage rollers knead the calves for extreme relaxation.

NAME AND FUNCTION OF EACH COMPONENTS

Heat

-Press HEAT from the Home Menu, press RIGHT button, then press UP or Down button to navigate among: 1,2,3 heat levels or OFF. (This deactivates the heat function)
Press OK to select the desired heat setting.
-Level 3 has the most heat.

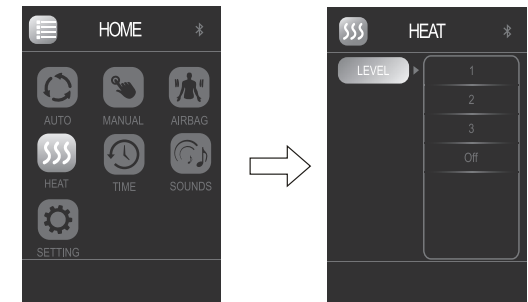


Time

-Press TIME from the Home Menu to show available timer: 10 mins, 20 mins, or 30 mins. Press RIGHT then press UP or DOWN button to navigate. Press OK button to enter the time you desire.



Sounds

-Press SOUNDS from HOME display, press RIGHT then press UP or DOWN to navigate among: SOUND, VOLUME, BEEP, AND DEMO VOICE press OK button to select the sound setting to control.

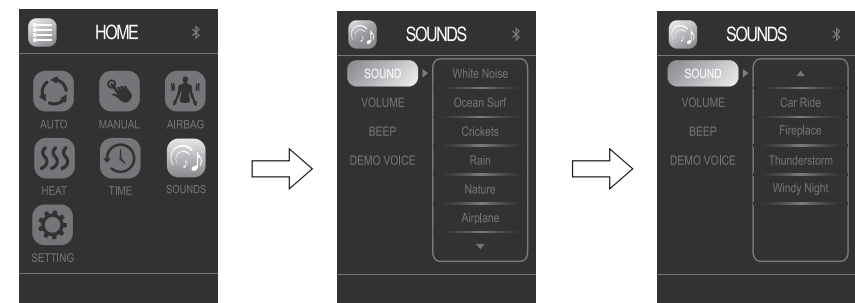

Sound

-Press SOUND from SOUNDS menu, press RIGHT then press UP or DOWN to navigate among: White Noise, Ocean Surf, Crickets, Rain, Nature, Airplane, Car Ride, Fireplace, Thunderstorm, or Windy Night. Press OK to Select the sound setting you desire.


Note

-Bluetooth must be ON for SOUND to work.

NAME AND FUNCTION OF EACH COMPONENTS


Volume

-Press VOLUME from SOUNDS menu, then press RIGHT then UP or DOWN to navigate among: 1, 2, 3, 4, or 5 volume settings, press OK to select the volume setting you desire.

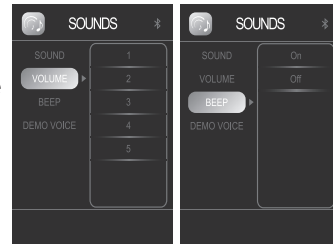


Beep

-Press BEEP from SOUNDS display, then press RIGHT. Use UP or DOWN to select ON or OFF for the remote control beep.

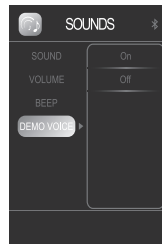

DEMO Voice

-Press DEMO VOICE from SOUNDS display, press RIGHT button, then press UP or DOWN to turn ON or OFF DEMO voice. Voice demo only works on the DEMO program.

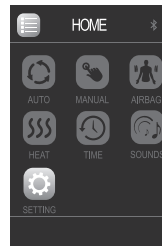


Setting

-Press HOME button, then choose SETTING. Use the up or down button to select LANGUAGE, VOICE, BRIGHTNESS, BUTTONS, BEEP, BLUETOOTH, or SLEEP mode settings.

LANGUAGE

-In the SETTING MENU select LANGUAGE. Press right button, then press up or down button to set the chairs language to CHINESE, ENGLISH, JAPANESE or KOREAN.

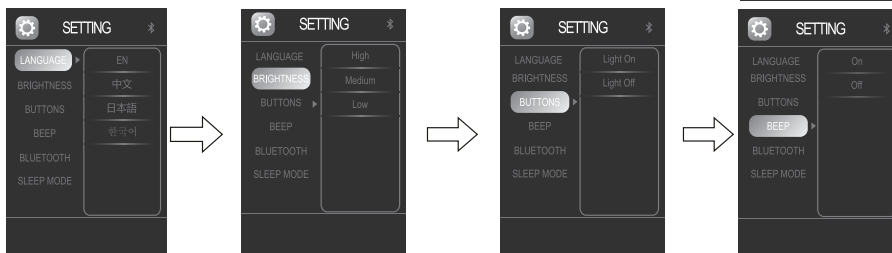

Brightness

-In the SETTING menu, select BRIGHTNESS, press RIGHT, then UP or DOWN to select the display screen brightness. Press OK to set the desired setting.

Buttons

-In the SETTING menu, select BUTTONS, press RIGHT then use UP or DOWN to toggle the remote backlight settings. Press OK to set the desired setting.

Beep

-In the SETTING menu, Select BEEP, Press UP or DOWN to START or STOP the remote beep sound.

NAME AND FUNCTION OF EACH COMPONENTS

Bluetooth

-In the SETTING menu, select BLUETOOTH, press UP or DOWN to turn on or off Bluetooth. Press OK to set the desired setting.




Sleep Mode


-In the SETTING menu, select SLEEP MODE, press RIGHT then select ON to stop the chair in the reclined position after Auto Programs. Select OFF to have the chair return fully upright after each Auto Program.

Note

-Under both Auto Mode and Manual Mode, the remote will return to the In-Use screen, as shown below, if it remains idle for 10 seconds.

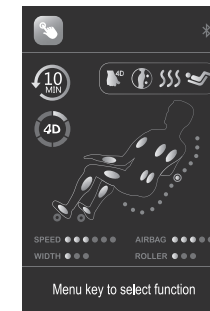

In-Use page in Manual Mode

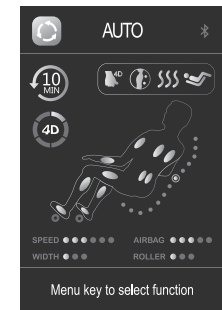

In-Use page in Auto Mode



Note:

Please do not spill water or other liquids on the controller in order to avoid key malfunctions and error codes. Please do not place heavy objects on the controller or power cord.