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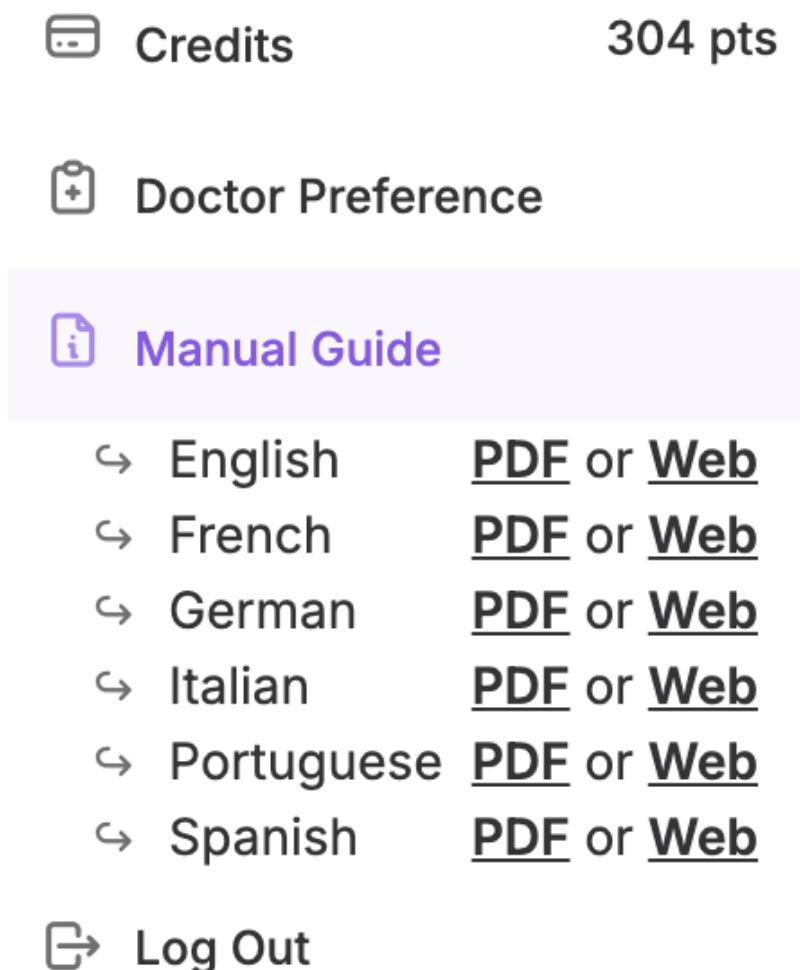
Inhalt

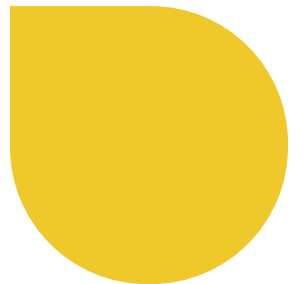
DR. ANNIE™ INSIGHTS

Here to provide personalized assistance and guidance to help you address any dental concerns you may have.

- ✓ AI Dental Findings
- ✓ Report Generating
- ✓ Timepoint Feature
- ✓ AnnieInsight+

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- **SIGN IN WITH GOOGLE SSO**
- **EMAIL & PASSWORD**

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password


Sign In

New to iSmart? [Create an account here](#)

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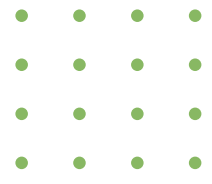The screenshot shows the 'iSmartOffice - Provider' registration page. At the top right, there are links for 'Register' and 'Login'. The main section is titled 'PROVIDER REGISTRATION FORM' and contains several input fields with labels in red: 'First Name', 'Last Name', 'Username' (with a hint 'Use email address as your username'), 'Password', 'Retype Password', and 'Referral Code' (with a hint 'if any'). At the bottom right of the form, there are two buttons: an orange 'Back' button and a blue 'Register' button. A red arrow points to the 'Register' button. In the bottom right corner of the page, there is a small red circular icon with a white bell.

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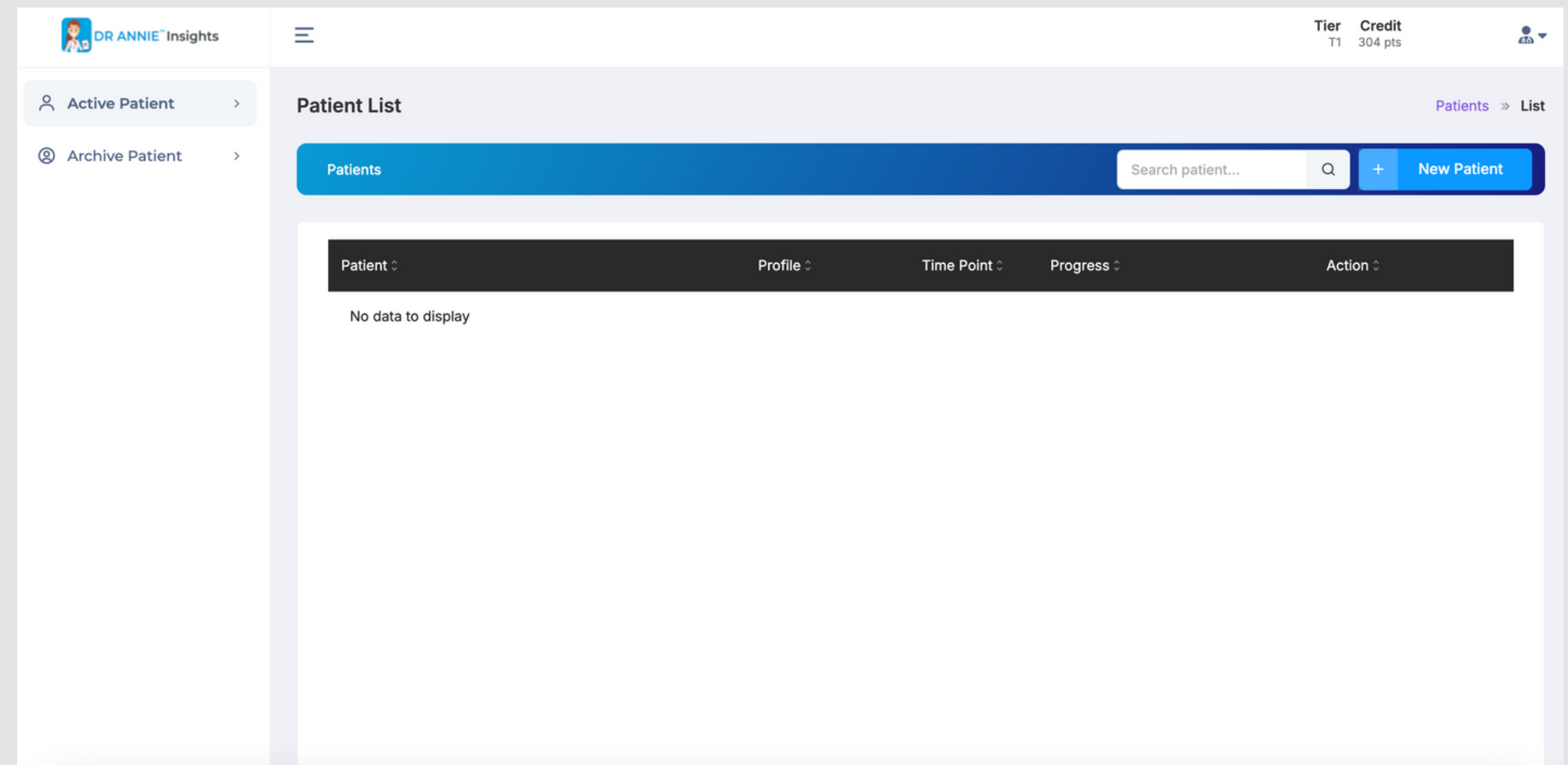
- **LEFT: ACTIVE & ARCHIVE PATIENT**
- **CENTER: PATIENT LIST**

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Before adding the patient's name, first add a credit so that DR. ANNIE™ can continue the analysis process.

For details, please see the [User Credit](#) or [Top Up](#) Section

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
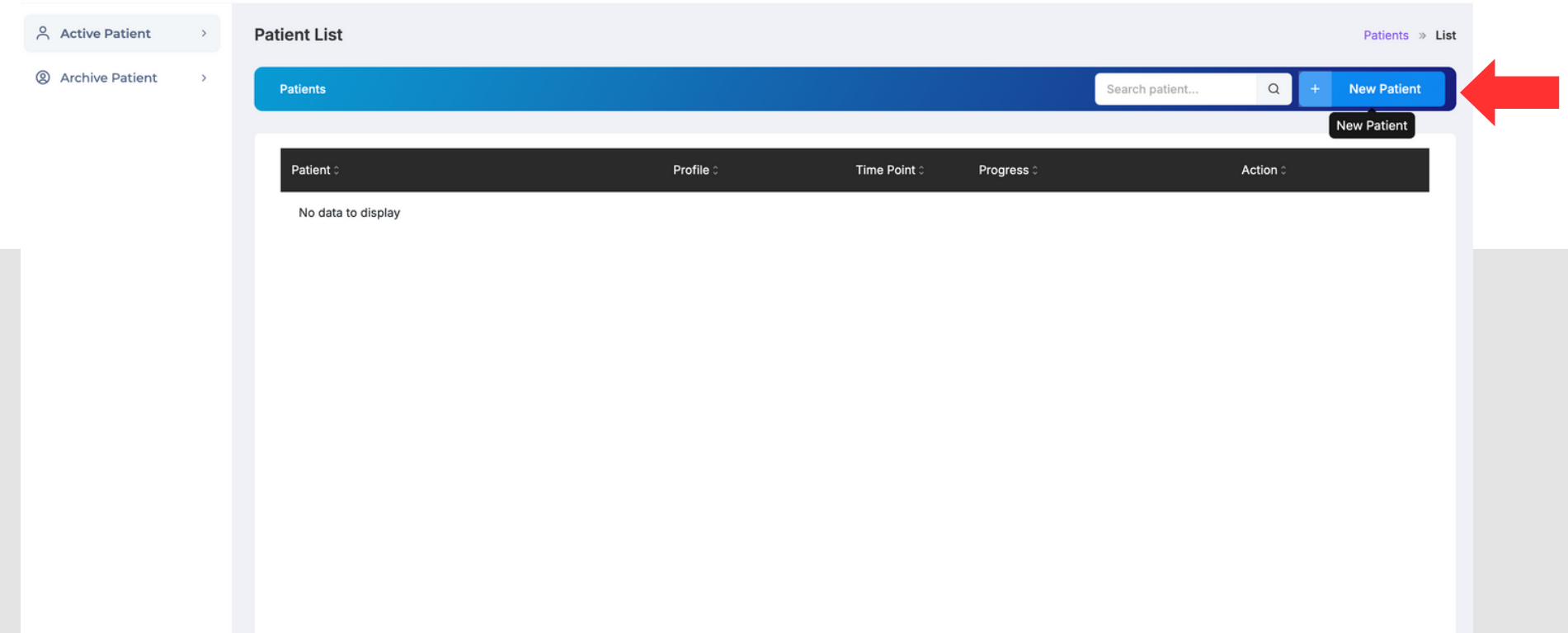
The screenshot shows the 'Patient List' interface. At the top, there are tabs for 'Active Patient' and 'Archive Patient'. Below the tabs, there is a search bar and a '+ New Patient' button, which is highlighted with a red arrow. The table below the search bar is empty, showing 'No data to display'.

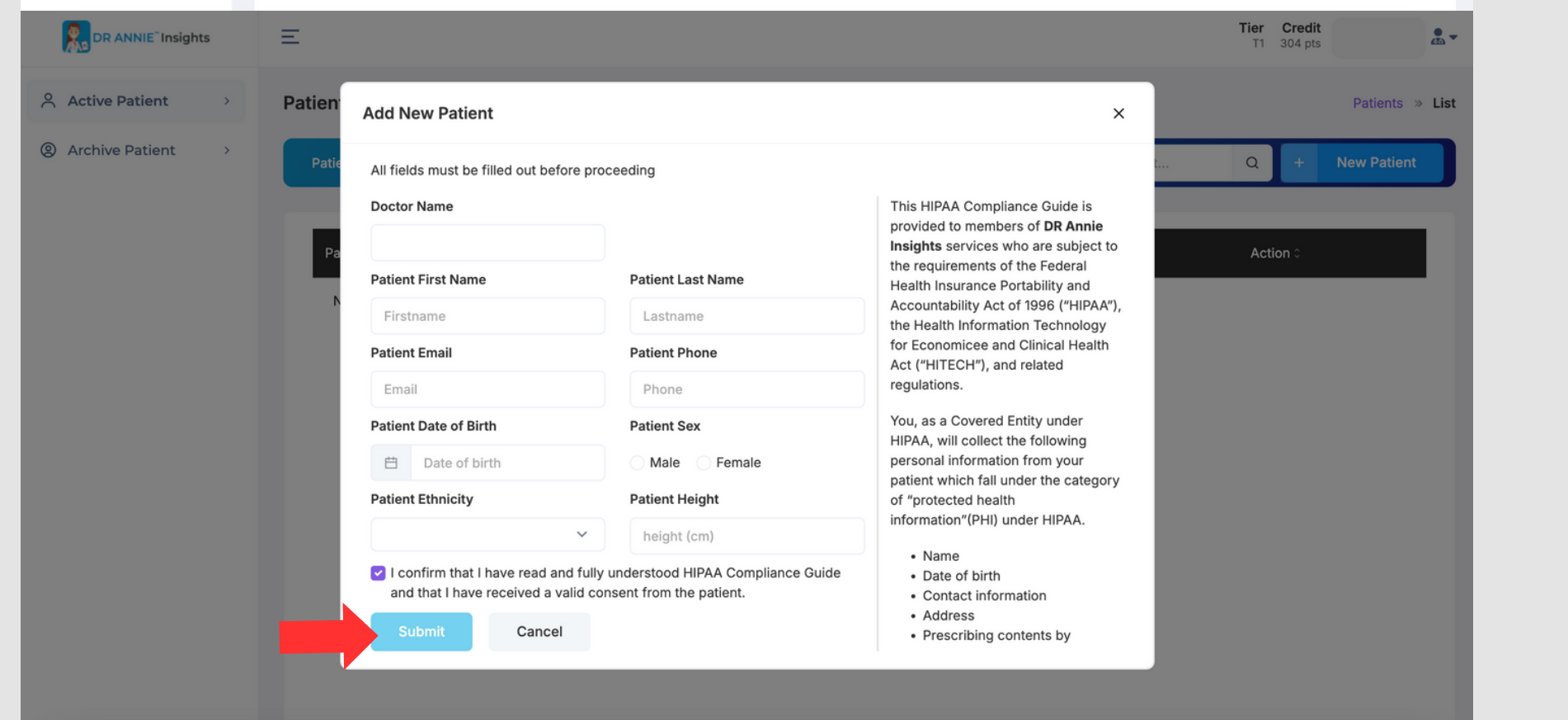The screenshot shows the 'Add New Patient' form. The form has a title bar 'Add New Patient' and a close button. Below the title bar, there is a message: 'All fields must be filled out before proceeding'. The form contains several input fields: 'Doctor Name', 'Patient First Name' (with a 'Firstname' placeholder), 'Patient Last Name' (with a 'Lastname' placeholder), 'Patient Email' (with an 'Email' placeholder), 'Patient Phone' (with a 'Phone' placeholder), 'Patient Date of Birth' (with a 'Date of birth' placeholder), 'Patient Sex' (with radio buttons for 'Male' and 'Female'), 'Patient Ethnicity' (with a dropdown menu), and 'Patient Height' (with a 'height (cm)' placeholder). At the bottom of the form, there is a checkbox with the text: 'I confirm that I have read and fully understood HIPAA Compliance Guide and that I have received a valid consent from the patient.' Below the checkbox are two buttons: 'Submit' and 'Cancel'. A red arrow points to the 'Submit' button. To the right of the form, there is a sidebar with a 'HIPAA Compliance Guide' section, which includes a list of items: 'Name', 'Date of birth', 'Contact information', 'Address', and 'Prescribing contents by'.

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Dental Chart

The screenshot shows the 'Dental Chart' interface. It has a top bar with 'Upper Arch' and 'Lower Arch' sections. Below these are grids for 'Permanent Teeth' and 'Deciduous Teeth'. Red arrows point to the 'Upper Arch' and 'Lower Arch' labels. Below the grids is a list of teeth (T01 to T10) and a 'Disclaimer' box.

Disclaimer:

- Certain dental condition, such as extreme missing teeth or anomalies, may not be accurately detected by the current version of our AI engine. Verification by a doctor is needed.
- Please note: Sequence Worksheet report will not be available if there are missing 1st molar(s) OR missing central incisor(s).

Prev Next

Dental Chart

The screenshot shows the 'Dental Chart' interface with two pop-up windows. The 'Arch Parameter' window is on the left, and the 'Tooth Status - Permanent - 01' window is on the right. Red numbers 1 and 2 are placed near the windows.

Arch Parameter

Single Arch Parameters

- ☒ None
- ☐ Partial denture
- ☐ Complete denture
- ☐ Denture cleaning
- ☐ Orthodontic

Save Cancel

Tooth Status - Permanent - 01

Adult No. 01

Tooth Parameter

- ☒ None
- ☐ Missing (this option will be used on report generation)
- ☐ Extraction needed
- ☐ Restoration crown needed

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Continue...

1 Intra Oral Photos 2 Extra Oral Photos 3 Dental Chart 4 X-Rays/Radiographs AI - Findings Reports

X-Rays / Radiographs

Lateral Cephalometric
Jan 16, 2025 9:54 AM

Frontal Cephalometric
Feb 14, 2025 2:52 AM

Dental Panoramic
Jan 16, 2025 9:55 AM

Step 9: Upload X-Ray Images & HLD Form / X-Ray-Bilder und HLD-Formular hochladen (2)

Warning: The automated HLD calculation will only work if you have uploaded the lateral image, frontal image, panoramic X-ray, and upper & lower arch images. Please ensure all required images are uploaded before proceeding with the form.

3

HLD Form

HLD Summary

===== HLD Summary =====

1. Is there an impacted maxillary central incisor? : No
2. What is the condition of the patient oral hygiene? : Good
3. Does the patient does have any dental deformity? : No
Detail :
4. Does the patient have any traumatic deviations? : No
Detail :

4

← Prev

Next →

HLD Form

Is there an impacted maxillary central incisor?

☒ No ☐ Yes

What is the condition of the patient's oral hygiene?

☐ Poor ☒ Good ☐ Excellent

Does the patient have any dental deformity?

☒ No ☐ Yes

Please describe the dental deformity in detail, if present.

Does the patient have any severe traumatic deviations?

☒ No ☐ Yes

If applicable, provide a brief description of the severe traumatic deviations.

Does the patient have surgical malocclusion?

☒ No ☐ Yes

What is the number of ectopic eruption teeth present?

0

Is there a posterior crossbite, and does at least one of the affected teeth have to be a molar?

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Continue...

The screenshot displays the 'AI-Findings' interface. At the top, a progress bar shows five steps: 1. Intra Oral Photos, 2. Extra Oral Photos, 3. Dental Chart, 4. X-Rays/Radiographs, and 5. AI - Findings (highlighted with an orange icon). A 'Reports' icon is at the end. On the left, a vertical sidebar shows icons for '+', '2', and '1'. The main content area is titled 'Intra Oral' and contains a grid of six dental images. Each image has a label and a green checkmark indicating verification status: 'Intra Oral Right' (Not available), 'Intra Oral Front' (Not available), 'Intra Oral Left' (Not available), 'Upper Arch' (Not available), and 'Lower Arch' (Not available). Below the grid, there are expandable sections for 'Extra Oral', 'Panoramic Radiographs', 'Cephalometric (Lateral & Frontal)', and 'Dental Arch'. A red-bordered warning box at the bottom states: 'Warning: Please verify the Cephalometric and Dental Arch sections before proceeding to the next step. Click the yellow 'Verify' button located below the Cephalometric and Dental Arch images to confirm the results. Ensure that you carefully review both the Cephalometric (Lateral & Frontal) and Dental Arch (Upper & Lower) sections, as they are critical components of treatment planning and staging. The AI-generated report will be based on this information. For detailed instructions, please refer to the Instruction Manual'. At the very bottom, there are 'Prev' and 'Next' navigation buttons.

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The screenshot displays a web interface for generating dental reports. At the top, a progress bar shows five steps: 1. Intra Oral Photos, 2. Extra Oral Photos, 3. Dental Chart, 4. X-Rays/Radiographs, and 5. AI - Findings. Below the progress bar, there are four buttons: 'Generate Standard Report', 'Generate Standard Plus Report', 'Generate Complete Report', and 'VIP Report Service'. A table below these buttons lists various report types with their creation dates and download actions.

Report Name	Date Create	Action
01 - Dental Evaluation	Apr 28, 2025, 10:01	
02 - Problem List	Apr 28, 2025, 10:01	
03 - Treatment Rationale	Apr 28, 2025, 10:01	
04 - Case Evaluation	Apr 28, 2025, 10:01	
05 - Dental Findings	Apr 28, 2025, 10:01	
06 - Skeletal Findings	Apr 28, 2025, 10:01	
The following reports are full comprehensive advanced reporting, will be available within 10-15 minutes. Thank you for your patience		
07 - Tracing Measurement	Apr 28, 2025, 10:02	
08 - Sequences Worksheet	Apr 28, 2025, 10:02	
09 - Full Comprehensive	Apr 28, 2025, 10:02	
10 - HLD Summary	Apr 28, 2025, 10:01	

At the bottom left of the table, there are 'Prev' and 'Next' navigation buttons.

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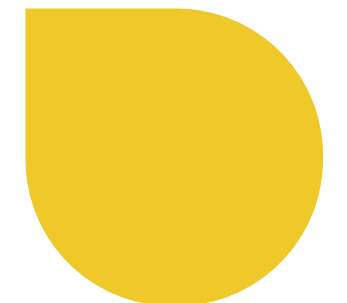
Dental Panoramic

No Image

By continuing, you agree to the VIP Report processing fee and acknowledge that any previous report will be overwritten. Are you sure to submit the data?

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1. DENTAL EVALUATION
2. PROBLEM LIST
3. TREATMENT RATIONALE
4. CASE EVALUATION
5. DENTAL FINDINGS
6. SKELETAL FINDINGS
7. TRACING MEASUREMENTS
8. SEQUENCES WORKSHEET
9. FULL COMPREHENSIVE ANALYSIS
10. HLD SUMMARY

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Lower difficulty

B. Orthodontic (Tooth Alignment):

Some teeth alignment are needed (esthetic)

C. Periodontic (Gum Health):

Some dental/ oral hygiene maintenance is recommended

D. Restorative (Cavity or Fillings):

Some dental/ oral restoration is recommended (possible filling, cavities)

E. Cost Estimation:

The total cost estimation based on these assessment is 1 out of 4 (affordable)

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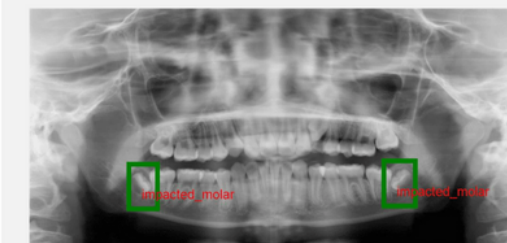
Bolton 12 ratio: 1.058

Airway Analysis

If the patient is clinically
diagnosed as a
mouthbreather, observed
mouthbreathing is probably
not caused by adenoid
blockage of the airway.

Transverse

No data available

Problem List

Problem List

Significant Considerations

Horizontal

Class II malocclusion 1

due to the lower molar 18

Severe Overjet 5

Vertical

Deep Bite 7

Skeletal Deep Bite 15

due to the mandible &
palate 39

Symmetry

Midline Asymmetry 10

Special Consideration

Upper anterior teeth
relatively small.

Upper arch tooth mass
relatively small.

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- Initial conditions
- Maximum permissible arch length increase (within doctor limits) due to contributing factors
- Resultant arch length discrepancy considering all possible increases (C + G)
- Final computer-generated decision
- Work-up presented is

Continue...

Treatment Rationale

Original Condition

Facial Pattern: -0.0 CD - Mesofacial

Lower Arch form: Ovoid

Lower arch length discrepancy (ALD): 1.2 mm SHORTAGE

Leeway Space: 0.0

Max use of leeway space: 0.0 mm

Total arch length discrepancy: 1.2 mm SHORTAGE

Computer Decision

UPPER ARCH: NON-EXTRACTION

Convexity Change: REDUCE 1.0 mm

LOWER ARCH: NON-EXTRACTION

Lower Incisor: FORWARD 0.3 mm

Buccal Expansion: GAIN 0.5 mm

Lower Molar: FORWARD 0.0 mm

Auxiliary Appliances

Activator: NOT INDICATED

Palate Separation: NOT INDICATED

Headgear: CERVICAL

Headgear: LIGHT FORCES

Post Treatment

Pentamorphic Arch Form: Ovoid

Arch Length Relapse: 2.3 mm

Lower third molar probabilities

Impaction: 15 %

Marginal: 43 %

Functional: 42 %

Treatment Rationale

1. Initial Conditions

A. Original Arch Length1.2 mm shortage

B. Useable Leeway Space0.0 mm

C. Total Initial Discrepancy(A+B)1.2 mm shortage

2. Maximum Permissible Arch Length Increase (Within Doctor Limits) Due To

D. Lower Incisor Repositioning4.2 mm increase

E. Buccal Expansion0.5 mm increase

F. Lower Molar Distal Movement0.0 mm

G. Total Possible Increase (D+E+F)4.7 mm increase

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- Convexity change
- Upper incisor tip movement for overbite/overjet ideal to lower
- First molar movement required
- First molar movement clinical limit
- Work-Up Presented Is

Treatment Rationale

REASON FOR UPPER ARCH DECISION

1. Convexity change	1.0 mm reduce
2. Upper incisor tip movement for overbite/overjet ideal to lower	5.0 mm backward
3. First molar movement required	1.5 mm backward
4. First molar movement clinical limit	5.8 mm backward
5. Work-Up Presented Is	NON-EXTRACTION

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Upper Analysis	
Movement of First Molar (non-ext.) Required for Class I	1.0 mm Distal
Clinical Limit	
Resulting Expected Space for 2nd & 3rd Molars at Maturity (non-ext.)	19.6 mm
Required Space for 2nd Molars	9.0 mm to 11.0 mm
Required Space for 2nd & 3rd Molars	18.0 mm to 22.0 mm

Upper Analysis	
Upper Arch	NON-EXTRACTION
Lower Arch	NON-EXTRACTION
Lower Incisor	Forward 0.3 mm
Buccal Expansion	Gain 0.5 mm
Lower Molar Movement	Forward 0.0 mm

Case Evaluation Detail

Auxiliary Appliances

Headgear:

CERVICAL

Activator:

NOT INDICATED

Palate Separation:

NOT INDICATED

Convexity Objective:

Reduce 1.0 mm

Lower Arch Length Discrepancy (original arch), Including useable leeway (E) space::

1.2 mm Shortage

Probability of Lower Third Molar

Impaction:

15.%

Eruption Questionable Function:

43.%

Eruption Functional:

42.%

Lower Arch

LOWER ANALYSIS	REQUIRED TOOTH MOVEMENT (LT)	REQUIRED TOOTH MOVEMENT (RT)	EFFECT ON ARCH LENGTH	RESULTING DISCREPANCY
Lower Incisor to Ideal	0.2 mm Bwd.	0.4 mm Bwd.	0.8 mm Decrease	2.0 mm Shortage

Teeth Type																
TEETH TYPE	L6	L5	L4	L3	L2	L1	R1	R2	R3	R4	R5	R6	TOTAL	SUM OF INCISOR	NORM	
Lower Arch	11.4	8.0	7.7	7.6	6.3	5.7	5.7	6.3	7.5	7.8	7.5	11.7	93.2	24.0 MM	22.2	
Upper Arch	8.9	6.7	6.9	6.4	6.8	7.9	8.1	6.4	7.5	7.2	6.3	9.0	88.1	29.2 MM	N/A	

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- Dental Relations
- Dental to Skeleton
- Esthetics
- Nasopharyngeal Airway
- Frontal Before Treatment

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- Dental Relations
- Dental to Skeleton

Dental Cephalometric Finding

Dental Cephalometric Finding

Lateral Before Treatment

Dental Relations

FACTOR	MEASURED VALUE	CLINICAL NORM	DEVIATION FROM	DEVIATION NORM
01 Molar Relation	1.0 mm	-3.0 mm	1.5	*
03 Canine Relation	0.9 mm	-2.0 mm	1.4	*
05 Incisor Overjet	10.5 mm	2.5 mm	3.2	***
07 Incisor Overbite	5.9 mm	2.5 mm	1.7	*
09 Mand Incisor Extrusion	4.7 mm	1.2 mm	1.7	*
#11 Interincisal Angle	114.5 dg	121.0 dg	-1.1	*

Dental Cephalometric Finding

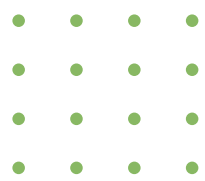
Dental Cephalometric Finding

Frontal Before Treatment

Dental Relations

FACTOR	MEASURED VALUE	CLINICAL NORM	DEVIATION FROM	DEVIATION NORM
02 Molar Relation Left (A6-B6)	0.9 mm	0.0 mm	0.5	
04 Molar Relation Right (A6-B6)	-0.2 mm	0.0 mm	-0.1	
#06 Intermolar Width (B6-B6)	56.6 mm	64.0 mm	-3.7	***
08 Inter canine Width (B3-B3)	26.5 mm	29.1 mm	-1.1	*
#10 Denture Midline	-2.1 mm	0.0 mm	-1.4	*

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Jaw to Cranium

FACTOR	MEASURED VALUE	CLINICAL NORM	DEVIATION FROM	DEVIATION NORM
--------	----------------	---------------	----------------	----------------

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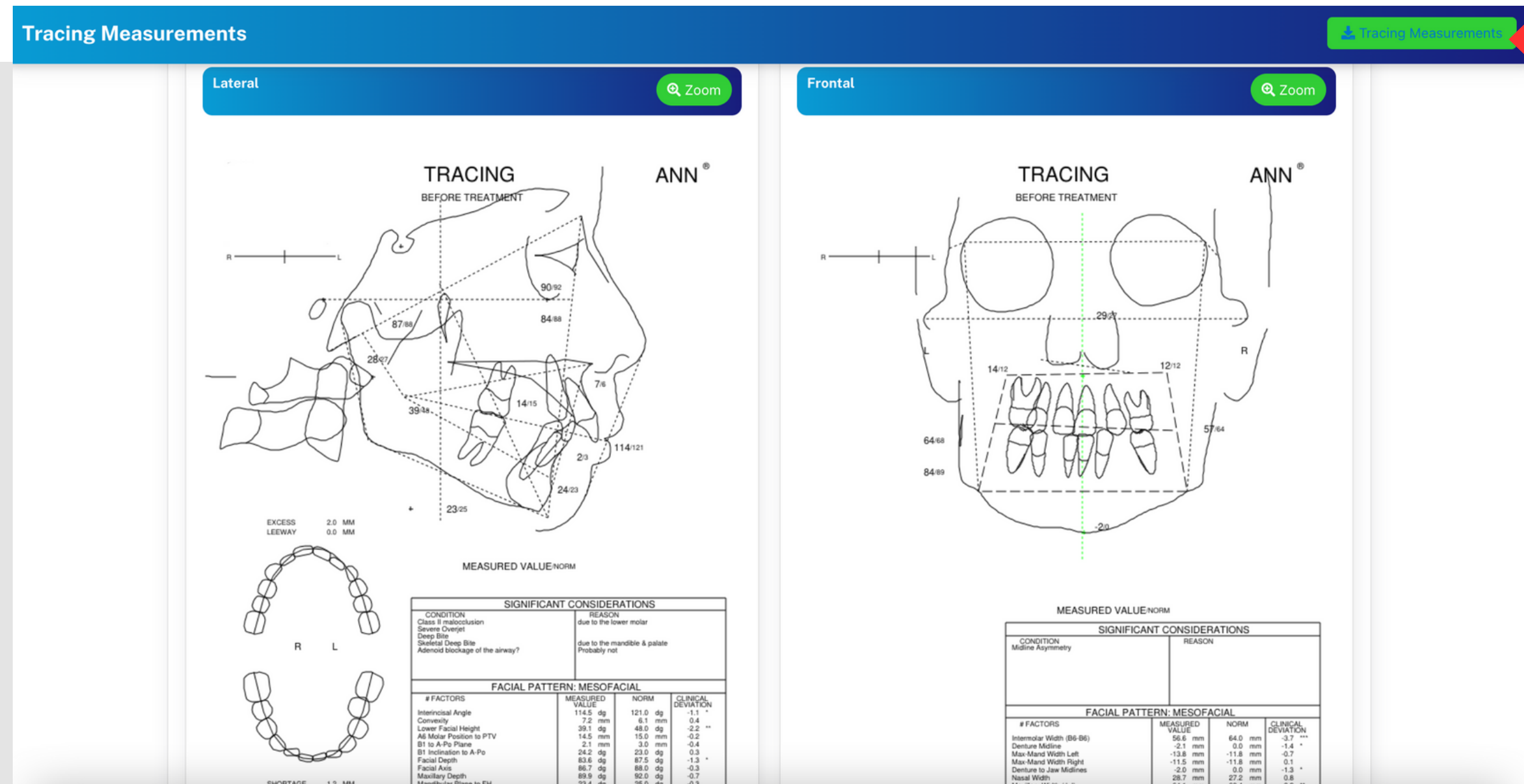
Continue...

Sequence Worksheet

Narrative (This output might take some time to produce, please check regularly)

English (US) Español (Mexicano) Português (Brasil)

Step-by-Step Treatment Summary for

Upper Arch Treatment Sequence:

1. **Month 1-4: Expand Arch**
 - The upper arch will be expanded. This helps to create space and improve the alignment of the teeth.
2. **Month 5-7: Correct Convexity and Distalize 6**
 - Simultaneously, we will correct the convexity (the outward curvature of your teeth) and move the molar teeth (6th teeth from the front) backwards to align properly with the rest of the teeth.
3. **Month 6-10: Intrude 1 and Retract 4 and/or 5**
 - From month 6, while continuing previous treatments, the front tooth (Incisor 1) will be moved downwards (intruded) to correct its height.
 - The canine and premolar teeth (4 and 5) will be pulled back to improve the bite and aesthetic appearance.
4. **Month 10-11: Retract 3**

Sequence Worksheet

Letter (This output might take some time to produce, please check regularly)

English (US) Español (Mexicano) Português (Brasil)

Subject: Treatment Design Review for

Dear Dr. Portal Demo,

I hope this message finds you well. After reviewing Vincent Ann's facial structure and anticipating how they might respond to dental treatment, our planned approach indicates a significant adjustment to the position of the front teeth is necessary.

We would appreciate your expertise in evaluating our treatment plan. Could you please consider whether standard dental treatments would be effective for this case? Alternatively, we have outlined a few less conventional methods that could be considered:

- Implementing mechanisms to help open the bite
- Introducing a phase of treatment using a functional appliance
- Aiming for a slightly deeper bite than usual

These methods are not typically standard, but given the unique requirements of this case, they might be worth considering. We strive to keep you informed when a case requires exceptional procedures.

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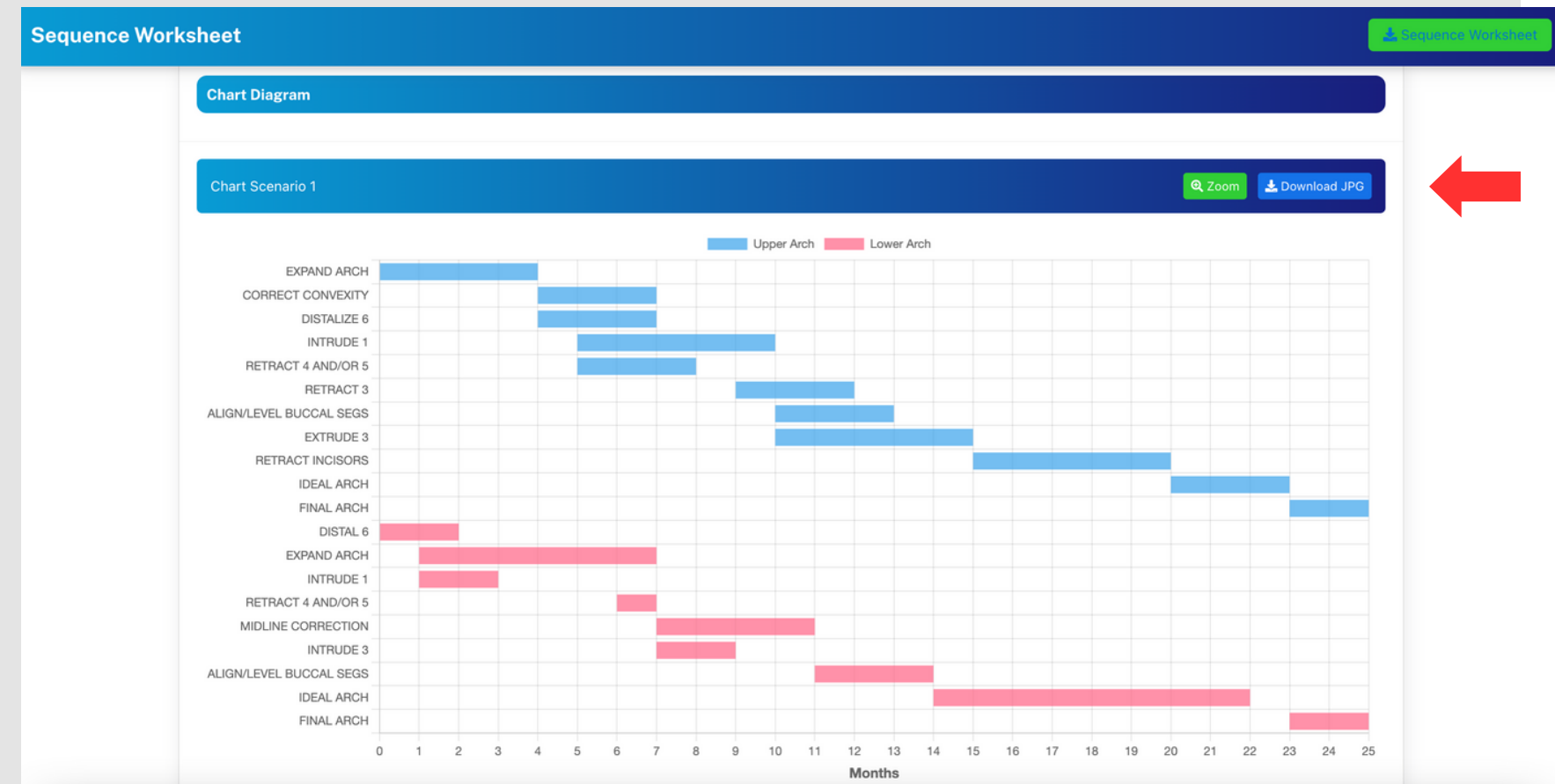
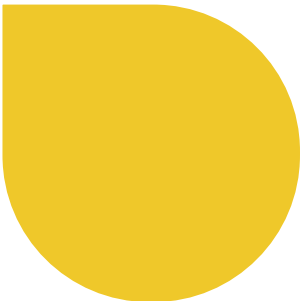
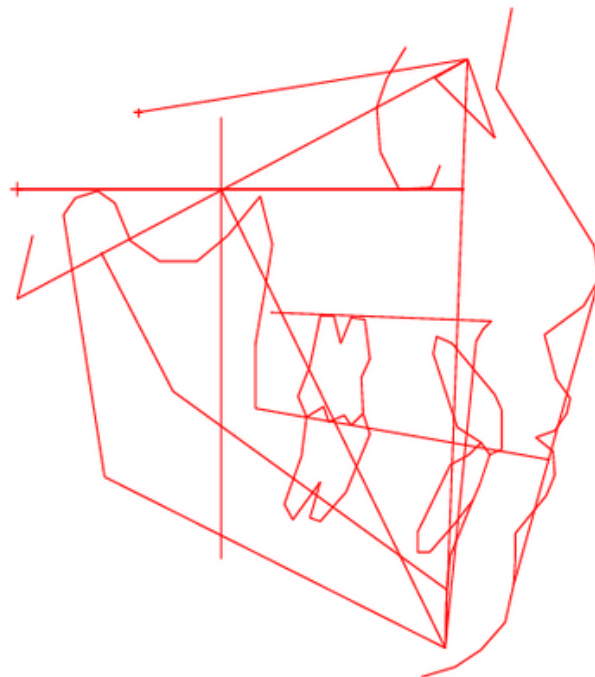
DENTAL TO SKELETON

DENTAL TO SKELETON

#18 A6 Molar Position to PTV	14.5 mm	15.0 mm	-0.2
#20 B1 to A-Po Plane	2.1 mm	3.0 mm	-0.4
#22 A1 to A-Po Plane	13.0 mm	5.5 mm	3.4 ***
#24 B1 Inclination to A-Po	24.2 dg	23.0 dg	0.3
26 A1 Inclination to A-Po	41.3 dg	36.0 dg	1.3 *
27 Occlusal Plane to Xi	0.9 mm	-0.5 mm	0.5
28 Inclination of Occl Plane	16.0 dg	25.0 dg	-2.2 **
54 B1 Inclination to FH	49.6 dg	65.0 dg	-3.1 ***

19 B6 to J-Ag Left	6.2 mm	5.4 mm	0.4
21 B6 to J-Ag Right	9.6 mm	5.4 mm	2.1 **
#23 Denture to Jaw Midlines	-2.0 mm	0.0 mm	-1.3 *
25 Occlusal Plane Tilt	-2.2 mm	0.0 mm	-1.1 *

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HANDICAPPING LABIO-LINGUAL DEVIATION INDEX (HLD) SCORE SHEET

A. Conditions 1-5 Are Automatic Qualifiers

- 1 Cleft Palate YES
- 2 Deep impinging bite with signs of tissue damage, not just touching palate NO
- 3 Anterior crossbite with gingival recession NO
- 4 Severe traumatic deviation (i.e., accidents, tumors, etc. attach description) NO
- 5 Impacted Maxillary Central Incisor NO

Continue to scoring below if there are no qualifying conditions checked above

B. Conditions 6-14 Must Score 28 Points or More to Qualify

Condition	Unit	Index	Score
6 Overjet (one upper central incisor to labial of the most labial lower incisor)	mm	x 1	11

HANDICAPPING LABIO-LINGUAL DEVIATION INDEX (HLD) SCORE SHEET

Guidelines and Rules for Applying the HLD Index

The provider is encouraged to score the case and exclude any case that obviously would not qualify for treatment. Upon completion of the HLD Index score sheet, review all measurements and calculations for accuracy.

General Instructions

- ☒ Indicate by checkmark next to A or B which criteria you are submitting for review.
-  Position the patient's teeth in centric occlusion.
-  Record all measurements in the order given and round off to the nearest millimeter.
-  Enter the score "0" if condition is absent.

A. Automatic Qualifiers (Conditions 1-5)

Indicate with an "X" if condition is present.

Cleft Palate
a cleft palate deformity must be demonstrated on the study model, if the deformity cannot be demonstrated on the study model, the condition must be diagnosed by properly credentialed experts and the diagnosis must be supported by documentation. If present, enter an "X".

Deep Impinging Bite
deep impinging overbite with signs of tissue damage of the palate, not just touching palate, including inflammation or damage of the gingiva palatal to the maxillary incisors. The condition must be clearly visible in the mouth and demonstrate on photos. On study models, the mandibular teeth must be clearly touching the palate and the tissue indentations or evidence of soft tissue damage or inflammation must be clearly visible. Tissue indentations without soft tissue damage do not meet the criteria. If present, enter an "X".

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PRICING AND TIER
UPGRADE GUIDELINES

Tier Upgrade Rules

Topping up \$1,000 in a single transaction will automatically upgrade you by 1 tier.
Topping up \$2,000 in a single transaction will upgrade you by 2 tiers.
Tier progression is calculated on an annual basis.

To Qualify for Tier 1 in the Following Year:

You must remain in Tier 2 for a full year and generate 100–200 cases. If you do not top up and generate fewer than 100 cases, you will be downgraded to Tier 3.

To Maintain Tier 1:

You must either top up \$1,000 or generate 200+ cases annually. If you do not top up and generate fewer than 200 cases, your tier will drop to Tier 2 or Tier 3, depending on your annual case volume.

TIER 3

Standard \$10

Standard + \$15

Complete \$20

0 - 100

Report(s) Annually

TIER 2

Standard \$8

Standard + \$10

Complete \$15

100 - 200

Report(s) Annually

TIER 1

Standard \$5

Standard + \$8

Complete \$10

200+

Report(s) Annually

VIP REPORTS \$50

VIP Reports will manually reviewed and processed by experienced professionals.

Your credits will be utilized to generate reports. The more reports you generate, the lower the cost per report, as outlined in our tiered pricing structure. When your credits are depleted, you can conveniently top up through our payment gateway and apply any available discount coupons.

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
The screenshot displays a user interface for managing credits. At the top right, the user's tier is 'T1' and their credit balance is '286 pts'. A dropdown menu is open, showing options: 'Credits' (highlighted with a red arrow), 'Doctor Preference', 'Manual Guide', and 'Log Out'. The main content area is divided into two sections: 'Top Up' and 'Credit'. The 'Top Up' section includes buttons for adding credit in increments of \$20, \$50, \$100, \$300, 1000, and 2000, along with 'BACK' and 'CHECKOUT' buttons. The 'Credit' section lists various statistics: Balance, Credit Spend (MTD) / (YTD), Credit Topup (YTD), Paid Report (MTD) / (YTD) (2 / 8), and Generated Report (MTD) / (YTD) (14 / 34). At the bottom, there are two tabs: 'Pricing & Tier Guidelines' (active) and 'Usage History'.

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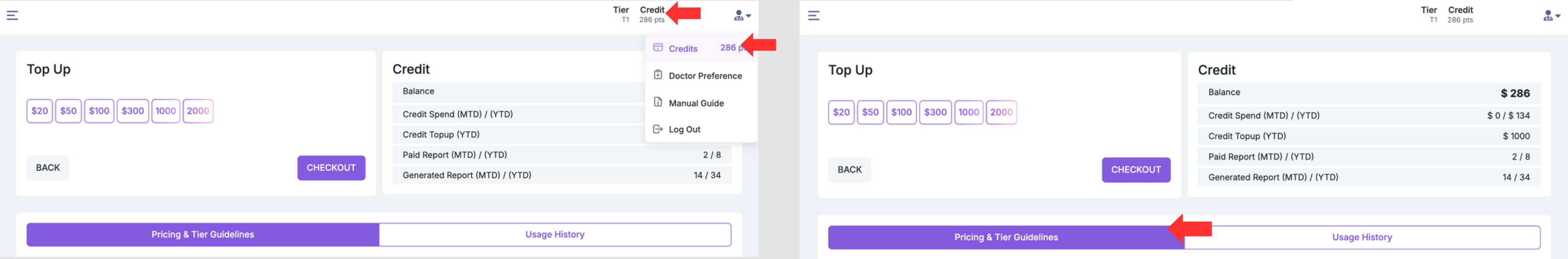PRICING AND TIER UPGRADE GUIDELINES

Tier Upgrade Rules
Topping up \$1,000 in a single transaction will automatically upgrade you by 1 tier.
Topping up \$2,000 in a single transaction will upgrade you by 2 tiers.
Tier progression is calculated on an annual basis.

To Qualify for Tier 1 in the Following Year:
You must remain in Tier 2 for a full year and generate 100-200 cases. If you do not top up and generate fewer than 100 cases, you will be downgraded to Tier 3.

To Maintain Tier 1:
You must either top up \$1,000 or generate 200+ cases annually. If you do not top up and generate fewer than 200 cases, your tier will drop to Tier 2 or Tier 3, depending on your annual case volume.

TIER 3	TIER 2	TIER 1
Standard \$10	Standard \$8	Standard \$5
Standard + \$15	Standard + \$10	Standard + \$8
Complete \$20	Complete \$15	Complete \$10
0 - 100 Report(s) Annually	100 - 200 Report(s) Annually	200+ Report(s) Annually

VIP REPORTS \$50
VIP Reports will manually reviewed and processed by experienced professionals.

Your credits will be utilized to generate reports. The more reports you generate, the lower the cost per report, as outlined in our tiered pricing structure. When your credits are depleted, you can conveniently top up through our payment gateway and apply any available discount coupons.

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Paid Report (MTD) / (YTD) 2 / 8

Generated Report (MTD) / (YTD) 14 / 34

Pricing & Tier Guidelines Usage History

Tier T1 Credit 286 pts

Credits 286 pts

Doctor Preference

Manual Guide

Log Out

Top Up

\$20 \$50 \$100 \$300 1000 2000

BACK CHECKOUT

Credit

Balance \$ 286

Credit Spend (MTD) / (YTD) \$ 0 / \$ 134

Credit Topup (YTD) \$ 1000

Paid Report (MTD) / (YTD) 2 / 8

Generated Report (MTD) / (YTD) 14 / 34

Pricing & Tier Guidelines Usage History

Date	Transaction	Topup	Spend
Jul 22, 2025 5:38 PM	Reporting		8
Jul 22, 2025 5:25 PM	New Case		5
Jul 15, 2025 8:02 AM	New Case		5
Jul 3, 2025 10:30 AM	Reporting		8
Jul 3, 2025 10:02 AM	New Case		5

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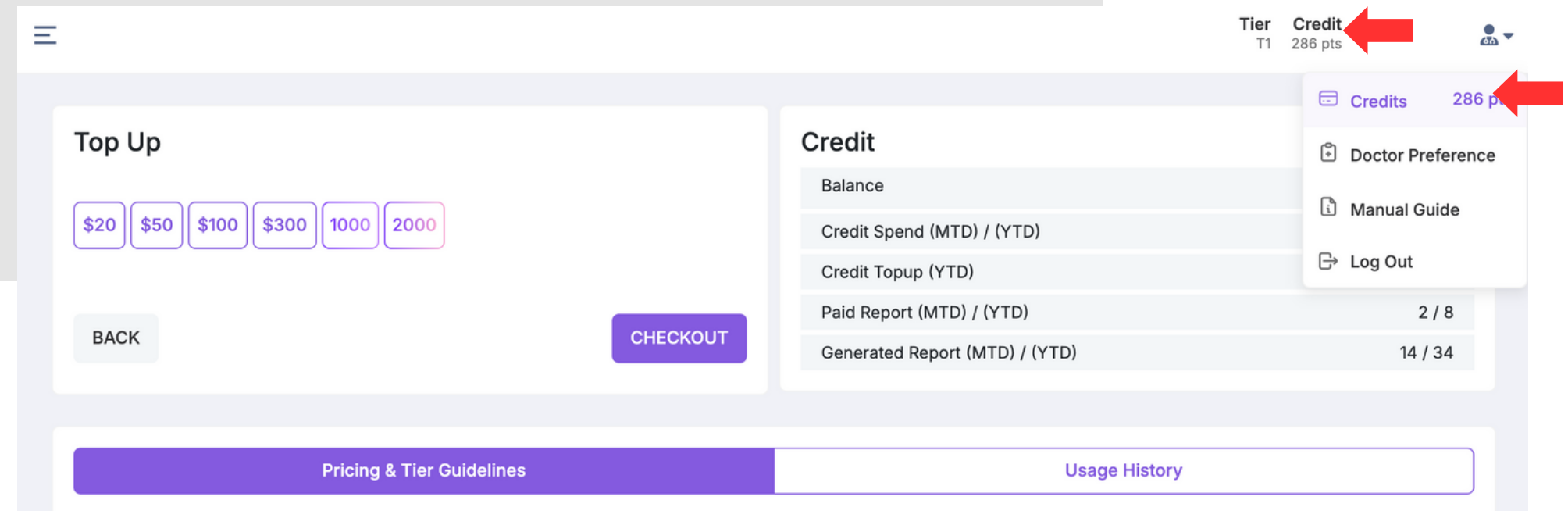
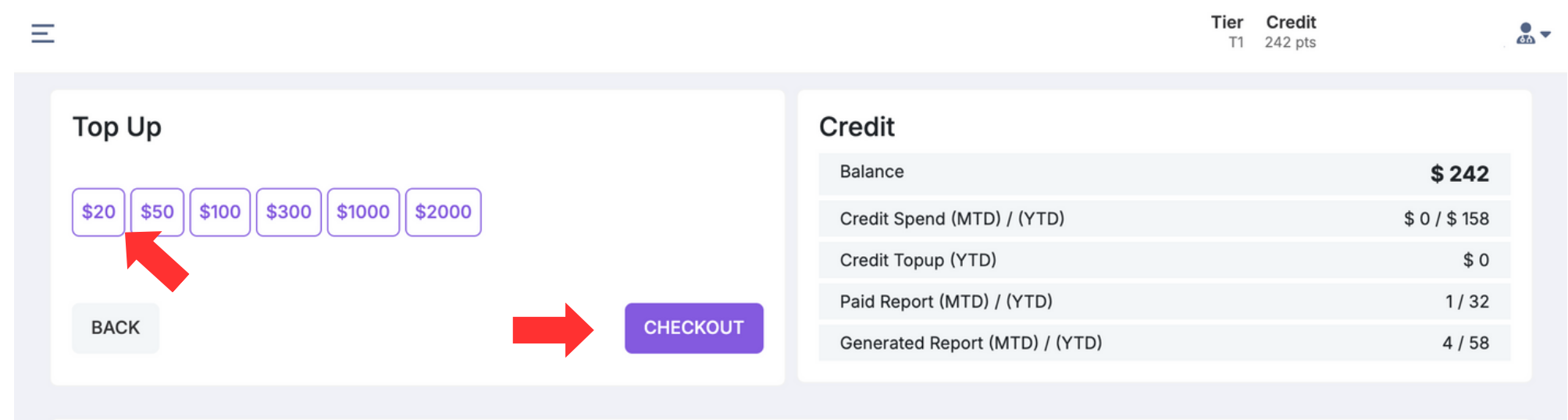
The screenshot shows the payment interface for DR ANNIE Insights. It includes a currency selection section, a payment summary table, and a payment method selection area. Red arrows and numbers 3 through 6 highlight specific steps: 3 points to the currency selection and the 'Add promotion code' link; 4 points to the 'Pay with link' button; 5 points to the card information fields; and 6 points to the 'Pay' button at the bottom.

DR ANNIE Insights

Choose a currency:

Exchange rate and fees of your bank may apply

Payment for Annie-Insights Credit 20 points. Trx# TXN12345	\$20.00
Subtotal	\$20.00
Total due	\$20.00

Pay with link

Or

Email

Card information

1234 1234 1234 1234

MM / YY

CVC

Cardholder name

Full name on card

Country or region

Securely save my information for 1-click checkout

Pay faster on Foundation for Modern Bioprogressive Orthodontics and everywhere Link is accepted.

Pay

Patient Case Search / Patientenfallsuche

2

The dashboard shows a patient case for DEMO, CLASSONE (ismart.patient@gmail.com). The Case ID is AN2001500011, and the Selected Timepoint is January 30, 2025. The status is In Progress. The workflow consists of six steps: 1. Intra Oral Photos, 2. Extra Oral Photos, 3. Dental Chart, 4. X-Rays/Radiographs, 5. AI - Findings, and 6. Reports. A red arrow points to the Case ID field.

1

The 'Search Patient Result' interface shows a search bar with the text 'CLASSONE' and a red arrow pointing to the search button. Below the search bar is a table with patient information.


Patient	Profile	Time Point	Progress	Action
DEMO, CLASSONE ismart.patient@gmail.com 0618812345	Jan 30, 2012 F 150 cm CH	2	Submitted Processed Complete	[Refresh] [Download] [Delete]

The second screenshot shows the same interface but with the search bar containing the Case ID 'AN2001500011' and a red arrow pointing to the search button.

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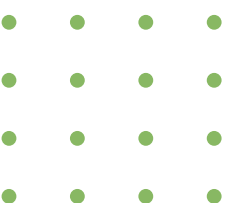Doctor Preference

Preferred Treatment

☒ Let DR.ANNIE suggest
The user does not have a specific preference and authorizes DR. ANNIE™ to recommend the most appropriate treatment approach, whether extraction or non-extraction.

☐ Force Non-Extraction
Based on the user's selection, DR. ANNIE™ will proceed with a treatment plan that is primarily non-extraction-based, where clinically appropriate.

☐ Force Extraction
Based on the user's selection, DR. ANNIE™ will proceed with a treatment plan that is primarily extraction-based, where clinically appropriate.

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