


UV-C MOBILE SURFACE DISINFECTION UNIT

QUICK START GUIDE

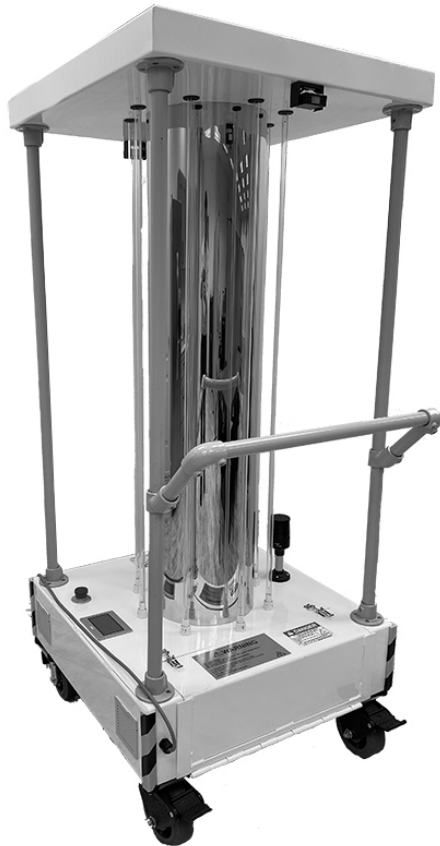

Models:

UVC-PKCART-ECO

UVC-PKCART-MAX-LP

UVC-PKCART-MAX

ERICSON QUICK START GUIDE

Step 1: Position and Plug-In

- Inspect PK-CART for damage prior to placement for use.
- Wear proper PPE when operating PK-CART
- Place PK-CART in treatment space
- Position PIR Sensor towards entry point
- Plug PK-CART into standard 120v NEMA 5-15 or NEMA 5-20 receptacle or connector


Step 2: Initiating Treatment Cycle

- Press "Perma-Kleen" logo for Home screen
 - Select "Start Cycle" to begin factory:
 - 30 Seconds Exit/Countdown Timer*
 - 10 Minute Treatment Cycle*
 - Optional: For custom programming, select "User Settings"
- *Times superseded after custom user input




Step 3: Custom Programming (optional)

- At Home screen, select "User Settings"
- Use Up/Down arrows to adjust Countdown and Cycle Timers
- Times will remain for subsequent cycles until manually adjusted
- Hour and Cycle counts visible here
- Return to Home screen to start treatment cycle

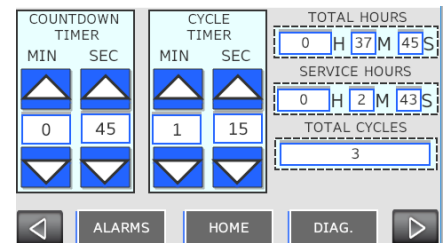

Step 4: Start Cycle

- Press "Start Cycle" from Home screen
- Observe Countdown timer action
- Leave treatment space immediately
- Close door to area and post Warning Signage
- Audible countdown alarm tone with increased frequency as timer nears zero

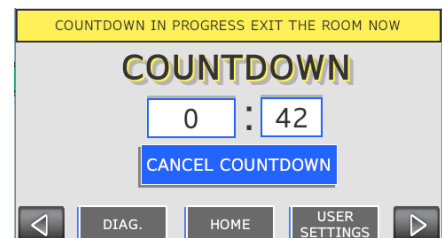

ERICSON QUICK START GUIDE



Step 5: In Process

- Treatment cycle "ON" with timer screen
- Cycle will proceed unless Safety PIR Sensor or Emergency Shutoff interruption
- Avoid direct exposure to UV light
- Do not look at UV light without approved safety glasses





Step 6a: PIR Sensor Fault

- "Person Detected" screen will appear if PIR Sensor activated
- Cycle deactivation will occur and event will be logged
- Select "Home" to restart cycle process

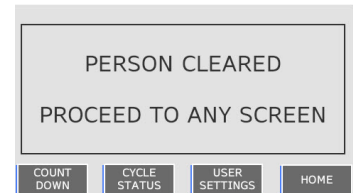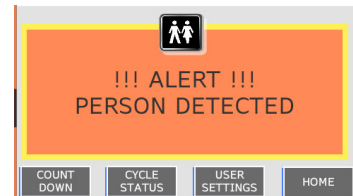

Step 6b: E-Stop Fault

- "Emergency Stop Active" screen will appear if E-Stop Button activated
- Cycle deactivation will occur and event will be logged
- Deactivate E-Stop Button clearing alarm log from red to green
- Select "Home" to restart cycle process

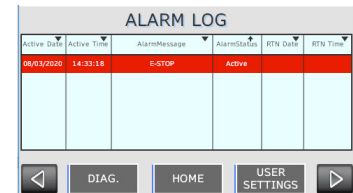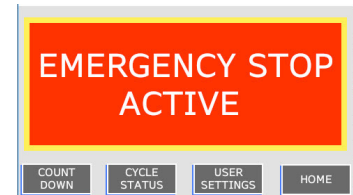

Additional Information

- Please contact manufacturer for:
 - Troubleshooting
 - FAQ's
 - Diagnostics



Electrical Manufacturing

4323 Hamann Parkway
Willoughby, OH 44094
info@ericson.com
ericson.com
1.800.374.2766
C8021242A

