



205 Westwood Ave, Long Branch, NJ 07740
Phone: 866-94 BOARDS (26273) / (732)-222-1511
Fax: (732)-222-7088 | E-mail: sales@touchboards.com

ASSEMBLY INSTRUCTIONS

Zido™ Gel Cup/Probe Holders




Product	Part#
Zido Gel Cup/Probe Holder Small	ZGCSSB
Zido Gel Cup/Probe Holder Large	ZGCLSB

NOTE: The Zido Base Unit is not included.



Anthro Corporation® | 10450 SW Manhasset Dr. | Tualatin, OR 97062
Toll-free: 800.325.3841 | Fax: 800.325.0045 | email: sales@anthro.com | anthro.com
Outside the U.S. | Tel: 503.691.2556 | Fax: 503.691.2409

Rev A November 2014

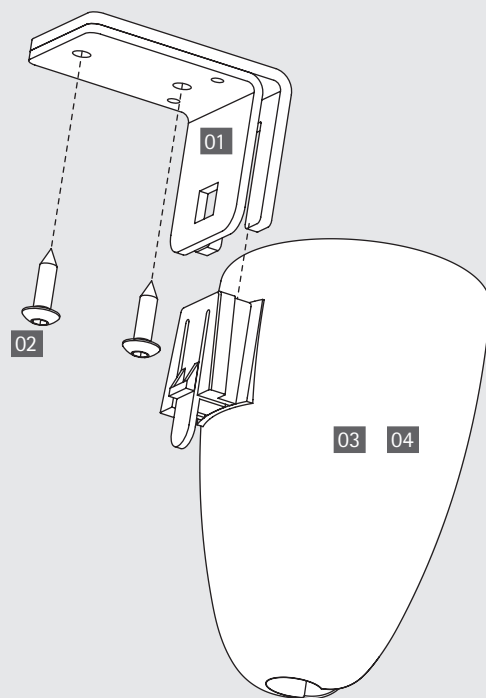

WELCOME

Thank you for purchasing
Zido Gel Cup/Probe Holder!

If you have any questions or if we
can help you in any way, please
contact us at 800.325.3841.

PARTS LIST

Before beginning assembly of your Zido accessory, please
review the parts list to verify that your shipment is complete.

A 5/32" hex driver (provided with your Zido cart) is required
for this assembly.




A 1/8" drill bit and electric drill are also required.

Product	Quantity	Part Number
01 Zido Gel Cup/Probe Holder Bracket	1	225-2657-00
02 3/4" MDF Wood Screws	2	325-5575-00
03 Long Gel Cup/Probe Holder	1	175-5243-00
04 Short Gel Cup/Probe Holder (not shown)	1	175-5244-00

STEP 1

Install the Bracket

- Select the installation location. Place the mounting bracket on the bottom of the shelf at the desired location so that the side with four holes is against the shelf.
- Using the Mounting Bracket as a template, mark the drill points through the two larger bracket holes onto the bottom of the shelf.
- Drill the screw holes using a 1/8" drill bit. Because the shelf is 3/4" thick, only drill the hole 1/2" deep. (Mark your drill bit with masking tape to be sure you don't drill too far.)
- Install the bracket onto the shelf using two 3/4" MDF wood screws.

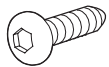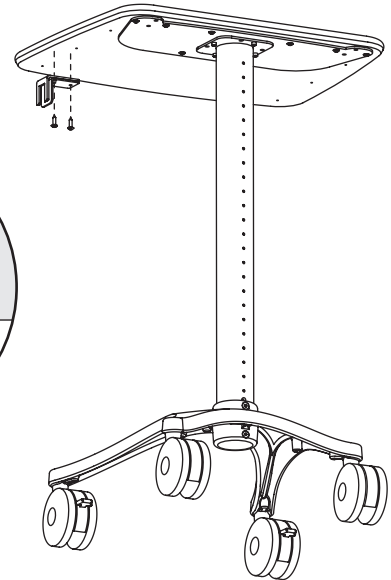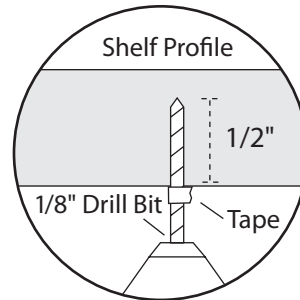


3/4" Button Hd PB Screw
325-5575-00

STEP 2

Install the Holder

- Slide the Gel Cup/Probe Holder over the bracket until it clicks into place.

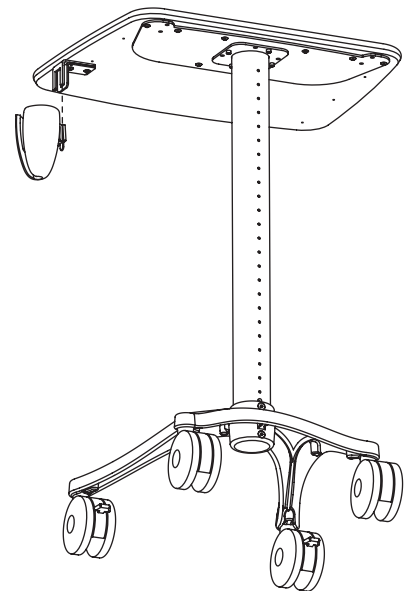

CONGRATULATIONS! YOUR HOLDER ASSEMBLY IS COMPLETE!



205 Westwood Ave, Long Branch, NJ 07740
Phone: 866-94 BOARDS (26273) / (732)-222-1511
Fax: (732)-222-7088 | E-mail: sales@touchboards.com



Technology Furniture®
– Fast and Friendly!

Anthro Corporation® | 10450 SW Manhasset Dr. | Tualatin, OR 97062
Toll-free: 800.325.3841 | Fax: 800.325.0045 | email: sales@anthro.com | anthro.com
Outside the U.S. | Tel: 503.691.2556 | Fax: 503.691.2409



300-5416-00

Warranty:

For warranty information, please visit: <http://www.anthro.com/support/warranty>

Notices:

Technology Furniture and Zido are trademarks of Anthro Corporation. Anthro reserves the right to modify the design and specifications without prior notice.