




FM 300 P SERIES IR CAMERA USER MANUAL

PLEASE READ THIS MANUAL BEFORE SWITCHING THE UNIT ON.
IMPORTANT SAFETY INFORMATION INSIDE.

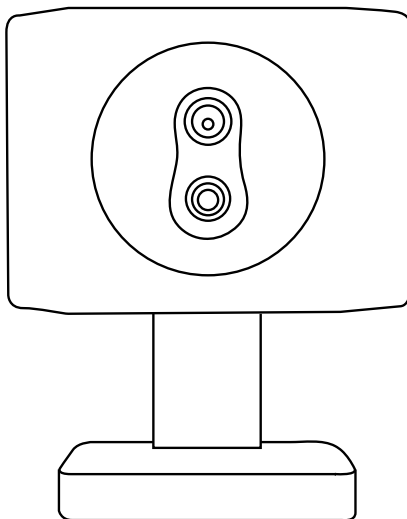

**THIS DEVICE IS INTENDED FOR ADJUNCTIVE USE.
NOT MEANT FOR STANDALONE DIAGNOSTICS.**

ICI cameras fall under US Federal Law and Export Control.

©Copyright 2021, Infrared Cameras, Inc. - All rights reserved. The contents of this document may not be reproduced in whole or in parts without the written consent of the copyright owner.

Printed in the United States of America.

2105 W. Cardinal Dr.
Beaumont, TX 77705
(866) 861-0788
www.infraredcameras.com

Revision: 2.2021-001

THE MANUFACTURER RESERVES THE RIGHT TO ALTER THE SPECIFICATIONS OF THE PRODUCT WITHOUT PRIOR NOTIFICATION. THE MANUFACTURER ALLOWS HIMSELF THE RIGHT TO MODIFY WITHOUT ANY PRELIMINARY OPINION THE TECHNICAL SPECIFICATIONS OF THE PRODUCT. THIS MANUAL MAY CONTAIN TECHNICAL INACCURACIES OR TYPOGRAPHICAL ERRORS.

NOTICE ALL STATEMENTS, INFORMATION, AND RECOMMENDATIONS IN THIS MANUAL ARE BELIEVED TO BE ACCURATE BUT ARE PRESENTED WITHOUT WARRANTY OF ANY KIND. NOTWITHSTANDING ANY OTHER WARRANTY HEREIN, ALL DOCUMENT FILES AND SOFTWARE ARE PROVIDED "AS IS" WITH ALL FAULTS. ICI DISCLAIMS ALL WARRANTIES, EXPRESSED OR IMPLIED, INCLUDING, WITHOUT LIMITATION, THOSE OF MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE AND NONINFRINGEMENT OR ARISING FROM A COURSE OF DEALING, USAGE, OR TRADE PRACTICE.

THE SOFTWARE LICENSE AND LIMITED WARRANTY FOR THE ACCOMPANYING PRODUCT ARE SET FORTH IN THE INFORMATION PACKET THAT SHIPPED WITH THE PRODUCT AND ARE INCORPORATED HEREIN BY THIS REFERENCE. IF YOU ARE UNABLE TO LOCATE THE SOFTWARE LICENSE OR LIMITED WARRANTY, CONTACT YOUR ICI REPRESENTATIVE FOR A COPY.

IN NO EVENT SHALL ICI BE LIABLE FOR ANY INDIRECT, SPECIAL, CONSEQUENTIAL, OR INCIDENTAL DAMAGES, INCLUDING, WITHOUT LIMITATION, LOST PROFITS OR LOSS OR DAMAGE TO DATA ARISING OUT OF THE USE OR INABILITY TO USE THIS MANUAL, EVEN IF ICI HAS BEEN ADVISED OF THE POSSIBILITY OF SUCH DAMAGES.

Contents

1.	General Description _____	4
2.	Safety Information _____	4
3.	Intended Use _____	4
4.	Technical Specifications _____	5
5.	Structure _____	6
6.	Package Contents _____	7
7.	Installation Instructions _____	8
8.	Computer Setup _____	11
9.	Cleaning and Maintenance _____	13
10.	Site and System Validation Checklist _____	14
11.	Troubleshooting _____	17
12.	About ICI _____	18

1. General Description

Our FM 300 P Series features ICI Dual Vision which gives you thermal and visible imaging in an all-in-one device. It provides audio alarms when it registers an elevated skin temperature measurement. Suitable for public places with orderly flows of people, such as airports, transportation hubs, shopping malls, schools and enterprises.

2. Safety Information

- This device must be installed by qualified service personnel or system installation personnel.
- Do take precautions to prevent the lens from being worn, scratched or broken. Avoid touching the lens to prevent it from being damaged or getting dirty.
- Given that the uncooled thermal infrared image camera uses a very sensitive thermal sensor, under no circumstances (powered on or off) should the lens be pointed directly at a strong radiation source (such as sun, direct or reflected laser beam, etc.), otherwise permanent damage will be caused to the uncooled thermal imager.
- This product is a precise electronic device that must be handled with care during use, storage, and transportation to prevent dangerous actions such as the device being hit by external force, or falling from heights.
- During transportation and storage, the ambient temperature must not be lower than -25 °C, and the original packaging box must be used during transportation.
- Prior to start of the device, make sure that the power supply is properly connected. If the power supply is connected incorrectly, the device may be damaged.
- Do not place any objects on the power cord, and do not place the device where the power cord can be easily touched.
- If the device operates abnormally, please contact the supplier and do not dismantle the device on your own.
- Do not drop or throw the device.
- Do not put the product into a fire.
- It is recommended to calibrate the device(s) annually.

3. Intended Use

FM 300 P Series IR cameras are designed for body surface temperature measurement of infants and adults without contacting the human body.

4. Technical Specifications


SPECIFICATIONS SUBJECT TO CHANGE WITHOUT NOTICE

	IR Camera Specifications
Detector Array	VOx
Spectral Band	7.5 μm ~ 13.5 μm
Thermal Sensitivity (NETD)	< 0.05 $^{\circ}\text{C}$ at 25 $^{\circ}\text{C}$ (50 mK)
Frame Rate	\leq 25 Hz
Dynamic Range	14-bit
Temperature Range	-30 $^{\circ}\text{C}$ to 45 $^{\circ}\text{C}$
Operation Range	-30 $^{\circ}\text{C}$ to 80 $^{\circ}\text{C}$
Storage Range	-40 $^{\circ}\text{C}$ to 70 $^{\circ}\text{C}$
Humidity	5% to 95% non-condensing
Accuracy	\pm 0.5 $^{\circ}\text{C}^*2$
Pixel Operability	> 99 %
Dimensions (without lens)	66 mm x 20 mm x 50 mm (L x W x H \pm .5 mm)
Power	DC 5V, < 700 mW
Weight (without lens)	< 122 g
Interface	Micro-USB
Shutter	Built-in shutter


	Visible Camera Specifications
Detector Array	1/4" CMOS
Pixel Pitch	3 μm
FOV	customizable
Camera	5 MP, 1296 x 732, 30 Hz
Chief Ray Angle	17.5 $^{\circ}$ - linear
RGB	color filter array
Shutter	Rolling shutter

5. Structure

5-1 Appearance

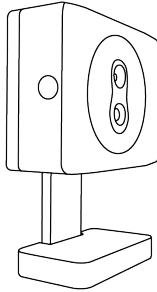


5-2 Definitions of Housing Interface

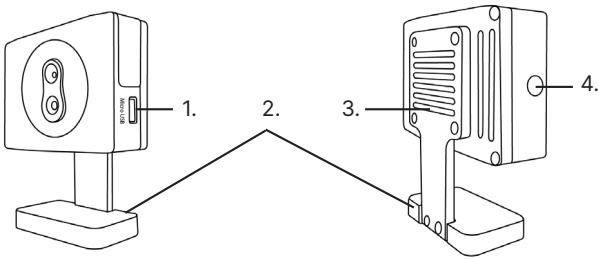


1. Micro-USB interface
2. FM 300 camera stand
3. Mounting bracket
4. 1/4"-20 tripod mount

FM 300 CAMERA STAND IS AN OPTIONAL ACCESSORY


6. Package Contents

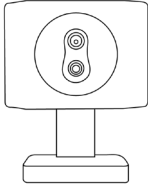


FM Series IR Camera
with camera stand (optional)

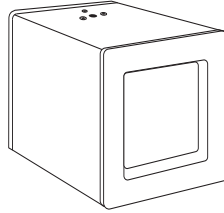


Temperature Reference

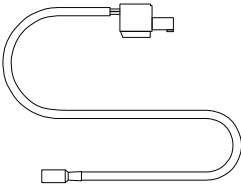


Power Cord

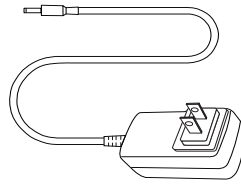

Temperature Reference

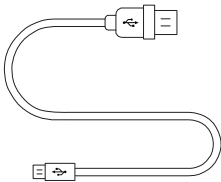

Micro-USB to USB Cable

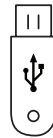

Software USB Drive




ENSURE ALL SYSTEM EQUIPMENT AND COMPONENT ITEMS ARE PRESENT BEFORE BEGINNING INSTALLATION

7. Installation Instructions

SELECT AN APPROPRIATE AREA FREE OF IMMEDIATE AIRFLOW FROM DOORWAYS AND AIR CONDITIONING/VENTILATION SYSTEMS. THE SELECTED AREA SHOULD HAVE A STABLE AMBIENT TEMPERATURE BETWEEN 20 °C AND 24 °C AND RELATIVE HUMIDITY RANGE FROM 10% TO 50%.


A.

Mount the Temperature Reference to a tripod using the ¼-20 mount.

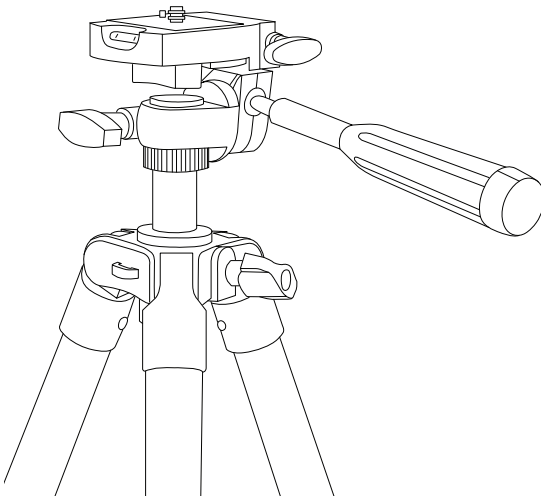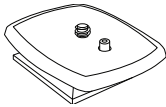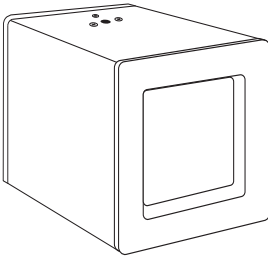

B.

Plug one power cord into Temperature Reference.

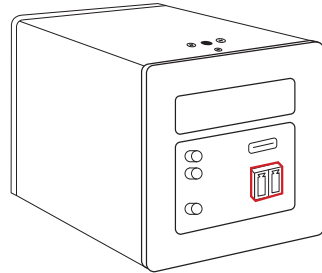


C.

Plug the jack end of the power extension cable into the barrel cable attached to the FM Series IR Camera.




D.

Plug power cord into a 110/120V electrical outlet.

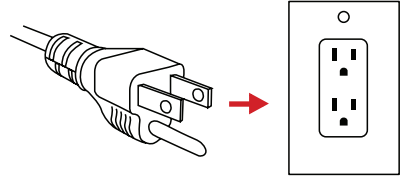


E.

Mount the FM Series IR Camera to a tripod using the 1/4-20 mount.

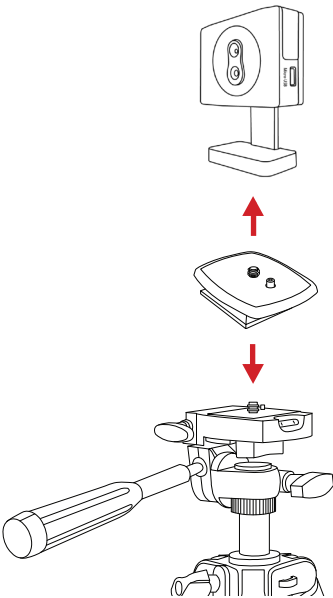

F.

Plug the Micro-USB end of the cable into the FM Series IR Camera.

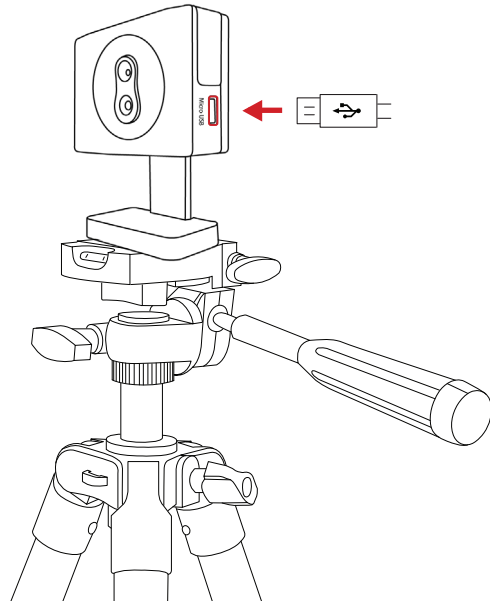

MAKE SURE TRIPODS DO NOT BLOCK THE DIRECT PATH OF PERSON(S) TO BE IMAGED TO ENSURE THE EQUIPMENT WILL NOT BE MOVED OR KNOCKED DOWN. USING A DIVIDING BARRIER WILL HELP KEEP TRIPODS SEPARATE FROM THE PATH.



ENSURE THE CAMERA LENS AND TEMPERATURE REFERENCE SOURCE ARE FACING EACH OTHER ON PARALLEL PLANES. MAKE SURE CAMERA TARGET AREA IS FACING A NON-REFLECTIVE BACKGROUND.

G

Plug the other end of the USB cable into a computer USB port.

H.


Power on devices.

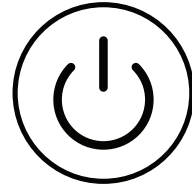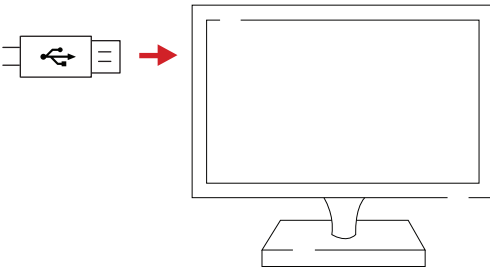

ALLOW AT LEAST 45 MINUTES FOR THE TEMPERATURE REFERENCE SOURCE TO WARM UP. THIS WILL PROVIDE THE MOST ACCURATE SKIN TEMPERATURE MEASUREMENTS.

SOFTWARE ARRIVES INSTALLED ON COMPUTERS AND TABLETS PURCHASED FROM ICI AS COMPLETE SYSTEMS AND IT WILL LAUNCH AUTOMATICALLY ON STARTUP. AFTER THE SOFTWARE LAUNCHES THE USER SHOULD ENSURE THE TARGET ZONE IS IN VIEW OF THE CAMERA AND THE TEMPERATURE REFERENCE SOURCE IS POSITIONED OFF CENTER TO KEEP THE PERSON BEING IMAGED AS CENTERED AS POSSIBLE.

7-1 Full Assembly Diagram

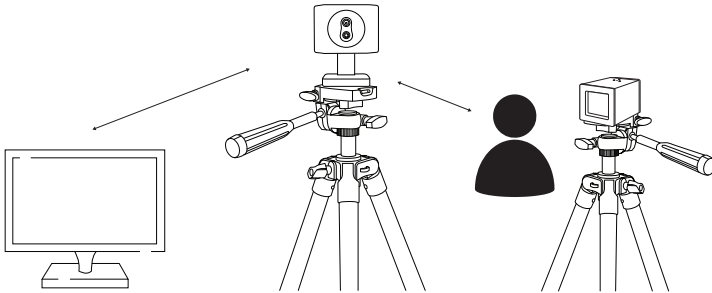

GRAPHIC FOR ILLUSTRATION PURPOSES ONLY

8. Computer Setup

8-1 IR Flash Skin Temperature Monitor Software (IR Flash STM)

Software arrives installed on computers and tablets purchased from ICI as complete systems and it will launch automatically on startup. A copy of the software is provided on the Software USB Drive. Documentation can be found online on the IR Flash STM web page under the Downloads section or at this address:

<https://infraredcameras.com/ir-flash-stm-manual/>

8-2 Recommended System Requirements

- **Processor:** i5 or above (Quad Core or better)
- **RAM:** 4 GB or above
- **OS:** Windows 8/8.1/10
- **Hard Drive:** 256 GB or above
- **Resolution:** 1920 x 1080

SOFTWARE DISPLAYS BEST WITH 1920 X 1080 RESOLUTION OR ABOVE.

CRITICAL INFORMATION

- The technology should be used to measure only one subject's temperature at a time.
- Measurements should not be solely, or primarily, relied upon to diagnose or exclude a diagnosis of any disease.
- Elevated body temperature should be confirmed with secondary evaluation methods (e.g., an NCIT or clinical grade contact thermometer).
- Signage should instruct for the removal of glasses, headwear, and masks.
- Read the current government guidance regarding the use of telethermographic systems for skin temperature measurements. Additional information can be found by reading IEC 80601-2-59:2017 Medical electrical equipment — Part 2-59: Particular requirements for the basic safety and essential performance of screening thermographs for human febrile temperature screening and ISO/TR 13154:2017 Medical electrical equipment - Deployment, implementation and operational guidelines for identifying febrile humans using a screening thermograph.

CRITICAL INFORMATION

9. Cleaning and Maintenance

9-1 Cleaning the Germanium Lens

Do not use corrosive chemicals on the optical glass components. The germanium window surface is coated with anti-reflection coating. Dust, grease, and fingerprints will produce harmful substances and lead to a decline in performance, or cause scratches. If dirt is found, please use the following methods:

1. Use a blown balloon or a soft brush to clean the lens surface to avoid dust particles scratching the anti-reflection film on lens surface during the wiping process.
2. Use a soft cotton cloth or lens wiping paper and dip in alcohol or lens wiping liquid. Gently wipe the lens surface from the middle to the edge, paying attention to not crack the lens, or use too much wiping liquid. If the lens is still not clean, replace the cloth and repeat operation.

9-2 Device Calibration

It is recommended to have your device(s) re-calibrated annually. Contact customer service to schedule maintenance.

10. Site and System Setup Validation Checklist

Site: _____ **System:** _____
Contact Person: _____ **Date:** _____

**All those tasked with the initial setup of the temperature screening equipment must be present during this training.

***This portion of the validation requires a live video feed with the instructors.

10-1 The Screening Area

- Ensure that this area is free from:
 - Any direct or indirect (reflected) sunlight
 - Warm or cold conductive airflow
 - HVAC vents/intakes
 - Any radiant energy from electrical sources
 - Direct or indirect lighting on individual being screened
- Room temperature is 20 °C - 24°C (68 °F - 75 °F)
- Relative humidity is within 10% – 50%

10-2 The Screening Background


- Placement is parallel and perpendicular to screening camera
 - At a distance of 3 - 4 feet
- Consists of a non-IR reflective homogeneous background
 - Minimum size is: 10' x 8'
 - White to gray flat finish
- The temperature reference source is in front of background (direct line of sight to the screening camera) and is framed within the screening image

Continues

10-3 The Traffic Flow

- Establish a guided pathway from the entrance to the screening area
 - Design this pathway so there is an equilibration time
 - No restrooms on ingress pathway
- Signage should instruct for removal of glasses, headwear, and masks
- Mark the point of measurement on the floor (“V/X” or a set of footprints)
 - The distance is unique to each screening system
- Ensure the line of sight from the temperature reference source to the camera is never blocked by an individual entering or leaving the point of measurement
- The secondary screening area should be setup near the screening area

10-4 Typical traffic flow illustration:

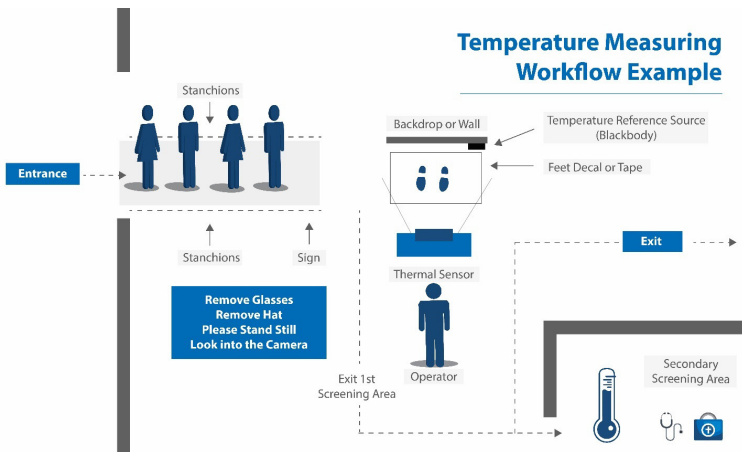

10-5 IR Camera Placement

- Level on tripod or other suitable support
- Perpendicular to floor and individual being screened

Continues

- Height of camera
 - Average male 70 ± 4 inches; Average female 65 ± 4 inches
 - 5'4" – 5'8" will image 4'8" to 6'4" individual
 - Larger will require a height adjustment to camera
- Distance of camera to individual being screened is: 2 - 3 feet
 - The individual's face and blackbody should be in the frame
- Distance to blackbody: 2 - 3 feet
 - Behind the individual being screened yet visible within the frame of the image
- Protect the camera from exposure to direct sunlight to avoid damage to the camera sensor

10-6 Temperature reference source

- On tripod or other stable stand, in front of background, and positioned for visibility within the frame of the image.
- Temperature is set at 35 °C (95 °F) or 37 °C (98.6 °F)
- Warm-up time is 15 minutes

10-7 Software

- Set up your software according to the instructions provided
- Documentation can be found online on the IR Flash STM web page under the Downloads section or at this address:

<https://infraredcameras.com/ir-flash-stm-manual/>

10-8 Final Step

Final validation is dependent upon a signed copy of this checklist from each person responsible for each site.

Upon completion please sign below:

ICI representative:

Company representative:

Signature: _____

Signature: _____

Print Name: _____

Print Name: _____

Date: _____

11. Troubleshooting

11-1 Camera(s) not showing, camera(s) lagging, or software crashing

- Close and reopen software
- Reconnect power & Ethernet cables
- Ensure that camera has been powered on for a few minutes
- Restart computer
- Verify that correct software is installed
- Uninstall & reinstall software, running as administrator
- Verify that firewall is not blocking software
- Try IR Flash STM software. If already using, try a newer version.

11-2 Temperature readings are incorrect or facial recognition is suboptimal

- Close and reopen software
- Check that software settings are correct:
 - Fixed temperature of 35 °C or 37 °C, depending on temperature reference source
 - Temperature reference source has crosshair over it
- Remove masks & glasses
- Camera is proper distance from person
- Camera is at proper height
- Temperature reference source in view, perpendicular to camera, but not blocked
- Person is looking directly at camera lens, not at an angle
- Reconnect the camera to the computer and restart the computer
- Reconnect the power to the temperature reference source
- Temperature reference source power switch is on
- Camera and temperature reference source powered on for several minutes
- Ensure temperatures on back of temperature reference source match
- Camera is away from direct sunlight or reflective light
- Plain background
- Try IR Flash STM software. If already using, try a newer version.

12. About ICI

Infrared Cameras, Inc.
2105 W. Cardinal Dr.
Beaumont, TX 77705

Phone: (409) 861-0788 | Toll Free: (866) 861-0788 | International: (409) 861-0788

General Inquiry: support@infraredcameras.com
Website: www.infraredcameras.com

You may reach a representative by phone or email Monday – Friday 8:00AM - 5:00PM CST.

ICI manufactures complete systems and software. We can provide complete engineering, software, and OEM solutions. Our Fortune 500 clients rely on us for infrared equipment and thermography training (which we offer through the Infrared Training Institute).

In addition to providing custom germanium, silica, and sapphire optics, we also build windows for enclosures, as well as custom pan and tilt units. We can even provide customizable explosion-proof systems.

Our knowledge and experience stems from years of using infrared imaging and temperature measurement instruments to provide solutions to: managers, engineers, scientists, inspectors and operators in space, power companies, medical, pulp and paper, food industry, research and development, and various process industries. You can see our products and services used in industrial, commercial, and government applications worldwide. Originally named Texas Infrared (still DBA), Infrared Cameras, Inc. has been in business since March, 1995.

Thank you for your dedicated and continued support.