SAFETY AND REGULATORY  
INFORMATION

# MEDICAL MONITOR

Please read this manual carefully before operating your set and retain it for future reference.

27HJ713S

- Be sure to see the owner's manual provided with the CD for more information about the product and its use.
  - The CD manual can be read on a computer with Adobe Reader installed.
  - You can download Adobe Reader free from the Adobe website.
- Do not modify or copy arbitrarily the owner's manual provided on CD.
- If you need a printed owner's manual, please contact the manufacturer.
  - At the request, the manufacturer provide a hard copy manual without the additional cost within 7 days.

Please note that this information is for proper use and safety of the equipment. The following symbols may indicate a hazardous situation in which, if not heeded, may result in serious injury or even death to the user or others, or damage to the equipment.

 <b>WARNING</b>	Indicates warning and safety instructions. If not adhered to, it could result in death or serious injury to the user or others.
 <b>CAUTION</b>	Indicates a hazardous situation which, if not heeded, may result in minor or moderate injury to the user or others, or damage to the equipment.

### **For users in the United States**

- United State federal law restricts this equipment to be used by or on the order of a physician.

### **For users in other countries**

- This equipment is to be used by or on the order of a licensed person under the related laws for each country.

### **WARNING**

- To avoid the risk of electric shock, this equipment must only be connected to a supply mains with protective earth.
- Do not modify this equipment without authorization of the manufacturer.

### **Intended use**

- This Medical Monitor is intended to provide color video displays and images from medical equipment which include laparoscopy and endoscopy systems for surgery and various medical imaging systems. This product does not support 3D.

Authorized representative in the European community.

**LG Electronics European Shared Service Center B.V.**

Krijgsman 1, 1186 DM Amstelveen, The Netherlands  
Tel: +31-20-456-3132

**LG Electronics Inc.**

77, Sanho-daero, Gumi-si, Gyeongsangbuk-do, 39381, Republic of Korea  
Tel: +82-1544-8777

# SAFETY INFORMATION

## Safety Standard

### Medical Device Classification

<b>Classification by protection type against Electric Shock</b>	Class I equipment
<b>Classification according to the degree of protection against ingress of water or particulate matter</b>	Front: IP35 Except for Front: IP32
<b>Mode of operation</b>	Continuous Operation
<b>Environment of Use</b>	This equipment is not suitable for use in the presence of flammable anesthetic or oxygen.

### Regulations

#### Safety and Electromagnetic Compatibility Information

This equipment has been tested and found to comply with the limits for medical devices in IEC 60601-1-2. These limits are designed to provide reasonable protection against harmful interference in a typical medical installation.

This equipment generates, uses and can radiate radio frequency energy. If not installed and used in accordance with the instructions, it may cause harmful interference to other devices in the vicinity.

However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to other devices, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures.

- Reorient or relocate the equipment.
- Increase the separation between the equipment.
- Connect the equipment into an outlet on a circuit different from that to which the other devices are connected.
- Contact manufacturer or authorized agent for help.

#### Radio Frequency compliance

##### FCC (For USA)

#### FCC NOTICE

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

**FCC CAUTION**

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment. This transmitter must not be co-located or operated in conjunction with any other antenna or transmitter.

**FCC WARNING**

This equipment may generate or use radio frequency energy changes or modifications to this equipment may cause harmful interference unless the modifications are expressly approved in the instruction manual.

The user could lose the authority to operate this equipment if an unauthorized change or modification is made.

Note: This equipment has been tested and found to comply with the limits for a Class A digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference when the equipment is operated in a commercial environment.

This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance of this equipment in a residential area is likely to cause harmful interference in which cause the user will be required to correct the interference at his own expense.

**For Europe****DECLARATION OF CONFORMITY**

The complete Declaration of Conformity may be requested through the following postal address:

LG Electronics European Shared Service Center B.V., Krijgsman 1, 1186 DM Amstelveen, The Netherlands

or can be requested at our dedicated DoC website: <http://www.lg.com/global/support/cedoc/cedoc#>

**Electro-Magnetic Compatibility Information****Electro-Magnetic Emissions**

This EUT is intended for use in the electromagnetic environment specified below.

The customer or the user of the EUT should assure that it is used in such an environment.

Immunity Test	Compliance	Electromagnetic Environment – Guidance
RF Emissions CISPR 11	Group 1	The EUT uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF Emissions CISPR 11	Class A	The EUT is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	A	
Voltage fluctuations/ Flicker emissions IEC 61000-3-3	Complies	

### Electro-Magnetic Immunity

This EUT is intended for use in the electromagnetic environment specified below.

The customer or the user of the EUT should assure that it is used in such an environment.

Immunity Test	IEC 60601-1-2 Test Level	Compliance Level	Electromagnetic Environment – Guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 8 kV contact ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air	± 8 kV contact ± 15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	± 2 kV 100 kHz repetition frequency	± 2 kV 100 kHz repetition frequency	Mains power quality should be that of a typical commercial or hospital environment.
Surge Line-to-line IEC 61000-4-5	± 0,5 kV, ± 1 kV	± 1 kV	Mains power quality should be that of a typical commercial or hospital environment.
Surge Line-to-ground IEC 61000-4-5	± 0,5 kV, ± 1 kV, ± 2 kV	± 2 kV	
Voltage dips IEC 61000-4-11	0 % $U_i$ ; 0,5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°	0 % $U_i$ ; 0,5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°	Mains power quality should be that of a typical commercial or hospital environment. If the user of the EUT image intensifier requires continued operation during power mains interruptions, it is recommended that the EUT image intensifier be powered from an uninterruptible power supply or a battery.
	0 % $U_i$ ; 1 cycle and 70 % $U_i$ ; 25/30 cycles Single phase: at 0°	0 % $U_i$ ; 1 cycle and 70 % $U_i$ ; 25/30 cycles Single phase: at 0°	
Voltage interruptions IEC 61000-4-11	0 % $U_i$ ; 250/300 cycle	0 % $U_i$ ; 250/300 cycle	
RATED power frequency magnetic fields (50/60Hz) IEC 61000-4-8	30 A/m	30 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

### ! NOTE

- $U_i$  is the A.C mains voltage prior to application of the test level.

Immunity Test	IEC 60601-1-2 Test Level	Compliance Level	Electromagnetic Environment – Guidance
Conducted disturbances induced by RF fields IEC 61000-4-6	3 V 0,15 MHz – 80 MHz 6 V in ISM bands between 0,15 MHz and 80 MHz 80 % AM at 1 kHz	3 V 0,15 MHz – 80 MHz 6 V in ISM bands between 0,15 MHz and 80 MHz 80 % AM at 1 kHz	Portable and mobile RF communications equipment should be used no closer to any part of the EUT, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.
Radiated RF EM fields IEC 61000-4-3	3 V/m 80 MHz – 2,7 GHz 80 % AM at 1 kHz	3 V/m 80 MHz – 2,7 GHz 80 % AM at 1 kHz	<p>Recommended separation distance:</p> $d = \left[ \frac{3,5}{V_1} \right] \sqrt{P}$ $d = \left[ \frac{3,5}{E_1} \right] \sqrt{P} \quad 80 \text{ MHz to } 800 \text{ MHz}$ $d = \left[ \frac{7}{E_1} \right] \sqrt{P} \quad 800 \text{ MHz to } 2.7 \text{ GHz}$ <p>Where P is the maximum output power rating of the transmitter in watts(W) according to the transmitter manufacturer and d is the recommended separation distance in meters(M).</p> <p>Field strengths from fixed RF transmitters as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range.</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol:</p> 

## ! NOTE

- At 80 MHz and 800 MHz, the higher frequency range applies.
- These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

- 1 Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the EUT is used exceeds the applicable RF compliance level above, the EUT should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the EUT.
- 2 Over the frequency range 150 kHz to 80 MHz, field strengths should be less than [V1] V/m.

**Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications equipment**

Test frequency (MHz)	Band <sup>a)</sup> (MHz)	Service <sup>a)</sup>	Modulation <sup>b)</sup>	Maximum power (W)	Distance (m)	IMMUNITY TEST LEVEL (V/m)
385	380 – 390	TETRA 400	Pulse modulation <sup>b)</sup> 18 Hz	1,8	0,3	27
450	430 – 470	GMRS 460, FRS 460	FM <sup>c)</sup> ± 5 kHz deviation 1 kHz sine	2	0,3	28
710	704 – 787	LTE Band 13, 17	Pulse modulation <sup>b)</sup> 217 Hz	0,2	0,3	9
745						
780						
810	800 – 960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation <sup>b)</sup> 18 Hz	2	0,3	28
870						
930						
1 720	1 700 – 1 990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3, 4, 25; UMTS	Pulse modulation <sup>b)</sup> 217 Hz	2	0,3	28
1 845						
1 970						
2 450	2 400 – 2 570	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation <sup>b)</sup> 217 Hz	2	0,3	28
5 240	5 100 – 5 800	WLAN 802.11 a/n	Pulse modulation <sup>b)</sup> 217 Hz	0,2	0,3	9
5 500						
5 785						

**! NOTE**

• If necessary to achieve the IMMUNITY TEST LEVEL, the distance between the transmitting antenna and the ME EQUIPMENT or ME SYSTEM may be reduced to 1 m. The 1 m test distance is permitted by IEC 61000-4-3.

a) For some services, only the uplink frequencies are included.

b) The carrier shall be modulated using a 50 % duty cycle square wave signal.

c) As an alternative to FM modulation, 50 % pulse modulation at 18 Hz may be used because while it does not represent actual modulation, it would be worst case.

## Symbols

Symbols	Descriptions
	Refer to instruction manual/ booklet
	Alternate current
	Direct current
	Off (power: disconnect from the main switch)
	On (power: connect from the main switch)
	Warning
	Caution
	UL classified mark of medical equipment with respect to electrical shock, fire and mechanical hazards only in accordance with ANSI/AAMI ES60601-1 (2005) + AMD 1 (2012), CAN/CSA-C22.2 No. 60601-1 (2014)
	Manufacturer
	Date of manufacture
	Serial number
	Non-ionizing radiation
	WEEE: Waste Electrical and Electronic Equipment
	Authorized representative in the European community.
	CE Marking

	Temperature limit
	Humidity limitation
	Pressure limitation
<b>Rx</b> ONLY	For the customers in the U.S.A. Caution Federal law (United States of America) restricts this device to sale by or on the order of a licensed healthcare practitioner.
<b>IPN<sub>1</sub>N<sub>2</sub></b>	Ingress of protection

## Warning

**WARNING:** To avoid the risk of electric shock, this equipment must only be connected to a supply mains with protective earth.

### Connection

Do not connect the monitor with anything other than specified.

Otherwise, it may result in fire or electric shock.

To avoid the risk of electric shock, this monitor must only be connected to supply mains with protective earth.

### Handling

Always be sure to keep checking the condition of the system and the patient to ensure they are normal during the use of the monitor. If any problem is found, take appropriate measures, such as stopping the operation of the monitor, as required.

Never disassemble or modify the monitor as it may result in fire or electric shock.

Also, since the monitor incorporates parts that may cause electric shocks and other hazardous parts, touching them may cause death or serious injury.

Do not hit or drop the monitor. The monitor may be damaged if it receives a strong jolt, which may result in fire or electric shock if the monitor is used without being repaired.

The operator must not touch connectors of the monitor and the patient simultaneously.

The product has lower breaking capacity type. So do not install at the building power system prospective short-circuit current exceeding 35 A.

## Caution

Do not install the ME Equipment in a location without easy disconnect accessibility.

### Environment of Use and Storage

Do not install the monitor in a location with the conditions listed below.

Otherwise, it may result in failure or malfunction, cause fire or injury.

- Close to facilities where water is used.
- Where it will be exposed to direct sunlight.
- Close to air-conditioner or ventilation equipment.
- Close to heat source such as a heater.
- Prone to vibration
- Unsecure location place.
- Dusty environment.
- Saline or sulfurous environment.
- High temperature or humidity.
- Freezing or condensation.

Do not place the storage case in a location with the conditions listed below.

- Where the cable of the monitor unit will be strongly pulled when the monitor is put into the case, otherwise, the cable may be damaged, resulting in fire or electric shock.
- Where someone might get their foot caught in the cable of the monitor.

### Handling

For safety reasons, be sure to turn off the power when monitor is not used.

This monitor is contraindicated for pregnant woman.

MANUFACTURER will make available on request circuit diagrams, component part lists, descriptions, calibration instructions, or other information that will assist SERVICE PERSONNEL to repair those parts of ME EQUIPMENT.

### Location of Cables

Make sure all cables are located so that they cannot be stepped on, tripped over, or otherwise subjected to damage or stress.

If the monitor is defective, do not disassemble the monitor randomly. Maintenance of the monitor should be done by a manufacturer.

# SAFETY PRECAUTIONS

The Safety Precautions are intended to prevent unexpected danger or harm by helping the user to use the product safely and for its intended purpose.

 <b>WARNING</b>	Failure to follow the instructions may result in serious injury or even death.
 <b>CAUTION</b>	Failure to follow the instructions may result in minor injury to people or damage to the product.

## Precautions When Installing the Product

### **WARNING**

- Do not place the product close to heat sources such as radiators, fires, stoves, etc.
- Be careful when disinfecting the product, as some disinfectants may cause fire.
- Do not install the product in a damp or dusty area.
- If the product emits smoke, strange odors or sounds, disconnect the power cord and contact the manufacturer.
- Do not leave the power or signal cable, etc. in walkways.
- Do not install the product onto a wall close to sources of oil or oil mist.
- If the product has been dropped or its case is damaged, turn off the power and unplug the power cord from the power outlet.

### **CAUTION**

- Ensure that the product is installed at least 100 mm (3.9 inches) away from the wall for good ventilation to prevent deformation of the product or a fire due to the increase in the internal temperature.
- Keep the product away from direct sunlight.
- Do not install the product close to hot objects, such as lights.
- Make sure the air vents are not blocked by a tablecloth or curtain.
- Do not install the product near magnetic field generating equipment, such as transformers or high-voltage power lines.

Installing the product in locations that do not meet the general conditions may seriously damage the product's picture quality, life cycle and appearance. Please check with our service engineer before installing in such locations.

**Locations that do not meet the general conditions:** places where the product is not intended for use, such as those where fine dust or oil mist is generated, places where chemicals are used, places where the temperature is too high or low, places where the humidity is too high, or places where the product is likely to be used for a long period of time (e.g. airports or train stations).

## Precautions for the AC/DC Adaptor and Power

### WARNING

- If water or a foreign substance enters the product (power cord or AC/DC adaptor), disconnect the power cord immediately and contact the manufacturer.
- Always connect the power cord to a grounded outlet.
- Do not touch the power plug or AC/DC adaptor with wet hands. If the pins of the plug are wet or dusty, clean and dry them before use.
- Insert the power plug securely so that it does not come loose. Do not use the product if the power outlet is loose.
- Make sure the power cord is fully inserted into the AC/DC adaptor.
- Do not insert any conductive objects into the device connection end of the power cord, while the power cord is inserted into the wall outlet. Avoid touching the power plug immediately after pulling from the wall outlet.
- Be sure to use the power cords and AC/DC adaptors provided or ones approved by LG Electronics, Inc.
- Use the product at the rated voltage only.
- Disconnect the power cable when not using the product for long periods.
- Always pull by the plug when unplugging the power cord. Do not bend the power cord with excessive force.
- Take care not to step on or place heavy objects (electronic appliances, clothing, etc.) on the power cord or AC/DC adaptor. Be careful not to damage the power cord or AC/DC adaptor with sharp objects.
- Never disassemble, repair or modify the power cord or AC/DC adaptor.
- To turn off the main power, remove the power cord, which should be positioned so it is easy to access for operation.
- If the product is connected to an AC wall outlet, it will not be disconnected from the AC power source even if you turn off the switch.

### CAUTION

- Do not remove the power cord when the product is in use.
- Be sure to keep the outlet, AC/DC adaptor and pins of the power plug clean from dust, etc.
- Do not turn the monitor on or off by plugging in or unplugging the power plug from the power outlet. (Do not use the power plug as a switch.)
- Keep the power cord away from heating devices.

## Precautions When Transporting the Product

### WARNING

- Before moving the product, disconnect the power cord and all other connected cables.
- When carrying the product, make sure the screen faces forward and hold it firmly in both hands.

 **CAUTION**

- Do not discard the delivery box with the original packaging. Put the product in the box when carrying.
- When moving or unpacking the product, make sure that two or more people safely lift it because it is heavy. Using the product after it has fallen may result in electric shock or fire. Contact the manufacturer.

## Precautions When Using the Product

 **WARNING**

- Do not disassemble, repair or modify the product by yourself. If the product needs to be checked, reset or repaired, contact the manufacturer.
- Do not allow water to enter the product and keep the product dry at all times.
- If there is a gas leak, do not touch the power outlet. Open the windows for ventilation.
- Do not impact the front or sides of the screen with a hard object, such as a metal object. Do not scratch the screen.
- If liquid or a foreign object falls into the product, switch the product off and unplug it from the wall outlet. Contact the manufacturer.
- Stop using the product if no image appears on the screen or no sound is heard. Switch the product off immediately. Unplug the product from the power outlet and contact the manufacturer.
- Do not use the product in any environment with excessively high temperatures or humidity.
- Do not use high-voltage electrical products near the product. This may result in product malfunction due to electric shock. (e.g. electrical mosquito swatter)

 **CAUTION**

- Do not use or store the product near inflammables.
- Take a break to protect your health and vision when using the product for long periods of time.
- Do not push hard against the screen surface or scratch it with your hands or sharp objects, such as a nail, pencil or pen.
- Keep the product clean.
- Refer to the Owner's Manual and set the product to the correct resolution and frequency. Otherwise, you may experience blurred vision.
- Do not leave the product where it can get wet. Do not place anything containing liquid on top of the product, such as a flower vase.
- Because the LCD requires high technology consisting of hundreds of pixels, pixelated spots (red, green, white or black) may appear on the screen during use. This is normal for an LCD screen and not an error, nor is it related to the monitor's performance.
- Given the nature of the viewing angle, the brightness and color of the LCD may vary between the left and right or top and bottom, depending on the viewing position. This is normal for an LCD screen and not an error, nor is it related to the monitor's performance.

## Precautions for Connecting to Other Medical Devices

Take the following precautions before using this product or connecting it to other medical devices.

- Stop using this device for medical purposes if you experience interruptions or discomfort in executing medical activities.
- Quick movements, shaking and the focus of the video displayed on the monitor, its distance from the user, the user's point of view, the user's physical fitness and other factors may cause discomfort (eye fatigue, dizziness, vomiting, nausea, etc.).
- Before using the product, check that the image of the connected device displays properly on the screen of this product.

## Precautions for Use with Electric Scalpels, etc.

- When this product is used with electric scalpels, etc., the image may appear unstable, distorted or abnormal due to the strong electromagnetic waves or voltage emitted by the device. This is not a malfunction.
- If you are using this device with equipment that emits strong radio waves or voltage, install this device in a direction that minimizes radio wave interference.

## Recommendation for Using Multiple Units

- Electronic products may fail unexpectedly. If using the monitor for medical, emergency or any other critical use, we strongly recommend that you use multiple monitors or have a spare monitor ready.

## Precautions When Cleaning the Product

The medical LCD monitor's front protective panel has been specially processed to reflect less light.

Using solvents such as thinner or benzene, acidic, alkaline or corrosive detergents, chemical cleaning fibers, etc. may damage the finish material on the surface.

- Unplug the power cord before cleaning.
- Never spray water directly onto the monitor or use thinner or benzene solvents, acidic, alkaline or corrosive detergents, or cleaning or sterilizing chemical cleaning fibers, etc. as they may damage the product.
- To clean the product, unplug the power cord and wipe it gently with a soft cloth. Do not spray with water or wipe with a wet cloth. When cleaning the product or the screen, do not use cleansers, automobile or industrial thinners, abrasives or wax, benzene, alcohol, etc., as this may damage the product.
- To clean the front frame, spray water onto a soft cloth two to four times and wipe in one direction only.
- Do not rub the product with a stained cloth using more force than necessary. This may scratch the surface of the product.
- Do not allow the product to remain in contact with rubber or vinyl resin for extended periods. The surface finish may deteriorate or the coating may peel off.
- If you do not clean the product, dust will accumulate inside and may create a fire hazard or malfunction.

## Image Sticking Precautions

- Displaying a still image for a prolonged period may damage the screen, resulting in image sticking. Most third-party products have the same issue. Damage resulting from long-term static images is not covered by product warranty.

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# INFORMATION FOR ENVIRONMENTAL PRESERVATION

## English

### Information for Environmental Preservation

LGE. announced the 'LG Declaration for a Cleaner Environment' in 1994, and this ideal has served as a guiding managerial principle ever since. The Declaration is a foundation that has allowed us to undertake environmentally friendly activities in careful consideration of economic, environmental, and social aspects.

We promote activities for environmental preservation, and we specifically develop our products to embrace the concept of environment-friendly. We minimize the hazardous materials contained in our products. For example, there is no cadmium to be found in our product.

### Information for recycling

This product may contain parts which could be hazardous to the environment. It is important that this product be recycled after use.

LGE. handles all waste product through an environmentally acceptable recycling method. There are several take-back and recycling systems currently in operation worldwide. Many parts will be reused and recycled, while harmful substances and heavy metals are treated by an environmentally friendly method.

If you want to find out more information about our recycling program, please contact your local LG vendor or a corporate representative of LG. We set our vision and policies on a cleaner world by selecting the issue of the global environment as a task for corporate improvement. Please visit our website for more information about our 'green' policies.  
<http://www.lg.com/global/sustainability/environment/take-back-recycling.jsp>

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### Ukraine Restriction of Hazardous Substances

The equipment complies with requirements of the Technical Regulation, in terms of restrictions for the use of certain dangerous substances in electrical and electronic equipment.

## EPEAT

"EPEAT registered where applicable/supported. EPEAT registration varies by country - see [www.epeat.net](http://www.epeat.net) for registration status by country"

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## ENGLISH

### Disposal of your old appliance

- 1 This crossed-out wheeled bin symbol indicates that waste electrical and electronic products (WEEE) should be disposed of separately from the municipal waste stream.
- 2 Old electrical products can contain hazardous substances so correct disposal of your old appliance will help prevent potential negative consequences for the environment and human health. Your old appliance may contain reusable parts that could be used to repair other products, and other valuable materials that can be recycled to conserve limited resources.
- 3 You can take your appliance either to the shop where you purchased the product, or contact your local government waste office for details of your nearest authorised WEEE collection point. For the most up to date information for your country please see [www.lg.com/global/recycling](http://www.lg.com/global/recycling)

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### WEEE

#### Disposal of your appliance

The symbol indicating separate collection for electrical and electronic equipment consists of the crossed-out wheeled bin.

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# INDIA ONLY

## Disposal of your old appliance (as per e-waste Rules)

1. When this crossed out wheeled bin symbol is depicted on the product and its operator's manual, it means the product is covered by the e-waste Management and Handling Rules, 2011 and are meant to be recycled, dismantled, refurbished or disposed off.

### 2. Do's

- a. The product is required to be handed over only to the authorized recycler for disposal.
- b. Keep the product in isolated area, after it becomes non-functional/unrepairable so as to prevent its accidental breakage.

### Don't

- a. The product should not be opened by the user himself/herself, but only by authorized service personnel.
  - b. The product is not meant for re-sale to any unauthorized agencies/scrap dealer/kabariwalah.
  - c. The product is not meant for mixing into household waste stream..
  - d. Do not keep any replaced spare part(s) from the product in exposed area.
3. Any disposal through unauthorized agencies/person will attract action under Environment (Protection) Act 1986.
4. "This product is complied with the requirement of Hazardous Substances as specified under Rule 13 (1) & (2) of the E-Waste (Management & Handling) Rules, 2011"
5. To locate a nearest collection centre or call for pick-up (limited area only) for disposal of this appliance, please contact Toll Free No. 1800-315-9999 / 1800-180-9999 for details. All collection centre and pick up facilities are done by third parties with LG Electronics India Pvt. Ltd. Merely as a facilitator.  
For more detailed information, please visit : <http://www.lge.com/in>.

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# CONFORMITY NOTICE

## ENGLISH

Hereby, LG Electronics European Shared Service Center B.V., declares that this product is compliant with the essential requirements. The complete Declaration of Conformity may be requested through the following postal address:

LG Electronics European Shared Service Center B.V.,  
Krijgsman 1, 1186 DM Amstelveen, The Netherlands  
or can be requested at our dedicated DoC website:  
<http://www.lg.com/global/support/cedoc/cedoc#>

**WARNING:** This equipment is compliant with Class A of CISPR 32. In a residential environment this equipment may cause radio interference.

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