


UVC-PKCART-ULTRA SURFACE DISINFECTION UNIT

QUICK START GUIDE

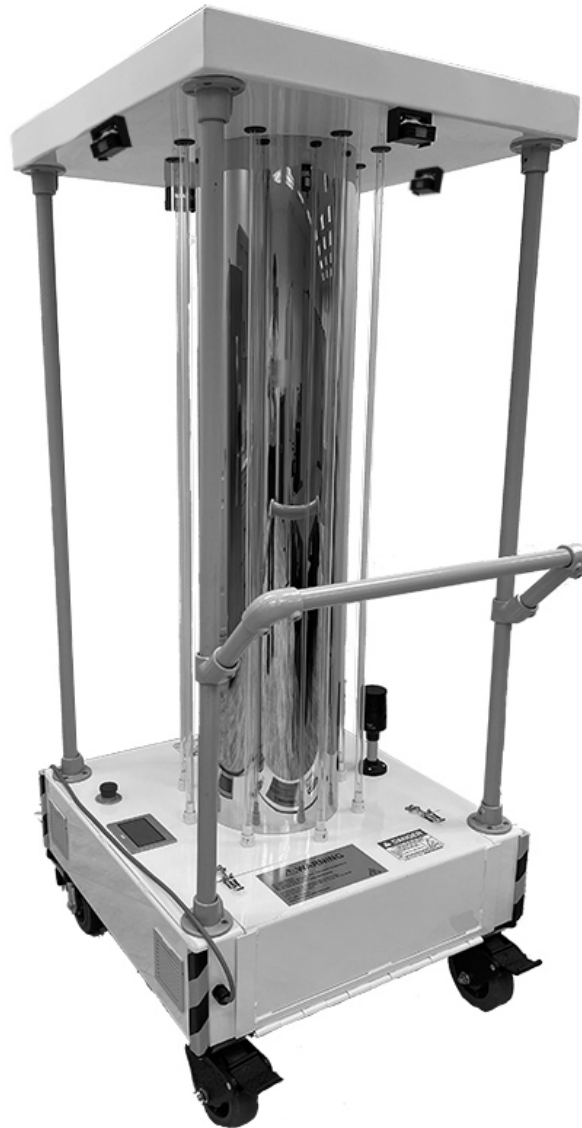

Models:
UVC-PKCART-ULTRA


ERICSON QUICK START GUIDE

Step 1: Position and Plug-In

- Inspect UVC-PKCART-ULTRA for damage prior to placement for use.
- Wear proper PPE when operating UVC-PKCART-ULTRA
- Place UVC-PKCART-ULTRA in treatment space
- Position a PIR Sensor toward entry point
- Plug UVC-PKCART-ULTRA into standard 120v NEMA 5-15 or NEMA 5-20 receptacle or connector


Step 2: Initiating Treatment Cycle

- Press "Bacteria" or "Spores" operation mode from Home screen (See device User Manual for more information)
- Select "Start Cycle" to begin factory 30 Seconds Exit/Countdown Timer*
- Device calculated treatment run time
- Optional: For custom programming, select "User Settings"
 - *Factory times superseded after custom user input

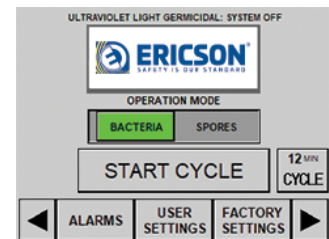


Step 3: Custom Programming (optional)

- At Home screen, select "User Settings"
- Use Up/Down arrows to adjust Countdown Timer
- Times will remain for subsequent cycles until manually adjusted
- Run Hours, Cycle Counts and Lamp Life visible here
- Return to Home screen to start treatment cycle

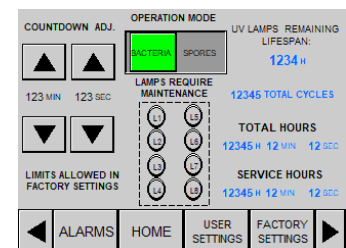

Step 4: Room Measure Viewing (optional)


- Press bottom Right Arrow from Home screen
- Observe (4) laser measured distances to wall
- Cycle time calculation based on Operation Mode input and longest distance measured
- Note: If device max distance measure (16') is registered on any sensing laser, device will automatically run for 30 min "Bacteria" or 60 min "Spores"

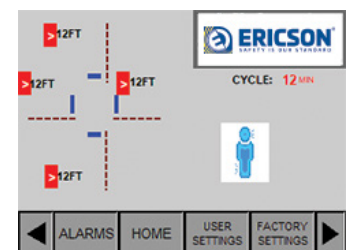

ERICSON QUICK START GUIDE


Step 5: Start Cycle

- Press "Start Cycle" from Home screen
- Observe Countdown timer action
- Leave treatment space immediately
- Close door to area and post Warning Signage
- Audible countdown alarm tone with increased frequency as timer nears zero

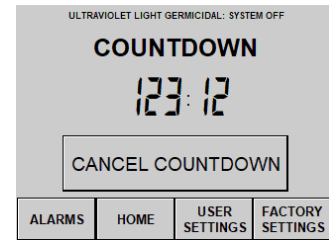

Step 6: In Process

- Treatment cycle "ON" with timer screen
- Cycle will proceed unless Safety PIR Sensor or Emergency Shutoff interruption
- Avoid direct exposure to UV light
- Do not look at UV light without approved safety glasses

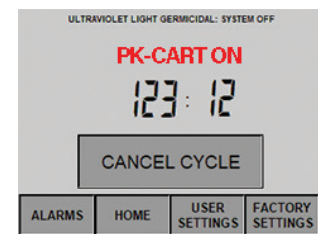

Step 6a: PIR Sensor Fault

- "Person Detected" screen will appear if PIR Sensor activated
- Cycle deactivation will occur and event will be logged
- Select "Home" to restart cycle process



Step 6b: E-Stop Fault

- "Emergency Stop Active" screen will appear if E-Stop Button activated
- Cycle deactivation will occur and event will be logged
- Deactivate E-Stop Button clearing alarm log from red to green
- Select "Home" to restart cycle process



Additional Information

- Please contact manufacturer for:
 - Troubleshooting
 - FAQ's
 - Diagnostics



Electrical Manufacturing

4323 Hamann Parkway
Willoughby, OH 44094
info@ericson.com
ericson.com
1.800.374.2766
C8021245A

