	TEXT PROPOSAL FOR PACKAGE LEAFLET	April 2021
	VIZOL S 0.21% FAST RELIEF VIZOL S 0.4% LONG EFFECT eye drops, 10 ml	page: 1/4

**Vizol S 0.21% FAST RELIEF
Vizol S 0.4% LONG EFFECT**
eye drops

Instruction for use - please read attentively

COMPOSITION

Sodium chloride, sodium dihydrogen phosphate dihydrate, disodium phosphate anhydrous, sodium hyaluronate, purified water.

Preservative-free.

PURPOSE

Vizol S 0.21% FAST RELIEF drops for dry eyes are intended for relieving **mild symptoms** of dry eyes.

Vizol S 0.4% LONG EFFECT drops for dry eyes are intended for relieving **severe symptoms** of dry eyes.

Eye drops serve to protect, moisten and lubricate the eye. Drops application provides fast relief of discomforts.

Dry eye symptoms can be alleviated and complications thus prevented.

Eye dryness may occur due to:

- disorder of the tear film caused by tear deficiency or their profuse evaporation which can lead to damage of the ocular surface and symptoms of discomfort
- environmental conditions (central heating, air conditioners, fluorescent lighting, smoke and wind)
- prolonged computer work
- use of decongestant eye drops
- ageing (75% of population over the age of 65 suffers from the dry eye syndrome)
- wearing contact lenses
- surgical treatments or damage to the eye
- gender (hormone-related changes in women during pregnancy, breastfeeding, menstruation or menopause)
- use of medicinal products (antidepressants, hypertension treatment, contraception)
- diseases (rheumatoid arthritis, diabetes, asthma, diseases of the thyroid gland, glaucoma).


Vizol S 0.21% FAST RELIEF drops for dry eyes can also be used while wearing contact lenses.

Vizol S 0.4% LONG EFFECT are viscous drops which makes sodium hyaluronate last longer on the ocular surface, thus providing prolonged moistening and enhanced protection. While using Vizol S 0.4% drops for dry eyes, it is recommended to take out contact lenses for 15 minutes during instillation.

Sodium hyaluronate is present in the aqueous and vitreous humour of the eye.

In the Vizol S drops for dry eyes, with each blink of the eye, sodium hyaluronate creates a protective film over the ocular surface. In that way it removes the sense of dryness, sandiness and irritation.

They are also used after surgical treatments and for dry eyes resulting from applying anti-inflammatory medical products to the eye (after consultations with the doctor).

	TEXT PROPOSAL FOR PACKAGE LEAFLET	April 2021
	VIZOL S 0.21% FAST RELIEF VIZOL S 0.4% LONG EFFECT eye drops, 10 ml	page: 2/4

WARNINGS AND PRECAUTIONS

Do not use Vizol S:

- in the case of hypersensitivity (allergy) to sodium hyaluronate or any other ingredient of Vizol S eye drops.
In the case of an allergic reaction, stop using the product immediately and contact a doctor.

Other remarks:

- If you notice that the Vizol S bottle or applicator is damaged do not use the product.
- To ensure safe application and hygiene, each bottle should be used by one person only.
- There are no known interactions between Vizol S eye drops and contact lens solutions, other eye drops or topical eye ointments. If you use another eye product, wait at least 15 minutes between applications.

Note:

Vizol S 0.4% can momentarily blur vision. It is therefore recommended that during this short period of time you refrain from driving a car or operating a vehicle or machine but wait until your sight becomes sharp again.

PRODUCT SAFETY

The dropper technology includes a filter which makes the product **safe for use** because it prevents the contamination of the liquid even after opening.

The product does not contain preservatives, which reduces the possibility of irritation and damage to the surface of the eye.


The product is generally well tolerated. However, if you notice any side effects, contact your doctor or pharmacist.

INSTRUCTIONS FOR USE


Drops are used several times a day, as necessary. With correct use, one bottle is enough for 30 days of use. For prolonged use, please consult your doctor.

Instil 1-2 drops into the eye, blink several times so that the solution is distributed equally over the ocular surface.

Do not contain preservatives and are therefore not harmful for application to the eye.

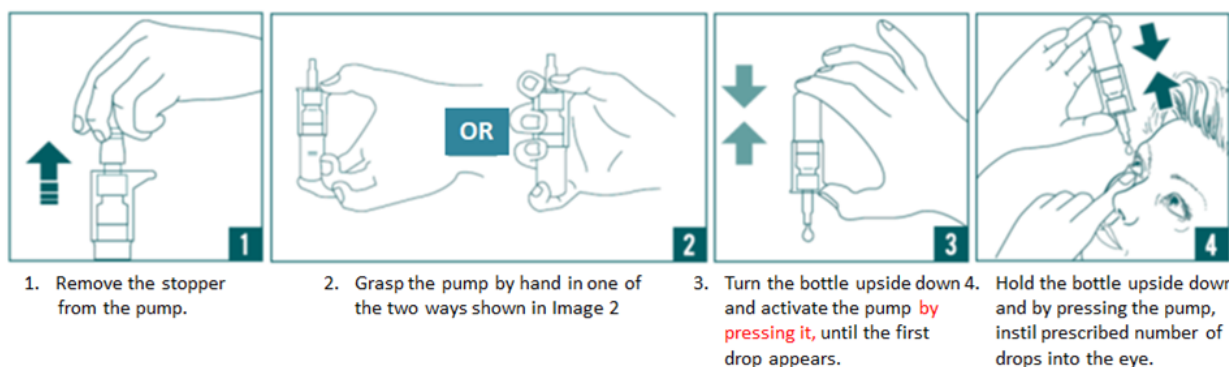

TRY THE OTHER DRY EYE DROPS FROM THE VIZOL S PRODUCT LINE:

	TEXT PROPOSAL FOR PACKAGE LEAFLET	April 2021
	VIZOL S 0.21% FAST RELIEF VIZOL S 0.4% LONG EFFECT eye drops, 10 ml	page: 3/4

PRODUCT	PURPOSE	COMPOSITION	CAUSE OF DISTURBANCES	MODE OF APPLICATION	USE WITH CONTACT LENSES	FEELING AFTER APPLICATION
				Apply 1-2 drops in the eye, several times a day, or as required.		
Vizol S 0.21% FAST RELIEF	Relieving mild dry eye symptoms	sodium hyaluronate 0.21%	Long-lasting computer work, eye fatigue while driving and traveling, wearing lenses, allergy, foreign object, external factors (dry air, smoke)		Yes	Immediate relief and refreshment. Long-lasting feeling of moistening.
Vizol S 0.4% LONG EFFECT	Relieving severe dry eye symptoms	sodium hyaluronate 0.4%	Aggressive external factors (wind, sun, dry air), centrally heated and air-conditioned spaces, aging, pharmacotherapy		No	Immediate relief and long effect of moistening. Long-lasting feeling of comfort and lubrication. The product can momentarily blur vision.
Vizol S INTENSIVE	Relieving dry eye symptoms and regenerating eye surface	sodium hyaluronate 0.15% dexpanthenol 2%	Mechanical eye irritants: wearing lenses, surgical or diagnostic interventions, medication.		Yes	Immediate refreshment and long-lasting feeling of comfort and moistening.

MANUFACTURER

JADRAN-GALENSKI LABORATORIJ d.d., Svilno 20, 51000 Rijeka, Croatia

EXPIRY DATE

Printed on the packaging.

Expiry date after opening is 180 days.

Do not use the product after its expiry date.

STORAGE

Store the product below 30°C in original packaging.

Keep out of sight and reach of children.

PACKAGING


10 ml



REVISION DATE

April 2021

Key to symbols on the product

	TEXT PROPOSAL FOR PACKAGE LEAFLET	April 2021
	VIZOL S 0.21% FAST RELIEF VIZOL S 0.4% LONG EFFECT eye drops, 10 ml	page: 4/4



Read the Instructions for use



Manufacturer



Expiry date



Batch number

CE CE mark



Sterile, aseptic production



30°C Upper limit of temperature



Caution, check the Instructions for use (for important warnings/precautions)

Additional symbols on the box:

Vizol S 0.21% FAST RELIEF



Immediate relief of dry eye symptoms due to extended periods of screen time.



Moisturizing and alleviation of eye fatigue while driving and traveling.



Alleviation of eye discomfort and irritation while wearing lenses.



Expiry date after opening 6 months

Vizol S 0.4% LONG EFFECT



Moisturizing and protection for severe dry eye symptoms.



Immediate relief from eye irritation and prolonged effect.



Moisturizing and eye protection in dry and air-conditioned areas.



Expiry date after opening 6 months