	TEXT PROPOSAL FOR LEAFLET	April 2021
	Vizol S ALLERGY eye drops, 10 ml	2 % Ectoin®

Vizol S ALLERGY EYE DROPS

Instructions for use - please read attentively

Read this information carefully before you start using Vizol S ALLERGY. **DO NOT THROW THIS LEAFLET AWAY!** It contains important information. If you have any questions about Vizol S ALLERGY please ask your doctor or pharmacist.

COMPOSITION

Ectoin®, hydroxyethyl cellulose, sodium chloride, water, citrate buffer.

PRESERVATIVE-FREE

PURPOSE

Vizol S ALLERGY isotonic eye drops is a Medical Device which is used to prevent and relieve the symptoms of allergic conjunctivitis. Symptoms such as itching, redness, or excessive watering of the eyes can be effectively alleviated.

Within 30 seconds a quick reduction of symptoms like itching, tearing and eye irritation will occur.

Vizol S ALLERGY protects against the harmful influences of allergens and supports the regeneration process of irritated and sensitive conjunctiva.

The product is suitable for contact lens wearers, children and people with sensitive eyes.

WHAT DO VIZOL S ALLERGY EYE DROPS CONTAIN?

Vizol S ALLERGY eye drops contain Ectoin®, a natural molecule obtained from microorganisms resistant to extreme conditions. To survive, these organisms produce extremolytes.

Ectoin® is a well-researched extremolyte that can be used to protect mucosal cells.


WARNINGS AND PRECAUTIONS

Do not use Vizol S ALLERGY:

- in the case of hypersensitivity (allergy) to Ectoin® or any other ingredient of Vizol S ALLERGY eye drops.
In the case of an allergic reaction, stop using the product immediately.
- if the product is damaged or punctured.
- for more than 3 months after opening.

Other remarks:

- To ensure safe application and hygiene, each bottle should be used by one person only.
- There are no data available regarding interactions between Vizol S ALLERGY and other eye drops or topical eye ointments, therefore the usage with other ophthalmological treatments is not recommended.

	TEXT PROPOSAL FOR LEAFLET	April 2021
	Vizol S ALLERGY eye drops, 10 ml	2 % Ectoin®

- Vizol S ALLERGY eye drops should be kept out of sight and reach of children.

PRODUCT SAFETY

The applicator technology includes a filter that makes the product **safe for use**, as it prevents contamination of the solution after opening.

The product does not contain preservatives, which reduces the possibility of irritation and damage to the surface of the eye.

The product is generally well tolerated. Up to now, no regularly occurring or persisting side effects are known. In isolated cases, it may cause short-term irritation, eye redness or conjunctivitis. If you notice any side effects that are associated with the use of Vizol S ALLERGY, contact your doctor or pharmacist.

If condition persists or worsens, contact your doctor or pharmacist.

ADDITIONAL INFORMATION

If recommended by your doctor, Vizol S ALLERGY may be used after eye surgery and trauma. Currently, there are no data available regarding the use of Vizol S ALLERGY during pregnancy and while breast-feeding.

Consult your doctor or pharmacist before using Vizol S ALLERGY, if you are:

- pregnant or breast-feeding
- not sure whether you are pregnant or not.


Children 10 years of age and younger and patients unable to use Vizol S ALLERGY on their own should be assisted during the application of Vizol S ALLERGY.

APPLICATION


Always use Vizol S ALLERGY according to these instructions for use. Consult your doctor or pharmacist before use if you are not sure. **The drops can be used several times a day as required. Apply 1-2 drops to the eye and blink several times to distribute the solution evenly across the surface of the eye.**

If necessary, Vizol S ALLERGY can be used multiple times per hour.

Before the first application, it is necessary to activate the bottle. For this purpose, press the dosage ring several times while holding the bottle upside down.

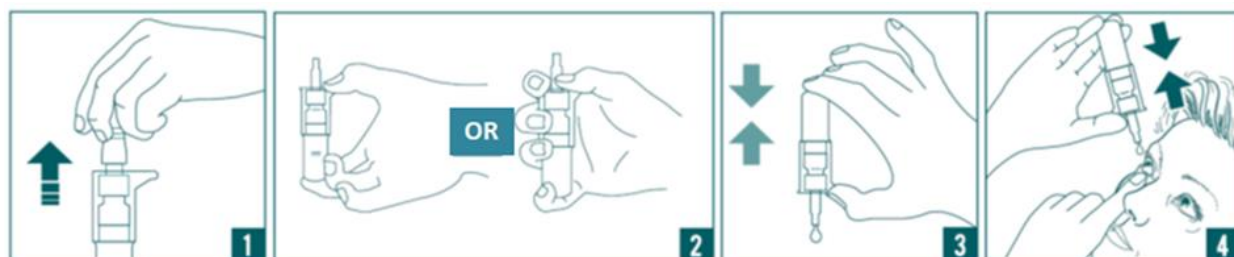

1. Remove the protective cap (stopper) from the pump. Be careful not to touch anything with the tip.

	TEXT PROPOSAL FOR LEAFLET	April 2021
	Vizol S ALLERGY eye drops, 10 ml	2 % Ectoin®

2. Grasp the pump with your hand as shown in Fig.2
 3. Turn the bottle upside down and activate the pump by pressing it until the first drop appears. Do not cut the tip.
 4. Hold the bottle upside down and by pressing the pump, put the prescribed number of drops into the eye. Avoid direct contact of the bottle tip with your fingers, eyes or contact lenses.
- After each use dry the tip and close tip with the protective cap.

STORAGE

Keep the product at a temperature from 2°C to 25°C.

Do not use the product after the expiry date indicated on the box and on the bottle.

PACKAGING

10 ml

TRY OTHER EYE DROPS FROM THE VIZOL S PRODUCT LINE:

PRODUCT	PURPOSE	COMPOSITION	CAUSE OF DISTURBANCES	MODE OF APPLICATION	USE WITH CONTACT LENSES	FEELING AFTER APPLICATION
Vizol S 0,21% FAST RELIEF	Relieving mild dry eye symptoms	sodium hyaluronate 0.21%	Long-lasting computer work, eye fatigue while driving and traveling, wearing lenses, allergy, foreign object, external factors (dry air, smoke)	Apply 1-2 drops in the eye, several times a day or as required	Yes	Immediate relief and refreshment. Long-lasting feeling of moistening.
Vizol S 0,4% LONG EFFECT	Relieving severe dry eye symptoms	sodium hyaluronate 0.4%	Aggressive external factors (wind, sun, dry air), centrally heated and air-conditioned spaces, aging, pharmacotherapy		No	Immediate relief and long effect of moistening. Long lasting feeling of comfort and lubrication. The product can momentarily blur vision.
Vizol S INTENSIVE	Relieving dry eye symptoms and regenerating eye surface	sodium hyaluronate 0.15% dexpanthenol 2%	Mechanical eye irritants: wearing lenses, surgical or diagnostic interventions, medication		Yes	Immediate refreshment and long-lasting feeling of comfort and moistening.

MANUFACTURER

bitop AG Stockumer Str. 28, 58453 Witten, Germany

DISTRIBUTOR


JGL d.d., Svilno 20, 51000, Rijeka, Croatia

Phone: +385 51 546 399

Website: www.jgl.hr

DATE OF LAST REVISION

April 2021

	TEXT PROPOSAL FOR LEAFLET	April 2021
	Vizol S ALLERGY eye drops, 10 ml	2 % Ectoin®

Key to symbols on the product




Manufacturer



Expiry date



Batch number

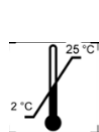

Sterile, aseptic processing technique

Keep at a temperature from 2°C to 25°C

CE mark with Notified Body Number



Please read the instructions for use



Please do not use damaged or punctured products

Additional symbols on the box:



Protects
from the
influence
of allergens



**Immediate
relief** from
eye
irritation



**Alleviation
of excessive
tearing,
itching and
eye redness.**



**Suitable
for contact
lens
wearers**