

VIZOL S HYDRO LIPID BALANCE

eye drops, emulsion

Instructions for use – please read attentively

Vizol S HYDRO LIPID BALANCE eye drops are intended to alleviate symptoms of moderate to severe dry eye due to deficient lipid layer of tear film, which leads to hyper evaporation of the aqueous layer and dryness of the eye.

COMPOSITION

Vizol S HYDRO LIPID BALANCE are sterile eye drops, emulsion which contains sodium hyaluronate 0.15%, castor oil 0.25%, tocophersolan (vitamin E TP GS), glycerol, polysorbate 80, macrogolglycerol hydroxystearate, trometamol, hydrochloric acid, disodium edetate, sodium chloride, purified water. Castor Oil, vitamin E TP GS and macrogolglycerol hydroxystearate form a Long Chain Triglyceride (LCT) complex.

Preservative-free.

Phosphate-free.

PURPOSE

Vizol S HYDRO LIPID BALANCE eye drops are intended to alleviate symptoms of moderate to severe dry eye. They restore the lipid layer of the tear film, stabilize the tear film, and optimally moisturize the eye surface.

Whether dry eye is caused by insufficient aqueous, or insufficient lipid component of tears, the dried surface of the eye causes unpleasant symptoms, which only worsen over time when left untreated adequately. Eye dryness can be manifested by a feeling of sand or foreign body in the eye, irritation, burning, itching, excessive tearing and redness of the eye.

Eye dryness is associated with the aging process, the use of certain medications, hormonal changes, eye strain (wearing contact lenses), various environmental influences (dry air, wind, air conditioning), and numerous risks of modern lifestyle such as prolonged viewing of screens (computers, mobile phones, TVs), when the number of blinks per minute is significantly reduced, and thus the optimal secretion of tears.

Vizol S HYDRO LIPID BALANCE contain nano-droplets of **LCT complex** (formed of castor oil, vitamin E and surfactants) **to restore and protect the lipid layer of tears, which is a natural barrier against moisture evaporation.**

Additionally, sodium hyaluronate contained in Vizol S HYDRO LIPID BALANCE eye drops effectively **moisturizes the eye** and immediately relieves symptoms of dryness.

Suitable for all types of dry eyes.

WARNINGS AND PRECAUTIONS

Do not use Vizol S HYDRO LIPID BALANCE:

- in case of hypersensitivity (allergy) to any ingredient of the product.

In case of an allergic reaction, stop using the product immediately and contact a doctor.

Other warnings:

- If you notice that the product bottle or applicator is damaged, do not use it.
- To ensure safe use and hygiene, each bottle should be used by one person only.
- There are no known interactions between Vizol S HYDRO LIPID BALANCE eye drops and other eye drops or eye ointments. If you use another eye product, consult your doctor before using Vizol S HYDRO LIPID BALANCE.

Vizol S HYDRO LIPID BALANCE is intended for use by adults aged 18 and older.

For paediatric use, please consult a healthcare professional before administering these eye drops to ensure they are suitable for your child.

Note:

The product may cause slight blurring of vision. In this case, it is recommended that during this short period of time you refrain from driving a car or operating a vehicle or machine but wait until your sight becomes sharp again.

PRODUCT SAFETY

Vizol S HYDRO LIPID BALANCE eye drops are **preservative** and **phosphate free**, which reduces the possibility of irritation and damage to the surface of the eye.

The dropper technology includes a filter which makes the product safe for use because it prevents the contamination of the eye drops and preserves sterility of the contents even 6 months after opening.

The product is generally very well tolerated, however if you notice any side effects, contact to your doctor or pharmacist.

HOW TO USE VIZOL S HYDRO LIPID BALANCE

Always use Vizol S HYDRO LIPID BALANCE according to these instructions for use. Consult your doctor or pharmacist before use if you are not sure.

If not otherwise prescribed by your doctor, instill 1-2 drops in the eyes, several times a day, as needed. Blink several times to distribute the drop evenly over the eye surface.

Avoid contact of the dropper tip with the surface of the eye or any other surface.


Before first application, it is necessary to activate the bottle. For this purpose, remove the stopper from the pump, turn the bottle upside down and **press the applicator several times until the first drop appears**.

Information for contact lens users


The product is compatible with contact lenses.

Instructions for use


1. Wash your hands thoroughly.
2. Remove the stopper from the pump (image 1).
3. Grasp the pump by hand as shown in image 2.
4. Turn the bottle upside down and press the applicator until the first drop appears (image 3).
5. With your head back, carefully instill 1 drop in the eye (image 4).
6. Replace the stopper on the pump.

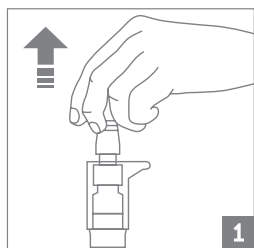


1. Remove the stopper from the pump.

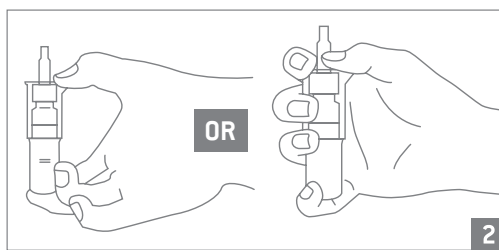

2. Grasp the pump by hand in one of the two ways shown in Image 2.

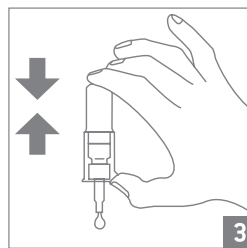

3. Turn the bottle upside down and activate the pump by pressing it, until the first drop appears.

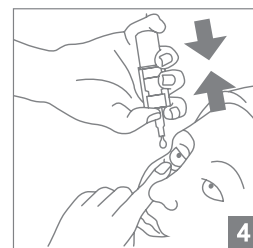

4. Hold the bottle upside down and by pressing the pump, instil prescribed number of drops into the eye.

MANUFACTURER

JADRAN-GALENSKI LABORATORIJ d.d. Svilno 20, 51000 Rijeka, Croatia

EXPIRY DATE

Printed on the packaging.

Period after opening is 6 months.

Do not use the product after its expiry date.

STORAGE

Store the product below 30°C in original packaging.

Keep out of sight and reach of children.

PACKAGING












10 ml



REVISION DATE

June 2024.

Key to symbols on the product

Symbol	Explanation	Symbol	Explanation
	Read Instructions for use		30°C Upper limit of temperature
	Medical device		Single patient multiple use
	Manufacturer		Do not use damaged or punctured products
	Expiry date		Caution, check the Instructions for use (for important warnings/precautions)
	Batch number		CE mark
	Sterile, aseptic processing technique		

Additional symbols on the box:

VIZOL S HYDRO LIPID BALANCE



Moisturizing and protection for moderate to severe dry eye



Replenishment and stabilization of the lipid and aqueous layer of the tear film



Restoration of the lipid layer of tears, which is a natural barrier against moisture evaporation



Expiry date after opening is 6 months