



Bovichek® Neospora
Neospora caninum Antibody Test Kit, ELISA

Evaluated by an independent reference laboratory
Biovet BOVICHEK® Neospora kit passed the test with flying colors!

Neospora caninum, a protozoal parasite, is a major cause of abortion in cattle. Once the animal is infected, the risk of aborting is 2- to 3-fold greater than in noninfected animals. Furthermore, if abortion does not occur, the animal will likely, 8 times out of 10, give birth to infected progeny. ELISA testing allows assessing exposure to the parasite.

Major advantage of Bovichek® Neospora assay

- Indirect ELISA • 2 plate format (2 x 90 tests) • results in less than 2 hours •

	BIOVET study	Saluvet* study (2012)
Sensitivity	99.4% (95% CI = 96.1 – 100%)	98.9% (95% CI = 97.6% – 100%)
Specificity	99.1% (95% CI = 98.1 – 99.6%)	98.9% (95% CI = 97.4% – 100%)

* Group SALUVET from the University of Madrid, Spain is a renowned research center for studies on Neospora.

IMPORTANT:

- Requires a permit to release veterinary biologics from CFIA in Canada. This product is not licensed by CFIA and any claims made have not been substantiated by CFIA.
- Requires a Research and Evaluation import permit from the USDA in the US. This product is not licensed by the USDA and any claims made have not been substantiated by the USDA.

USA
1-877-8BIOVET
 (824-6838)

Biovet®
 order@biovet-inc.com

International & Canada
1-888-8BIOVET
 (824-6838)

Bovichek® Neospora

Kit components	Quantity
• 12 strips of 8 wells coated with <i>N. caninum</i> Ag	2
• Ready-to-use positive control	2.5 mL
• Ready-to-use negative control	2.5 mL
• Concentrated conjugate	125 µL
• Concentrated wash solution (10X)	2 X 100 mL
• Ready-to-use substrate	30 mL
• Ready-to-use stop solution	30 mL

The materials provided are sufficient for testing up to 180 samples.

Materials required but not provided:

- Purified water
- Adjustable single- and multi-channel pipettes
- Single-use pipette tips
- Test tubes for sample dilution
- ELISA 96-well microplate reader equipped with 405 nm filter
- Containers for preparation of solutions

Rev. 2017-06-19

For further information visit www.biovet-inc.com or contact customer service.

USA
1-877-8BIOVET
(824-6838)


order@biovet-inc.com

International & Canada
1-888-8BIOVET
(824-6838)