


2015-04-30

OVUCHECK® Rapid Well

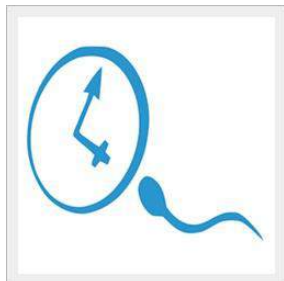

PRODUCT CODE: TRM-548

DESCRIPTION

OVUCHECK® RAPID WELL is used to measure the level of progesterone in a drop of cow's milk. The quantity of progesterone present is indicated by a change in colour, which is compared with a standard. OVUCHECK® RAPID WELL is a simple and rapid test which provides a reliable aid to cow fertility management.

SUMMARY AND EXPLANATION

The amount of progesterone is almost zero at oestrus (heat) and then increases and remains at a high level. The progesterone level falls suddenly 17 to 20 days after the last heat, indicating the onset of the next cycle. This fall does not occur if the animal is pregnant.

CHARACTERISTICS

- Qualitative test
- Rapid (Less than 30 minutes)
- Competitive binding of progesterone
- Tests per kit: 31 possible tests

ASSAY PROCEDURE

INCUBATION TIME/ TEMP.

1. Incubation of samples, standard and conjugate	5 min / AT
2. Successive washes	
3. Incubation of substrate B and C	5 min / AT
4. Interpretation of the results	

AT: Ambient Temperature



Biovet inc.
4375, Beaudry Ave
St-Hyacinthe QC J2S 8W2
Canada
T 1-888-8BIOVET
F 450 771-2996
order@biovet-inc.com

www.biovet.ca