


2014-12-23

Poultry Check MP[®] MS-MG

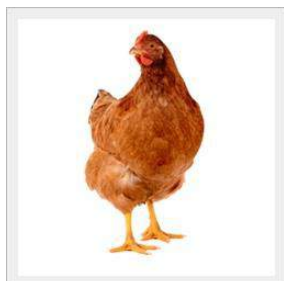

PRODUCT CODE: TRM-555

DESCRIPTION

Poultry Check MP[®] Ms-Mg Assay is a multiplexed fluorometric immunoassay (MFIA) intended for detecting antibodies to *Mycoplasma synoviae* (MS) and *Mycoplasma gallisepticum* (MG) in chicken serum.

SUMMARY AND EXPLANATION

Mycoplasma synoviae (MS) and *Mycoplasma gallisepticum* (MG) are two major bacterial pathogens for poultry flocks. The surveillance of poultry flocks for MS and MG relies on regular testing of serum samples for the presence of specific antibodies.

CHARACTERISTICS

- Multiplexed fluorometric immunoassay (MFIA)
- Results in less than 2 hours
- 460 tests per kit

ASSAY PROCEDURE

INCUBATION TIME/ TEMP.

| | |
|--|---------------------------|
| 1. Preparation of samples and reagents | |
| 2. Incubation of samples and controls | 60 minutes / 19°C to 26°C |
| 3. Serial washings | |
| 4. Incubation of conjugate | 15 minutes / 19°C to 26°C |
| 5. Serial washings | |
| 6. Revelation with substrate | 15 minutes / 19°C to 26°C |
| 7. Serial washings | |
| 8. Plate reading with a special reader | |



Biovet inc.
4375, Beaudry Ave
St-Hyacinthe QC J2S 8W2
Canada
T 1-888-8BIOVET
F 450 771-2996
order@biovet-inc.com

www.biovet.ca

IMPORTANT :

- Requires a permit to release veterinary biologics from CFIA in Canada. This product is not licensed by CFIA and any claims made have not been substantiated by CFIA.
- Requires a Research and Evaluation import permit from the USDA in the US. This product is not licensed by the USDA and any claims made have not been substantiated by the USDA.