


# AQUACEL<sup>®</sup> Pro Foam

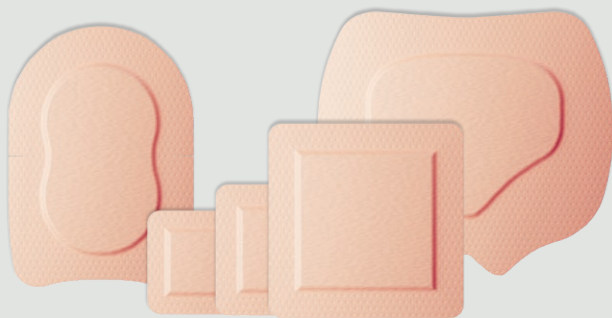

## PRODUCT DESCRIPTION:<sup>1</sup>

AQUACEL<sup>®</sup> Foam Pro dressing is a sterile, Hydrofiber<sup>™</sup> foam wound dressing, consisting of:

- a waterproof, breathable outer polyurethane film which provides a bacterial and viral barrier
- a soft, multi-layered, absorbent central pad containing polyurethane foam and Hydrofiber<sup>™</sup> (Sodium carboxymethylcellulose). The pad absorbs and retains moisture, exudates and bacteria
- a perforated silicone adhesive wound/skin contact layer to adhere the dressing to the patient

## AQUACEL<sup>™</sup> Foam Pro Dressings

Dressing Size PAD	Max wound size	CVT number	pieces/package
AQUACEL <sup>®</sup> Foam Pro			
8 x 8 cm	5 x 5 cm	422359	10
10 x 10 cm	6,5 x 6,5 cm	422357	10
15 x 15 cm	11 x 11 cm	422358	10
AQUACEL <sup>®</sup> Foam Pro Heel/Multisite			
19,8 x 14 cm	14 x 8,7 cm	422356	10
AQUACEL <sup>®</sup> Foam Pro Sacral			
20 x 16,9 cm	11,4 x 13,5 cm	421579	5
24 x 21.5 cm	16,4 x 13,8 cm	421580	5

If the dressing is cut, additional tape may be required to secure it in place.



**convatec**  
— forever caring —

# CLINICAL BENEFITS:<sup>1</sup>

AQUACEL® Foam Pro is designed to protect fragile skin from moisture, shear and friction damage with a gentle but effective adhesive silicone contact layer.

AQUACEL® Foam Pro is also designed to provide a moist wound healing environment, whilst effectively managing excess exudate levels which may further damage the wound bed and surrounding skin.

## PRECAUTIONS AND OBSERVATIONS

Sterility is guaranteed unless the pouch is damaged or opened prior to use. Do not use the device if the sterile barrier is damaged or opened prior to use and dispose of the device according to local regulations.

Do not use after stated expiry date.

AQUACEL® Foam Pro dressings are for single use only, do not reuse. Reuse may lead to increased risk of infection and cross-contamination and delayed healing.

This dressing should not be used with other wound care products without first consulting a healthcare professional. AQUACEL® Foam Pro dressings are water resistant however, should not be submerged during bathing or swimming. AQUACEL® Foam Pro dressings are not compatible with oil-based products.

A healthcare professional should be consulted if any of the following are observed during dressing changes, irritation (reddening, inflammation), maceration (whitening of skin), hyper-granulation (excess tissue formation), signs of infection (increased pain, bleeding, warmth/redness of surrounding tissue, wound exudate), or a change in wound colour and/or odour.

## DIRECTIONS FOR USE

Always check instructions for use please see IFU in product box before clinical use



## ADDITIONAL INFORMATION

Can be used with compression therapy <sup>2</sup> .	YES
Help facilitate easy removal <sup>2</sup>	The perforated silicone layer provides skin-friendly adhesion and supports atraumatic removal.
MRI safe	YES
Exudate levels	AQUACEL® Foam Pro is designed to provide a moist wound healing environment, whilst effectively managing excess exudate levels which may further damage the wound bed and surrounding skin.

## INTENDED USE:

AQUACEL® Foam Pro dressings may be used, with the consultation of a healthcare professional, for the management of exuding, non-exuding wounds and protection of intact skin.

AQUACEL® Foam Pro dressing may be included in a comprehensive protocol of care to protect intact skin against breakdown.

## INDICATIONS FOR USE:

AQUACEL® Foam Pro dressing is indicated for the treatment of:

- Leg ulcers
- Pressure Ulcers
- Diabetic Foot Ulcers
- Surgical Wounds
- Traumatic Wounds.

## INTENDED PATIENT POPULATION:<sup>2</sup>

AQUACEL® Foam Pro is intended to be used on patients with chronic or acute wounds or those who require protection from skin breakdown.

## DURATION OF USE

AQUACEL® Foam Pro dressings can be worn for up to 7 days, dressings should be changed earlier if clinically indicated, and are safe for continuous use.




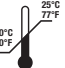


Do not re-use

## CONTRAINDICATIONS

AQUACEL® Foam Pro dressings should not be used on individuals who are sensitive to or who have had an allergic reaction to the dressing or its components.

<sup>2</sup>Associated Products and Therapies:  
AQUACEL® Foam Pro dressing may be used alone or in combination with other wound care products, and retention bandages, as directed by a Health Care Professional. Appropriate supportive measures should be taken when used in combination with adjunct therapies e.g. compression bandaging or pressure relieving equipment.

STERILIZATION:	
Sterile	YES
Method	Sterilized using ethylene oxide
CERTIFICATIONS:	
CE mark	CE 2797
MDR 2017/745	Class IIb
MATERIAL SPECIFICATIONS:	
List of ingredients	 Sodium carboxymethylcellulose, Polyurethane Film, Polyurethane Foam and Silicone
STORAGE AND HANDLING:	
Before Use   	Keep away from sunlight  Keep dry.  Store at room temperature (10°C - 25°C/50°F - 77°F).
After use	After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local state and federal laws and regulations.

**References:**

1. Aquacel® Foam Pro IFU Revised: 2023-05. 1727787V1
2. Clinical Evaluation Report Name: RPT-006796 Title: AQUACEL Foam Pro\_DHF704\_CER (2021) Convatec Data on file

**Convatec Limited**

First Avenue,  
Deeside Industrial Park Deeside,  
Flintshire CH5 2NU,  
UK



**UNOMEDICAL A/S**

Aaholmvej 1-3, Osted  
4320 LEJRE  
DENMARK



**Convatec Denmark A/S**

CVR 31477093,  
Transformervej 14, 1. , 2860 Soborg,  
Denmark  
Tlf.: +45 48 16 74 74  
www.convatec.dk

**Convatec Norway AS**

Org. nummer: MVA 992 780 118  
Visiting address:  
Nils Hansens Vei 8, 0667 Oslo  
Postadresse:  
Postboks 6464 Etterstad, 0605 Oslo  
Tlf.: +47 21 09 67 90  
www.convatec.no

**Convatec (Sweden) AB**

Organisationsnummer: 556754-8473,  
Box 3096, 16973 Solna, Sweden  
Visiting address:  
Gustav III:s Boulevard 42, plan 9,  
16973 Solna, Sweden  
Tlf.: +46 020-21 22 22  
www.convatec.se

**Convatec Oy**

Y-tunnus 1644589-9,  
Aviapolis, Karhumäentie 3, 01530 VANTAA,  
Finland  
P: +358 020 7659 600  
www.convatec.fi