

# AQUACEL<sup>®</sup>



## PRODUCT DESCRIPTION<sup>1</sup>:

AQUACEL<sup>™</sup> Ribbon Dressing with Strengthening Fibre is a soft, sterile, non-woven ribbon dressing composed of sodium carboxymethylcellulose and regenerated cellulose fibre for strengthening. This conformable and highly absorbent dressing absorbs wound fluid and transforms into a soft gel, which maintains a moist environment to support the body's healing process and aid in the removal of nonviable tissue from the wound (autolytic debridement), without damaging newly formed tissue.

### Aquacel<sup>™</sup> Ribbon Dressing

Dressing Size	CVT number	Pieces/package
1cm x 45cm	420127	5
2cm x 45cm	403770	5

## CLINICAL BENEFITS<sup>1</sup>:

AQUACEL™ Ribbon Dressings absorb wound fluid and bacteria, providing a moist wound healing environment, aiding autolytic debridement and removing dead-space between the wound and dressing interface.

AQUACEL™ Ribbon Dressings are designed to manage exudate levels which may further damage the wound bed and surrounding skin.

AQUACEL™ Ribbon Dressings provide a barrier to protect the wound bed from contamination.

### INTENDED USE<sup>1</sup>:

AQUACEL™ Ribbon Dressings have been designed to be used as a primary dressing for cavity wounds. They are intended to be used under the direction of a healthcare professional as per the indications for use.

### INTENDED USER<sup>1</sup>:

AQUACEL™ Ribbon Dressings are intended to be used by healthcare professionals, carers and patients under the direction of a Healthcare Professional.

### DIRECTIONS FOR USE<sup>1</sup>:

Always check instructions for use please see IFU in product box before clinical use



### ADDITIONAL INFORMATION<sup>1</sup>:

Can be used with compression therapy	YES
Help facilitate easy removal <sup>2</sup>	YES
Exudate management	Low to heavy*
MRI safe	YES
Waterresistant	NO

\* Place the AQUACEL™ Ribbon Dressing with Strengthening Fibre on the wound and wet with sterile water or saline over the wound area only.

### INDICATIONS FOR USE<sup>1</sup>:

AQUACEL™ Ribbon Dressings are indicated for:

- Diabetic foot ulcers
- Pressure ulcers/injuries
- Surgical wounds
- Traumatic wounds

### INTENDED PATIENT POPULATION<sup>1</sup>:

AQUACEL™ Ribbon Dressings are intended to be used on patients with one of the wound types listed in the indications for use.

### DURATION OF USE<sup>1</sup>:

AQUACEL™ Ribbon Dressings can be worn for up to 7 days, dressings should be changed earlier if clinically indicated.





Do not re-use

Aquacel™ Ag+ Ribbon Dressings are for single-use only and should not be re-used. Re-use may lead to increased risk of infection, cross-contamination or delayed healing.

### CONTRAINDICATIONS<sup>1</sup>:

AQUACEL™ Ribbon Dressings should not be used on individuals who are sensitive to or who have had an allergic reaction to the dressing or any of its components.

STERILIZATION <sup>1</sup> :	
Sterile	YES
Method	Sterilized using irradiation
CERTIFICATIONS <sup>1</sup> :	
CE mark	CE 2797
MDR 2017/745	Class IIb
MATERIAL SPECIFICATIONS <sup>1</sup> :	
List of ingredients	 Non-woven ribbon dressing composed of sodium carboxymethylcellulose and regenerated cellulose fibre for strengthening.
STORAGE AND HANDLING <sup>1</sup> :	
Before Use	Keep away from sunlight
	Keep dry.
	Store at room temperature (10°C - 25°C/50°F - 77°F).
After use	After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local state and federal laws and regulations.

**References:**

1. Aquacel™ Ribbon Dressing IFU Issued 2023-03 1738658V1
2. CER, version 10.0

**Convatec Limited**

First Avenue,  
Deeside Industrial Park Deeside,  
Flintshire CH5 2NU,  
UK



**UNOMEDICAL A/S**

Aaholmvej 1-3, Osted  
4320 LEJRE  
DENMARK



**Convatec Denmark A/S**

CVR 31477093,  
Transformervej 14, 1. , 2860 Soborg,  
Denmark  
Tlf.: +45 48 16 74 74  
[www.convatec.dk](http://www.convatec.dk)

**Convatec Norway AS**

Org. nummer: MVA 992 780 118  
Visiting address:  
Nils Hansens Vei 8, 0667 Oslo  
Postadresse:  
Postboks 6464 Etterstad, 0605 Oslo  
Tlf.: +47 21 09 67 90  
[www.convatec.no](http://www.convatec.no)

**Convatec (Sweden) AB**

Organisationsnummer: 556754-8473,  
Box 3096, 16973 Solna, Sweden  
Visiting address:  
Gustav III:s Boulevard 42, plan 9,  
16973 Solna, Sweden  
Tlf.: +46 020-21 22 22  
[www.convatec.se](http://www.convatec.se)

**Convatec Oy**

Y-tunnus 1644589-9,  
Aviapolis, Karhumäentie 3, 01530 VANTAA,  
Finland  
P: +358 020 7659 600  
[www.convatec.fi](http://www.convatec.fi)