

# AQUACEL™ Extra™



## PRODUCT DESCRIPTION<sup>1</sup>:

AQUACEL™ EXTRA™ Dressings are soft, sterile, dressings made from two layers of sodium carboxymethylcellulose stitched together with strengthening fibres.

This dressing absorbs high amounts of wound fluid and bacteria and creates a soft, cohesive gel that intimately conforms to the wound surface, maintains a moist environment to support the body's healing process and aids in the removal of non-viable tissue from the wound (autolytic debridement).

## AQUACEL™ Extra™ Dressings

Dressing Size	CVT number	Pieces/ package	Can be cut
5cm x 5cm	420671	10	yes
10cm x 10cm	420672	10	yes
15cm x 15cm	420673	5	yes
4cm x 10cm	420820	10	yes
4cm x 20cm	420821	10	yes
4cm x 30cm	420822	10	yes



**convatec**  
— forever caring —

## CLINICAL BENEFITS<sup>1</sup>:

AQUACEL™ EXTRA™ Dressings absorb wound fluid and bacteria, providing a moist wound healing environment, aiding autolytic debridement and removing dead-space between the wound and dressing interface.

AQUACEL™ EXTRA™ Dressings are designed to manage excess exudate levels, which may further damage the wound bed and surrounding skin.

AQUACEL™ EXTRA™ Dressings provide a barrier to protect the wound bed from contamination.

### INTENDED USE<sup>1</sup>:

AQUACEL™ EXTRA™ Dressings have been designed to be used as a primary dressing under the direction of a healthcare professional as per the indications for use.

### INTENDED USER<sup>1</sup>:

AQUACEL™ EXTRA™ Dressings are intended to be used by healthcare professionals, carers and patients under the direction of a Healthcare Professional.

### DIRECTIONS FOR USE<sup>1</sup>:

Always check instructions for use please see IFU in product box before clinical use.



### ADDITIONAL INFORMATION<sup>1</sup>:

Can be used with compression therapy	YES
Help facilitate easy removal	YES
Exudate levels	Low to high
MRI safe	YES

### INDICATIONS FOR USE<sup>1</sup>:

AQUACEL™ EXTRA™ Dressings are indicated for:

- Leg ulcers, including:
  - Venous ulcers
  - Arterial ulcers
- Diabetic ulcers including diabetic foot ulcers
- Pressure ulcers/injuries
- Surgical wounds
- Traumatic wounds
- Malignant wounds

### INTENDED PATIENT POPULATION<sup>1</sup>:

AQUACEL™ EXTRA™ Dressings are intended to be used on patients with one of the wound types listed in the indications for use.

### DURATION OF USE<sup>1</sup>:


AQUACEL™ EXTRA™ Dressings can be worn for up to 7 days, dressings should be changed earlier if clinically indicated.



Do not re-use

### CONTRAINDICATIONS<sup>1</sup>:

AQUACEL™ EXTRA™ Hydrofiber™ Dressing with Strengthening Fibre should not be used on individuals who are sensitive to or who have had an allergic reaction to the dressing or its components

STERILIZATION <sup>1</sup> :	
Sterile	YES
Method	Sterilized using irradiation
CERTIFICATIONS <sup>1</sup> :	
CE mark	CE 2797
MDR 2017/745	Class IIb
MATERIAL SPECIFICATIONS <sup>1</sup> :	
List of ingredients	Two layers of sodium carboxymethylcellulose stitched together with strengthening fibres
STORAGE AND HANDLING <sup>1</sup> :	
Before Use	Keep away from sunlight
	Keep dry.
	Store at room temperature (10°C - 25°C/50°F - 77°F).
After use	After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations.

**References:**

1, Aquacel™ Extra™ IFU Issued 2023-03  
1738657V1

**Convatec Limited**

First Avenue,  
Deeside Industrial Park Deeside,  
Flintshire CH5 2NU,  
UK



**UNOMEDICAL A/S**

Aaholmvej 1-3, Osted  
4320 LEJRE  
DENMARK



**Convatec Denmark A/S**

CVR 31477093,  
Transformervej 14, 1., 2860 Soborg,  
Denmark  
Tlf.: +45 48 16 74 74  
[www.convatec.dk](http://www.convatec.dk)

**Convatec Norway AS**

Org. nummer: MVA 992 780 118  
Visiting address:  
Nils Hansens Vei 8, 0667 Oslo  
Postadresse:  
Postboks 6464 Etterstad, 0605 Oslo  
Tlf.: +47 21 09 67 90  
[www.convatec.no](http://www.convatec.no)

**Convatec (Sweden) AB**

Organisationsnummer: 556754-8473,  
Box 3096, 16973 Solna, Sweden  
Visiting address:  
Gustav III:s Boulevard 42, plan 9,  
16973 Solna, Sweden  
Tlf.: +46 020-21 22 22  
[www.convatec.se](http://www.convatec.se)

**Convatec Oy**

Y-tunnus 1644589-9,  
Aviapolis, Karhumäentie 3, 01530 VANTAA,  
Finland  
P: +358 020 7659 600  
[www.convatec.fi](http://www.convatec.fi)

For full description and instructions for use please see IFU in product box  
The information in this document is subject to change without notice.  
(C) 2023 ConvaTec. ® /TM indicate trademarks of the ConvaTec group of companies  
AP-66155-FIN-ENG-v1 (v1.0)

