









Instructions for Use

UNITY[™]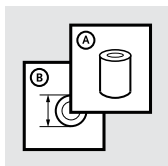

		3
EN	Instructions for Use	5
DE	Gebrauchsanweisung	9
FR	Notice d'utilisation	14
ES	Instrucciones para el uso	18
IT	Istruzioni per l'uso	22
DA	Brugsanvisning	26
SV	Bruksanvisning	30
EL	Οδηγίες Χρήσης	34
FI	Käyttöohjeet	38
NL	Gebruiksaanwijzing	42
PT	Instruções de Utilização	46
JA	取扱説明書	50
ZH	中文说明书	54
KO	사용 설명서	58

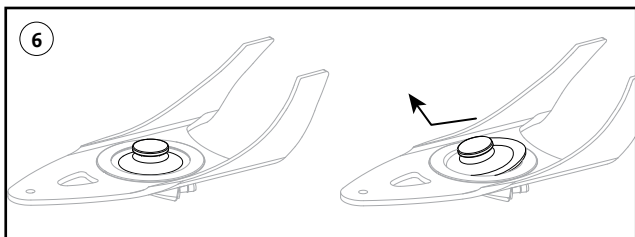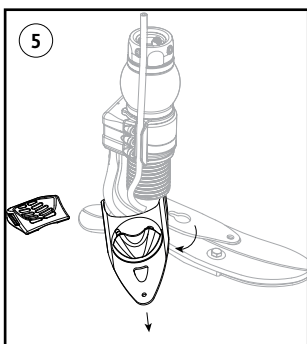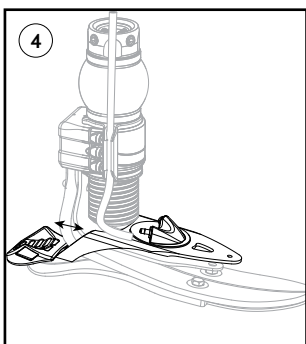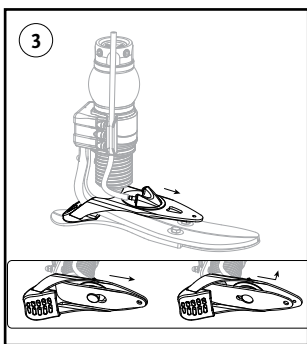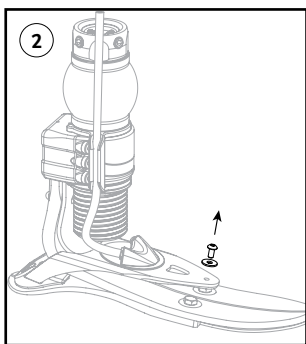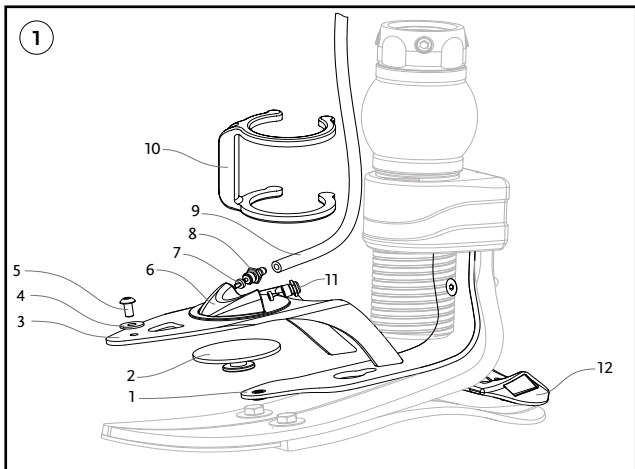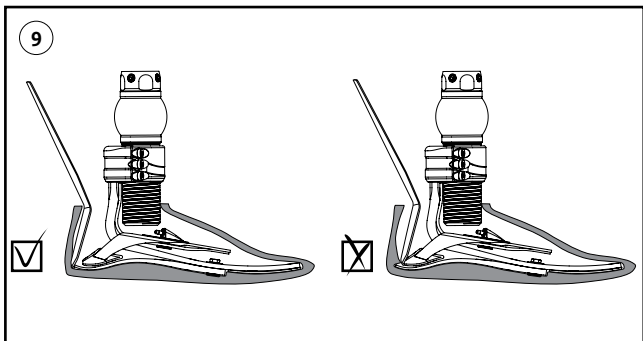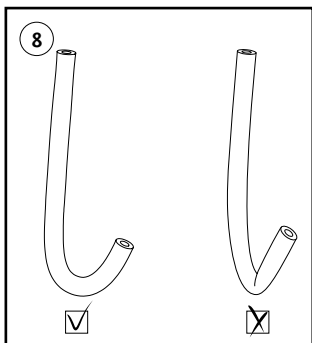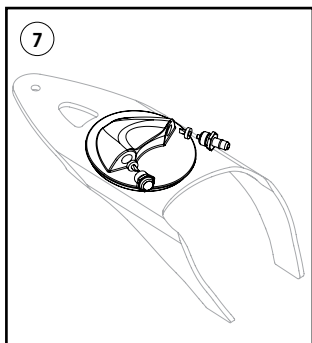UNITY™ SYSTEM COMPONENTS (Figure 1)

1. Lower blade
2. Membrane
3. Upper blade
4. Washer
5. Screw
6. Membrane housing
7. Duckbill
8. Inlet port
9. Tube
10. Tube fixture
11. Outlet port
12. Heel

Re-Flex Rotate / Re-Flex Shock

See IFU for the Re-Flex foot kits

SAFETY PRECAUTIONS

Please follow the safety precautions as failure to do so can lead to malfunction of the Unity™ System and risk injury to the user.

- This product is to be used exclusively for lower part of the body.
- Use the Unity™ Socket Valve Kit to connect the system to the socket.
- Make sure that the socket is a close fit to prevent blistering and/or wounds.
- Make sure the tube is not folded, bent, or closed off in any way (**Figure 8**).
- Make sure the tube is secured firmly away from any sharp objects or components.

Clinical specialists should make sure their clients are aware of and understand the following:

- The tube must be well secured at all times.
- Never operate this product if it is not working properly. In such cases the user should return the product to the clinical specialist.

CAUTION: Federal law requires this device to be sold by a prosthetist and by the order of a physician. The clinical specialist must provide each user with training for appropriate use and proper application of the Unity™ System.

UNITY™ SYSTEM OVERVIEW

The Unity™ System is designed to provide elevated vacuum for socket suspension. Expected attainable vacuum that the Unity™ System achieves is ranging from 15-18 inHg.

With the Unity™ System, assisted suspension is obtained through the act of a pump which draws air out from the socket in each step creating an elevated vacuum inside the socket interface.

The Unity™ System is provided preassembled to a Re-Flex Rotate / Re-Flex Shock foot module.

The Unity™ System is designed to function in any temperature and humidity that the user encounters during normal use.

NOTE: The use of wedges can affect the number of steps needed for full vacuum.

NOTE: The Unity™ System is designed to be used with Iceross Seal-In liners, other applications are sole responsibility of the provider.

LOW TO HIGH IMPACT USE

Description: Community and workplace ambulatory with ability or potential for cadence variation and ramp/stair descent.

Activity examples: Shopping, gardening, longer walks, golfing, hiking.

CONTRAINDICATIONS FOR USE:

Make sure that the socket is well fitting before any use; a loose fitting or leaking socket may never be used with the Unity™ System.

STRUCTURAL SPECIFICATION

Net weight: approx.110g including tube.

Frame construction: Carbon fiber frame

Clearance and dimensions: See IFU for Re-Flex Rotate and Re-Flex Shock. Re-Flex Shock lowest build height setting is 20mm taller than without the Unity™ System.

Follow local ordinances or regulations for proper disposal of device, accessories and packaging.

FITTING THE UNITY™ SYSTEM

NOTE: To connect the system to a socket a Unity™ Socket Valve Kit is recommended, the use of other applications is the sole responsibility of the provider.

1. Fit the user with the foot containing the system
2. Cut the tube to a proper length and connect to the Unity™ System and the socket (using the Unity™ Socket Valve kit)
3. Use the supplied tube fixture to secure the tube against the foot. The tube should be located medially rather than laterally to avoid damage during ambulation.

Once the Unity™ System has been connected make sure the pump is working properly; creating vacuum during gait and not leaking air.

DISASSEMBLING / ASSEMBLING THE UNITY™ SYSTEM

The Unity™ System can be disassembled in case it needs to be cleaned or for troubleshooting purposes.

CAUTION: The Unity™ System should only be disassembled / assembled by a clinical specialist.

HOW TO DISASSEMBLE THE UNITY™ SYSTEM

CAUTION: The inlet and outlet (**Figure 7**) ports on the membrane housing should not be disassembled unless absolutely necessary. This part of the system contains miniature duckbill valves that are easily lost and can cause the Unity™ System to malfunction if not correctly reassembled.

1. Use the Flex-foot shoehorn (or equivalent) to remove the foot from the foot cover (**Figure 9**).
2. Disconnect the tube from the Unity™ Membrane housing.
3. Untighten the screw that fastens the lower and upper blade, remove the screw (**Figure 2**).
4. Release the upper blade by sliding it towards the toe and release it from the keyhole on the lower blade (**Figure 3**).
5. Remove the heel pad (**Figure 4**).
6. Rotate the upper blade to either side and separate it from the foot (**Figure 5**).
7. Gently pull the membrane out of its housing by pulling the aluminum

insert sideways (**Figure 6**).

HOW TO ASSEMBLE THE UNITY™ SYSTEM

1. Lubricate the membrane before sliding it into the housing with Iceross Clean&Simple spray or equivalent.
2. Mount the upper blade back on the foot and slide the membrane insert into the lower blade keyhole.
3. Fasten the upper blade with the screw using medium strength Loctite 243 or equivalent (tighten the screw with 2Nm torque) take care not to over tighten the screw.
4. Snap the heel pad back in place.
5. Connect the tube to the inlet port and use the tube fixture to secure the tube against the foot.
6. Make sure that the pump is functioning properly and the system is not leaking.

FINISHING / LINER OPTIONS

Össur recommends that the Unity™ System is used with the Iceross Seal-In V liner.

CAUTION: Make sure the Flex-Foot sock does not interfere with the Unity™ System as it could compromise the function of the pump.

MAINTENANCE

The following components of the Unity™ System might need cleaning or replacing during the lifetime of the product. This is to be considered normal wear and is not covered by the Limited Product Warranty.

- Membrane
- Duckbill valves
- Tube

It is recommended to have the product inspected every 6 months by a clinical specialist.

Clean the components with mild soap and rinse with water, isopropyl alcohol or Iceross Clean&Simple spray.

DO NOT use solvents stronger than isopropyl alcohol.

Please refer to the current Össur printed catalog or online pages via www.ossur.com for replacement parts available and appropriate item numbers.

IN CASE THE UNITY™ SYSTEM IS NOT FUNCTIONING PROPERLY

Check the following:

- Is there a leakage in the socket?
- Is the tube punctured?
- Is the membrane punctured or torn?
- Has the membrane been pulled out of the membrane housing?
- Is there dust or grit inside the system?
- Is the tube fully inserted on the inlet port? (Check both at the Unity™ Socket valve and at the Unity™ Membrane housing)
- Are the inlet and outlet ports tightened?
- Is the Unity™ Socket valve leaking?
- Is the sock inside the foot cover affecting the systems mechanical movements?

AFTER DISASSEMBLY / ASSEMBLY

Check the following:

- Are the duckbill valves oriented in the correct direction (**Figure 7**)?

- Is the membrane fully inserted into the housing?
- Are the inlet and outlet ports tightened?
- Is the tube fully inserted on the inlet port? (Check both at the Unity™ Socket valve and at the Unity™ Membrane housing)

THE TUBE

The tube is the most sensitive part of the Unity™ System and special attention should be paid to it in case of any problems.

Make sure the tube is NOT:

- Folded (**Figure 8**).
- Pressed up against sharp objects.
- Punctured.
- Dangling during gait.
- Located laterally, as there is more likelihood of damage.

WARRANTY

The Unity™ System has a 24 month limited warranty, excluding the membrane.

The membrane has a 12 month limited warranty.

The following activities will void the warranty:

- Exposure to seawater or extreme corrosive environment.
- Usage with Copolymer Gel liners.
- Usage without liners or other means which can lead to sweat or other fluids than Iceross Lubricant entering the system.
- Exposure to environment with extreme airborne particle contamination.
- Abuse and damage caused by abnormal impact.
- Tampering with the lower blade connection to foot module.
- Dents and scratches on sealing surfaces.

A weatherproof device allows use in a wet and/or humid environment but does not allow submersion. Fresh water splashing against the enclosure from any direction shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

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EN – Caution: Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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