

# BRACING & SUPPORT CATALOGUE



2020 BRACING AND SUPPORT CATALOGUE | ÖSSUR



# ÖSSUR BRACING & SUPPORT CATALOGUE

## TABLE OF CONTENT

<b>SPINE</b> .....	<b>9</b>
MIAMI J® .....	11
MIAMI J® SELECT .....	13
MIAMI JTO® THORACIC EXTENSION .....	14
OCCIAN® BACK .....	15
MIAMI Jr® .....	16
PHILADELPHIA® ADJUSTABLE TRACHEOTOMY COLLAR .....	17
PHILADELPHIA® TRACHEOTOMY COLLAR .....	18
NECLOC® .....	19
NECLOC® KIDS .....	20
MIAMI LSO® .....	21
MIAMI TLSO 464 .....	23
MIAMI TLSO 456 .....	25
MIAMI LUMBAR® POSTEO .....	27
PAPOOSE® .....	28
ÖSSUR FORMFIT® PRO BACK .....	29
<b>FORMFIT® BACK SUPPORT AIR</b> .....	<b>30</b>
<b>FORMFIT® BACK SUPPORT</b> .....	<b>31</b>
<b>HIP</b> .....	<b>33</b>
REBOUND® HIP .....	35
UNLOADER® HIP .....	37
<b>UPPER EXTREMITY</b> .....	<b>39</b>
<b>FORMFIT® SHOULDER BRACE</b> .....	<b>41</b>
PREMIUM SHOULDER IMMOBILISER .....	42
BERGSCHULTZ UNIVERSAL ARM SLING .....	42
PAEDIATRIC UNIVERSAL ARM SLING .....	43
FORMFIT® PRO ELBOW .....	44
INNOVATOR X® POST-OP ELBOW .....	45
FORMFIT® TENNIS ELBOW .....	46
FORMFIT® PRO WRIST .....	47
FORMFIT® WRIST / FORMFIT® WRIST & FOREARM .....	48
FORMFIT® UNIVERSAL WRIST & THUMB .....	49
FORMFIT® THUMB SPICA .....	50
<b>KNEE</b> .....	<b>51</b>
UNLOADER ONE® X .....	52
UNLOADER ONE® .....	56
UNLOADER ONE® CUSTOM .....	59
REBOUND® CARTILAGE .....	62
CTI® OA .....	64
REBOUND® DUAL .....	66
REBOUND® PCL .....	68
REBOUND® ACL .....	70
CTI® OTS .....	72
CTI® CUSTOM .....	75
REBOUND® POST-OP .....	78
<b>FORMFIT POST-OP KNEE</b> .....	<b>79</b>

# ÖSSUR BRACING & SUPPORT CATALOGUE

FORMFIT® KNEE IMMOBILISER .....	80
ÖSSUR FORMFIT® PRO KNEE .....	81
FORMFIT® HINGED LATERAL J .....	83
FORMFIT® KNEE HINGED .....	84
FORMFIT® KNEE MCL .....	85
FORMFIT® KNEE ROM BRACE .....	86
NEOPRENE HINGED KNEE SUPPORT .....	87
NEOPRENE WRAPAROUND HINGED KNEE SUPPORT .....	88
NEOPRENE KNEE SLEEVE .....	89
<b>FOOT AND ANKLE .....</b>	<b>91</b>
REBOUND® DIABETIC WALKER .....	93
DH OFFLOADING WALKER® .....	95
REBOUND® AIR WALKER .....	96
FORMFIT® WALKER AIR .....	98
FORMFIT® WALKER .....	99
PAEDIATRIC WALKER .....	100
AFO DYNAMIC .....	101
AFO LIGHT .....	103
AFO LEAF SPRING .....	104
REBOUND FOOT-UP® .....	105
FOOT-UP® .....	106
ÖSSUR FORMFIT® PRO ANKLE .....	107
FORMFIT® ANKLE BRACE .....	109
AIRFORM® UNIVERSAL INFLATABLE ANKLE STIRRUP .....	110
FORMFIT® NIGHT SPLINT .....	111
DH OFFLOADING POST-OP STYLE SHOE .....	112
MESH TOP POST-OP SHOE .....	113
SOFT TOP POST-OP SHOE .....	113
<b>CASTING &amp; TRACTION .....</b>	<b>115</b>
TECHFORM® PREMIUM .....	117
NOTES .....	118



## ÖSSUR OA SOLUTIONS

Enhance quality of life and mobility for people living with osteoarthritis.

These product solutions have been built on decades of experience and are designed to enhance the quality of life of people living with osteoarthritis by reducing pain and improving mobility. Through innovation leadership, OA expertise and partnerships with medical professionals, we offer comprehensive, proven and effective treatment options.

### UNLOADER®

The Össur Unloader® brand name has become synonymous with osteoarthritis bracing, representing OA Solutions' technology as the industry standard, with more than 15 clinical studies demonstrating its efficacy.

Össur strives to maintain its position as the innovation leader for OA Solutions by continuing to develop simple, yet effective clinically proven treatment solutions that increase people's quality of life.



## ÖSSUR INJURY SOLUTIONS

For people recovering from fractures, ligament injuries or in need of post-operative treatment solutions.

These product solutions have been built on decades of experience and are designed to enhance the healing process of bone and soft tissue injuries. Through innovation leadership, proven biomechanical approaches, and partnerships with leading medical professionals, we strive to offer comprehensive and effective treatment options.

Össur Injury Solutions are globally recognised, primarily under the brand names CTi®, Rebound® and Miami®, all of which are developed to assist healing and increase people's quality of life.

Our expertise in Injury Solutions has driven us to take the next step forward by developing indication-specific products which are designed to optimise healing while maintaining function and mobility. These Functional Healing products incorporate Össur proprietary technology and are designed to enhance clinically relevant outcomes.



FUNCTIONAL HEALING –  
WHERE MOBILITY MEANS RECOVERY.

SPINE



TABLE OF CONTENT

**SPINE** ..... 9

MIAMI J® ..... 11

MIAMI J® SELECT ..... 13

MIAMI JTO® THORACIC EXTENSION ..... 14

OCCIAN® BACK ..... 15

MIAMI Jr® ..... 16

PHILADELPHIA® ADJUSTABLE TRACHEOTOMY COLLAR ..... 17

PHILADELPHIA® TRACHEOTOMY COLLAR ..... 18

NECLOC® ..... 19

NECLOC® KIDS ..... 20

MIAMI LSO® ..... 21

MIAMI TLSO 464 ..... 23

MIAMI TLSO 456 ..... 25

MIAMI LUMBAR® POSTEO ..... 27

PAPOOSE® ..... 28

ÖSSUR FORMFIT® PRO BACK ..... 29

FORMFIT® BACK SUPPORT AIR ..... 30

FORMFIT® BACK SUPPORT ..... 31

MIAMI J®

Known as the superior c-spine immobiliser through multiple independent studies, the Miami J's patented design meets the specialised needs of individuals with cervical injuries. Bioengineered to minimise pressure points in key known areas: Chin, occiput, trapezius, and clavicle. Miami J offers features that enhance compliance, while providing immobilisation, inhibition of skin deterioration, and patient comfort.



INDICATIONS FOR USE

Situations requiring gross immobilisation of the cervical spine, these may include:

- Pre and post c-spine surgery
- C-spine precaution for trauma patients


FEATURES AND BENEFITS

- Sorbatex™ padding is antibacterial and clinically proven to inhibit microbial growth, which enhances comfort and helps assure patient compliance
- Comfort adjustment buttons for occiput and sternal pad is removable when supine, lying and swallowing
- MRI, CT and X-ray lucent, MR compatible

REQUIRED MEASUREMENTS

- Sizing based on 6 phenotypes to ensure correct fit and optimum immobilisation

Phenotype-driven sizing system makes size selection simple while drastically reducing margin of error in fitting a patient. Each collar has a bioengineered front with a specific back designed to work together for superior immobilisation. For paediatric sizing, please see Miami Jr in this catalogue.

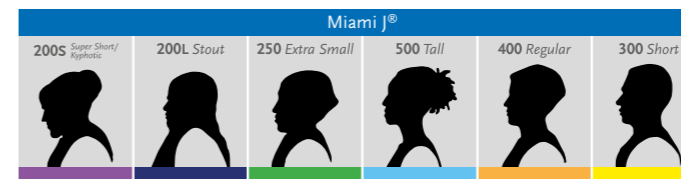

## MIAMI J®

### ORDERING INFORMATION

#### MIAMI J

Size	Collar set*	Collar	Collar Back	Collar Front
Super Short	MJR-200S	MJ-200S	MJ-400BK	MJ-200SFRT
Stout	MJR-200L	MJ-200L	MJ-200LBK	MJ-200LFRT
XS	MJR-250	MJ-250	MJ-250BK	MJ-250FRT
Short	MJR-300	MJ-300	MJ-300BK	MJ-300FRT
Regular	MJR-400	MJ-400	MJ-400BK	MJ-400FRT
Tall	MJR-500	MJ-500	MJ-500BK	MJ-500FRT
Bariatric	MJR-200B	MJ-200B		

\*Collar Sets come with one additional set of replacement pads.  
Miami J Front and back panels may be interchanged to accommodate patient anatomy, e.g. a Short front may be used with a Tall back.  
Choose collar size based on patient neck height, not patient body size. Super Short collars are packaged with regular back panels

### ACCESSORIES

#### REPLACEMENT PADS

Size	Part#
Super Short	MJP-100
Stout	MJP-200L
XS	MJP-250
Short	MJP-100
Regular	MJP-100
Tall	MJP-100
Bariatric	MJP-200B

## MIAMI J® SELECT

The Miami J Select cervical collar by Össur offers the widest range of height adjustability in the market with a single, universal design that replicates the comfort, immobilisation and quality you expect from a Miami J product. Featuring a patented, first-to-market locking mechanism to encourage patient compliance and anti-microbial Sorbatex padding, Miami J Select is designed to support successful patient outcomes.



### INDICATIONS FOR USE

- Situations requiring gross immobilisation of the cervical spine, these may include:
  - Pre and post c-spine surgery
  - C-spine precaution for trauma patients

### FEATURES AND BENEFITS

- Easy height adjustability with a tamper-proof locking mechanism – ‘Left to Lock’
- Anti-microbial & biocompatible Sorbatex™ padding prevents bacterial build-up
- Universal design that doesn't compromise immobilisation or comfort
- 7 precise fitting sizes with opportunity to fine-tune
- Miami J® inspired chin tray
- One-piece, anatomically inspired collar back with Flex Edge® designed to reduce occipital pressure
- Easy to don and doff with patented Reproducible Fit option
- Intuitive blue Patient Touch-Points to send patients home with confidence – Blue is You™
- Simplified comfort adjustments with integrated and patented Sternal-Relief Dial
- Large tracheal opening – improved function for ICU patients
- Easy to clean and replace multi-coloured padding
- X-ray and CT lucent, MR-safe
- Compatible with Occian™ Back & Miami JTO® accessories

### ORDERING INFORMATION

#### COLLAR SET

Description	Part#
Collar set	MJSR-101

#### MIAMI J SELECT COLLAR

Description	Part#
Miami J Select Collar	MJS-101

### OTHER ACCESSORIES

Description	Part#
Miami J Select Replacement Pads	MJSP-101
Miami J Select Tabs Reproducible Fit Tabs (10 pack)	MJST-101

## MIAMI JTO® THORACIC EXTENSION

The Miami JTO Thoracic Extension is designed in conjunction with leading spine surgeons, orthotists and nurses. The orthosis is used exclusively with all Miami J® collars when even greater stability at the cervical junctions is indicated.

Miami JTO Thoracic Extension is a versatile, tool-free device with a patented chin strut, providing more control than a collar alone can provide, designed to treat cervical and high-thoracic injuries.



### INDICATIONS FOR USE

- Low c-spine and high t-spine precaution for trauma patients
- Immobilisation for pre and post low c-spine and high t-spine surgery
- Other conditions requiring gross immobilisation of the low-cervical and/or high-thoracic-spine
- Conditions requiring gross immobilisation of the low cervical and/or high-thoracic-spine

### FEATURES AND BENEFITS

- Patented design with 'floating back' permits maximum immobilisation without the risk of compensatory motion
- Snap on and off without tools
- Sorbatex™ II padding is antibacterial and clinically proven to inhibit microbial growth, which enhances comfort and helps assure patient compliance
- MRI, CT and X-ray lucent

### REQUIRED MEASUREMENTS

- Universal, one size

### ORDERING INFORMATION

#### MIAMI JTO THORACIC EXTENSION

Description	Part#
Miami JTO	MT-900
Miami JTO with replacement softgoods	MTR-900

Compatible with all sizes of the Miami J®, Miami J® Select and the P3 Miami Jr®. Miami J collar is not included with Miami JTO Thoracic Extension. Miami J collar needs to be purchased also

### ACCESSORIES

Description	Part#
Replacement softgoods*	MTP-950

\*Contains 2 x 4" Shoulder Strap Pads / 4 x 8" Waist Strap Pads / 4 x Snap Rivet Sets

## OCCIAN® BACK

The Occian Back is designed exclusively for the specialised needs of the multi-system trauma patient in the ICU. Universally sized, specifically engineered and designed with advanced, Intuitex™ pressure-relieving memory foam, the Occian Back helps caregivers manage c-spine patients who are at high risk of occipital breakdown.



### INDICATIONS FOR USE

- Patients not awake and alert within 24 - 48 hours after admission
- Patients with pre-existing skin injuries and/or lacerations to the head/neck
- Burn patients who must wear a collar
- Elective surgery patients desiring night time post-operative comfort

### FEATURES AND BENEFITS

- Intuitex pressure relieving memory foam
- Maintains c-spine stability
- Provides unique pressure-relieving comfort by replacing the back section of a Miami J Collar.
- Easy to clean
- MRI, CT and X-ray lucent

### REQUIRED MEASUREMENTS

- Universal, one size

### ORDERING INFORMATION

Description	Part#
Occian Back	ACB-100
Occian Back Junior	ACB-200

Compatible with all sizes of the Miami J®, Miami J® Advanced, Miami J® Select and the Miami Jr®. Collar front panel not included.



## MIAMI Jr®

Miami Jr is engineered to fit the unique anatomies of patients aged 12 and under. The Miami Jr line, which spans the growth cycle, is sized according to age. Össur is the only company that has a paediatric line of collars based on the anatomical differences between adults and children. We realise that children are not just "little adults" - they require innovative design.



### INDICATIONS FOR USE

Situations requiring gross immobilisation of the cervical spine, these may include:

- pre and post c-spine surgery
- c-spine precaution for trauma patients

### FEATURES AND BENEFITS

- Designed exclusively for children 12 years and under
- Sorbatex™ II pads make the collar "skin friendly" and protect skin integrity during extended wear
- MRI, CT and X-ray lucent, MR compatible

### REQUIRED MEASUREMENTS

- Age

### ORDERING INFORMATION

#### MIAMI JR

Size	Collar set*	Collar
0 - 6 months	MJR-P0	MJ-P0
6 - 24 months	MJR-P1	MJ-P1
2 - 6 years	MJR-P2	MJ-P2
6 - 12 years	MJR-P3	MJ-P3

\*Collar sets come with one additional set of replacement pads.

### ACCESSORIES

#### REPLACEMENT PADS

Size	Part#
0 - 6 months	MJP-P0
6 - 24 months	MJP-P1
2 - 6 years	MJP-P2
6 - 12 years	MJP-P3

## PHILADELPHIA® ADJUSTABLE TRACHEOTOMY COLLAR

Philadelphia tracheotomy collars have been an industry standard for decades. Philadelphia Adjustable is constructed from the same high-quality materials as the original, and combines convenient size adjustability with immobilisation and comfort.

### INDICATIONS FOR USE

Situations requiring gross immobilisation of the cervical spine, these may include:

- Pre and post c-spine surgery
- C-spine precaution for trauma patients

### FEATURES AND BENEFITS

- Circumference adjustable and height adjustable with permanent lock
- Anatomically shaped Plastazote® foam leaving clearance for cervical veins
- Well ventilated and water-resistant for bathing and hydro-therapy
- MRI, CT and X-ray lucent, MR compatible

### REQUIRED MEASUREMENTS

- Universal, one size



### ORDERING INFORMATION

Description	Part#
Philadelphia Adjustable Tracheotomy Collar	PHP-AA

## PHILADELPHIA® TRACHEOTOMY COLLAR

The Philadelphia Tracheotomy Collar is a hospital and rehabilitation collar that is designed to maintain neutral alignment of the cervical spine after injury to the neck. The collar contains a large tracheal opening, allows carotid pulse monitoring and emergency tracheotomies.

### INDICATIONS FOR USE

- C-spine precaution for trauma patients
- Immobilisation for pre c-spine surgery
- Other conditions requiring gross immobilisation of the mid-cervical spine

### FEATURES AND BENEFITS

- Anterior and posterior rigid plastic reinforcement supports limit movement
- Total cervical arch support maintains neutral alignment of the cervical spine
- Water-resistant for bathing and hydro-therapy
- Easily customised for individual requirements
- Does not contain natural rubber latex, non-toxic and hypoallergenic Plastazote® foam reduces skin irritation

### REQUIRED MEASUREMENTS

- Neck circumference and height from tip of chin to sternal notch



## ORDERING INFORMATION

Size	Neck circumference	6cm height	8cm height	11cm height	13cm height
X-Small	< 25cm	PHP-T2XS			
Small	25 - 33cm	PHP-T2S	PHP-T3S	PHP-T4S	PHP-T5S
Medium	33 - 41cm	PHP-T2M	PHP-T3M	PHP-T4M	PHP-T5M
Large	41 - 48cm	PHP-T2L	PHP-T3L	PHP-T4L	PHP-T5L
X-Large	> 48cm	PHP-T2XL	PHP-T3XL	PHP-T4XL	PHP-T5XL

Size	Neck circumference	Part#
Infant	15 - 20cm	PHP-TI
Paediatric	20 - 28cm	PHP-TP

## ACCESSORIES

### PHILADELPHIA COOLMAX COTTON LINER

Size	Neck circumference	Part#
Child	20 - 28cm	PHP-LCP
Small	25 - 33cm	PHP-LCS
Medium	33 - 41cm	PHP-LCM
Large	41 - 48cm	PHP-LCL

### PHILADELPHIA TRACHEOTOMY ATTACHMENTS

Size	Stabiliser	Halo
Paediatric	PHP-OSP	PHP-OHP
Adult	PHP-OS	PHP-OH

## NECLOC®

### INDICATIONS FOR USE

Situations requiring gross immobilisation of the cervical spine, these may include:

- Pre and post c-spine surgery
- C-spine precaution for trauma patients

### FEATURES AND BENEFITS


- Ergonomic two-piece clamshell design and retaining strap secure flexion during application
- Five unique phenotype-driven sizes make it easy to select the correct size
- Minimal manipulation of patient during application
- X-ray and CT lucent. MR safe

### REQUIRED MEASUREMENTS

- Sizing based on 5 phenotypes to ensure correct fit and optimum immobilisation

## SIZING

- Phenotype-driven sizing system makes size selection simple while drastically reducing margin of error in fitting a patient. Each collar has bioengineered size specific front and back sections designed to work together for superior immobilisation. For paediatric sizing, please see NecLoc Kids.

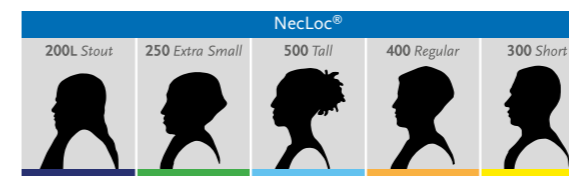

## ORDERING INFORMATION

Size	Part#
Stout	NL-200EL
XS	NL-250E
Small	NL-300E
Medium	NL-400E
Large	NL-500E

## NECLOC® KIDS

### INDICATIONS FOR USE

Situations requiring gross immobilisation of the cervical spine, these may include:

- Pre and post c-spine surgery
- C-spine precaution for trauma patients

### FEATURES AND BENEFITS

- Ergonomic two-piece clamshell design and retaining strap secure flexion during application
- Three paediatric sizes ensure optimal airway and cervical spine immobilisation
- Minimal manipulation of patient during application to children
- Meets the unique developing anatomical requirements of children
- X-ray and CT lucent. MR Safe



### REQUIRED MEASUREMENTS

- Age

### ORDERING INFORMATION

Age	Part#
0 - 2 years	NL-P1
2 - 6 years	NL-P2
6 - 12 years	NL-P3

## MIAMI LSO®

Miami LSO is a spinal orthosis designed for pain relief, protection of injured ligaments or muscles and post-surgical immobilisation. It provides low-profile, anatomical immobilisation, easy fit and wear, options for modular support and an advanced pulley system with a length-adjustable pulley cord.



### INDICATIONS FOR USE

Conditions requiring gross immobilisation of the trunk in the lumbar region such as:

- Stable non-displaced spinal fractures (L1-L5)
- Spinal Stenosis
- Herniated Disks
- Degenerative Spinal Pathologies
- Spondylolysis
- Spondylolisthesis

### FEATURES AND BENEFITS

- Slim, breathable material
- Proven Miami double-pulley system with length-adjustable pulley cord
- Intuitive fitting without trimming with fold-over belt arms & Precise-Size™ landmarks
- Optional fit tool for perfect fitting directly on the patient
- Universal belt sizing accommodates waists up to 178cm/70"
- New Universal Posterior panel to accommodate more patients with less inventory
- Contoured posterior panels sized S-XL for a precise, anatomical fit
- Quick-Fit pocket enclosure and blue highlights for patient ease of use

### REQUIRED MEASUREMENTS

- Waist circumference

## MIAMI LSO®

### ORDERING INFORMATION

#### MIAMI LSO BELT

Size	Waist Circumference	Part#
6" Universal (Slim)	26-50" (66-127cm)	MLSO-1E
8" Universal (Standard)	26-50" (66-127cm)	MLSO-2E



#### OPTIONAL BELT EXTENDERS (2 PCS)

Size	Waist Circumference	Part#
6" (Slim) Belt Extension	50-70" (127-177cm)	MLSO-1EXT
8" (Standard) Belt Extension	50-70" (127-177cm)	MLSO-2EXT

#### STANDARD POSTERIOR PANEL

Size	Part#
Standard Posterior Panel	MLSO-P56



#### POSTERIOR PANEL WITH LATERAL SUPPORTS

Size	Waist Circumference	Part#
Small (S)	26-34" (66-86cm)	MLSO-P12
Medium (M)	34-38" (86-96cm)	MLSO-P22
Large (L)	38-50" (96-127cm)	MLSO-P32
Extra Large (XL)	50"-70" (127-178 cm)	MLSO-P42



\*Non-lateral and non-lordotic panels available on request.

#### ANTERIOR PANEL

Size	Part#
6" Panel (Slim)	MLSO-PA6
8" Panel (Standard)	MLSO-PA8



#### QUICK FIT TOOL

Size	Part#
Quick fit tool	B-040100000



## MIAMI TLSO 464

The Miami TLSO from Össur directly addresses the needs of patients and practitioners, simplifying preparation without compromising on fit or performance. This easy-to-use pre-assembled Miami TLSO brace offers three configurations in one.

**Compliance:** Easily applied in over the shoulder or underarm configuration, donning and doffing is further simplified with the conveniently located quick release buckles.

**Adjustability:** The universal belt designs allows for setting the belt length on the patient for a more accurate sizing experience.

**Comfort:** Truly shaped to fit the anatomy, the anatomically contoured design better immobilises the spine while maximizing patient comfort.

**Ease of Use:** The Miami TLSO brace requires minimal preparation and is designed to provide optimal patient comfort.



### INDICATIONS FOR USE

Conditions requiring gross immobilisation of the thoracic and lumbar spine. Such conditions may include:

- Post-surgical immobilisation
- Spinal support TLSO attachment is up to T7
- Spinal stenosis
- Herniated disc
- Degenerative spinal fractures
- Spondylolithesis
- Spondylolysis

### CONTRAINDICATIONS

- Unstable, displaced fractures

### FEATURES AND BENEFITS

- Significantly superior comfort and ease of donning/doffing
- Fully assembled for fitting with ease
- Accommodates heights 155-193 cm (5'1" – 6'5")
- Posterior thoracic extension option with ergonomic axillary strapping
- Step-down to an LSO as one progresses through rehabilitation
- One-touch telescoping anterior-thoracic extension option with single or double-hinge
- The convenience of a one-size Universal or individualise with a uniquely-sized anatomical design
- Improve comfort and immobilisation with multiple strapping configurations and removable anchors

### REQUIRED MEASUREMENTS

- Waist circumference

## MIAMI TLSO 464

### ORDERING INFORMATION

#### ATE/PTE, LORDOTIC

Size	Part#	Waist circumference cm	Waist circumference inch
Universal	MTLSO-2351	66 - 177	26 - 70
S	MTLSO-2111	66 - 86	26 - 34
M	MTLSO-2121	86 - 107	34 - 42
L	MTLSO-2131	107 - 127	42 - 50
XL	MTLSO-2141	127 - 177	50 - 70

\*Non-lateral and non-lordotic available on request

\*Belt extensions are required for waist circumferences larger than 127 cm / 50"

#### ATE/PTE, LORDOTIC, SLIM

Size	Part#	Waist circumference cm	Waist circumference inch
Universal	MTLSO-1351	66 - 177	26 - 70
S	MTLSO-1111	66 - 86	26 - 34
M	MTLSO-1121	86 - 107	34 - 42
L	MTLSO-1131	107 - 127	42 - 50
XL	MTLSO-1141	127 - 177	50 - 70

\*Belt extensions are required for waist circumferences larger than 127 cm / 50"

### OTHER ACCESSORIES

Description	Part#
5MM Hex Long Wrench	MTLSO-0004
Axillary strap accessory kit	MTLSO-0003
Quick fit tool	B-040100000
Belt extension, standard (Recommended for waist circumferences 127-167 cm (50"-66"). Fits size XL of Miami LSO.)	MLSO-2EXT
Belt extension, slim (Recommended for waist circumferences 127-167 cm (50"-66"). Fits size XL of Miami LSO.)	MLSO-1EXT

## MIAMI TLSO 456

The Miami TLSO 456 from Össur directly addresses the needs of patients and practitioners, simplifying preparation without compromising on fit or performance. This easy-to-use pre-assembled Miami TLSO brace offers the convenience of a one-size Universal or a uniquely-sized anatomical design.

**Compliance:** Blue is You™ patient touch-points help send patients home with confidence.

**Adjustability:** The universal belt design allows for setting the belt length on the patient for a more accurate sizing experience. Heat moldable posterior panels allow a customised fit, while the modular design allows step-down support to an LSO.

**Comfort:** Featuring the Miami LSO™ Advanced Compression pulley system with lumbar compression you can feel, and an anatomically contoured design that better immobilises the spine while maximising patient comfort.

**Ease of Use:** Easily applied with over-the-shoulder straps, the Miami TLSO brace requires minimal preparation and is designed to provide optimal patient comfort.



### INDICATIONS FOR USE

Conditions requiring gross immobilisation of the thoracic and lumbar spine. Such conditions may include:

- Post-surgical immobilisation
- Spinal support TLSO attachment is up to T7
- Spinal stenosis
- Herniated disc
- Degenerative spinal fractures
- Spondylolithesis
- Spondylolysis

### CONTRAINDICATIONS

- Unstable, displaced fractures

### FEATURES AND BENEFITS

- Significantly superior comfort and ease of donning/doffing (when compared to Miami Lumbar TLSO)
- Fully assembled for fitting with ease
- Accommodates heights 155-193 cm (5'1" – 6'5")
- Step-down to an LSO as one progresses through rehabilitation
- The convenience of a one-size Universal or individualise with a uniquely-sized anatomical design

### REQUIRED MEASUREMENTS

- Waist circumference

## MIAMI TLSO 456

### ORDERING INFORMATION

#### PTE LORDOTIC

Size	Part#	Waist circumference cm	Waist circumference inch
Universal	MTLSO-2352	66 - 177	26 - 70
S	MTLSO-2112	66 - 86	26 - 34
M	MTLSO-2122	86 - 107	34 - 42
L	MTLSO-2132	107 - 127	42 - 50
XL	MTLSO-2142	127 - 177	50 - 70

\*Belt extensions are required for waist circumferences larger than 127 cm / 50"

#### PTE LORDOTIC, SLIM

Size	Part#	Waist circumference cm	Waist circumference inch
Universal	MTLSO-1352	66 - 177	26 - 70
S	MTLSO-1112	66 - 86	26 - 34
M	MTLSO-1122	86 - 107	34 - 42
L	MTLSO-1132	107 - 127	42 - 50
XL	MTLSO-1142	127 - 177	50 - 70

\*Non-lateral and non-lordotic available on request.

\*Belt extensions are required for waist circumferences larger than 127 cm / 50"

### OTHER ACCESSORIES

Description	Part#
Quick fit tool	B-040100000
Belt extension, standard (Recommended for waist circumferences 127-167 cm (50"-70")). Fits size XL of Miami LSO)	MLSO-2EXT
Belt extension, slim (Recommended for waist circumferences 127-167 cm (50"-70")). Fits size XL of Miami LSO)	MLSO-1EXT

## MIAMI LUMBAR® POSTEO

Designed with patient comfort and compliance in mind, the Miami Lumbar Posteo provides functional relief of vertebral fracture pain, and facilitates return to mobility. The lumbar compression system offers both pain relief via active vertebral offloading and better grip of patient's hips for greater ease of use and compliance. The Miami Lumbar Posteo includes an innovative approach to ease of use and consistent donning.



### INDICATIONS FOR USE

- Back pain caused by stable vertebral compression fractures (VCF)
- Post-operative immobilisation

### FEATURES AND BENEFITS

- Active offloading via thoracic extension and lumbar compression
- Integrated pulley system for customised fit
- Trimmable axillary straps with adjustable tension for optimal stabilisation
- Cold-moldable aluminum PTE to fit different anatomies
- Extra cushioning around the aluminum strut for minimised skin pressure

### REQUIRED MEASUREMENTS

- Back length measured from the top of the sacrum up to C7 vertebrae, following the spinal curvature

### SIZING

Please note that the Miami Lumbar Posteo includes a universal lumbar belt which can be trimmed down to the appropriate size.

### ORDERING INFORMATION

#### MIAMI LUMBAR POSTEO

Size	Back Length	Part#
X-Small	37 - 44cm	LMB-POSTEO-XS
Small	44 - 51cm	LMB-POSTEO-SM
Medium	49 - 56cm	LMB-POSTEO-MD
Large	55 - 61cm	LMB-POSTEO-LG

### ACCESSORIES

Description	Part#
Strap Pad	LMB-POSTEO ST PD

## PAPOOSE®

Papoose is an innovative design for newborns up to 3 months (Broselow Gray). This true cervical spine immobiliser cradles the baby's body, securely while gently immobilising the head and spine in the proper anatomical airway and spinal alignment to help achieve optimal patient outcome while allowing for continued intimacy between caregiver and child.



### INDICATIONS FOR USE

- C-Spine precaution for infants
- Immobilisation for pre and post c-spine surgery
- Other conditions requiring gross immobilisation of the infant cervical spine

### FEATURES AND BENEFITS

- X-ray & CT Lucent. MRI Safe

### REQUIRED MEASUREMENTS

- Universal, one size

### ORDERING INFORMATION

#### PAPOOSE

Description	Part#
Papoose without front collar	PO-100
Papoose without front with replacement pads	POR-100
Papoose with front collar	PA-100
Papoose with front with replacement pads	PAR-100

### ACCESSORIES

#### REPLACEMENT PADS

Description	Part#
Papoose replacement pads (with front)	PAP-100
Papoose replacement pads (without front)	POP-100

## ÖSSUR FORMFIT® PRO BACK

Össur Formfit® Pro Back provides relief for patients suffering from pains and strains in the lower back. Manufactured with our dynamic, 3D-Motion Tech™ knitting technology, our belts feature superior moisture wicking capabilities and are highly breathable. The patented back panel design utilises our StayBilizer™ technology to provide optimum shape and fit, while conveniently placed finger loops make donning easy. The SmartPad™ provides a targeted support and cushioning and can be removed if desired.



MotionTech™



CoolVent™



StayBilizer™



SmartPad™



### INDICATIONS FOR USE

- For lower back conditions that may benefit from compression and gross motion restriction, such as:
- Lower back pain (Lumbago)
- Lumbosacral sprain/strain
- Degenerative joint/disc disease

### CONTRAINDICATIONS

- Must not be used by individuals for whom compression is contraindicated.

### FEATURES AND BENEFITS

- MotionTech™ for compression and dynamic fit
- Removable lumbar SmartPad™ for targeted support and cushioning
- Lightweight, malleable aluminum stays are shaped to the anatomy of the lower back

### REQUIRED MEASUREMENTS

- Measure waist circumference

### ORDERING INFORMATION

Size	Part#	Waist circumference	
		cm	inch
XS	FP500001	70 - 80	27½ - 31½
S	FP500002	80 - 90	31½ - 35¾
M	FP500003	90 - 100	35¾ - 39¾
L	FP500004	100 - 115	39¾ - 45¼
XL	FP500005	115 - 130	45¼ - 51½



## FORMFIT® BACK SUPPORT AIR

The Form Fit Back Support with Air provides compression and support for those suffering from lower back pain. The integrated pneumatic system is intuitive and conveniently located so that adjustments can be made as demands on the body change throughout the day. The double wrap design improves compression, while durable finger loops make applying the brace easy. Made from breathable materials, the Form Fit Back Support provides compression and support for those suffering from lower back pain. The double wrap design improves compression, while the durable finger loops make applying the brace easy.

### INDICATIONS FOR USE

Lower back conditions that may benefit from gross motion restriction, such as;

- Muscular low back pain
- Lumbosacral strains
- Chronic low back pain
- Sciatica

### FEATURES AND BENEFITS

- Self-contained pneumatic system is low-profile and easy to use
- Mesh panels and ventilation holes increase breathability and enhance comfort
- Overlapping double pull strap design provides additional support to the lumbar and abdominal areas
- Flexible stays provide optimum support in the lumbar and abdominal areas
- Gripper strips prevent migration
- Adjustable air bladder provides a custom fit and offers focused compression in the lumbar area

### REQUIRED MEASUREMENTS

- Waist circumference

### ORDERING INFORMATION

Size	Waist circumference	Standard
Small	63 - 76cm	209133
Medium	76 - 86cm	209135
Large	86 - 97cm	209137
X-Large	97 - 107cm	209138
XX-Large	107 - 142cm	209139

### ACCESSORIES

Description	Part#
Gel pad	GB28



## FORMFIT® BACK SUPPORT

Made from breathable materials, the Formfit Back Support provides compression and support for those suffering from lower back pain. The double wrap design improves compression, while the durable finger loops make applying the brace easy.

### INDICATIONS FOR USE

- Muscular low back pain
- Lumbosacral strains
- Chronic low back pain
- Sciatica

### FEATURES AND BENEFITS

- Anti-migration system reduces slippage
- Double wrap design improves compression
- Finger loops for easy application
- Hot/Cold Gel pad available separately

### REQUIRED MEASUREMENTS

- Waist circumference

### ORDERING INFORMATION

Size	Waist Circumference	Standard
Small	25-30"	209143
Medium	30-34"	209145
Large	34-38"	209147
X-Large	38-42"	209148
XX-Large	42-56"	209149
XXX-Large	56-60"	209151

### ACCESSORIES

Description	Part#
Gel pad	GB28





HIP



TABLE OF CONTENTS

HIP .....  
33

REBOUND® HIP .....  
35

UNLOADER® HIP .....  
37

REBOUND® HIP

Rebound Hip delivers gentle controlling forces and effective functional ROM restriction in extension / flexion and abduction / adduction to promote optimal post-surgical outcomes. The low profile, lightweight design supports patient comfort and compliance. The universal frame and softgood are easy to stock, assemble and quickly fit.



INDICATIONS FOR USE

Hip conditions that could benefit from motion restriction such as:

- Postoperative after (total) hip arthroplasty
- Postoperative after revision of (total) hip arthroplasty
- Postoperative hip arthroscopy
- Post-op following FAI arthroscopic surgery
- Post-op following gluteus medius repair surgery
- Post-op following hamstring repair surgery
- Inoperable hip abnormalities
- Preventively when there is risk of hip dislocation
- Conventional treatment after dislocation/reduction

FEATURES AND BENEFITS

- Low-profile, lightweight design that supports patient comfort and compliance
- Quick-release pegs easily adjust the desired range-of-motion needed for the patient's optimal rehab
- Quick-fit with sliding mechanism on frame to adjust for patient's height
- Optional, adjustable ROM hinge enables adduction/abduction from 0° to 10°, 20° or 30°
- OR kit offers patient protection of ankle and perineum skin during intraoperative traction on a surgical table

REQUIRED MEASUREMENTS

- Circumference at iliac crest height
- Length from greater trochanter to mid patella

ORDERING INFORMATION

REBOUND HIP

Description	Part#
Rebound Hip	98013XRB
Rebound Hip with adjustable hinge	98012XRB

X = 0 (Preset to left) or 5 (Preset to right)  
\*Frame: Universal; Softgood: Universal fitting to 48" (122cm) hip circumference

## REBOUND® HIP

### ACCESSORIES

Description	Part#
OR Kit	980301
Belt Extender (Fitting 49"-67" / 122 - 170cm hip circumference)	980121RB
Rotation Strap	RHK-100024
Fitting Tool	B-04010000

## UNLOADER® HIP

Unloader Hip is designed to improve activities of daily living by optimizing load dispersion for patients suffering from mild and moderate osteoarthritis of the hip, thus contributing to hip stability and improved mobility.

Unloader Hip delivers proprioceptive support during the gait cycle by providing compression with a proven pulley system design. If required it can decrease the internal rotation and adduction moment with its unique and clinically proven Dynamic Rotation Straps™.



### INDICATIONS FOR USE

- Mild and moderate osteoarthritis of the hip
- Conditions benefitting from repositioning of the hip and/or compression around the greater trochanters

### FEATURES AND BENEFITS

- The Dynamic Rotation Straps with SmartDosing dial allows individual adjustment of the external rotation force, to achieve optimum pain relief for each patient
- Easy donning and doffing thanks to pocket wrap closure system and zipper
- Pulley system for improved stability
- Premium quality Lycra® for breathability and a snug fit


### REQUIRED MEASUREMENTS

- Measure the hip circumference at the anterior superior iliac-spine.

### ORDERING INFORMATION

#### UNLOADER HIP

Size	Hip Circumference	Woman	Men
Small	80 – 88cm	B-465350030	B-465350031
Medium	88 – 96cm	B-465350032	B-465350033
Large	96 – 104cm	B-465350034	B-465350035
X-Large	104 – 114cm	B-465350036	B-465350037
XX-Large	114 – 124cm	B-465350038	B-465350039

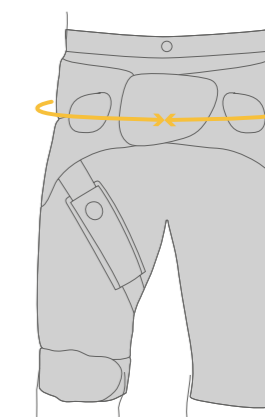

## UNLOADER® HIP

### ACCESSORIES/ OPTIONS

#### REPLACEMENT TROUSERS

Size	Woman	Men
Small	B-465350077	B-465350078
Medium	B-465350079	B-465350080
Large	B-465350081	B-465350082
X-Large	B-465350083	B-465350084
XX-Large	B-465350085	B-465350086

#### OTHER ACCESSORIES

Description	Part#
Rotation strap	B-465350064
Washbag	B-465350065
Trochanter pad	B-465350066
Thigh strap S	B-465350067
Thigh strap M	B-465350068
Thigh strap L	B-465350069
Thigh strap XL	B-465350070
Thigh strap XXL	B-465350071
Bilateral kit* S	B-465350072
Bilateral kit* M	B-465350073
Bilateral kit* L	B-465350074
Bilateral kit* XL	B-465350075
Bilateral kit* XXL	B-465350076

\*The bilateral kit consists of a rotation control strap, a thigh strap and a trochanter pad

## UPPER EXTREMITY

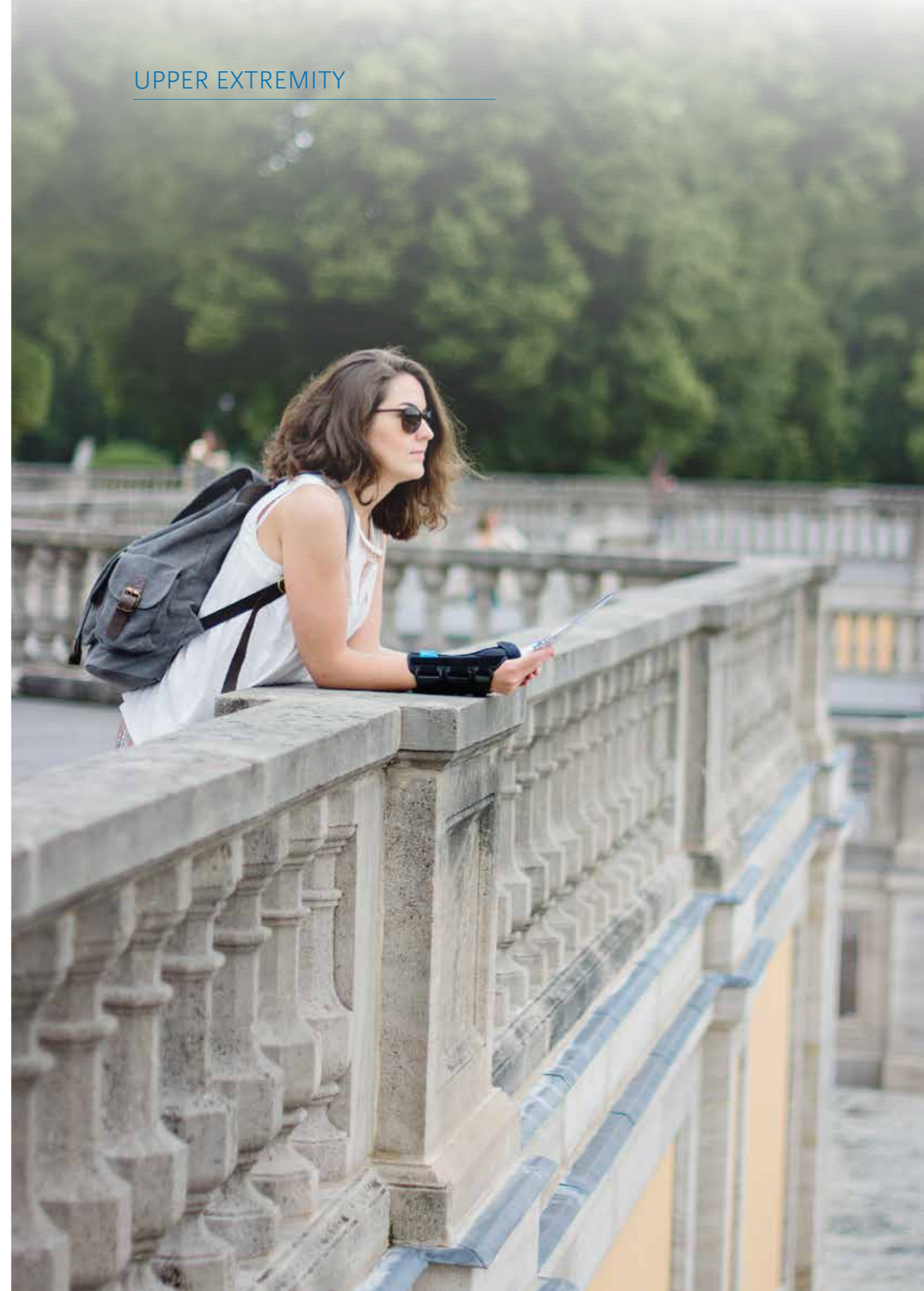


TABLE OF CONTENTS

UPPER EXTREMITY ..... 39

FORMFIT® SHOULDER BRACE ..... 41

PREMIUM SHOULDER IMMOBILISER ..... 42

BERGSCHULTZ UNIVERSAL ARM SLING ..... 42

PAEDIATRIC UNIVERSAL ARM SLING ..... 43

FORMFIT® PRO ELBOW ..... 44

INNOVATOR X® POST-OP ELBOW ..... 45

FORMFIT® TENNIS ELBOW ..... 46

FORMFIT® PRO WRIST ..... 47

FORMFIT® WRIST / FORMFIT® WRIST & FOREARM ..... 48

FORMFIT® UNIVERSAL WRIST & THUMB ..... 49

FORMFIT® THUMB SPICA ..... 50

FORMFIT® SHOULDER BRACE

The Formfit Shoulder Brace is designed to provide immobilisation following shoulder injury/surgery. With its streamlined fit, sporty look and breathability it offers excellent comfort and ease of use. Same as its predecessor, the SmartSling®, the Formfit Shoulder Brace also provides abduction of the shoulder (when the abduction pillow is attached), or external rotation of the shoulder (when the external rotation pillow is attached).

Optional axilla pillow helps distribute weight over a larger surface area.

INDICATIONS FOR USE

- Immobilisation for rotator cuff repairs
- Bankart repairs
- Capsular shifts
- Glenohumeral dislocations/subluxations
- Soft tissue repairs/strains
- Shoulder instabilities/injuries
- Post-surgical rehabilitation

FEATURES AND BENEFITS

- Breathable material
- Quick-release buckles
- Offloading strap
- Extra padding
- Universal thumb loop

REQUIRED MEASUREMENTS

- Olecranon to end of closed fist

ORDERING INFORMATION

FORMFIT SHOULDER BRACE

Size	Length	Part
Small	up to 29cm	SSS-SO
Medium	29 - 40cm	SSR-SO
Large	40 - 44.5cm	SSE-SO

FORMFIT SHOULDER BRACE WITH ABDUCTION

Size	Length	Part
Small	up to 29cm	SSS
Medium	29 - 40cm	SSR
Large	40 - 44.5cm	SSL

FORMFIT SHOULDER BRACE WITH EXTERNAL ROTATION

Size	Length	Part
Small	up to 29cm	SSS-ER
Medium	29 - 40cm	SSR-ER
Large	40 - 44.5cm	SSE-ER

Description	Part
Axilla Pillow	SSAXP



Formfit Shoulder Brace

Axilla Pillow

## PREMIUM SHOULDER IMMOBILISER

### INDICATIONS FOR USE

For shoulder conditions that may benefit from immobilizing the shoulder, such as:


- Pre or post-operative
- Soft tissue repairs/strains
- Shoulder instabilities/injuries

### FEATURES AND BENEFITS

- Effectively controls abduction and rotation
- Humerus cuff, wrist cuff and shoulder strap may be placed in desired position for optimum fit/comfort
- One size for reduced stockholding
- Fits sizes S-XL

### REQUIRED MEASUREMENTS

- Universal, one size (for upper waist diameter up to 106cm)

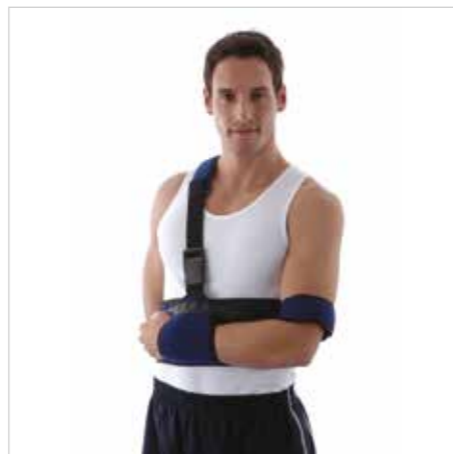

### ORDERING INFORMATION

Description	Part#
Premium Shoulder Immobiliser	0380
Premium Shoulder Immobiliser XL	0380XL

## BERGSCHULTZ UNIVERSAL ARM SLING

### INDICATIONS FOR USE

- Post-trauma
- Post-operative

### FEATURES AND BENEFITS

- Designed by Jim Bergschultz. R.O.T.
- Blue heavy denim material with contact closure

### REQUIRED MEASUREMENTS

- Universal, one size



### ORDERING INFORMATION

Description	Part#
Bergschulz Universal Arm Sling	204070

## PAEDIATRIC UNIVERSAL ARM SLING

### INDICATIONS FOR USE

- Post upper limb trauma
- Post upper limb surgery

### FEATURES AND BENEFITS

- Prong buckle closure correct fit without slippage

### REQUIRED MEASUREMENTS

- Universal, one size



### ORDERING INFORMATION

Description	Part#
Paediatric Universal Arm Sling Packed in boxes of 6	204020

## FORMFIT® PRO ELBOW

Formfit® Pro Elbow provides relief from elbow pain. This 3D compressive elbow sleeve is manufactured using our MotionTech™ technology and features two integrated epicondyle SmartPads™ for more targeted support and cushioning. The RadialStrap™ provides additional compression and support.



MotionTech™



CoolVent™



SmartPad™



FoldLock™


### INDICATIONS FOR USE

- For elbow conditions that may benefit from compression, such as:
- Medial and lateral epicondylitis
- Elbow pain and swelling

### CONTRAINDICATIONS

- Must not be used by individuals for whom compression is contraindicated.

### REQUIRED MEASUREMENTS

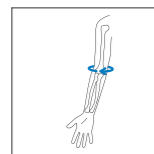

Measure arm circumference exactly at the elbow. Elbow in full extension.

### ORDERING INFORMATION

Size	Elbow circumference		Black	Blue
	cm	inch		
XS	17 - 20	6¾ - 7⅞	FP300001	FP301001
S	20 - 23	7⅞ - 9	FP300002	FP301002
M	23 - 26	9 - 10¼	FP300003	FP301003
L	26 - 29	10¼ - 11¾	FP300004	FP301004
XL	29 - 32	11¾ - 12⅝	FP300005	FP301005



## INNOVATOR X® POST-OP ELBOW

Benefiting from our proven, patented technologies, Innovator X, with extending struts, can help improve patient outcomes. With superior comfort, clean aesthetics and the most user-friendly hinge in post-op bracing, Innovator X is designed to increase patient satisfaction and compliance.



### INDICATIONS FOR USE

- Indications requiring elbow immobilisation and/or range of motion restriction. This may include:
- Stable fractures of the elbow, distal humerus, proximal radius or ulna
- Tendon and ligament injuries or repairs
- Collateral ligament reconstructions
- Elbow hyperextension
- Triceps tendon repair
- Chronic elbow injuries
- Tennis elbow release

### FEATURES AND BENEFITS

- OneTouch™ easy to adjust ROM hinge
- Slide-to-Size™ struts lead to a more customised fit
- Sensil® silicone padding on the strapping system reduces slipping and malleable aluminium cuffs allow for customisation
- Patented Form Fit® 3-dimensional padding and lycra lining provide superior comfort and compliance
- Comfort pad at the neck and shoulder
- Optional anti-supination/pronation arm bar kit is available

### REQUIRED MEASUREMENTS

- Universal, one size

### ORDERING INFORMATION

#### INNOVATOR X

Side	Part#
Left	W-823101
Right	W-823100

### ACCESSORIES

Description	Part#
Innovator X Bar Kit	W-A82310
Replacement softgoods Left	W-823111
Replacement softgoods Right	W-823110

## FORMFIT® TENNIS ELBOW

Formfit Tennis Elbow is made with a patented, ergonomically-shaped plastic. There are two blue cushions with ribbed surface on each side that reduce traction on the tendons. The durable design and materials enable everyday use for both work and athletic activities.

### INDICATIONS FOR USE

- Tendonitis of the elbow
- Prevention during work, daily life and sport
- Epicondylitis

### FEATURES AND BENEFITS

- Adjustable strap
- Patented ergonomically-shaped plastic eliminates tourniquet effect associated with traditional designs.
- Twin blue cushions provide counterforce, reducing traction on the tendons and thus inflammation
- Ribbed surfaces reduce vibration to the tendon, increasing comfort
- Durable design and materials enable everyday use for both work and athletic activities

### REQUIRED MEASUREMENTS

- Circumference of the elbow



### ORDERING INFORMATION

Size	Circumference	Part#
Small	22-26cm	B-247000002
Medium	27-32cm	B-247000003
Large	33-38cm	B-247000004

## FORMFIT® PRO WRIST

Formfit Pro Wrist is constructed using our superior signature 3D MotionTech knit, and ideal for injuries that may benefit from compression. Malleable aluminum stays can be adjusted to conform to an individual's anatomy. CoolVent™ technology in the basal thumb area enhances breathability and comfort.



MotionTech™



CoolVent™



StayBilizer™



FoldLock™

### INDICATIONS FOR USE

- For wrist conditions that may benefit from compression, such as:
- Rheumatoid Arthritis (RA) and other chronic pain conditions
- Wrist sprain and strains
- Overuse and pain conditions
- Wrist tendonitis


### CONTRAINDICATIONS

- Must not be used by individuals for whom compression is contraindicated.

### FEATURES AND BENEFITS

- MotionTech™ for compression and dynamic fit has been proven to be superior to the leading brands at moisture wicking and breathability, keeping patients cool during wear
- Malleable aluminum stay for a customised fit
- CoolVent™ technology in thumb base area for extra breathability and comfort

### REQUIRED MEASUREMENTS

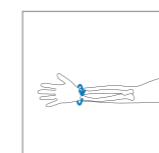

Measure wrist circumference at the level of the radial styloid with the hand outstretched. Wrist in 0 degrees of flexion.

### ORDERING INFORMATION

Size	Wrist circumference		Black		Blue	
	cm	inch	Left	Right	Left	Right
XS	14 - 15	5½ - 5⅞	FP400201	FP400101	FP401201	FP401101
S	15 - 16	5⅞ - 6¼	FP400202	FP400102	FP401202	FP401102
M	16 - 17	6¼ - 6¾	FP400203	FP400103	FP401203	FP401103
L	17 - 18	6¾ - 7⅞	FP400204	FP400104	FP401204	FP401104
XL	18 - 19	7⅞ - 7½	FP400205	FP400105	FP401205	FP401105
XXL	19 - 20	7½ - 7⅞	FP400206	FP400106	FP401206	FP401106





## FORMFIT® WRIST / FORMFIT® WRIST & FOREARM

Formfit Wrist, and Formfit Wrist & Forearm, provides immobilisation and protection for patients suffering from tendonitis, select soft tissue injuries – including mild to moderate sprains and strains, as well as for post-cast healing. The lightweight, breathable material and contour fit make this comfortable product an excellent choice.



### INDICATIONS FOR USE

- Mild to moderate sprains and strains
- Tendonitis
- Post-cast healing
- Other selected soft tissue injuries

### FEATURES AND BENEFITS

- Breathable material
- Contoured fit
- Contact closure straps
- Customisable stays

### REQUIRED MEASUREMENTS

- Wrist circumference

## ORDERING INFORMATION

### FORMFIT WRIST & FOREARM

Size	Wrist Circumference	8"		10"	
		Left	Right	Left	Right
XSmall	13 - 16cm	317082	317072	617082	617072
Small	16 - 18.5cm	317083	317073	617083	617073
Medium	18.5 - 20.5cm	317085	317075	617085	617075
Large	20.5 - 23.5cm	317087	317077	617087	617077
XLarge	23.5 - 25.5cm	317088	317078	617088	617078

## ACCESSORIES

### GEL PAD FOR HOT/COLD THERAPY

Description	Left	Right
Form Fit Wrist (8" version)*	GB20L	GB20R
Form Fit Wrist & Forearm (10" version)	GB22L	GB22R



\* Fits XSmall/Small. For Medium-XLarge, please use Form Forearm 10" part numbers



## FORMFIT® UNIVERSAL WRIST & THUMB

Formfit Universal Wrist and Thumb products are designed to restrict motion of the wrist and the thumb. Providing immobilisation and protection from select soft tissue injuries, mild to moderate sprains and strains, including gamekeeper's thumb and post cast healing. The lightweight, universal design also reduces inventory needs.



### INDICATIONS FOR USE

- Mild to moderate sprains and strains
- Tendonitis
- Post-cast healing
- Other selected soft tissue injuries

### FEATURES AND BENEFITS

- Breathable material
- Quick-Pull close strap
- Positional dorsal stay
- Customisable thumb stay
- Malleable palmar stay

### REQUIRED MEASUREMENTS

- Wrist circumference

## ORDERING INFORMATION

### FORMFIT WRIST

Size	Wrist Circumference	Left	Right
Paediatric	3.5 - 5"	B-252603301	B-252603300
8" Universal	4.5 - 9.5"	B-252603109	B-252503109
8" X-Large	7.5 - 11"	B-252603106	B-252603105

### FORMFIT WRIST & FOREARM UNIVERSAL

Size	Wrist Circumference	Left	Right
10" Universal	4.5 - 9.5"	B-252603210	B-252603209

### FORMFIT THUMB SPICA UNIVERSAL

Size	Wrist Circumference	Left	Right
Paediatric	3.5 - 5"	B-253603301	B-253603300
8" Universal	4.5 - 9.5"	B-253603109	B-253503109
8" X-Large	7.5 - 11"	B-253603106	B-253603105

## FORMFIT® THUMB SPICA

The Formfit Thumb Spica is designed to restrict wrist and thumb motion. It provides immobilisation and protection for patients suffering from tendonitis (e.g. DeQuervain's Tendonitis), select soft tissue injuries – including mild to moderate sprains and strains and Gamekeeper's Thumb –, as well as for post-cast healing. The lightweight, breathable material and contour fit make this comfortable product an excellent choice.



### INDICATIONS FOR USE

- Mild to moderate sprains and strains
- Tendonitis  
- e.g., DeQuervain's Tendonitis
- Post-cast healing
- Other selected soft tissue injuries  
- e.g., Gamekeeper's Thumb

### FEATURES AND BENEFITS

- Breathable material
- Contoured fit
- Contact closure straps
- Customisable stays

### REQUIRED MEASUREMENTS

- Wrist circumference

### ORDERING INFORMATION

Size	Wrist Circumference	Left	Right
XSmall	13 - 16cm	3020	3120
Small	16 - 18.5cm	3030	3130
Medium	18.5 - 20.5cm	3050	3150
Large	20.5 - 23.5cm	3070	3170
XLarge	23.5 - 25.5cm	3080	3180

### ACCESSORIES

#### GEL PAD FOR HOT/COLD THERAPY

Description	Left	Right
Form Fit Wrist (8" version)*	GB20L	GB20R
Form Fit Wrist & Forearm (10" version)	GB22L	GB22R



\* Fits XSmall/Small. For Medium-Xlarge, please use Form Forearm 10" part numbers

## KNEE

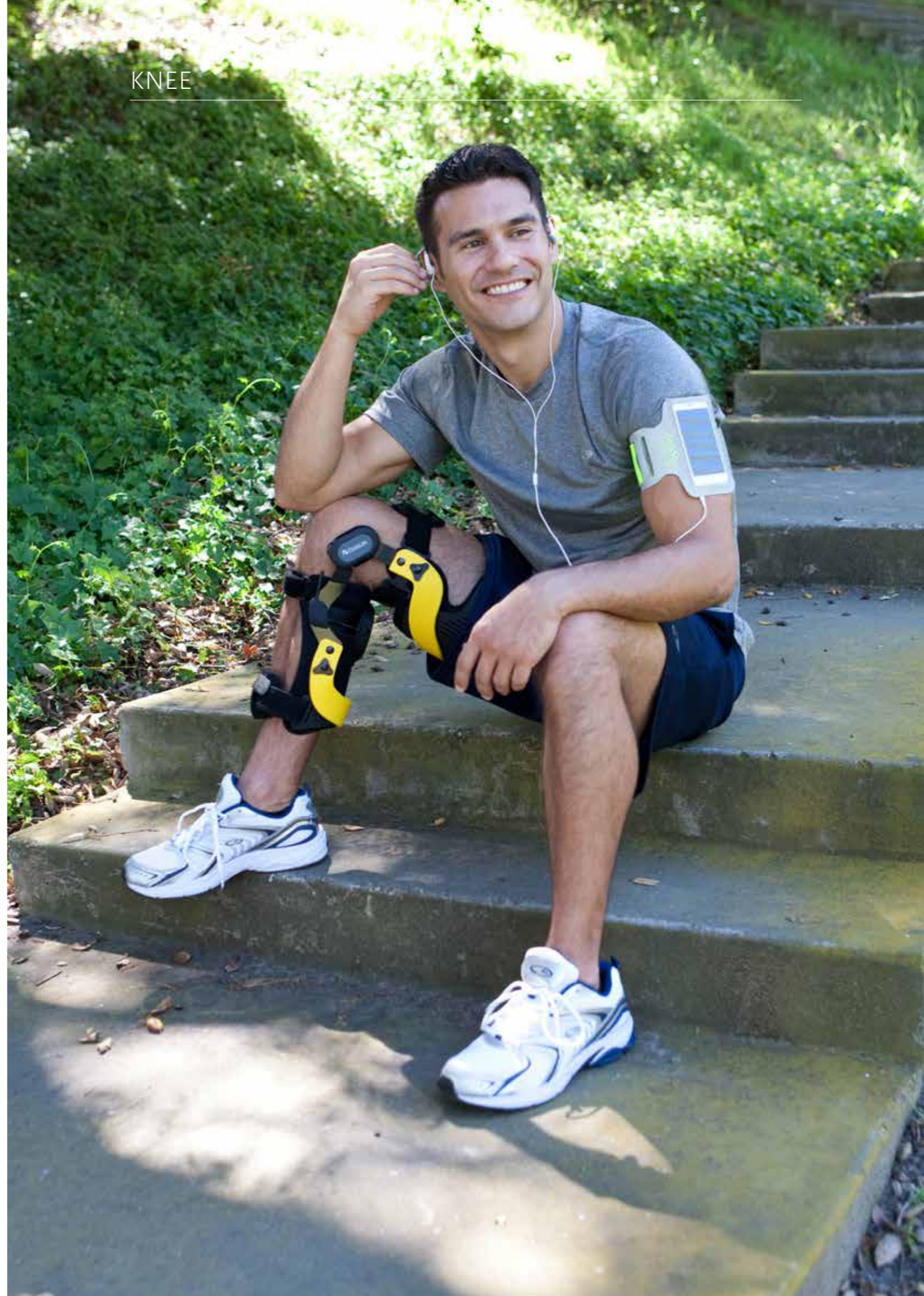


TABLE OF CONTENTS

**KNEE** ..... 51

UNLOADER ONE® X ..... 52

UNLOADER ONE® ..... 56

UNLOADER ONE® CUSTOM ..... 59

REBOUND® CARTILAGE ..... 62

CTI® OA ..... 64

REBOUND® DUAL ..... 66

REBOUND® PCL ..... 68

REBOUND® ACL ..... 70

CTI® OTS ..... 72

CTI® CUSTOM ..... 75

REBOUND® POST-OP ..... 78

FORMFIT POST-OP KNEE ..... 79

FORMFIT® KNEE IMMOBILISER ..... 80

ÖSSUR FORMFIT® PRO KNEE ..... 81

FORMFIT® HINGED LATERAL J ..... 83

FORMFIT® KNEE HINGED ..... 84

FORMFIT® KNEE MCL ..... 85

FORMFIT® KNEE ROM BRACE ..... 86

NEOPRENE HINGED KNEE SUPPORT ..... 87

NEOPRENE WRAPAROUND HINGED KNEE SUPPORT ..... 88

NEOPRENE KNEE SLEEVE ..... 89

UNLOADER ONE® X

Unloader One X improves mobility by providing pain relief from knee osteoarthritis and degenerative meniscal tears. It features Össur's unique patented and clinically proven 3-point Leverage System that generates unicompartmental unloading of the knee joint.



INDICATIONS FOR USE

- Moderate to severe unicompartmental knee osteoarthritis
- Degenerative meniscal tears
- Other knee conditions that may benefit from unicompartmental unloading

FEATURES AND BENEFITS

- 3-Point Leverage System consisting of two Dynamic Force System (DFS) Straps, rigid shells and a single upright hinge contribute to a unicompartmental unloading of the knee joint.
- The unloading is effective only when needed, no condyle pressure during sitting or swing phase.
- A slender perforated frame with soft edges, contours easily to individual anatomies and remains discreet whether worn over or under clothing.
- Skin friendly liners and pads as well as a smooth inner surface of shells reduce probability of pressure points.
- The gastroc strap is positioned over thickest part of calf to help prevent migration.
- Tool-free SmartDosing® dials enable the patient to be in control of their pain level by fine-tuning the strap tension.
- Color coded touchpoints and Quick-Fit buckles increase user-friendliness.
- Alignment guide on hinge facilitates correct positioning on the leg.
- Entire brace is machine washable (washbag included with product).
- Easily adaptable to individual needs and anatomical shapes.
- Field serviceable.

REQUIRED MEASUREMENTS

Size	Calf circumference 15 cm (6") below mid patella	
	cm	inch
XS	25 - 31	9¾ - 12¼
S	31 - 36	12¼ - 14
M	36 - 43	14 - 16¾
L	43 - 51	16¾ - 20
XL	51 - 62	20 - 24¾



Measure calf circumference 15 cm (6") below mid-patella. Leg in extension.

# UNLOADER ONE® X

## ORDERING INFORMATION

### STANDARD

Size	Right Medial	Left Medial	Right Lateral	Left Lateral
XS	UOX0518711	UOX0618711	UOX0528711	UOX0628711
S	UOX0518712	UOX0618712	UOX0528712	UOX0628712
M	UOX0518713	UOX0618713	UOX0528713	UOX0628713
L	UOX0518714	UOX0618714	UOX0528714	UOX0628714
XL	UOX0518715	UOX0618715	UOX0528715	UOX0628715

Recommended for patients between 1.65 and 1.92 meters tall (approx. 5'5" - 6'4"). Hinge is placed on the side of the affected compartment.

### SHORT

Size	Right Medial	Left Medial	Right Lateral	Left Lateral
XS	UOX1518711	UOX1618711	UOX1528711	UOX1628711
S	UOX1518712	UOX1618712	UOX1528712	UOX1628712
M	UOX1518713	UOX1618713	UOX1528713	UOX1628713
L	UOX1518714	UOX1618714	UOX1528714	UOX1628714
XL	UOX1518715	UOX1618715	UOX1528715	UOX1628715

Recommended for patients under 1.65 meters tall (approx. 5'5"). Hinge is placed on the side of the affected compartment.

## ACCESSORIES

### EASE-OF-USE KIT

Description	Part#
Kit	UOX800001

Sundial  
Buckle finger loop

### THIGH EXTENSION LINER, GRAY

Right Medial, Left Lateral	Right Lateral, Left Medial
UOX546101	UOX545101

### ACTIVEGRIP CALF LINER, GRAY

Size	Right Medial, Left Lateral	Right Lateral, Left Medial
XS-S	UOX516153	UOX515153
M-XL	UOX516163	UOX515163

### DOESKIN CALF LINER, GRAY

Size	Right Medial, Left Lateral	Right Lateral, Left Medial
XS-S	UOX516151	UOX515151
M-XL	UOX516161	UOX515161

### SLEEVE

Size	Part#
XS	UOX60001
S	UOX60002
M	UOX60003
L	UOX60004
XL	UOX60005
XXL	UOX60006

# UNLOADER ONE® X

Description	Part#
Hinge pad, gray	UOX700011
Extension stop kit	B-301040047
Sensil strap pad, gray (10 pcs)	UOX700193
Sundial	UOX1200001

## REPLACEMENT PARTS

Description	Part#
Popliteal pad, gray	UOX700092
Alligator clip for DFS, gray	UOX900002
SmartDosing Dial, gray (1pc)	UOX1000002

### UPPER DFS AND CALF STRAP WITH BUCKLE, GRAY

Right Medial, Left Lateral	Right Lateral, Left Medial
UOX406101	UOX405101

Calf & DFS strap with buckle

### LOWER DFS AND THIGH STRAP WITH BUCKLE, GRAY

Right Medial, Left Lateral	Right Lateral, Left Medial
UOX406102	UOX405102

Thigh & DFS strap with buckle

### SENSIL CALF LINER, GRAY

Size	Right Medial, Left Lateral	Right Lateral, Left Medial
XS-S	UOX516152	UOX515152
M-XL	UOX516162	UOX515162

### DOESKIN THIGH LINER, GRAY

Right Medial, Left Lateral	Right Lateral, Left Medial
UOX526101	UOX525101

# UNLOADER ONE®

Unloader One® is designed to relieve osteoarthritis knee pain and wearing it may help improve mobility. It features Össur's unique, patented 3-point leverage system, which combines a single hinge with dual Dynamic Force System (DFS) straps to provide unloading of the knee joint. Clinical studies demonstrate that this system may relieve osteoarthritis knee pain and improve function and quality of life while being worn. Colour-coded Quick Fit buckles make donning and doffing simple, while flexible shells and breathable liners are designed to improve comfort and reduce migration, thus helping to ensure compliance.



## INDICATIONS FOR USE

- Moderate to severe unicompartmental knee osteoarthritis
- Degenerative meniscal tears
- Other knee conditions that may benefit from unicompartmental unloading, such as Articular cartilage defect repair, Avascular necrosis, Tibial plateau fracture, Bone marrow lesions (bone bruises)


## FEATURES AND BENEFITS

- Dual Dynamic Force Straps® designed to reduce pressure on sensitive bony surfaces of the knee, resulting in reduction of pain
- Lightweight frame, perforated for flexibility and ventilation, with flexible, low profile upright that contours to patient anatomy and allows use underneath clothing
- Tool-free SmartDosing® dial enables fine-tuned pain management and can be adjusted even when wearing the brace under clothing
- Colour-coded quick fit buckles enable easy application and removal
- Proprietary Sensil® silicone liners designed to reduce migration
- Available in standard and short version, for a perfect fit on shorter legs, too

## REQUIRED MEASUREMENTS

- Circumference measurement: 6"/15cm below mid-patella

Size	Measurements
Small	31 - 35.5cm
Medium	35.5 - 42.5cm
Large	42.5 - 51cm
XLarge	51 - 61.5cm

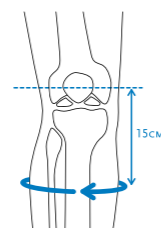

# UNLOADER ONE®

## ORDERING INFORMATION

### UNLOADER ONE WITH SMARTDOSING

Size	Calf circumference	Left medial	Right medial	Left lateral	Right lateral
Small	31 - 35.5cm	B-240618712	B-240518712	B-240628712	B-240528712
Medium	35.5 - 42.5cm	B-240618713	B-240518713	B-240628713	B-240528713
Large	42.5 - 51cm	B-240618714	B-240518714	B-240628714	B-240528714
X-Large	51 - 61.5cm	B-240618715	B-240518715	B-240628715	B-240528715

### UNLOADER ONE SHORT WITH SMARTDOSING

Size	Calf circumference	Left medial	Right medial	Left lateral	Right lateral
X-Small	25 - 31cm	B-241618711	B-241518711	B-241628711	B-241528711
Small	31 - 35.5cm	B-241618712	B-241518712	B-241628712	B-241528712
Medium	35.5 - 42.5cm	B-241618713	B-241518713	B-241628713	B-241528713
Large	42.5-51cm	B-241618714	B-241518714	B-241628714	B-241528714
Xlarge	51 - 61.5cm	B-241618715	B-241518715	B-241628715	B-241528715

### UNLOADER ONE WITH RATCHET

Size	Calf circumference	Left medial	Right medial	Left lateral	Right lateral
Small	31 - 35.5cm	B-240619712	B-240519712	B-240629712	B-240529712
Medium	35.5 - 42.5cm	B-240619713	B-240519713	B-240629713	B-240529713
Large	42.5 - 51cm	B-240619714	B-240519714	B-240629714	B-240529714
X-Large	51 - 61.5cm	B-240619715	B-240519715	B-240629715	B-240529715

### UNLOADER ONE SHORT WITH RATCHET

Size	Calf circumference	Left medial	Right medial	Left lateral	Right lateral
X-Small	25 - 31cm	B-241619711	B-241519711	B-241629711	B-241529711
Small	31 - 35.5cm	B-241619712	B-241519712	B-241629712	B-241529712
Medium	35.5 - 42.5cm	B-241619713	B-241519713	B-241629713	B-241529713
Large	42.5 - 51cm	B-241619714	B-241519714	B-241629714	B-241529714
Xlarge	51 - 61.5cm	B-241619715	B-241519715	B-241629715	B-241529715

Standard version is recommended for patients above 1.65m  
Short version is recommended for patients under 1.65m tall.  
Hinge is placed on the side of the affected compartment

## ACCESSORIES

### ACCESSORY KITS

Description	SMARTDOSING		RATCHET	
	RM/LL	RL/LM	RM/LL	RL/LM
Strap & Strap Pad & Buckle Clasp Kit	B-705277003	B-705278003	B-701217008	B-701218008
Strap & Strap Pad Kit	B-705270003		B-701210006	
Strap Kit	B-701270013		B-701210004	
Strap Kit Short	B-701240012			
Pad Kit	B-702210003		B-702210003	

KNEE SOLUTIONS

KNEE SOLUTIONS

# UNLOADER ONE®

## ACCESSORIES

### ACCESSORY KITS

Description	Right medial/Left lateral	Right lateral/Left medial
Buckle Clasp Kit	B-708217003	B-708218003
Alligator Clip Kit (included in all strap kits)	B-704210003	
Extension Stop Kit	B-301040046	
Twist-Fit Hook & Loop Attachment Kit	B-721240003	

### UNDERSLEEVE

Size	Thigh Circumference	Part#
X-Small	25.4 - 30.9cm	B-250150201
Small	30.9 - 35.6cm	B-250150202
Medium	35.6 - 42.5cm	B-250150203
Large	42.5 - 50.8cm	B-250150204
X-Large	50.8 - 61cm	B-250150205
XX-Large	61 - 81.3cm	B-250150206

When using an undersleeve we advice to use the suspension strap kit to minimise migration



### SENSIL LINERS

Size	Tibia		Femur	
	RM/LL*	RL/LM**	RM/LL*	RL/LM**
X-Small	B-419700311	B-419800311	B-409700321	B-409800321
Small	B-419700312	B-419800312	B-409700322	B-409800322
Medium	B-419700313	B-419800313	B-409700323	B-409800323
Large	B-419700314	B-419800314	B-409700324	B-409800324
X-Large	B-419700315	B-419800315	B-409700325	B-409800325

\* Right medial/Left lateral / \*\* Right lateral/Left medial



### DOESKIN LINERS

Size	Tibia		Femur	
	RM/LL*	RL/LM**	RM/LL*	RL/LM**
X-Small	B-424703721	B-424803721	B-412701721	B-412801721
Small	B-424703722	B-424803722	B-412701722	B-412801722
Medium	B-424703723	B-424803723	B-412701723	B-412801723
Large	B-424703724	B-424803724	B-412701724	B-412801724
X-Large	B-424703725	B-424803725	B-412701725	B-412801725

When using a doeskin tibia liner we advice to use a suspension strap kit to minimise migration / \* Right medial/Left lateral / \*\* Right lateral/Left medial



### SENSIL AMS (ANTI-MIGRATION SYSTEM) WRAPS

Size	Right medial/Left lateral	Right lateral/Left medial
X-Small	B-715217031	B-715218031
Small	B-715217032	B-715218032
Medium	B-715217033	B-715218033
Large	B-715217034	B-715218034
X-Large	B-715217035	B-715218035



### SUSPENSION STRAP KIT

Description	Part#
Suspension Strap Kit	UNL00212



# UNLOADER ONE® CUSTOM

## ORDERING INFORMATION

### UNLOADER ONE CUSTOM WITH SMARTDOSING

Colour	Thigh circumference	Calf circumference	Left	Right
Black	34.3 - 73.5cm	25 - 61.5cm	B-115600112	B-115500112
Grey	34.3 - 73.5cm	25 - 61.5cm	B-115601712	B-115501712

### UNLOADER ONE CUSTOM

Colour	Thigh circumference	Calf circumference	Left	Right
Black	34.3 - 73.5cm	25 - 61.5cm	B-115600111	B-115500111
Grey	34.3 - 73.5cm	25 - 61.5cm	B-115601711	B-115501711



### SMARTMEASURE®

A digital measuring system for custom knee braces on the iPhone or iPad. The most intuitive, efficient and accurate way to measure for any Össur custom knee brace, including the Unloader One. Utilizes a section-based, step-by-step approach to guide you through the entire brace selection, measuring and order process.



### DIGITAL MEASUREMENT SYSTEM (DMS)

This system requires minimal physical measurements and two digital photos, which can then be sent to customer service. This system is a quick, easy and reliable method of measuring for an Unloader One® Custom without the need for training.

\*For custom accessories, please contact customer care on 08450 065 065.

### STANDARD COLOURS (CUSTOM ONLY)



## UNLOADER ONE® LITE

Unloader One Lite is a slim and low profile knee brace that relieves pain from degenerative meniscal tears and knee OA by means of the clinically proven Dynamic Force System that provides uni-compartmental unloading of the knee joint.

With an integrated Lycra® sleeve and shells with overmold, the brace is an optimal combination of comfort and sleek look.



### INDICATIONS FOR USE

- Mild to moderate unicompartmental osteoarthritis
- Degenerative meniscus
- Other knee conditions that may benefit from unicompartmental unloading

### FEATURES AND BENEFITS

- 3-Point Leverage System provides clinically relevant pain relief of affected compartment
- Single SmartDosing dial allows fine-tuned pain management
- Helps keep patients mobile during daily activities
- Discreet hinge and sleeve design ensure a snug fit underneath clothing
- Lightweight and easy to use

### REQUIRED MEASUREMENTS

- Circumference measurement: 6"/15cm above and below mid-patella



## UNLOADER ONE® LITE

### ORDERING INFORMATION

#### UNLOADER ONE LITE WITH ALLIGATOR CLIP DFS

Size	Circumference 6"/15cm above mid patella	Circumference 6"/15cm below mid-patella	Left medial	Right medial	Left lateral	Right lateral
X-Small	12 - 15" (30 - 38cm)	10 - 12" (25 - 30cm)	UOL02300	UOL01300	UOL02400	UOL01400
Small	15 - 17.75" (38 - 45cm)	12 - 13.75" (30 - 35cm)	UOL02301	UOL01301	UOL02401	UOL01401
Medium	17.75 - 22" (45 - 56cm)	13.75 - 16.5" (35 - 42cm)	UOL02302	UOL01302	UOL02402	UOL01402
Large	22 - 26" (56 - 66cm)	16.5 - 20" (42 - 51cm)	UOL02303	UOL01303	UOL02403	UOL01403
X-Large	26 - 29" (66 - 73cm)	20 - 24" (51 - 61cm)	UOL02304	UOL01304	UOL02404	UOL01404

\*Hinge is placed on the side of the affected compartment

### ACCESSORIES

Description	Part#
Washbag	UNH6180
Hinge pad	B-304120501-10

#### STRAP KITS

Size	Part#
Alligator DFS Strap Kit	UNF01502
Calf strap X-Small	UNF01402
Calf strap Small	UNF01404
Calf strap Medium	UNF01406
Calf strap Large	UNF01408
Calf strap X-Large	UNF01410

\*Both calf strap and alligator strap kit required for full strap kit

## REBOUND® CARTILAGE

Cartilage and meniscal injuries usually lead to significant limitations of knee function and knee pain. As surgery cannot be done on the day of diagnosis, prehabilitation is indicated to limit knee pain and keep the patient functional until surgery is scheduled. In addition, post-operatively, the success of meniscal tear and articular cartilage repair is dependent upon patient compliance during the rehabilitation process and the protocol is based on restricted weight bearing combined with gradual increase in activity. Many rehabilitation protocols suggest partial weight-bearing as the standard of care for successful outcomes. However, patient compliance to partial weight-bearing instructions with crutches has been shown to be inconsistent.

Rebound Cartilage is a protective functional solution designed for prehabilitation / pre-surgical management and rehabilitation of cartilage injuries and meniscal lesions. By protecting the healing tissue from overload, it fits into accelerated rehab protocols.

### FUNCTIONAL HEALING - WHERE MOBILITY MEANS RECOVERY



### INDICATIONS FOR USE

- Articular cartilage defect repair needing unicompartmental load reduction or range of movement restriction
- Cartilage knee trauma benefiting from unloading or movement restriction
- Meniscal repair
- Avascular necrosis
- Condylar bone marrow lesions (bone bruises)
- Tibial plateau fractures
- May also be beneficial for other unicompartmental knee conditions that require unloading

### FEATURES AND BENEFITS

- Dual Cartilage Protection Straps™ (CPS) maintain joint unloading in flexion for protection of the cartilage during the healing process
- Breathable wrap around sleeve integrated with Sensil® silicone coated calf liners and doeskin thigh liners provide increased comfort while ensuring adequate suspension
- Colour coded Quick Fit buckles enable easy application and removal
- Optional modular Flexion Control kit when needed per prescribed rehab protocol for range of movement restriction.

### REQUIRED MEASUREMENTS

- Circumference measurement: 15cm below mid-patella



## REBOUND® CARTILAGE

### ORDERING INFORMATION

#### REBOUND CARTILAGE

Size	Circumference	Left medial	Right medial	Left lateral	Right lateral
Small	31 - 35.5cm	B-254610002	B-254510002	B-254620002	B-254520002
Medium	35.5 - 42.5cm	B-254610003	B-254510003	B-254620003	B-254520003
Large	42.5 - 51cm	B-254610004	B-254510004	B-254620004	B-254520004
X-Large	51 - 61.5cm	B-254610005	B-254510005	B-254620005	B-254520005

#### REBOUND CARTILAGE - WITH FLEXION CONTROL

Size	Circumference	Left medial	Right medial	Left lateral	Right lateral
Small	31 - 35.5cm	B-254810002	B-254710002	B-254820002	B-254720002
Medium	35.5 - 42.5cm	B-254810003	B-254710003	B-254820003	B-254720003
Large	42.5 - 51cm	B-254810004	B-254710004	B-254820004	B-254720004
X-Large	51 - 61.5cm	B-254810005	B-254710005	B-254820005	B-254720005

Note: Circumference limitation: Thigh circumferences smaller than 34.3cm or larger than 73.5cm. Calf circumferences smaller than 25cm or larger than 61.5cm.

### ACCESSORIES

#### WRAPAROUND SLEEVE (WITH INTEGRATED SILICONE CALF LINER AND DOESKIN THIGH LINER)

Size	Right medial/Left lateral	Right lateral/Left medial
Small	B-254900072	B-254900082
Medium	B-254900073	B-254900083
Large	B-254900074	B-254900084
X-Large	B-254900075	B-254900085

Optional sleeveless liner kit	Right medial/Left lateral	Right lateral/Left medial
Silicone calf liner, doeskin thigh liner, strap pads	B-254900070	B-254900080
Doeskin calf and thigh liners, suspension strap kit, strap pads	B-427705000	B-427705001

#### STRAP KITS

Size	Complete strap kit (incl. 4 straps)	CPS strap kit (incl. 2 straps)
Small	B-701290002	B-700290002
Medium	B-701290003	B-700290003
Large	B-701290004	B-700290004
X-Large	B-701290005	B-700290005

#### UNIVERSAL FLEXION CONTROL KIT

Description	Part#
Left medial	B-705296100
Right medial	B-705295100
Left lateral	B-705296200
Right lateral	B-705295200

#### OTHER ACCESSORIES

Description	Part#
Flexion Stop Kit	B-729000000
Twist-Fit Hook & Loop Attachment Kit	B-705290000



## CTi® OA

CTi braces provide the ultimate combination of stabilisation and protection of the knee joint. CTi uses Accutrac® hinges with extension stops to glide with the knee and breathable liners coated with bio-inert Sensil® silicone to ensure the CTi remains properly positioned on the leg, for optimal functional benefit. Allowing for varus or valgus correction, the custom made CTi OA is an excellent, dual hinged hybrid OA and ligament knee brace for your more active patients.



### INDICATIONS FOR USE

- For knee conditions that may benefit from increased A-P and M-L stability around the knee, such as: ACL, MCL, LCL, PCL, rotary and combined instabilities
- Also treats mild to severe unicompartmental osteoarthritis

### FEATURES AND BENEFITS

- Reversible field-serviceable d-rings and trimmable straps facilitate fast adjustments and customised fit
- Corrosion resistant materials, ideal for water sports
- Anatomically-correct Accutrac® hinges with extension stops
- Proprietary Sensil® silicone liners designed to reduce migration

### REQUIRED MEASUREMENTS

- Custom-to-measurement
- 5 point knee measurement (Smartmeasure)



## CTi® OA

### ORDERING INFORMATION

#### CTi OA

Description	Left	Right
CTi OA	B-116630010	B-116530010

For accessories see CTi Custom.



### SMARTMEASURE®

A digital measuring system for custom knee braces on the iPhone or iPad. The most intuitive, efficient and accurate way to measure for any Össur custom knee brace, including the CTi OA. Utilises a section-based, step-by-step approach to guide you through the entire brace selection, measuring and order process.



### DIGITAL MEASUREMENT SYSTEM (DMS)

This system simply requires minimal physical measurements and two digital photos, which can then be sent to customer service. This system is a quick, easy and reliable method of measuring for a CTi OA without the need for training.

Please contact customer care for a free DMS Kit. Item number - 89100

## REBOUND® DUAL

Osteoarthritis is a very common result of a history of ligament damage. The Rebound® DUAL offers a potent mix of pain relief and stabilisation to address the functional needs of a patient with both unicompartmental osteoarthritis and ligament damage. The brace combines Dynamic Force Strap (DFS) technology from the Unloader One® with a mouldable aluminium frame to provide ligament stability. The Össur OA range ensures we have an effective solution to suit the varied needs of your patients.



### INDICATIONS FOR USE

- For knee conditions that may benefit from increased AP and ML stability around the knee, such as: ACL, MCL, LCL, PCL, rotary and combined instabilities.

### FEATURES AND BENEFITS

- Polycentric hinge on an aluminium frame for stability, durability, and low-profile design
- Adjustable tibia strap captures tibia for optimal stability
- SmartDosing® kit for fine-tuned pain management and offloading
- Breathable ActiveGrip™ calf liner with anti-migration capabilities
- Acculign technology allows adjustment of femoral shell length from 16.5 cm to 20 cm
- Frame adjustable to accommodate both varus and valgus angulations

### REQUIRED MEASUREMENTS

- Caliper measurement: Medio-laterally at joint width in a standard, weight-bearing position
- Thigh Circumference: 6" above mid-patella

## REBOUND® DUAL

### ORDERING INFORMATION

#### REBOUND DUAL (OA) WITH SMARTDOSING STRAP

Size	Caliper	Circumference	Right medial	Left medial	Right lateral	Left lateral
XSmall	7.6 - 8.9cm	38.1 - 41.9cm	B-242510001M	B-242610001M	B-242510001L	B-242610001L
Small	8.9 - 10.2cm	41.9 - 47cm	B-242510002M	B-242610002M	B-242510002L	B-242610002L
Medium	10.2 - 11.4cm	47 - 52.1cm	B-242510003M	B-242610003M	B-242510003L	B-242610003L
Large	11.4 - 12.7cm	52.1 - 57.2cm	B-242510004M	B-242610004M	B-242510004L	B-242610004L
X-Large	12.7 - 14cm	57.2 - 62.2cm	B-242510005M	B-242610005M	B-242510005L	B-242610005L
XX-Large	14 - 15.2cm	62.2 - 69.9cm	B-242510006M	B-242610006M	B-242510006L	B-242610006L
XXX-Large	15.2 - 16.5cm	69.9 - 76cm	B-242510007M	B-242610007M	B-242510007L	B-242610007L

M/L at joint width in a standard, weight-bearing position. Thigh Circumference: 6" above mid-patella

#### LINERS & KITS

Size	Thigh liner kit	Calf liner kit	Doeskin calf kit	Activegrip thigh liner
X-Small	B-701090010	B-701090017	B-701090028	B-701090041
Small	B-701090011	B-701090018	B-701090029	B-701090042
Medium	B-701090012	B-701090019	B-701090030	B-701090043
Large	B-701090013	B-701090020	B-701090031	B-701090044
X-Large	B-701090014	B-701090021	B-701090032	B-701090045
XX-Large	B-701090015	B-701090022	B-701090033	B-701090046
XXX-Large	B-701090016	B-701090023	B-701090034	B-701090047

### ACCESSORIES

Description	Part#
Tibial Wedge Kit	B-701090001
Flexion Stop Kit	B-701090004
XS-LG Strap Kit	B-701090005
XL-XXXL Strap Kit	B-701090006
D-Ring Kit	B-701090007
Extension Stop Kit	B-701090008

Description	Part#
Condyle Pad Kit	B-701090009
Strap Pad Kit	B-701090037

## REBOUND® PCL

The PCL, compared to the ACL, has the intrinsic ability to heal and regain continuity after an injury. The PCL experiences variable tensile forces during knee flexion. Insufficient healing after a PCL rupture leads to significant pathologies, such as, chronic anterior knee pain or knee OA. Healing of the PCL in an elongated position can lead to chronic instability and disability. Therefore, physiologically correct positioning of the PCL during conservative or post-surgical rehabilitation is important. The Rebound PCL is the world's First Dynamic Force PCL Brace. It applies a physiologically correct dynamic force to the PCL during rehabilitation.


The dynamic load on the tibia is generated by applying an anterior directed dynamic force on the calf area, and opposing counter forces on the anterior aspect of the leg.



### FUNCTIONAL HEALING - WHERE MOBILITY MEANS RECOVERY

### INDICATIONS FOR USE

- Non-surgical treatment of PCL ruptures
- Post-surgical rehabilitation for:
  - Partial PCL tears
  - PCL reconstruction / augmentation
  - PCL revision

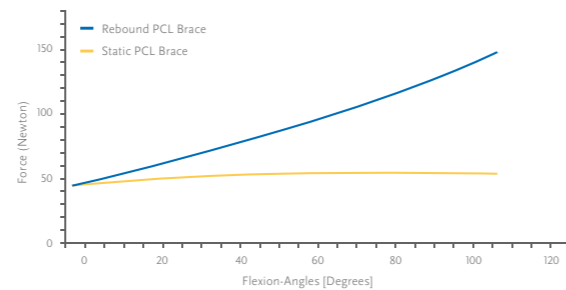

LaPrade RF, Smith SD, Wilson KJ, Wijdicks CA. Quantification of functional brace forces for posterior cruciate ligament injuries on the knee joint: an in vivo investigation. Knee Surg Sports Traumatol Arthrosc [Internet]. 2014 Aug 22 [cited 2014 Dec 11]; Available from: <http://www.ncbi.nlm.nih.gov/pubmed/25145947>

### FEATURES AND BENEFITS

- Dynamic Tension System (DTS) is designed to increase the load on the tibia as the knee goes through flexion, providing an anterior drawer force necessary to decrease load on the healing PCL to allow it to heal
- 3 options to customise the biomechanically correct posterior tibial load for each patient by using shear knobs. And having the possibility to re-adjust the posterior tibial load, e.g. in case of swelling.
- Polycentric hinge with extension and flexion stops allowing for range of motion adjustment.
- Lightweight brace with proprietary Sensil® silicone calf liner and doeskin thigh liner reduce migration while providing comfort to the patient
- Moulded posterior frame for optimised fit and support, with open anterior for ease of donning/doffing
- Available in 12 colours

### REQUIRED MEASUREMENTS

- Circumference 15cm above mid patella
- Circumference 15cm below mid patella
- Medio/lateral width of the knee using a Constant Force (CF) caliper (Contact Customer Care if you require a CF caliper)
- Custom- made to measure version also available

## REBOUND® PCL

### ORDERING INFORMATION

#### REBOUND PCL

Size	Caliper M/L*	Calf Circ.**	Thigh Circ.***	Left	Right
Small	8.9 - 10.2cm	29 - 34.5cm	42 - 45cm	B-124501161	B-124501158
Medium	10.2 - 11.4cm	34.5 - 38cm	45 - 51cm	B-124501162	B-124501159
Large	11.4 - 12.7cm	38 - 42cm	51 - 58.5cm	B-124501163	B-124501160

\* M/L measurement: In a standard, weight bearing position - Using a Caliper: M/L at joint width, Using a Constant Force Caliper: M/L at knee axis line  
 \*\* Calf circumference measurement: 15cm below knee centre  
 \*\*\* Thigh circumference measurement: 15cm above knee centre



### SMARTMEASURE®

A digital measuring system for custom knee braces on the iPhone or iPad. The most intuitive, efficient and accurate way to measure for any Össur custom knee brace. Utilises a section-based, step-by-step approach to guide you through the entire brace selection, measuring and order process.



### DIGITAL MEASUREMENT SYSTEM (DMS)

This system simply requires minimal physical measurements and two digital photos, which can then be sent to customer service. This system is a quick, easy and reliable method of measuring for a custom knee brace without the need for training.

Please contact customer care for a free DMS Kit. Item number - 89100

#### REBOUND PCL CUSTOM

Description	Part#
Rebound PCL Custom Left	B-124501155
Rebound PCL Custom Right	B-124501154

### ACCESSORIES

#### STRAPS

Size	Part#
X-Small - Large	PCL00061
X-Large - XX-Large	PCL00062

#### OTHER ACCESSORIES

Description	Part#
Extension Stop Kit	B-705130042
Flexion Stop Kit	B-705130043
Shear Knob Kit	PCL00030
Condyle Pad Kit	B-705130046
PCL Pad	B-705000036



Shear Knob Kit



PCL Pad



## REBOUND® ACL

Partial anterior cruciate ligament (ACL) tears can lead to rotary laxity or abnormal anteroposterior laxity of the knee. Acute ACL injury may lead to unsatisfactory knee function, decreased activity and poor knee related quality of life. Insufficient healing after an ACL rupture leads to significant pathologies, such as chronic knee pain or knee OA. Therefore, anatomically correct positioning during conservative treatment or after surgery is important.

The Rebound ACL applies a physiologically correct dynamic force to the ACL during rehabilitation. The dynamic load on the femur is generated by applying an anterior directed dynamic force on the thigh area, and opposing counter forces on the anterior aspect of the tibia. By counteracting the anatomical loading characteristics of the ACL, the Dynamic Tensioning System decreases the load on the ACL to allow it to heal.

FUNCTIONAL HEALING - WHERE MOBILITY MEANS RECOVERY




### INDICATIONS FOR USE

Post-surgical

- Partial ACL tears
- Proximal ACL tears
- ACL reconstruction / augmentation
- ACL revision

Anterior cruciate ligament (ACL) injury, including reconstruction or conservative, non-surgical treatment

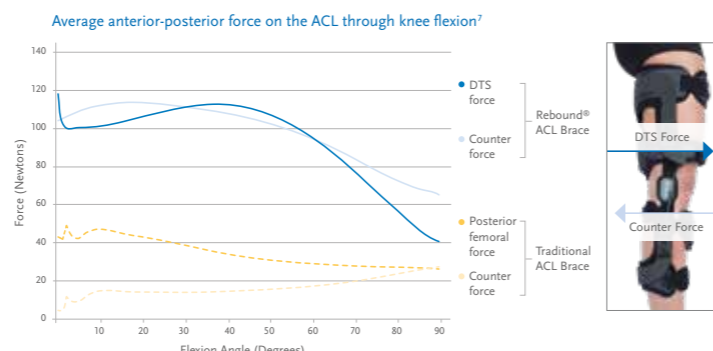

### FEATURES AND BENEFITS

- Dynamic Tension System (DTS) is designed to increase the load on the femur as the knee goes from flexion into extension providing an anterior drawer force necessary to decrease load on the ACL to allow it to heal.
- Polycentric hinge with extension and flexion stops allowing for range of motion adjustment.
- Lightweight brace with proprietary Sensil® silicone calf liner and doeskin thigh liner reduce migration while providing comfort to the patient
- Moulded posterior frame for optimised fit and support, with open anterior for ease of donning/doffing
- Available in 12 Colours

### REQUIRED MEASUREMENTS

- Circumference 15cm above mid patella
- Circumference 15cm below mid patella
- Medio/lateral width of the knee using a
- Constant Force (CF) caliper (Contact Customer Care if you require a CF caliper)
- Custom- made to measure version also available

#### STANDARD COLOURS (CUSTOM ONLY)



## REBOUND® ACL

### ORDERING INFORMATION



#### SMARTMEASURE®

A digital measuring system for custom knee braces on the iPhone or iPad. The most intuitive, efficient and accurate way to measure for any Össur custom knee brace, including the Rebound ACL. Utilises a section-based, step-by-step approach to guide you through the entire brace selection, measuring and order process.



#### DIGITAL MEASUREMENT SYSTEM (DMS)

This system simply requires minimal physical measurements and two digital photos, which can then be sent to customer service. This system is a quick, easy and reliable method of measuring for a Rebound ACL without the need for training.

### REBOUND ACL


Description	Right	Left	Caliper M/L	Calf circumference	Thigh circumference
Small	B-124501188	B-124601188	8.9 - 10.2cm	29 - 34.5cm	42 - 45cm
Medium	B-124501189	B-124601189	10.2 - 11.4cm	34.5 - 38cm	45 - 51cm
Large	B-124501190	B-124601190	11.4 - 12.7cm	38 - 42cm	51 - 58.5cm

### REBOUND ACL CUSTOM

Description	Part#
Rebound ACL Custom Left	B-124501157
Rebound ACL Custom Right	B-124501156

### ACCESSORIES

Description	Part#
Extension Stop Kit	B-705130042
Flexion Stop Kit	B-705130043
Shear Knob Kit	B-705130044
Strap Kit	B-705130045
Condyle Pad Kit	B-705130046
Pre-tibial Hexpad	B-705130049
Tibial Pad	B-705130047

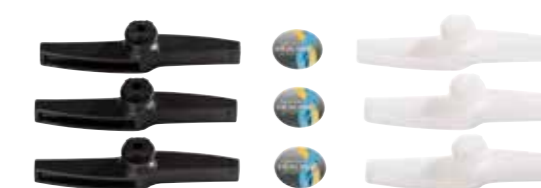

2 types of shear knob  
 3x White - Standard tension 125 Newtons  
 3x Black - Low tension 88 Newtons

KNEE SOLUTIONS

KNEE SOLUTIONS

## CTI® OTS

The proven technology of the CTi knee brace is available with off-the-shelf convenience, providing adjustability and an excellent out-of-the-box fit while maintaining the level of support and protection that you expect from a CTi knee brace.

Outfitted with anatomically-correct Accutrac® hinges and flexible subshells and d-rings, the rock-solid CTi carbon frame provides a rigid exoskeleton that stabilises the knee joint. Breathable liners coated with bio-inert Össur Sensil® Silicone ensure CTi OTS remains properly positioned on the leg, for optimal functional benefit.



### INDICATIONS FOR USE

- ACL, MCL, LCL, PCL, rotary and combined instabilities

### FEATURES AND BENEFITS

- Anatomically-correct Accutrac hinges with extension stops to glide with the knee
- Flexible subshell and Adjustable Tibial Strap capture the tibia and allow adjustments in hinge depth for a more customised fit
- Breathable liners coated with bio-inert Össur Sensil Silicone to reduce migration and skin irritation
- Total Support System™ for ACL, MCL, LCL, PCL, rotary and combined instabilities
- Corrosion resistant materials, ideal for water sports
- Available in 2 models: Standard model for moderate to high impact levels, Pro Sport model for the highest impact levels

### REQUIRED MEASUREMENTS

- Caliper measurement: Medio-laterally at joint width in a standard, weight-bearing position
- Circumference: Just below the distal border of the patella



## CTI® OTS

### ORDERING INFORMATION

#### CTI

Size	Caliper	Circumference	Standard Left	Standard Right
Small	8.9 - 10.2cm	28 - 34cm	B-238600112	B-238500112
Medium	10.2 - 11.4cm	34 - 38cm	B-238600113	B-238500113
Large	11.4 - 12.7cm	38 - 41cm	B-238600114	B-238500114
X-Large	12.7 - 14.0cm	41 - 44cm	B-238600115	B-238500115
XX-Large	14.0 - 15.2cm	44 - 48cm	B-238600116	B-238500116

Size	Pro Sport Left	Pro Sport Right	CTi PCL left	CTi PCL right
Small	B-239600112	B-239500112	Small, B-238640112	Small, B-238540112
Medium	B-239600113	B-239500113	Medium, B-238640113	Medium, B-238540113
Large	B-239600114	B-239500114	Large, B-238640114	Large, B-238540114
X-Large	B-239600115	B-239500115	X-Large, B-238640115	X-Large, B-238540115
XX-Large	B-239600116	B-239500116	XX-Large, B-238640116	XX-Large, B-238540116

Note: M/L measurements are the preferred and most accurate form of measurement for the CTi. Circumferential measurements are approximate.

### ACCESSORIES

#### LINERS

Size	Silicone tibia cuff	Silicone calf	Silicone thigh	Lycra thigh
Medium	B-304030030	B-425580002	B-425571002	B-425570002
Large		B-425580004	B-425571004	B-425570004
X-Large, XX-Large		B-425580005	B-425571004	B-425570004

#### STRAPS KITS

Size	Part#
Small, Medium	B-701140103
Large, X-Large, XX-Large	B-701140104

#### PCL KITS

Size	Part#
Small, Medium	B-705000132
Large, X-Large, XX-Large	B-705000235

#### STRAP PADS

Size	Part#
Small	B-304010057
Medium, Large	B-304010058
X-Large, XX-Large	B-304010059

#### AMS (ANTI-MIGRATION SYSTEM) WRAPS

Description	Part#
Silicone AMS Wrap Small, Medium	B-715260002
Silicone AMS Wrap Large, X-Large, XX-Large	B-715260004
Neoprene AMS Wrap Small, Medium	B-410000001
Neoprene AMS Wrap Large, X-Large	B-410000002

## CTI® OTS

### ACCESSORIES

#### MX GEAR

Description	Part#
Patella Cup	B-711130011
Gear Guards	B-705000003
MX Kit S (patella cup, gear guards, undersleeve, stickers)	B-711140002
MX Kit M (patella cup, gear guards, undersleeve, stickers)	B-711140003
MX Kit L (patella cup, gear guards, undersleeve, stickers)	B-711140004
MX Kit XL (patella cup, gear guards, undersleeve, stickers)	B-711140005

#### SLEEVES

Circumference 15cm above mid patella	Softsleeve	Undersleeve	Oversleeve	Sportsleeve
< 38cm	20643	20251	20051	20682
38 - 45cm	20644	20252	20052	20683
45 - 49.5cm	20644	20253	20053	20684
49.5 - 54.5cm	20645	20254	20054	20685
54.5 - 62cm	20645	20255	20055	N/A
62 - 71cm	20646	20256	20056	N/A
71 - 86cm	20647	20257	20057	N/A

#### OTHER ACCESSORIES

No.	Description	Part#
1	CTi D-Ring Kit Standard 2x 1" and 2x 1.5" d-rings	B-704132600
2	CTi D-Ring Kit 2" 2x 2" d-rings	B-704132602
3	CTi Condyle Pad Kit	B-704130150
4	Extension Stop Kit	22130
5	Flexion Stop Kit	19254



## CTI® CUSTOM

CTi braces provide the ultimate combination of stabilisation and protection of the knee joint. This truly custom made brace, with rock-solid CTi carbon frame, uses Accutrac® hinges with extension stops to glide with the knee and breathable liners coated with bio-inert Sensil® silicone to to ensure the CTi remains properly positioned on the leg, for optimal functional benefit. CTi custom braces are available in different models and can be further customised with a variety of options, colours, designs and accessories.



### INDICATIONS FOR USE

For knee conditions that may benefit from increased AP and ML stability around the knee, such as: ACL, MCL, LCL, PCL, rotary and combined instabilities

### FEATURES AND BENEFITS

- Reversible field-serviceable d-rings and trimmable straps facilitate fast adjustments and customised fit (optional)
- Corrosion resistant materials, ideal for water sports
- Anatomically-correct Accutrac® hinges with extension stops
- Proprietary Sensil® silicone liners designed to reduce migration
- Available in 3 different models: Standard model for moderate to high impact levels, Pro Sport model for the highest impact levels and Vapor model for low to moderate impact levels

#### STANDARD COLOURS



### REQUIRED MEASUREMENTS

- 5 point knee measurement (Smartmeasure)



## CTI® CUSTOM

### ORDERING INFORMATION

#### CTI CUSTOM

Description	Left	Right
Standard	B-116600010	B-116500010
Pro Sport	B-117600010	B-117500010
Vapor	B-118600010	B-118500010



#### SMARTMEASURE®

A digital measuring system for custom knee braces on the iPhone or iPad. The most intuitive, efficient and accurate way to measure for any Össur custom knee brace, including the CTi. Utilises a section-based, step-by-step approach to guide you through the entire brace selection, measuring and order process.



#### DIGITAL MEASUREMENT SYSTEM (DMS)

This system simply requires minimal physical measurements and two digital photos, which can then be sent to customer service. This system is a quick, easy and reliable method of measuring for a CTi without the need for training.

Please contact customer care for a free DMS Kit. Item number - 89100

### ACCESSORIES

#### CALF LINERS

Description	Part#
CTi Custom liner	B-411500115

#### STRAPS KITS

Size	Part#
Small, Medium	B-701130104
Large, X-Large, XX-Large	B-701130108

#### PCL KITS

Size	Part#
Small, Medium	B-705000132
Large, X-Large, XX-Large	B-705000235

## CTI® CUSTOM

### ACCESSORIES

#### STRAP PADS

Size	Part#
Small	B-304010057
Medium, Large	B-304010058
X-Large, XX-Large	B-304010059

#### AMS (ANTI-MIGRATION SYSTEM) WRAPS

Description	Part#
AMS Wrap EVA	22095
AMS Wrap Neo	22094
AMS Wrap Neo XL	22152

#### MX GEAR

Description	Part#
Patella Cup	B-711130011
Gear Guards	B-705000003
MX Kit S (patella cup, gear guards, undersleeve, stickers)	B-711140002
MX Kit M (patella cup, gear guards, undersleeve, stickers)	B-711140003
MX Kit L (patella cup, gear guards, undersleeve, stickers)	B-711140004
MX Kit XL (patella cup, gear guards, undersleeve, stickers)	B-711140005

#### SLEEVES

Size	Circumference 15cm above mid patella	Softsleeve	Undersleeve	Oversleeve	Sportsleeve
Small	< 38cm	20643	20251	20051	20682
Medium	38 - 45cm	20644	20252	20052	20683
Large	45 - 49.5cm	20644	20253	20053	20684
X-Large	49.5 - 54.5cm	20645	20254	20054	20685
XX-Large	54.5 - 62cm	20645	20255	20055	N/A
XXX-Large	62 - 71cm	20646	20256	20056	N/A
XXXX-Large	71 - 86cm	20647	20257	20057	N/A

#### OTHER ACCESSORIES

Description	Part#
CTi D-Ring Kit Standard 2x 1" and 2x 1.5" d-rings	B-704132600
CTi D-Ring Kit 2" 2x 2" d-rings	B-704132602
CTi Condyle Pad Kit	B-704130150
Extension Stop Kit	22130
Flexion Stop Kit	19254

## REBOUND® POST-OP

The simple, versatile post-operative knee bracing solution that is easy for everyone. Whether adjusting ROM, setting the drop lock, or altering the brace length, you'll appreciate the simplicity of the Rebound Post-op knee brace.



### INDICATIONS FOR USE

- ACL, PCL, MCL, LCL repairs/injuries
- Meniscal repairs
- Tibial plateau fractures
- Patellar tendon repairs
- Osteochondral repairs
- Condylar fractures
- Sprains/strains of the knee
- HTOs (High Tibial Osteotomy)

### FEATURES AND BENEFITS

- Tool-free design and multi-purpose Smart-Fit™ packaging
- Available in universal, left or right contour and paediatric
- Telescoping frame from 47cm-68cm (Adult) and 39.5cm - 54cm (paediatric)
- OneTouch™ easy to adjust ROM hinge with lock-out capability
- Colour-coding and numbered frame for patient education
- Quick-fit, gripping buckles for easy application

### REQUIRED MEASUREMENTS

- Circumference: 6" (15cm) above mid-patella

### ORDERING INFORMATION

#### REBOUND POST-OP

Description	Thigh circumference	Cool	Full foam
Paediatric	12"-25" (30.5cm - 63.5cm)	B-8381470	
Universal, Left contoured	16"-32" (41cm - 81cm)	B-8381680	B-8382680
Universal, Right contoured	16"-32" (41cm - 81cm)	B-8381580	B-8382580
Universal	16"-32" (41cm - 81cm)	B-8381480	B-8382480
Universal XL	16"-38" (41cm - 96,5cm)	B-8381490	B-8382490

#### ACCESSORIES

Description	Part#	Description	Part#
ROM stop locks	B-8380005	Standard buckle & pad kit	B-8381096
Alligator hook & loop kit	B-8380006	AMS Kit Paediatric	B-8381093
Adhesive Hook & Loop kit Paediatric	B-8380008	AMS Kit Regular	B-8381094
Adhesive Hook & Loop kit universal / universal XL	B-8380007	AMS Kit XL	B-8381095

#### REPLACEMENT SOFTGOODS

Description	Cool	Full foam
Paediatric	B-8381072	
Universal*	B-8381082	B-8382082
Universal XL	B-8381092	B-8382092

\*For universal, left contoured, right contoured

## FORMFIT POST-OP KNEE

### INDICATIONS FOR USE

- Post-op/post-injury ROM control or immobilisation
- Indicated for ligament and fracture-related surgeries and when controlled range of motion is necessary

### FEATURES AND BENEFITS

- Breakaway struts for easy, tool free length adjustment
- OneTouch™ easy to adjust ROM hinge with lock-out capability from -10° to 30° (10° incremental)
- Extension control from -10° to 90° (10° incremental)
- Flexion control from 0° to 120° (10° incremental)
- Available in cool or foam models, 3 lengths, and two sizes (Regular and XL)



### REQUIRED MEASUREMENTS



Measure thigh circumference 15 cm (6") above mid-patella. Leg in extension.

Size	Thigh circumference 15 cm (6") above mid patella	
	cm	inch
XL	69 - 89	27 1/8 - 35
Regular	0 - 69	0 - 27 1/8

### ORDERING INFORMATION

#### FULL FOAM, 61 CM (24")

Size	Part#
XL	FPOK403202
Regular	FPOK403200

#### FULL FOAM, 61 CM (24"), BAG

Size	Part#
XL	FPOK403201
Regular	FPOK4032001

#### COOL FOAM, 61 CM (24")

Size	Part#
XL	FPOK403102
Regular	FPOK403100

#### COOL FOAM, 61 CM (24"), BAG

Size	Part#
XL	FPOK403101
Regular	FPOK4031001



## FORMFIT® KNEE IMMOBILISER

### PRODUCT DESCRIPTION:

Formfit Knee Immobiliser features a universal 3-panel design that can be adjusted to accommodate most leg circumferences. Adjustable medial and lateral stays and a rigid posterior stay provide stability, while the breathable foam material helps keep patients cool.

### INDICATIONS FOR USE

- Traumatic knee injuries
- Emergency immobilisation
- Pre and post-surgical knee immobilisation

### FEATURES AND BENEFITS

- Adjustable medial/lateral stays and rigid posterior stay provide stability
- Soft breathable foam keeps patients cool
- Universal 3-piece design simplifies inventory needs
- 20" model is MRI Compatible

### REQUIRED MEASUREMENTS

- Universal - fits leg inseams of 46-61cm



### ORDERING INFORMATION

Length	Part#
12"	200009
14"	200010
16"	200016
18"	200118
20"	200020
22"	200022
24"	200030

### ACCESSORIES

Description	Part#
Hot/Cold Therapy Gel Pad	GB7
PCL Pad	B-705000036



## ÖSSUR FORMFIT® PRO KNEE

Össur Formfit® Pro Knee is a compressive knee sleeve that features a SmartPad™ buttress for targeted support and cushioning. Our patented design realigns the patella, suitable for conditions such as Patella Tendonitis (Jumper's Knee) and Chondromalacia Patella. The dynamic, 3-D MotionTech™ knit is supportive, meets certified medical grade compression requirements and has been proven to be superior to the leading brands at moisture wicking and breathability, keeping your knee cool during wear. Lightweight, removable spiral stays ensure optimum shape and fit, while distinct patella graphics make donning intuitive.



MotionTech™



CoolVent™



StayBilizer™



SmartPad™

### INDICATIONS FOR USE

For knee conditions that may benefit from compression, such as;

- Patella tendonitis
- Chondromalacia Patella
- Mild sprains and strains
- Knee pain
- Feeling of instability


### CONTRAINDICATIONS

- Must not be used by individuals for whom compression is contraindicated.


### FEATURES AND BENEFITS

- MotionTech™ for compression and dynamic fit with superior moisture wicking and breathability, keeping your knee cool during wear
- Certified medical grade compression (20-30 mmHg)
- Lightweight CoolVent™ knit in popliteal area for added breathability and comfort
- Patented SmartPad™ for targeted support and cushioning
- Patella graphics for intuitive placement
- Patented FoldLock™ system for easier and more secure application
- Lightweight, removable spiral steel stays

### REQUIRED MEASUREMENTS

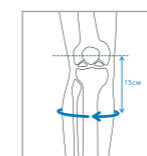

Measure calf circumference 15 cm (6") below mid-patella. Leg in extension.

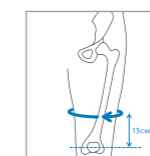

Measure thigh circumference 15 cm (6") above mid-patella. Leg in extension.

Size	Calf circumference 15 cm (6") below mid patella		Thigh circumference 15 cm (6") above mid patella	
	cm	inch	cm	inch
XS	26 - 31	10¼ - 12¼	36 - 41	14½ - 16½
S	31 - 36	12¼ - 14½	41 - 46	16½ - 18½
M	36 - 41	14½ - 16½	46 - 51	18½ - 20½
L	41 - 46	16½ - 18½	51 - 56	20½ - 22
XL	46 - 50	18½ - 19¾	56 - 60	22 - 23½

# ÖSSUR FORMFIT® PRO KNEE

## ORDERING INFORMATION

BLACK		BLUE	
Size	Part#	Size	Part#
XS	FP100001	XS	FP101001
S	FP100002	S	FP101002
M	FP100003	M	FP101003
L	FP100004	L	FP101004
XL	FP100005	XL	FP101005



# FORMFIT® HINGED LATERAL J

## PRODUCT DESCRIPTION:

Formfit Knee Hinged Lateral J is a sleeve design knee brace with a breathable Lycra-material and open popliteal area with two removeable hinges and J-shaped buttress providing support to the patella. The knee brace is designed for patients that would benefit from realignment of the patella, and/or patients that would benefit from an added lateral support of the patella.



## INDICATIONS FOR USE

Patients that would benefit from realignment of the patella, and/or patients that would benefit from an added lateral support of the patella.

- Lateral patellar subluxations and dislocations.
- Lateral patellofemoral malalignment.

## FEATURES AND BENEFITS

- Breathable material
- Removable hinges
- Easy pull-on
- Lateral J buttress
- Open popliteal area

## REQUIRED MEASUREMENTS

- Thigh Circumference: 6"/15cm above mid-patella
- Calf Circumference: 6"/15cm below mid-patella

## ORDERING INFORMATION

Size	Thigh circumference	Knee circumference	Calf circumference	Right leg	Left leg
X-Small	13 - 15.5" (33 - 39.4cm)	12-13" (30.5-33cm)	10-12" (25.4-30.5cm)	709152	709162
Small	15.5-18.5" (39.4-47cm)	13-14" (33-35.6cm)	12-14" (30.5-35.6cm)	709154	709164
Medium	18.5-21" (47-53.3cm)	14-15" (35.6-38.1cm)	14-16" (35.6-40.6cm)	709155	709165
Large	21-23.5" (53.3-59.7cm)	15-17" (38.1-43.2cm)	16-18" (40.6-45.7cm)	709157	709167
X-Large	23.5-26.5" (59.7-67.3cm)	17-19" (43.2-48.3cm)	18-20" (45.7-50.8cm)	709158	709168
XX-Large	26.5-29.5" (67.3-74.9cm)	19-21" (48.3-53.3cm)	20-22" (50.8-55.9cm)	709159	709169

## FORMFIT® KNEE HINGED

The Formfit Knee brace is for patients suffering from mild to moderate strains, sprains or ligament tears (ACL/PCL/MCL/LCL). It provides excellent stability, protection and compression during injury recovery. Made from strong and breathable materials it offers comfort while remaining highly durable. The open popliteal area reduces material bulging and potential irritation, while the malleable rigid hinge arms provide an intimate fit.

The Formfit Knee brace comes in either a wrap or a sleeve version.

### INDICATIONS FOR USE

- Mild to moderate ACL and PCL tears
- MCL and LCL tears
- Combined mild knee sprains and strains
- Instabilities of the knee

### FEATURES AND BENEFITS

- Breathable material
- Easy pull-on
- Malleable hinges
- Universal buttress
- Open popliteal area

### REQUIRED MEASUREMENTS

- Knee circumference at mid-patella

### ORDERING INFORMATION

Size	Knee Circumference	Sleeve	Wrap
XSmall	33cm to 39cm	504252	
Small	39cm to 47cm	504253	503253
Medium	47cm to 53cm	504255	503255
Large	53cm to 60cm	504257	503257
XLarge	60cm to 67cm	504258	503258
XXLarge	67cm to 75cm	504259	503259
XXXLarge	75cm to 81cm		503260



## FORMFIT® KNEE MCL

The Formfit Knee MCL provides external stability and protection for low to medium contact/impact activities and while recovering from injury. A unique, single hinged, hybrid wrap/sleeve design sets this brace apart from the competition. The lateral, range of motion hinge keeps the medial side free of obstruction. The Dynamic Force Strap aids in the resistance of valgus forces during knee extension, while the adjustable tibial strap improves suspension by minimising slippage.

### INDICATIONS FOR USE

- Mild to Moderate Medial Collateral Ligament (MCL) sprains (Grade I and Grade II)

### FEATURES AND BENEFITS

- Single, steel ROM lateral hinge (0° or 20° extension stop, 70° flexion stop)
- Dynamic Force Strap resist valgus forces
- Hybrid wrap/sleeve for easy application
- Adjustable tibial strap for improved suspension

### REQUIRED MEASUREMENTS

- Knee circumference at mid-patella

### ORDERING INFORMATION

Size	Thigh Circumference	Formfit Knee MCL Left	Formfit Knee MCL Right
XSmall	13-15.5"/33 - 39cm	B-237600001	B-237500001
Small	15.5-18.5"/39 - 47cm	B-237600002	B-237500002
Medium	18.5-21"/47 - 53cm	B-237600003	B-237500003
Large	21-23.5"/53 - 60cm	B-237600004	B-237500004
XLarge	23.5-26.5"/60 - 67cm	B-237600005	B-237500005



## FORMFIT® KNEE ROM BRACE

### INDICATIONS FOR USE

- Mild to moderate ACL and PCL tears
- MCL and LCL tears
- Combined mild knee sprains and strains
- Instabilities of the knee

### FEATURES AND BENEFITS

- With 6 options to choose from, you can confidently meet the unique needs of any patient
- When you need to limit range-of-motion, our ROM hinge provides the control you need. When you don't, you can opt for a more streamlined, polycentric hinge
- ROM braces available in 12" or 16" Length
- A removable, universal buttress is included with every sleeve version of the Formfit Knee Brace for additional, customisable patella support, if necessary

### REQUIRED MEASUREMENTS

- Knee circumference at mid-patella

### ORDERING INFORMATION

Size	Knee Circumference	Sleeve Short	Wrap Short	Sleeve Long	Wrap Long
XSmall	33cm to 39cm	508252	507252	508352	507352
Small	39cm to 47cm	508253	507253	508353	507353
Medium	47cm to 53cm	508255	507255	508355	507355
Large	53cm to 60cm	508257	507257	508357	507357
XLarge	60cm to 67cm	508258	507258	508358	507358
XXLarge	67cm to 75cm	508259	507259	508359	507359
XXXLarge	75cm to 81cm		507260		507360



## NEOPRENE HINGED KNEE SUPPORT

### INDICATIONS FOR USE

For patients suffering from knee conditions that may benefit from additional knee support, such as:

- Patellar tendonitis
- Subluxation dislocation
- Post-operation rehabilitation
- Post arthroscopic surgery
- Chondromalacia
- Bursitis
- Arthritis
- Tendonitis

### FEATURES AND BENEFITS

- Dual action hinges are moulded into the brace to maintain a low profile
- Accordion molding in the popliteal area reduces bunching behind the knee
- Lycra liner offers a breathable contact area for comfortable and extensive use
- Varied areas of thickness give additional protection

### REQUIRED MEASUREMENTS

- Knee circumference at mid-patella

### ORDERING INFORMATION

Size	Knee circumference	Part#
Small	29 - 33cm	302153BLK
Medium	33 - 37cm	302155BLK
Large	37 - 41cm	302157BLK
X-Large	41 - 44cm	302158BLK
XX-Large	44 - 48cm	302159BLK



## NEOPRENE WRAPAROUND HINGED KNEE SUPPORT

### INDICATIONS FOR USE:

For patients with unstable or injured knees that may benefit from additional knee support, such as: post-operation rehabilitation, tendonitis, bursitis, arthritis, sprains and strains.

### FEATURES AND BENEFITS

- Dual action hinges are removable for easy cleaning
- Open popliteal area eliminates bunching and increases comfort
- Front contact closures allow for hassle-free application
- Wraparound style accommodates all leg shape variations

### REQUIRED MEASUREMENTS

- Knee circumference at mid-patella



### ORDERING INFORMATION

Size	Knee circumference	Part#
Small	33 - 35.5cm	212003
Medium	35.5 - 38cm	212005
Large	38 - 40.5cm	212007
X-Large	40.5 - 43cm	212008
XX-Large	43 - 45.5cm	212009
XXX-Large	45.5 - 58.5cm	212010
XXXX-Large	58.5 - 63.5cm	212011

## NEOPRENE KNEE SLEEVE

A neoprene sleeve-design knee support providing compression and warmth

### INDICATIONS FOR USE

- For patients suffering from weak and/or injured knees that may benefit from additional knee support.

### FEATURES AND BENEFITS

- Unique accordion design enhances comfort by eliminating bunching in the popliteal area
- Open or closed patella

### REQUIRED MEASUREMENTS

- Knee circumference at mid-patella



### ORDERING INFORMATION

Size	Circumference	0.3cm Neoprene		0.6cm Neoprene	
		Open Patella	Closed Patella	Open Patella	Closed Patella
Small	29 - 33cm	402013BLK	412013BLK	302013BLK	312013BLK
Medium	33 - 37cm	402015BLK	412015BLK	302015BLK	312015BLK
Large	37 - 41cm	402017BLK	412017BLK	302017BLK	312017BLK
X-Large	41 - 44cm	402018BLK	412018BLK	302018BLK	312018BLK
XX-Large	44 - 48cm	402019BLK	412019BLK	302019BLK	312019BLK

FOOT AND ANKLE

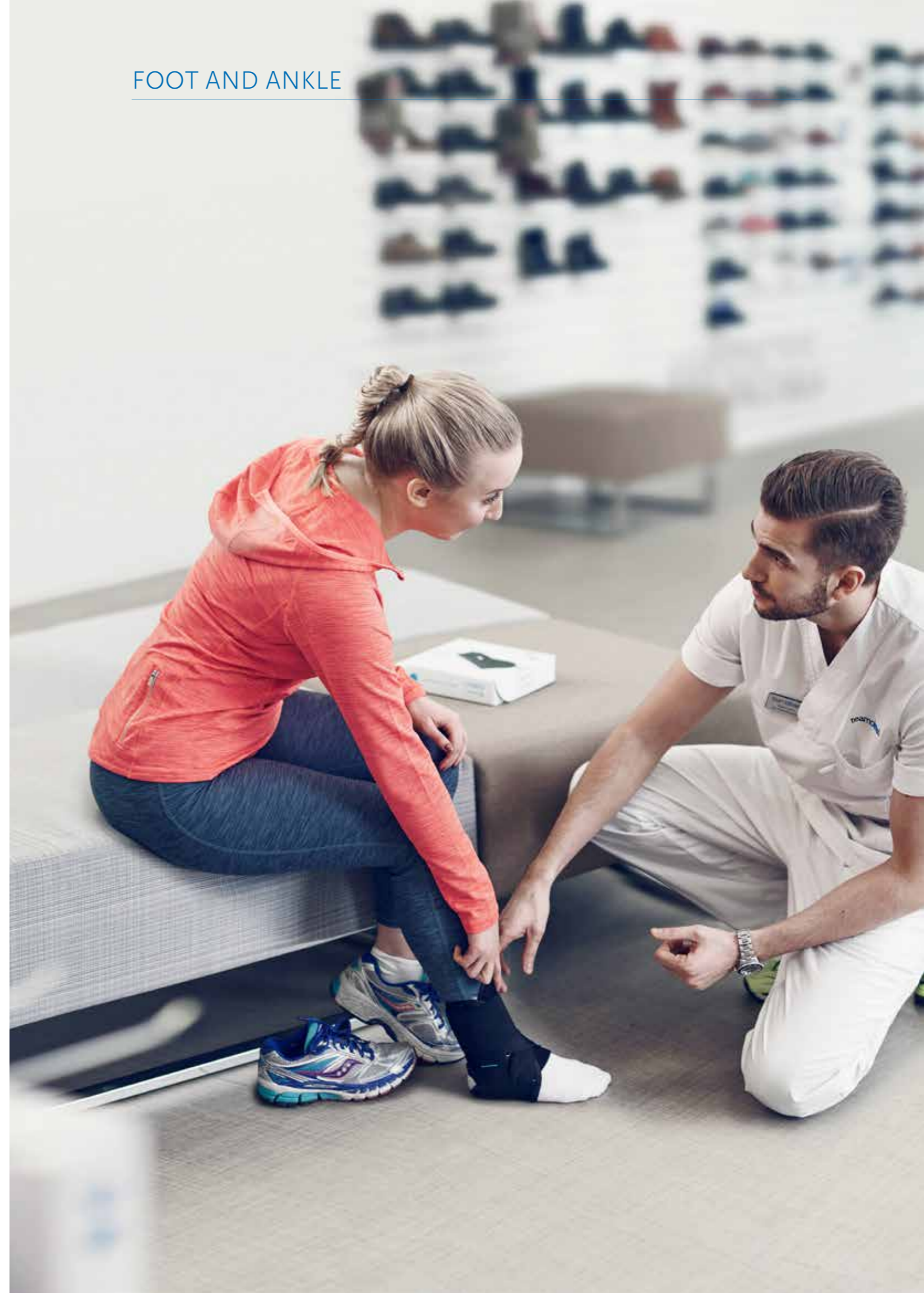


TABLE OF CONTENTS

FOOT AND ANKLE ..... 91

REBOUND® DIABETIC WALKER ..... 93

DH OFFLOADING WALKER® ..... 95

REBOUND® AIR WALKER ..... 96

FORMFIT® WALKER AIR ..... 98

FORMFIT® WALKER ..... 99

PAEDIATRIC WALKER ..... 100

AFO DYNAMIC ..... 101

AFO LIGHT ..... 103

AFO LEAF SPRING ..... 104

REBOUND FOOT-UP® ..... 105

FOOT-UP® ..... 106

ÖSSUR FORMFIT® PRO ANKLE ..... 107

FORMFIT® ANKLE BRACE ..... 109

AIRFORM® UNIVERSAL INFLATABLE ANKLE STIRRUP ..... 110

FORMFIT® NIGHT SPLINT ..... 111

DH OFFLOADING POST-OP STYLE SHOE ..... 112

MESH TOP POST-OP SHOE ..... 113

SOFT TOP POST-OP SHOE ..... 113

REBOUND® DIABETIC WALKER

Offloading has proven to be a key factor in foot ulcer treatment and entails reducing pressure of the plantar area in general and the ulcer site in particular.

The Rebound Diabetic Walker is uniquely designed for effective treatment of plantar foot ulcers, with easy access and re-fitting during wound care, while improving the patient's mobility during the healing process.

The Rebound Diabetic Walker has major advantages over diabetic shoes and shoe inserts in its healing effectiveness and over total contact casting due to its ease of use, speed of application and low cost.

FUNCTIONAL HEALING - WHERE MOBILITY MEANS RECOVERY



INDICATIONS FOR USE

- Plantar foot ulcers and wound prevention

FEATURES AND BENEFITS

- Effective tri-laminate insole with removable shock-absorbing hexagons that promote ulcer offloading and a Plastazote, heat-moldable top layer that evenly distributes the pressure and minimises friction
- Unique pressure relief valve for preventing over-inflation, while balancing compression with customised fit
- Positioning Portals allow the clinician to check the fit of the walker at the heel bottom, in order to ensure appropriate positioning of the foot in the walker
- Anatomically designed shell that minimises shear forces by immobilising the foot and ankle at 90 degrees while providing protection
- Flex Edge® technology for relief of pressure points
- Rocker Bottom outsole for a smooth roll over while allowing for individual natural toe out
- Soft and breathable, low friction, machine washable liner for hygiene promotion and prevention of infection
- Tamper evident strapping for compliance monitoring

REQUIRED MEASUREMENTS

- Shoe size



## REBOUND® DIABETIC WALKER

### ORDERING INFORMATION

#### REBOUND DIABETIC WALKER

Size	Shoe size UK	Shoe size EU	Part#
Small	4 - 6.5	36 - 39	B-242500002
Medium	7 - 10	39.5 - 44	B-242500003
Large	10.5 - 12	44.5 - 46.5	B-242500004
X-Large	12.5 - 14.5	47 - 50	B-242500005

### ACCESSORIES

#### REPLACEMENT SOFTGOODS

Size	Liners	Insoles
Small	B-418500102	B-418500202
Medium	B-418500103	B-418500203
Large	B-418500104	B-418500204
X-Large	B-418500105	B-418500205

#### WALKER COVERS

Size	1. Hygiene covers (2 pcs)	2. Durable weather cover	3. Disposable weather cover (20 pcs)
Small	B-418500602	B-418500502	B-418500512
Medium	B-418500603	B-418500503	
Large	B-418500604	B-418500504	B-418500514
X-Large	B-418500605	B-418500505	



#### DIABETIC SOCKS (2 PCS)

Size	Part#
Small, Medium	B-418500703
Large, X-Large	B-418500704

#### COMPLIANCE STRAPS (5 PCS)

Size	Part#
Universal	B-418500300

#### EVENUP

Size	Part#
Small	10350
Medium	10351
Large	10351



## DH OFFLOADING WALKER®

Assists in the healing of plantar ulcers with a patented pressure relief insole

#### INDICATIONS FOR USE

- Treatment of plantar ulcers

#### FEATURES AND BENEFITS

- Shear force reduction by stabilising foot and ankle at 90°
- Wide low profile rocker bottom provides a smooth roll-through and increased stability for altered gaits
- Designed around Össur's unique Active Offloading Insole
- The softgood's patented inner bootie prevents the foot from migrating on the insole
- Provides a safe environment for the simultaneous use of wound care material
- Create an Instant TCC using a single roll of cast tape for patient compliance

#### REQUIRED MEASUREMENTS

- Shoe size



### ORDERING INFORMATION

Size	Shoe Size UK	Shoe Size EU	Part#
X-Small	1.5 - 3.5	33 - 35.5	DH0200BLK
Small	4 - 6.5	36 - 39	DH0400BLK
Medium	7 - 10	39.5 - 44	DH0600BLK
Large	10.5 - 12	44.5 - 46.5	DH0800BLK
X-Large	12.5 - 14.5	47 - 50	DH1000BLK

### ACCESSORIES

#### REPLACEMENT PARTS

Size	Softgoods	Insoles
X-Small		11742C
Small	10958	PP04C
Medium	10959	PP06C
Large	10960	11740C
X-Large		11741C



## REBOUND® AIR WALKER

Rebound Air Walker is a circumferential walker with fully integrated air pump and release valve that provides easy, tool-free inflation and deflation for fine-tuned, consistent compression. It features a host of details designed to provide added protection, simplified application and usage.



### INDICATIONS FOR USE

Foot and ankle conditions that may benefit from motion restriction, such as:

- Grade 2 and 3 ankle sprains
- Stable fractures
- Tendon or ligamentous sprain
- During postoperative or rehab phase. If used with the Achilles Wedge Kit, Achilles ruptures can be treated as well.
- Rebound Air Walker Low Top
- Foot and ankle conditions that may benefit from motion restriction, such as:
- Stable foot fractures
- Tendon or ligamentous sprain
- During postoperative or rehab phase.

### FEATURES AND BENEFITS

- Linked medial and lateral air bladders with a single inflation point provide balanced compression and are shaped to cradle the Achilles, holding the heel in place
- Rocker bottom promotes a natural, stable gait, while a trimmable, semi-soft toe bumper provides added protection
- Moisture-wicking softgood helps keep skin dry and comfortable
- Straps fasten in front for easy application, adjustment and removal
- EVA material provides greater shock absorption in the heel
- Fully-integrated air pump and release valve provides easy, tool-free inflation and deflation for fine-tuned, consistent compression
- Flex Edge™ overmolding conforms to each patient's unique anatomy to help limit the formation of pressure points
- Ventilated panels wick moisture to prevent microbial buildup and improve patient comfort

### REQUIRED MEASUREMENTS

- Shoe size



## REBOUND® AIR WALKER

### ORDERING INFORMATION

#### REBOUND AIR WALKER

Size	Shoe Size UK	Shoe Size EU	Standard Liner / Standard	Standard Liner / Low Top	Slim Liner / Standard	Slim Liner / Low Top
X-Small	1.5 - 3.5	33 - 35.5	B-242900001E	B-242900061E	B-242900001SE	B-242900061SE
Small	4 - 6.5	36 - 39	B-242900002E	B-242900062E	B-242900002SE	B-242900062SE
Medium	7 - 10	39.5 - 44	B-242900003E	B-242900063E	B-242900003SE	B-242900063SE
Large	10.5 - 12	44.5 - 46.5	B-242900004E	B-242900064E	B-242900004SE	B-242900064SE
X-Large	12.5 - 14.5	47 - 50	B-242900005E	B-242900065E	B-242900005SE	B-242900065SE

### ACCESSORIES

#### REPLACEMENT SOFTGOODS

Size	Standard Liner / Standard	Standard Liner / Low Top	Slim Liner / Standard	Slim Liner / Low Top
X-Small	B-418000101	B-418000161	B-418030101	B-418030161
Small	B-418000102	B-418000162	B-418030102	B-418030162
Medium	B-418000103	B-418000163	B-418030103	B-418030163
Large	B-418000104	B-418000164	B-418030104	B-418030164
X-Large	B-418000105	B-418000165	B-418030105	B-418030165

#### ACHILLES HEEL WEDGE KIT

Size	Description	Part#
Small	Wedge kit for walkers XS-S	AWK0001
Medium	Wedge Kit for walkers M	AWK0005
Large	Wedge kit for walkers L-XL	AWK0009
N/A	5 layer Wedge kit	AWK00015

#### EVENUP

Size	Part#
Small	10350
Medium	10351
Large	10351

#### WALKER COVERS

Size	Hygiene covers (2 pcs)	Durable weather cover	Disposable weather cover (20 pcs)
Small	B-418500602	B-418500502	B-418500512
Medium	B-418500603	B-418500503	
Large	B-418500604	B-418500504	B-418500514
X-Large	B-418500605	B-418500505	

#### OTHER ACCESSORIES

Description	Part#
Heel Wedge	B-242902511
Adhesive Insole for Wedge	WHW-W0107
Strap Extension	B-502100101
Rebound Hightop Sock SM/MD SM/MD	B-418500703
Rebound Hightop Sock LG/XL LG/XL	B-418500704
Rebound Lowtop Sock SM/MD SM/MD	B-418500903
Rebound Lowtop Sock LG/XL LG/XL	B-418500904

## FORMFIT® WALKER AIR

The Formfit Walker Air provides immobilisation for patients suffering from soft tissue injury; grades 2 & 3 strains and sprains, and stable fractures. The patented pneumatic system is designed to increase stability while decreasing pain and swelling. The intuitive pump and clearly marked decompression valve make air/pressure adjustments easy.

### INDICATIONS FOR USE

For foot and ankle conditions that may benefit from motion restriction, such as:

- Grade 2 and 3 ankle sprains
- Stable fractures
- Soft tissue injuries
- During postoperative or rehab phase

### FEATURES AND BENEFITS

- Available in low top or standard height
- Integrated pneumatic system
- Low, wide rocker bottom promotes a stable, natural gait
- Shock absorbing sole for reduced impact at heel strike
- Breathable, seamless liner

### REQUIRED MEASUREMENTS

- Shoe size

## ORDERING INFORMATION

Size	Shoe Size UK	Shoe Size EU	Standard	Low Top
X-Small	1.5 - 3.5	33 - 35.5	A-W0200BLK	A-W0300BLK
Small	4 - 6.5	36 - 39	A-W0400BLK	A-W0500BLK
Medium	7 - 10	39.5 - 44	A-W0600BLK	A-W0700BLK
Large	10.5 - 12	44.5 - 46.5	A-W0800BLK	A-W0900BLK
X-Large	12.5 - 14.5	47 - 50	A-W1000BLK	A-W1100BLK

## ACCESSORIES

### REPLACEMENT SOFTGOODS

Size	Standard	Low Top
X-Small	12349	12356
Small	10675	12357
Medium	10678	12358
Large	10681	12359
X-Large	12347	12360

### OTHER ACCESSORIES

Description	Part#
10cm softgood width extension	WKSG-1BLK
Tibia and calf plates for use when additional splinting is required	WKSP100
Hot/Cold therapy insert for safe and reusable cryotherapy	WKGEL100
Heel wedge	AWK0005
Adhesive insole for heel wedge	WHW-W0107

## FORMFIT® WALKER

The Formfit Walker provides immobilisation for patients suffering from soft tissue injury; grades 2 & 3 strains and sprains; and stable fractures.

### INDICATIONS FOR USE

For foot and ankle conditions that may benefit from motion restriction, such as:

- Grade 2 and 3 ankle sprains
- Stable fractures
- Soft tissue injuries
- During postoperative or rehab phase

### FEATURES AND BENEFITS

- Available in low top or standard height
- Low, wide rocker bottom promotes a stable, natural gait
- Shock absorbing sole for reduced impact at heel strike
- Breathable, seamless liner

### REQUIRED MEASUREMENTS

- Shoe size

## ORDERING INFORMATION

Size	Shoe Size UK	Shoe Size EU	Standard	Low Top
X-Small	1.5 - 3.5	33 - 35.5	W0200BLK	W0300BLK
Small	4 - 6.5	36 - 39	W0400BLK	W0500BLK
Medium	7 - 10	39.5 - 44	W0600BLK	W0700BLK
Large	10.5 - 12	44.5 - 46.5	W0800BLK	W0900BLK
X-Large	12.5 - 14.5	47 - 50	W1000BLK	W1100BLK

## ACCESSORIES

### REPLACEMENT SOFTGOODS

Size	Standard	Low Top
X-Small	WKSG200BLK	WKSG0300BLK
Small	WKSG400BLK	WKSG500BLK
Medium	WKSG600BLK	WKSG700BLK
Large	WKSG800BLK	WKSG900BLK
X-Large	WKSG1000BLK	WKSG1100BLK

### OTHER ACCESSORIES

Description	Part#
10cm softgood width extension	WKSG-1BLK
Tibia and calf plates for use when additional splinting is required	WKSP100
Hot/Cold therapy insert for safe and reusable cryotherapy	WKGEL100
Heel wedge	AWK0005
Adhesive insole for heel wedge	WHW-W0107



## PAEDIATRIC WALKER

Paediatric Walkers are designed to stabilise and protect the lower leg, ankle and foot. They are easy to operate and comfortable to wear, making them ideal for an active child who requires stability following a sprain, stress fracture, or surgery.

### INDICATIONS FOR USE

- Soft tissue injuries
- Grade 2 and 3 sprains
- Stable fractures
- Post-operative ankle stabilisation
- Trauma and rehab use

### FEATURES AND BENEFITS

- High quality rigid outer skeleton for protection
- Dense foam lining within the sole to provide comfort and support
- Shock absorbing sole to improve comfort and protects the user from slips, providing additional security in each step
- Uprights give additional stability by restricting motion of the leg
- Lightweight design perfectly designed for kids to keep active

### REQUIRED MEASUREMENTS

- Shoe size



## ORDERING INFORMATION

### PAEDIATRIC WALKER

Size	Shoe Size	Part#
Small	Child 6.5 - 9.5	PEW0400
Medium	Child 10 - Youth 1	PEW0600
Large	Adult 1.5 - 2.5	PEW0800

### REPLACEMENT LINERS

Size	Shoe Size	Part#
Small	Child 6.5 - 9.5	PEW0001
Medium	Child 10 - Youth	PEW0002
Large	Adult 1.5 - 2.5	PEW0003

## AFO DYNAMIC

The AFO Dynamic provides a strong, lightweight solution for patients with varying degrees of drop foot secondary to a variety of neurological conditions. Össur has taken the carbon fibre technology from its Flex-Foot® product and incorporated it into the new AFO Dynamic to provide increased support over traditional ankle foot orthoses. The energy storing properties of carbon fibre make it an ideal material for this dynamic ankle foot orthosis because it provides real energy return. Best for those people leading busy lives, the AFO Dynamic will ensure a more efficient walk for the user.



### INDICATIONS FOR USE

Drop foot secondary to:

- Cerebro Vascular Accident (CVA)
- Other neurological conditions
- Multiple Sclerosis
- Mild knee instability
- Charcot Marie Tooth disease (HMSN)

Contraindications:

- Severe ankle - foot deformities
- Fluctuating oedema
- Severe spasticity
- Neuropathy

### FEATURES AND BENEFITS

- Carbon fibre design provides excellent strength-to-weight ratio and dynamic motion
- The tapered carbon heel stores energy and manages forces at heel strike
- Full-length toe lever provides full support while foot is loaded, as well as reduced shock to the contralateral limb
- Creates medio-lateral stability at mid-stance
- Reduces energy consumption by reducing hip hike
- Medial strut makes it less visible in shoe
- Open anterior tibia shell provides superior anatomical fit, and reduces heat and perspiration
- Weight limit: 120 kg

### REQUIRED MEASUREMENTS

- Shoe size
- Calf height at widest part of the calf



## AFO DYNAMIC

### ORDERING INFORMATION

#### AFO DYNAMIC

Size	Calf Height	Shoe Size UK		Shoe Size EU		Left	Right
		Male	Female	Male	Female		
X-Small	30 - 34cm	2.5 - 4.5	< 4	34 - 37	< 37	O-102021	O-102011
Small	32 - 36cm	5 - 7	4.5 - 6.5	37.5 - 40	37.5 - 40	O-102022	O-102012
Medium	34 - 38cm	7.5 - 10	7 - 9.5	40.5 - 44.5	40.5 - 44	O-102024	O-102014
Large	36 - 40cm	10.5 - 13.5		45 - 48.5		O-102026	O-102016

### ACCESSORIES

#### ACCESSORY KITS

Size	Part#
X-Small	O-102101
Small	O-102102
Medium	O-102104
Large	O-102106

Note: Accessory kits come with foam padding, upper and lower Hook & Loop straps, and Hook & Loop dots.

## AFO LIGHT

AFO Light provides support only where it is needed, eliminating unnecessary bulk and providing a cosmetically-appealing look. Using the best of Flex-Foot® technology and the energy storing properties of carbon fibre, AFO Light offers a strong and lightweight solution for people with mild to moderate drop foot.



### INDICATIONS FOR USE

Drop foot secondary to:

- Cerebro Vascular Accident (CVA)
  - Other neurological conditions
  - Multiple Sclerosis
  - Mild knee instability
  - Charcot Marie Tooth disease (HMSN)
  - Lower extremity overuse injuries
- Contraindications:
- Severe ankle - foot deformities
  - Fluctuating edema
  - Severe spasticity
  - Neuropathy

### FEATURES AND BENEFITS

- Energy return and propulsion assistance of the limb at toe off
- Supports the foot for ground clearance during swing phase
- Carbon fibre design provides excellent strength-to-weight ratio and dynamic motion
- Gradual plantar flexion at initial contact
- Stiffness gradient at heel and the toe correspond to ground reaction forces as they are applied to the foot
- Easy to don and doff
- Weight limit: 100kg

### REQUIRED MEASUREMENTS

- Shoe size
- Calf height at widest part of the calf

### ORDERING INFORMATION

#### AFO LIGHT

Size	Calf height	Shoe Size UK		Shoe Size EU		Left	Right
		Male	Female	Male	Female		
Small	32 - 36cm	< 7	4.5 - 6.5	< 40	37.5 - 40	O-121022	O-121012
Medium	34 - 38cm	7.5 - 10	7 - 9.5	40.5 - 44.5	40.5 - 44	O-121024	O-121014
Large	36 - 40cm	10.5 - 13.5		45 - 48.5		O-121026	O-121016
X-Large	38 - 42cm	11.5 - 14.5		46 - 50		O-121028	O-121018

### ACCESSORIES

Description	Part#
Refurb Kit	O-122101

## AFO LEAF SPRING

AFO Leaf Spring is a pre-fabricated polypropylene ankle-foot orthosis designed to support flaccid drop foot. It provides a semirigid section for toe clearance and support, yet has a thin, flexible foot plate for easy trimming. The absence of a heel section makes the Leaf Spring more comfortable to wear and provides a better fit in shoes.

### INDICATIONS FOR USE

- Drop foot
- Mild drop foot secondary to other neurological pathologies

### FEATURES AND BENEFITS

- Lightweight injection-molded polypropylene
- Thin, flexible foot part which is easy to trim with scissors
- Absence of heel section for improved comfort

### REQUIRED MEASUREMENTS

- Shoe size



## ORDERING INFORMATION

Size	Shoe Size US	Shoe Size UK	Height	Left	Right	Foot Plate Length
Small-Medium	7.5-10.5	3.5-6.5	10.75" (27cm)	O-110227	O-110127	24cm
Large	11-14	7-10.5	12" (30cm)	O-110230	O-110130	28cm

## REBOUND FOOT-UP®

Rebound Foot-Up is a lightweight ankle/foot orthosis (AFO) that offers dynamic support for drop-foot or related disorders that require dorsiflexion support. The Rebound Foot-Up supports the ankle dorsiflexion during swing phase. Dynamic support is provided by an elastic strap connected to the shoe by either a transparent plastic inlay placed between shoelaces, or by a nut and bolt fastened through the eyelets. The positioning of the fastening allows for individualising the support by providing the option of positioning the dorsiflexion support on the medial or lateral side. When using the plastic inlay, the dorsiflexion support becomes neutral.

The Rebound Foot-Up Ankle Cuff has a smart clamp that holds the cuff in place and facilitates one-handed donning and doffing.

Optional: Rebound Foot-Up Foot Wrap can be used when barefoot or in open shoes or sandals. The adjustable elastic strap is attached to the Foot Wrap using a nut and bolt fastened through the eyelets.



### INDICATIONS FOR USE


- Drop-foot
- Disorders that require dorsiflexion support

### FEATURES AND BENEFITS

- Dynamic and discreet support for drop-foot or related disorders that require dorsiflexion support
- Improved function, ease of use, comfort, and quality
- Allows for voluntary plantar flexion
- The elastic strap is adjustable and easily replaceable
- The Rebound Foot-Up Ankle Cuff and Rebound Foot-Up Foot Wrap are made from breathable material that can be worn comfortably for long periods of time
- The Rebound Foot-Up is hand-washable
- Positioned medial or lateral support
- Contoured to the shape of leg
- Adjustable strap length
- Smart clamp facilitates one-handed donning / doffing
- Optional Rebound Foot-Up Foot Wrap can be used without shoes or in open shoes or sandals

### REQUIRED MEASUREMENTS

- Circumference of ankle, measured around the malleoli

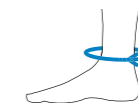

## ORDERING INFORMATION

### ANKLE CUFF

Description	Size	Ankle Circumference	Part #
Rebound Foot-Up Ankle Cuff	S/M	18 - 25cm (7 - 10")	RFU50001
Rebound Foot-Up Ankle Cuff	L/LX	26 - 33cm (10 - 13")	RFU50002

### FOOT WRAP

Description	Size	Foot Circumference	Part #
Rebound Foot-Up Foot Wrap	One Size	20 - 28cm (8 - 11")	RFU50101

### ACCESSORIES

Description	Part #
Rebound Foot-Up Attachment Kit (includes plastic inlay, elastic strap, and 1 x nut and screw)	RFU50201
Rebound Foot-Up Replacement Nuts And Bolts (5 nuts included: 2 short and 3 long screws)	RFU50202

Description	Size	Foot Circumference	Part #
Rebound Foot-Up Bundle	S/M	18 - 25cm	RFU500011
Rebound Foot-Up Bundle	L/LX	26 - 33cm	RFU500021

## FOOT-UP®

Foot-Up is a lightweight AFO that offers dynamic support for drop foot or similar complaints. The orthosis provides visible improvement in gait by supporting the foot the moment it is raised.



### INDICATIONS FOR USE

- Drop foot
- Support of dorsiflexion

### FEATURES AND BENEFITS

- Streamlined, lightweight and invisible when worn
- Easy to fit
- Breathable material, padded and comfortable to wear
- Available in black and beige
- Attached to the shoe, or available as shoeless model

### REQUIRED MEASUREMENTS

- Basic version: Circumference of ankle, measured around the malleoli
- Shoeless version: Circumference of foot immediately behind the ball of the foot

### ORDERING INFORMATION

#### FOOT-UP

Size	Ankle Circumference	Beige	Black
Medium	18 - 21cm	07811 -1	07810-1
Large	22 - 26cm	07811 -2	07810-2
X-Large	27 - 33cm	07811 -3	07810-3

#### SHOELESS VERSION\*

Size	Foot Circumference	Beige	Black
Small	< 20cm	27811-1	27810-1
Medium	20 - 24cm	27811-2	27810-2
Large	24 - 28cm	27811-3	27810-3
X-Large	> 28cm	27811-4	27810-4

\*Shoeless version item comprises of foot wrap only and must be ordered with the Foot-Up unless the patient already has one.

### ACCESSORIES

Description	Beige	Black
Foot-Up plastic insert	17811	17810

Description	Beige	Black
Foot-Up with Shoeless add-on Bundle	07811xy	07810xy

x = size of main foot-up part  
 y = size of shoeless add-on  
 e.g. 0781012 = size 1 Foot-up with a size 2 Shoeless add-on in Black

## ÖSSUR FORMFIT® PRO ANKLE

Össur Formfit® Pro Ankle is a compressive ankle sleeve with 2 integrated malleoli pads, designed to be worn after mild ankle injuries. The dynamic 3-D MotionTech™ knit is supportive and meets certified medical grade compression requirements. Proven superior to leading brands for moisture wicking and breathability, your ankle stays cool during wear. CoolVent™ technology in the area in front of the ankle further enhances breathability and comfort.



MotionTech™



CoolVent™



SmartPad™



FoldLock™


### INDICATIONS FOR USE

- For ankle conditions that may benefit from compression, such as:
- Ankle injury rehabilitation
- Mild sprains and strains
- Mild instability

### CONTRAINDICATIONS

- Must not be used by individuals for whom compression is contraindicated.

### REQUIRED MEASUREMENTS

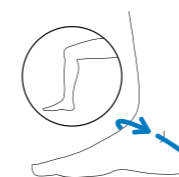

Measure ankle circumference diagonally over the base of the heel and across the ankle bone. Ankle in 90 degrees.

Size	Ankle circumference cm
XS	18 - 20
S	20 - 23
M	23 - 26
L	26 - 29
XL	29 - 32

### FEATURES AND BENEFITS

- MotionTech™ for compression and dynamic fit with superior moisture wicking and breathability, keeping your ankle cool during wear
- Certified medical grade compression (20-30 mmHg)
- CoolVent™ technology in the area in front of the ankle for added breathability and comfort
- 2 integrated malleoli SmartPads™ for targeted support and cushioning
- Patented FoldLock™ system for easier and more secure application

## ÖSSUR FORMFIT® PRO ANKLE

### ORDERING INFORMATION

BLUE			BLACK		
Size	Left	Right	Size	Left	Right
XS	FP201201	FP201101	XS	FP200201	FP200101
S	FP201202	FP201102	S	FP200202	FP200102
M	FP201203	FP201103	M	FP200203	FP200103
L	FP201204	FP201104	L	FP200204	FP200104
XL	FP201205	FP201105	XL	FP200205	FP200105



## FORMFIT® ANKLE BRACE

The Formfit Ankle brace provides stability and protection for the ankle – whether used prophylactically, for chronic instability or following an injury. A hybrid between a rigid ankle stirrup and a soft ankle support, the Formfit Ankle ensures a close fit which limits inversion, eversion, flexion and extension movements. The low profile design fits easily into shoes and the quick lace up feature makes it easy to don and doff. Breathable material ensures it can be comfortably worn all day. The Formfit Ankle with Figure 8 strap version, offers an even greater level of support, by further limiting the range of motion of the ankle.



#### INDICATIONS FOR USE


- Ankle injury rehabilitation/immobilisation (Grade I/II sprains)
- Support chronic instability
- Prophylactic use

#### FEATURES AND BENEFITS

- Breathable material
- Quick lace Up
- Inversion/Eversion Control
- Figure 8 strap version

#### REQUIRED MEASUREMENTS

- Measure ankle circumference diagonally over the base of the heel and across the ankle bone.

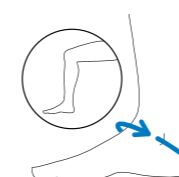

### ORDERING INFORMATION

Size	Ankle Circ.	Form Fit Ankle Brace	Form Fit Ankle with Figure 8
X-Small	28 – 30.5cm	B-212010001	B-212000001
Small	30.5 – 33cm	B-212010002	B-212000002
Medium	33 – 35.5cm	B-212010003	B-212000003
Large	35.5 – 38cm	B-212010004	B-212000004
X-Large	38 – 40.5cm	B-212010005	B-212000005

## AIRFORM® UNIVERSAL INFLATABLE ANKLE STIRRUP

### INDICATIONS FOR USE

- Mild to moderate ankle sprains
- Chronic ankle instability
- Post-cast and post walker support
- Ankle injury rehabilitation and/or post-surgical use

### FEATURES AND BENEFITS

- Universal design fits both left and right ankles, reducing inventory needs
- Soft Flex-Edge molding provides a comfort barrier between the shell and ankle
- Inflatable air bladders provides adjustable pneumatic compression
- Adjustable heel strap allow for easy width adjustments



### ORDERING INFORMATION

Size	White Single	White 6 pack	Black Single
Paediatric	80251	802516	
Adult	80250	802506	80350

## FORMFIT® NIGHT SPLINT

The Formfit Night Splint, designed to help relieve the pain associated with plantar fasciitis by providing a gentle overnight stretch. Straps can be adjusted from 0° up to 10° degrees as needed. The soft breathable liner and padded straps provide comfort and help keep patient cool. Straps adjust from 0°-10° dorsiflexion

### INDICATIONS FOR USE

Conditions that may benefit from resting with the ankle in a neutral position, such as:

- Plantar fasciitis
- Achilles tendinopathy

### FEATURES AND BENEFITS

- Straps adjust from 0°-10° of dorsiflexion
- Contoured for a comfortable fit
- Soft foam interior is washable and breathable quick release buckles for easy application
- Slip resistant tread

### REQUIRED MEASUREMENTS

- Shoe size



### ORDERING INFORMATION

Size	Shoe Size UK		Shoe Size EU		Standard with tread
	Male	Female	Male	Female	
Small	Up to 6.5	Up to 5	Up to 39	Up to 38	50023
Medium	7 - 9.5	5.5 - 8	39.5 - 44	38.5 - 43	50025
Large	10 and up	8.5 and up	44.5 and up	43.5 and up	50027

### ACCESSORIES

Description	Part#
Foam Wedge	10700
Replacement Liner S	11760
Replacement Liner M	11761
Replacement Liner L	11762



## DH OFFLOADING POST-OP STYLE SHOE

Designed for use in the treatment of plantar wounds, this shoe can be an excellent alternative to post-op style shoes with static pads (wool, felt, foam, etc.). It accommodates the ambulatory and compliant patient for short-term treatment (1 to 4 months).

### INDICATIONS FOR USE

- Plantar wounds
- Post-operative use
- Plantar foot ulcers

### FEATURES AND BENEFITS

- Designed to accommodate either left or right foot
- Designed around Össur's unique Active Offloading Insole
- Designed to accommodate bulky dressings
- Super soft nylon upper with reinforced heel counter provides extra comfort and stability
- Patented removable toe pieces aides in wound accessibility for daily wound site inspection

### REQUIRED MEASUREMENTS

- Shoe size



### ORDERING INFORMATION

Size	Shoe Size UK		Shoe Size EU		Part#
	Male	Female	Male	Female	
Small	3.5 - 5.5	4 - 6	35.5 - 38	37 - 39	10340
Medium	6 - 8	6.5 - 8.5	38.5 - 42	39.5 - 43.5	10341
Large	8.5 - 10.5	9 - 10	42.5 - 45	44 - 45	10342
X-Large	11 - 13.5	10.5+	45.5 - 48.5	45.5-48.5	10343

### ACCESSORIES

#### REPLACEMENT PARTS

Size	Insoles
Small	12365
Medium	12366
Large	12367
X-Large	12368

## MESH TOP POST-OP SHOE

### INDICATIONS FOR USE

- For recovery after soft tissue procedures and post trauma

### FEATURES AND BENEFITS

- Breathable mesh upper with reinforced heel counter and padded heel collar provides increased stability overall to the rear of the foot while reducing slippage
- Semi-rigid sole provides superior protection and control after osteotomies
- Extended strap and dual side closure system accommodates extra bulky dressings

### REQUIRED MEASUREMENTS

- Shoe size



### ORDERING INFORMATION

Size	Shoe Size UK		Shoe Size EU		Male	Female
	Male	Female	Male	Female		
Small	3.5 - 5.5	2.5 - 3.5	35.5 - 38	35 - 36	17013	17003
Medium	6 - 8	4 - 5.5	38.5 - 42	36.5 - 38.5	17015	17005
Large	8.5 - 10.5	6 +	42.5 - 45	39 +	17017	17007
X-Large	11 - 13.5		45.5 - 48.5		17018	

## SOFT TOP POST-OP SHOE

### INDICATIONS FOR USE

- Post-operative use or minor trauma wounds
- Geriatric or insensate foot

### FEATURES AND BENEFITS

- The cotton-poly blend upper allows the patient's foot to breathe and provides added comfort
- Designed specifically for recovery after soft tissue procedures post trauma
- The contact closure system provides even torsion and is designed for ease of use

### REQUIRED MEASUREMENTS

- Shoe size



### ORDERING INFORMATION

Size	Shoe Size UK		Shoe Size EU		Male	Female
	Male	Female	Male	Female		
Small	3.5 - 5.5	2.5 - 3.5	35.5 - 38	35 - 36	18013	18003
Medium	6 - 8	4 - 5.5	38.5 - 42	36.5 - 38.5	18015	18005
Large	8.5 - 10.5	6 +	42.5 - 45	39 +	18017	18007
X-Large	11 - 13.5		45.5 - 48.5		18018	

CASTING & TRACTION

---



TABLE OF CONTENTS

CASTING & TRACTION ..... 115  
 TECHFORM® PREMIUM ..... 117

TECHFORM® PREMIUM

Techform Casting Tape was designed by and for orthopaedic casting professionals who look for a superior handling experience and a smooth, strong finished cast. It incorporates a patented rub-responsive formula that works with our unique fabric to produce the best finished cast you'll ever see



INDICATIONS FOR USE

- Immobilisation

FEATURES AND BENEFITS

- Glides off the roll for easier application
- Soft, conformable rolls allow for easy handling
- “Rub-responsive” resin formula adapts to the technician's technique - the more it is rubbed, the smoother the cast gets
- Excellent lamination ensures a strong and durable cast
- Smoother finished cast is non-abrasive and will not snag on clothing
- Precision welded ends do not fray and lay down easily
- Finger holes in the pouch allow for easy opening

ORDERING INFORMATION

WIDTH	WH	BK	OR	RD	PK	HP	LB	DB	PR	GN	RB
1.5"	154WH										
2"	204WH	204BK	204OR	204RD	204PK	204HP	204LB	204DB	204PR	204GN	204RB
3"	304WH	304BK	304OR	304RD	304PK	304HP	304LB	304DB	304PR	304GN	304RB
4"	404WH	404BK	404OR	404RD	404PK	404HP	404LB	404DB	404PR	404GN	404RB
5"	504WH										

Note: Packed in boxes of 10

# NOTES

---