4R11 Quickchange Adapter

DE	Gebrauchsanweisung (Fachpersonal)	7
EN	Instructions for use (qualified personnel) ...	12
FR	Instructions d'utilisation (personnel spécialisé) ..	17
IT	Istruzioni per l'uso (personale tecnico specializzato) ..	22
ES	Instrucciones de uso (personal técnico especializado) ..	27
PT	Instruções de utilização (pessoal técnico) ..	33
NL	Gebruiksaanwijzing (deskundig personeel) ..	38
SV	Bruksanvisning (fackpersonal)	43
DA	Brugsanvisning (faguddannet personale) ...	48
NO	Bruksanvisning (fagpersonell)	53
FI	Käyttöohje (ammattihenkilöstö)	57
PL	Instrukcja używania (personel fachowy)	62
HU	Használati útmutató (szakszemélyzet)	67

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INFORMATION

Date of last update: 2026-03-31

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.

▶ Please keep this document for your records.

The 4R11 Quickchange adapter makes it possible to quickly change prosthetic components without tools. The adapter consists of a base and various connectors that are plugged together and locked into place.

2 Safety-related information

CAUTION!

Risk of injury and risk of product damage

- ▶ To prevent mechanical damage, treat the product with care.
- ▶ Observe the information on product combinations in the instructions for use for the products.
- ▶ Observe the maximum lifetime of the product (see page 13).
- ▶ Observe the product's operating conditions (see page 13).
- ▶ Do not expose the product to restricted environmental conditions (see page 13).
- ▶ Check the product for damage and readiness for use prior to each use.
- ▶ Do not use the product if it is damaged or if you are not sure whether it is working properly. Have the product inspected, repaired or replaced by the manufacturer or a specialist workshop.
- ▶ Keep the product away from devices and items that are sensitive to magnetic fields. (e.g. pacemakers, electronic devices, swipe cards). The magnets in the product may affect or damage other devices and objects.
- ▶ Pay attention to the manufacturer's information regarding magnetic fields (e.g. for a medical implant).

Signs of changes in or loss of functionality during use

Among other factors, changes in functionality can be indicated by an altered gait pattern, a change in the positioning of the prosthetic components relative to each other and by the development of noises.

3 Intended use

3.1 Indications for use

The product is to be used **solely** for lower limb exoprosthetic fittings.

3.2 Operating conditions

- The maximum approved body weight is specified in the technical data (see page 16).

3.3 Environmental conditions

Storage and transport

Storage temperature: -20 °C to +60 °C, relative humidity: 20 % to 90 %, no vibrations or impacts

Allowable environmental conditions

Temperature during use: -10 °C to +45 °C

Chemicals/liquids: Fresh water, salt water, perspiration, urine, soapsuds, chlorine water

Moisture: Submersion: max. 1 h in 3 m depth, relative humidity: no restrictions

Solids: Dust, occasional contact with sand

Clean the product after contact with humidity/chemicals/solids, in order to avoid increased wear and tear and damage (see page 16).

Prohibited environmental conditions

Solids: Highly liquid-binding particles (e.g. talcum), continuous contact with sand

Chemicals/liquids: Acids, continuous use in liquid media

3.4 Lifetime

This product was tested by the manufacturer with 3 million load cycles. Depending on the user's activity level, this corresponds to a maximum lifetime of 5 years.

Recreational sports: The maximum lifetime is 3 years.

Competitive sports: The lifetime is 1 year.

3.5 Product combinations

This prosthetic component is compatible with Ottobock's system of modular connectors. Functionality with components of other manufacturers that have compatible modular connectors has not been tested.

Limited combination options

- Combination with a rotation adapter (4R57, 4R57=WR):** Do not install the rotation adapter on the base of the Quickchange adapter.
- Connect to a pyramid of the modular system for children:** Insert the reducer pyramid receiver 4X907=1 into the pyramid receiver of the Quickchange adapter base.
- Foot adapter 2R40=1:** Do not connect the Quickchange adapter directly to the foot adapter.

4 Scope of delivery

Quantity	Designation	Reference number
1	Quickchange – base with locking ring	4R11=R-3-2
4	Set screw	506G3=M8x14
1	Instructions for use (qualified personnel)	647G1789=*
1	Instructions for use (user)	647H1089=*

- Provide the user with the instructions for use.

5 Accessories

Designation	Reference number
Quickchange connection: Pyramid	4R11=P-1-1
Quickchange connection: Pyramid receiver	4R11=R-1-1
Quickchange connection: Threaded connector, modular system	4R11=ST-1-1
Quickchange connection: Threaded connector, modular system for children	4R11=ST-2-1

Designation	Reference number
Reducer pyramid receiver	4X907=1

6 Preparing the product for use

⚠ CAUTION

Incorrect alignment or assembly

Risk of injury due to damaged prosthetic components

- Observe the alignment and assembly instructions.

⚠ CAUTION

Improper assembly of the screw connections

Risk of injury due to breakage or loosening of the screw connections

- Clean the threads before every installation.
- Apply the specified torque values.
- Follow the instructions regarding the length of the screws and about how to secure the screws.

INFORMATION

Observe the alignment of the prosthesis for all combinations.

- Position the adapter in the prosthesis in a way that makes it easy to adjust the structural height (e.g. close to the socket).
- Perform the "basic configuration," "static alignment" and "dynamic optimisation" steps for each prosthetic component combination.

Aligning the adapter

The base of the Quickchange adapter can be aligned up or down as required see fig. 2. The build height of the adapter (above/below the knee joint) determines the alignment of the balls in the locking mechanism.

> **Prerequisite:** The adapter is connected and locked into place.

- 1) **INFORMATION: A dot on the base indicates the position of each ball.**

Align the adapter according to the build height in the prosthesis:

- **Below the knee joint or a prosthetic knee joint:** Align two balls in the posterior and one ball (marked by a dot) in the **anterior** (see fig. 3).
 - **Above a prosthetic knee joint:** Align two balls in the anterior and one ball (marked by a dot) in the **posterior** (see fig. 4).
- 2) Carry out the prosthetic alignment.
 - 3) **For assembly above a prosthetic knee joint:** Flex the prosthetic knee joint and check whether the adapter is touching the joint (see fig. 4). Take measures if the adapter is touching the joint, for example:
 - Increase the distance between the adapter and prosthetic knee joint.
 - Insert a flexion stop.

Connecting the pyramid adapter and pyramid receiver

INFORMATION

Titanium set screws are available as optional accessories for waterproof products. They improve corrosion resistance.

- **Fitting:** Always use the steel set screws for fitting. They are easier to install.
- **Definitive prosthesis:** Continue using the steel set screws or insert titanium set screws when adjustments are no longer needed.

The set screws of the pyramid receiver secure the pyramid. The alignment of the prosthesis can be adapted by screwing the set screws in at different depths.

- > **Required materials:** torque wrench (e.g. 710D20), 636K13 Loc-tite 241
- 1) **Connect to a pyramid of the modular system for children:** Insert the reducer pyramid receiver 4X907=1 into the pyramid receiver of the Quickchange adapter base.
- 2) **Fitting:** Screw in and tighten the set screws.
Do not use Loctite during the fitting.
 Modular system for children: **9 Nm**
 Modular system: **10 Nm**
- 3) **Final assembly:** Secure the set screws with Loctite and tighten them.
 Modular system for children: **9 Nm**
 Modular system: First pull all set screws forward: **10 Nm**. Then tighten: **15 Nm**
- 4) **Replace set screws:** if they protrude too far or are recessed so deeply that they no longer engage correctly in the thread (see selection chart).

Selection table for set screws		
	Steel, galvanised	Titanium
12 mm	506G3=M8X12-V	506G5=M8X12
14 mm	506G3=M8X14	506G5=M8X14
16 mm	506G3=M8X16	506G5=M8X16

- **Disassembling the prosthetic components:** Only unscrew the two set screws that are screwed in the furthest. This maintains the set angles.
- **Replacing set screws:** Replace the set screws individually to obtain the set angles.

7 Use

Both parts of the adapter can be pulled apart after they are unlocked. After the prosthetic components are changed, the adapter is locked once more.

Open. Clean. Close.

Loosening the canted adapter

Watch the videos on YouTube for practical examples. Scan the QR codes or use the links

Open. Clean. Close:

<https://www.youtube.com/watch?v=K1G025W5M8w>

Loosen the angled adapter:

<https://www.youtube.com/watch?v=6sILNVZeTHE>

INFORMATION: Help children to lock the adapter.

- > **Prerequisite:** The user is sitting or the prosthesis is not being worn.
- > **Prerequisite:** The parts of the adapter are clean and free of dust or sand.
- 1) Rotate the locking ring to unlock the adapter (see fig. 5).
- 2) Pull apart both parts of the adapter.
- 3) Use a dry cloth to remove any soiling.
- 4) Make the change and plug the adapter back together.
- 5) Rotate the locking ring to lock the adapter (see fig. 5).
- 6) Retighten the locking ring slightly while the prosthesis is under load.

If the adapter cannot be pulled apart after unlocking

Sometimes the two parts of the adapter cannot be easily pulled apart after unlocking. This can happen, for example, if you have not unlocked the adapter for a long time. This is easy to fix:

- 1) Stand in a place where you can hold on tightly.
- 2) Unlock the adapter. Lift the prosthetic leg slightly to the side and then tap the floor with the inside of the prosthetic foot until the adapter loosens.

8 Cleaning

⚠ CAUTION

Use of unsuitable cleaning agents or disinfectants

Impairment of functionality and damage due to incorrect cleaning agents or disinfectants

- ▶ Clean the product only according to the instructions given in this section.
- ▶ For the prosthesis, observe the cleaning instructions for all prosthetic components.

Solids such as dust or sand can prevent the two parts of the adapter from connecting and locking together. As a result, special attention must be paid when removing dust and sand during cleaning.

Cleaning with water

- 1) Unlock the adapter and pull it apart.
- 2) Either rinse the two parts of the adapter with clean water or use a damp cloth.
- 3) Dry the parts with a soft cloth and allow them to air dry.

Cleaning with compressed air

Oil-free compressed air can be used for in-depth cleaning, for example while checking the prosthesis.

- ▶ Blow out the adapter with oil-free compressed air.

9 Maintenance

NOTICE! The individual components of the Quickchange adapter show signs of wear differently depending on frequency and intensity of use. Always bring all of the components that can be connected to each other for inspection.

- ▶ **After the first 30 days of use:** Perform a visual inspection and functional test of the product and the prosthesis.
- ▶ **Every 12 months:** Check the function of the product and the prosthesis and inspect for signs of wear.

10 Disposal

Do not dispose of the product in household waste. Improper disposal is harmful to health and the environment. Observe the instructions of the local authority regarding return, collection and disposal.

11 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

11.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

11.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

11.3 Local Legal Information

Legal information that applies **exclusively** to specific countries is written in the official language of the respective country of use after this chapter.

12 Technical data

System height and build height together with 4R11=R-3-2		
	System height [mm]	Build height [mm]
4R11=P-1-1	44	44
4R11=R-1-1	91	55
4R11=ST-1-1	62	41
4R11=ST-2-1 (with 4X907=1)	62 (64)	41 (43)

	Weight [g]	Max. body weight [kg]
4R11=R-3-2	220	150
4R11=P-1-1	150	150
4R11=R-1-1	150	150
4R11=ST-1-1	110	150
4R11=ST-2-1	105	45

4R11=R-3-2, 4R11=P-1-1, 4R11=R-1-1, 4R11=ST-1-1: For use in a sport prosthesis, the maximum body weight is 125 kg

	Material
4R11=R-3-2	Titanium/stainless steel
All connections	Stainless steel

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