## 12K41, 12K42, 12K44, 12K50

<b>DE</b> Gebrauchsanweisung (Fachpersonal) .....	2
<b>EN</b> Instructions for use (qualified personnel) .....	20
<b>FR</b> Instructions d'utilisation (personnel spécialisé) .....	38
<b>IT</b> Istruzioni per l'uso (personale tecnico specializzato) .....	57
<b>ES</b> Instrucciones de uso (personal técnico especializado) .....	76
<b>PT</b> Instruções de utilização (pessoal técnico) .....	94
<b>NL</b> Gebruiksaanwijzing (deskundig personeel) .....	113
<b>SV</b> Bruksanvisning (fackpersonal) .....	131
<b>DA</b> Brugsanvisning (faguddannet personale) .....	149
<b>NO</b> Bruksanvisning (fagpersonell) .....	167
<b>FI</b> Käyttöohje (ammattihenkilöstö) .....	185
<b>PL</b> Instrukcja użytkowania (personel fachowy) .....	203
<b>HU</b> Használati útmutató (szakszemélyzet) .....	222
<b>CS</b> Návod k použití (odborný personál) .....	240
<b>TR</b> Kullanım kılavuzu (uzman personel) .....	258
<b>EL</b> Οδηγίες χρήσης (Τεχνικό προσωπικό) .....	276
<b>RU</b> Руководство по применению (для специалистов) .....	295
<b>UK</b> Інструкція щодо використання (для кваліфікованого персоналу) .....	314
<b>JA</b> 取扱説明書(有資格担当者) .....	333

# 1 Vorwort

## INFORMATION

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### INFORMATION

Date of last update: 2024-09-17

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

The product "Ergo Arm 12K41=\*<sup>\*</sup>, ErgoArm plus 12K42=\*<sup>\*</sup>, ErgoArm Hybrid plus 12K44=\*<sup>\*</sup>, ErgoArm Electronic plus 12K50=\*<sup>\*</sup>" is referred to as the product/elbow joint below.


These instructions for use provide you with important information on the use, adaptation and handling of the product.

Only put the product into use in accordance with the information contained in the accompanying documents supplied.

## 2 Product description

### 2.1 Construction and Function

#### 2.1.1 Design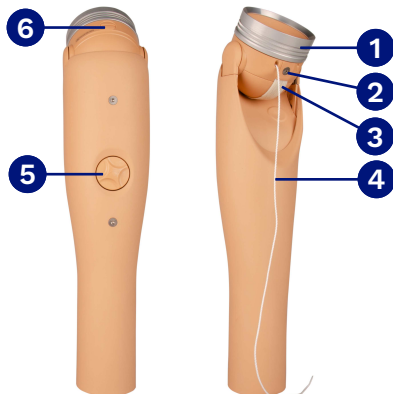

1. Easy Plug
2. Band clamp
3. Lifter band
4. Switch cable (mechanical unlocking mechanism)
5. Handwheel
6. Humeral rotation feature adjusting screw

#### 2.1.2 Function

The product is a mechanically/electronically driven elbow joint. It helps the user perform everyday tasks in combination with other prosthetic components (see page 22).

Various control programmes make optimal adaptation to the individual needs and abilities possible.

The ErgoArm models differ in the following characteristics:


Models	AFB	Easy Plug	Lock	Slip-stop
<b>ErgoArm 12K41=*</b>	-	-	Mechanical	Mechanical
<b>ErgoArm plus 12K42=*</b>	x	-	Mechanical	Mechanical
<b>ErgoArm Hybrid plus 12K44=*</b>	x	x	Mechanical	Mechanical
<b>ErgoArm Electronic plus 12K50=*</b>	x	x	Mechanical or Electronic	Mechanical or Electronic

##### 2.1.2.1 Definition of terms

#### AFB (Automatic Forearm Balance)

AFB is a mechanical flexion aid. It stores energy when the arm is extended and uses it to minimise the force required for flexion. The level of support can be adjusted to the weight of the prosthetic forearm and clothing with a handwheel.

#### Easy Plug

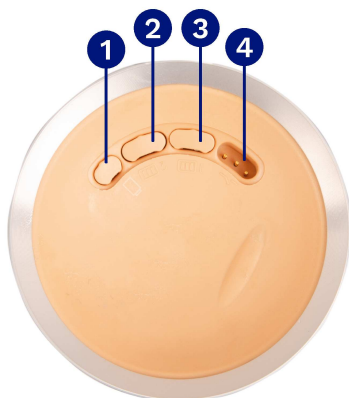1. Battery connection
2. Electrode connection CLOSED
3. Electrode connection OPEN
4. Switch connection

Easy Plug is a plug connection for myoelectrically controlled prostheses integrated in the 12K44=\* and 12K50=\* elbow joints.

The connecting cables of the electrodes, switches and batteries are connected to the Easy Plug. Its interior cable routing minimises the risk of cable breaks.

## Lock

The product is either locked mechanically or electronically (see page 21). The product can be locked and unlocked in any position, even under load, by activating the switch cable. The product has a load capacity of up to 177 N in the locked state. The lock will slip under higher loads.

### Mechanical lock

The product can be manually unlocked and locked by pulling hard on the switch cable.

This function is also possible when the product is switched off or the rechargeable battery is drained. In this way, the forearm can be brought into the desired position. Mechanical unlocking is also possible under load.

### Electronic lock

The lock can be released and engaged using electrode signals or switch signals (depending on the chosen switching version).

### Slip-stop function

The slip-stop function makes it possible to lower the forearm in a controlled manner without having to fully release the lock.

#### Slip-stop function, mechanical

By pulling slightly on the switch cable (no audible click), the product unlocks so the forearm can be moved continuously. When the switch cable is released, the product is locked again.

#### Slip-stop function, electronic

The slip-stop function is activated by the electrode signals (depending on the chosen control version). When the electrode signal stops, the product is locked again.

## 2.2 Combination possibilities

### Wrist joints

- Electric wrist rotator: 10S17 (only for 12K44=\* and 12K50=\*)
- Ball ratchet wrist joint: 10V8
- MovoWrist Flex: 10V39=\*
- MyoRotronic: 12E205 (only for 12K44=\* and 12K50=\*)
- Ottobock wrist joint: 10V18=\*
- Robo-Wrist, mechanical wrist joint: 10V41=\*

### Terminal devices

- All-purpose hook: 10A3
- All-purpose hook: 10A12=\*
- All-purpose ring: 10A4

### Accessories

- Battery
- Battery charger: 757L20
- MyoCharge Integral: 757L35

- bebionic hand EQD: 8E70=\* (only for 12K44=\* and 12K50=\*)
- MovoHook 2Grip: 10A71=\*
- MovoHook 2Grip: 10A81=\*
- Ottobock system hand -passive-: 8K18=\*
- Ottobock system hand -passive-: 8K19=\*
- VO hand: 8K22=\*
- VO hand: 8K23=\*
- VC hand: 8K26=\*
- VC hand: 8K27=\*
- System Electric Hand: 8E38=\* (only for 12K44=\* and 12K50=\*)
- System Electric Greifer: 8E33=\* (only for 12K44=\* and 12K50=\*)

## 3 Intended use

### 3.1 Indications for use

The product is intended **exclusively** for upper limb exoprosthetic fittings.

### 3.2 Conditions of use

The product is suitable for users with unilateral or bilateral upper arm amputations.

The product was developed for everyday use and must not be used for unusual activities. These activities include, for example, sports/extreme sports with high strain on the wrist and/or shocks such as push-ups, downhill racing, mountain biking, free climbing, paragliding, etc.

The product is intended **exclusively** for use on one user. Use of the product by a further person is prohibited by the manufacturer.

### 3.3 Contraindications

- All conditions which contradict or go beyond the specifications listed in the sections on "Safety" (see page 23) and "Intended use" (see page 22).

### 3.4 Qualification

Treatment may only be carried out by trained, qualified personnel.

## 4 Safety

### 4.1 Explanation of warning symbols

 <b>WARNING</b>	Warning regarding possible serious risks of accident or injury.
 <b>CAUTION</b>	Warning regarding possible risks of accident or injury.
 <b>NOTICE</b>	Warning regarding possible technical damage.

### 4.2 General safety instructions

<p> <b>CAUTION</b></p> <p><b>Electrode lifting off</b> Injury due to unexpected product behaviour as a result of uncontrolled activation of the prosthesis component.</p> <ul style="list-style-type: none"> <li>▶ Inform the user that uncontrolled movement of the prosthesis component may occur if the electrode is lifted off.</li> </ul>
<p> <b>CAUTION</b></p> <p><b>Damage to connecting cable due to kinking or small radii</b> Risk of injury due to unexpected product behaviour as a result of a malfunction.</p> <ul style="list-style-type: none"> <li>▶ Make sure there are no tight angles or kinks when installing the cables.</li> <li>▶ Replace the connecting cable immediately if it is damaged.</li> </ul>
<p> <b>CAUTION</b></p> <p><b>Incorrect electrode settings due to muscle fatigue</b> Injury due to faulty control or malfunction of the product.</p> <ul style="list-style-type: none"> <li>▶ Allow the user to rest during the adjustment of the electrodes.</li> </ul>
<p> <b>CAUTION</b></p> <p><b>Incorrect donning and doffing of the prosthesis</b> Risk of injury due to sudden flexion of the prosthesis.</p> <ul style="list-style-type: none"> <li>▶ Switch off the prosthesis before donning or doffing.</li> <li>▶ The prosthesis must always be flexed for donning and doffing.</li> </ul>
<p> <b>CAUTION</b></p> <p><b>Risk of pinching in the joint flexion area</b> Injuries due to pinching of body parts.</p> <ul style="list-style-type: none"> <li>▶ Ensure that fingers and other body parts are not in this area when bending the joint.</li> </ul>
<p> <b>CAUTION</b></p> <p><b>Independent user manipulation of system components</b> Injury due to faulty control or malfunction of the product.</p> <ul style="list-style-type: none"> <li>▶ Manipulations to the product other than the tasks described in these instructions for use are not permitted.</li> <li>▶ The product and any damaged components may only be opened and repaired by authorised, qualified Ottobock personnel.</li> </ul>

**⚠ CAUTION**

**Unintentional unlocking of the terminal device**

Risk of injury due to releasing the terminal device from the forearm (e.g. while carrying objects).

- ▶ When using a quick-disconnect wrist unit, position the terminal device so that slight twisting does not release the terminal device from the forearm.

**⚠ CAUTION**

**Insufficient skin contact of the electrodes**

Injury due to faulty control or malfunction of the product.

- ▶ The electrodes are to be placed on intact skin only and with as much electrode-skin contact as possible.
- ▶ In the case of strong interference from electronic devices, the position of the electrodes should be checked and changed if necessary.
- ▶ If the interference cannot be eliminated or if you do not achieve the expected results by adjustment or selection of the appropriate control programme, please contact the Ottobock branch responsible for your country.

**⚠ CAUTION**

**Improper handling**

Injury due to loss of product functionality.

- ▶ Instruct the user in the proper use of the product.

**⚠ CAUTION**

**Injury due to incorrect adjustment of the product.**

Loosening of the components.

- ▶ In case of a product with a quick-disconnect wrist, position the terminal device before use so that a slight turn cannot disconnect the terminal device from the prosthesis.

**⚠ CAUTION**

**Signs of wear and tear on the product components**

Injury due to faulty control or malfunction of the product

- ▶ In the interest of preventing patient injuries and to maintain product quality, we recommend performing regular service.
- ▶ Please see the service pass for more information about the service intervals. This is included in the purchased extended warranty packages.

**⚠ CAUTION**

**Use of inappropriate components**

Risk of injury due to malfunction of the product.

- ▶ Operate the product only with components recommended by the manufacturer. You can find the list of components in the "Combination possibilities" section (Combination possibilities).
- ▶ Operate the product only with accessories that have been recommended by the manufacturer. You can find the list of components in the "Scope of delivery" section (Scope of delivery).

**NOTICE**

**Damage to the battery**

Damage to the battery due to using both connection options.

- ▶ Only use one of the two connections for the battery (connection on the forearm cable or Easy Plug connection).

**NOTICE**

**Coating, gluing or painting the prosthesis**

Damage or fracture due to chemical processes.

- ▶ The prosthesis must not be coated, glued or painted.

**NOTICE****Improper product care**

Damage to the product due to the use of incorrect cleaning agents.

- ▶ Clean the product only as described in the section "Cleaning and care" (Cleaning and Care).

## 5 Scope of Delivery and Accessories

### 12K41 ErgoArm scope of delivery

- 1 pc. Lamination protection cover (disc with arrow)
- 1 pc. 12K41=\* ErgoArm
- 1 pc. instructions for use (user)
- 1 pc. Instructions for use (qualified personnel)
- 2 pc. bracket cover
- 1 pc. 60 x 2.5 mm O-ring
- 1 pc. Cable lock

### 12K42 ErgoArm plus scope of delivery

- 1 pc. Lamination protection cover (disc with arrow)
- 1 pc. 12K42=\* ErgoArm plus
- 1 pc. instructions for use (user)
- 1 pc. Instructions for use (qualified personnel)
- 2 pc. bracket cover
- 1 pc. 60 x 2.5 mm O-ring
- 1 pc. Cable lock

### Accessories

- Adapter: 13Z68
- Battery connecting cable: 13E51=2
- Alignment aid for ErgoArm: 743A23

### Options for programme settings of the 12K50

- 1 pc. 757T13 MyoSelect

### 12K44 ErgoArm Hybrid plus scope of delivery

- 1 pc. Lamination protection cover (hollow ball)
- 1 pc. Lamination protection cover (disc with arrow)
- 1 pc. 12K44=\* ErgoArm Hybrid plus
- 1 pc. instructions for use (user)
- 1 pc. Instructions for use (qualified personnel)
- 2 pc. bracket cover
- 1 pc. 60 x 2.5 mm O-ring
- 1 pc. Cable lock

### 12K50 ErgoArm Electronic plus scope of delivery

- 1 pc. Lamination protection cover (hollow ball)
- 1 pc. Lamination protection cover (disc with arrow)
- 1 pc. 12K50=\* ErgoArm Electronic plus
- 1 pc. instructions for use (user)
- 1 pc. Instructions for use (qualified personnel)
- 1 pc. spherical cord lock
- 2 pc. bracket cover
- 1 pc. 60 x 2.5 mm O-ring
- 1 pc. Cable lock

- Clamp stopple set: 21A207
- Battery receptacle set: 757Z184=1, 757Z190=1
- Battery receptacle set, screwable: 757Z184=2, 757Z190=2
- Coding plug set: 13E182

## 6 Preparing the product for use

### 6.1 Required machines, tools and materials

- 701P250=1 Belt sander (1x 230 V, 50/60 Hz)
- 756B2 Electric jigsaw (1x 230 V N/PE, 50 Hz)
- 636W60 Adhesive (Loctite 243)
- Knives
- Pattex Repair
- 89Z1 Tweezers
- Set of screwdrivers
- 633F11 silicone grease
- 633F30 Special grease

### 6.2 Laminating instructions

**INFORMATION**

Laminate the socket according to the state of the art (see 646T333=\*\_INT technical information).

### 6.3 Removing the lamination ring

Complete the following steps to remove the lamination ring:

- 1) Set the flexion assist to the minimum using the handwheel.
- 2) Secure the lifter band (e. g. using artery forceps).
- 3) Remove the band clamp.

- 4) Loosen the rotation lock of the lamination ring with the adjusting screw of the humeral rotation feature.
- 5) Unscrew the lamination ring from the elbow ball.
- 6) Carefully remove the electronics housing from the lamination ring.

## 6.4 Sealing the product

Complete the following steps to seal the product:

- 1) Clean the lamination ring so it is free of grease and dirt.

- 2) Lubricate the groove, inner edges and thread of the lamination ring with special grease.


### Additional steps for the products 12K44 and 12K50

#### NOTICE

#### Misalignment during installation of the electronics housing

Misalignment during installation results in the risk of loss of contact or short-circuiting of the spring contacts. The mechanical strength can also be reduced due to thread misalignment. The circuit board and the edge of the lamination ring must be parallel to each other.

Complete the following additional steps to seal the product:

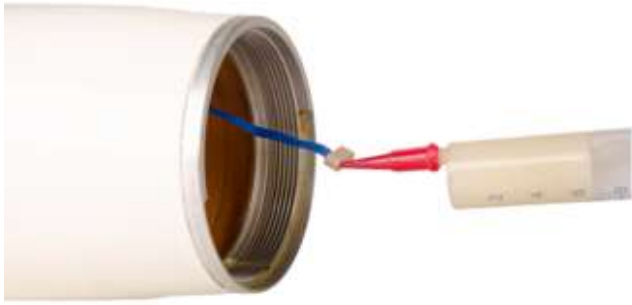- 1) Thoroughly lubricate the plugs of the connecting cables (electrodes, switch, rechargeable battery) with silicone grease.
- 2) Remove the blanking covers from the electronics housing.
- 3) Connect the plugs to the electronics housing.
- 4) Thoroughly seal the plugs and remaining blanking covers from above with silicone grease.
- 5) Push the electronics housing, including the connected cables, into the lubricated lamination ring and press firmly.

**INFORMATION: A loud popping sound must be audible.**

- 6) Check the fit of the electronics housing.  
**NOTICE! The circuit board of the electronics housing and the edge of the lamination ring are flush/parallel to each other.**

- 7) Connect the desired programming plug to the electronics housing if necessary.


## 6.5 Removing the elbow joint

### Removing the elbow joint for the 12K41 product

Complete the following additional steps to remove the elbow joint:

- 1) Extend the arm.


- 2) Loosen the two cover caps ❶.
- 3) Loosen the mounting screws from the elbow joint.
- 4) Pull off the elbow joint.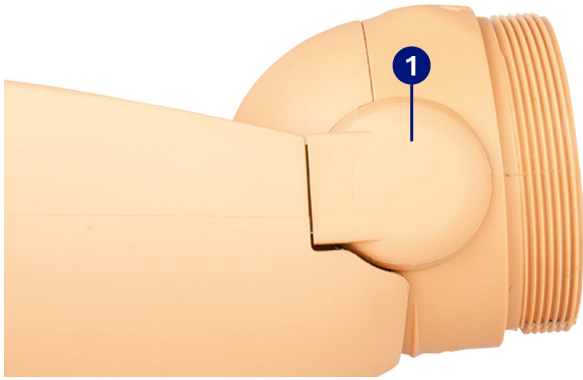

**Removing the elbow joint for the 12K42, 12K44, 12K50 products**

Complete the following steps to remove the elbow joint:

- 1) Flex the arm.
- 2) Set the flexion force to the minimum level using the handwheel.
- 3) Extend the arm again.
- 4) Lock the elbow joint in this position to fix the removal position and installation position.
- 5) Loosen the two cover caps ❶.

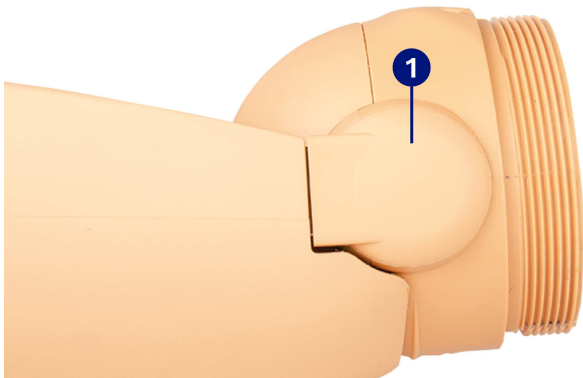- 6) Pry out the cable plugs.
- 7) Disconnect the cables.

- 8) Loosen the mounting screws from the elbow joint.
- 9) Unscrew the band clamp.
- 10) Pull out the lifter band.


- 11) Pull off the elbow joint.
- 12) Screw the elbow joint into the upper arm.

### 6.6 Removing the flexion aid (AFB)

Complete the following steps to remove the flexion aid:

- 1) Unscrew the two mounting screws on the forearm.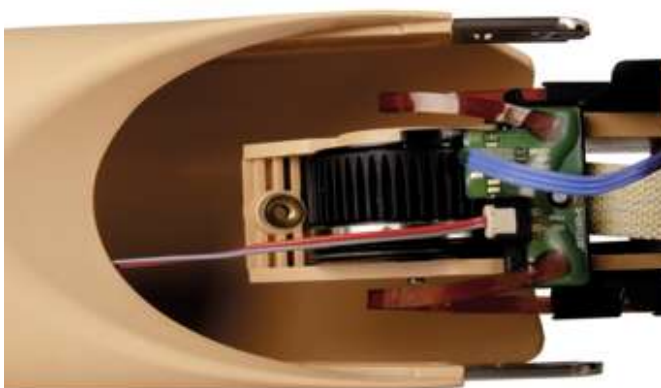

- 2) Pull out the flexion aid.

## 6.7 Shortening the forearm


### INFORMATION

Wrap the elbow area with plastic wrap to prevent swarf from entering it during cutting.

Complete the following steps to shorten the forearm:

- 1) Add 10 mm to the length that was determined and mark this on the forearm.

**INFORMATION: The forearm length is measured from the distal end of the lamination ring to the middle of the elbow axle.**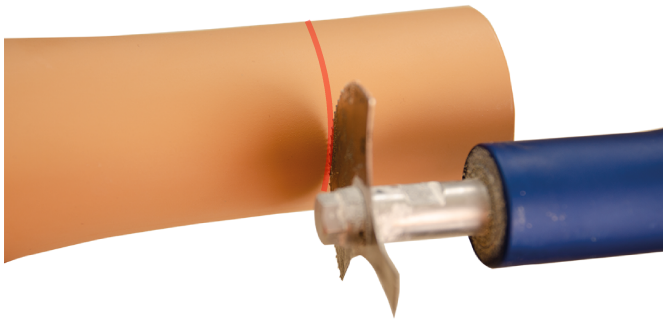

- 2) Shorten the forearm at the distal end to the user's individual dimensions (trim or grind).

**NOTICE! The minimum length of the forearm must be 220 mm after shortening.**

- 3) Sand all sides of the cut edges at a right angle.
- 4) Deburr and slightly round the inside and outside edges.

## 6.8 Optional: Positioning the battery receptacle

Complete the following steps to position the battery receptacle:


- 1) Position the battery receptacle.

**INFORMATION: Make sure the battery receptacle does not collide with the flexion aid (AFB) and electric wrist rotator.**

- 2) Connect the plug of the battery connection cable to the flexion aid (AFB) circuit board.

## 6.9 Installing the flexion aid (AFB)

Complete the following steps to install the flexion aid:

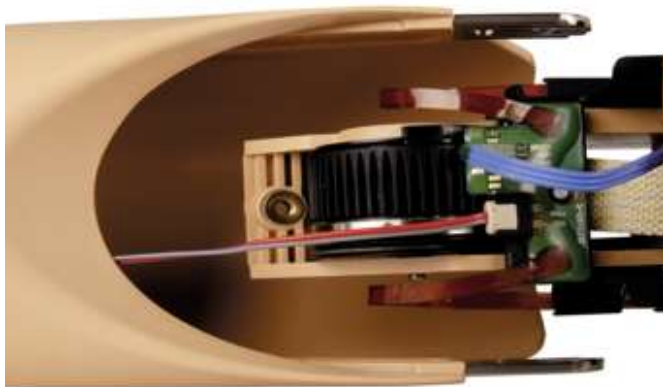- 1) Insert the flexion aid in the forearm.

- 2) Tighten the mounting screws on the forearm.

## 6.10 Converting the switch cable

Complete the following steps to convert the switch cable:

- 1) Pull the ball cap off the elbow ball.

- 2) Disengage the spring telescope.
- 3) Remove the ladder lock buckle.
- 4) Pull out the spring telescope.

- 5) Remove the dummy and insert it on the opposite side.


- 6) Thread the spring telescope through on the opposite side.
- 7) Engage the spring telescope on the protruding metal axle.
- 8) Put on the ball cap.
- 9) Attach the ladder lock buckle.

## 6.11 Finishing the switch cable

Complete the following additional steps to finish the switch cable:

- 1) Thread the switch cable through the cable lock.
- 2) Secure the end of the switch cable with a knot.
- 3) Tighten the switch cable and make a loop.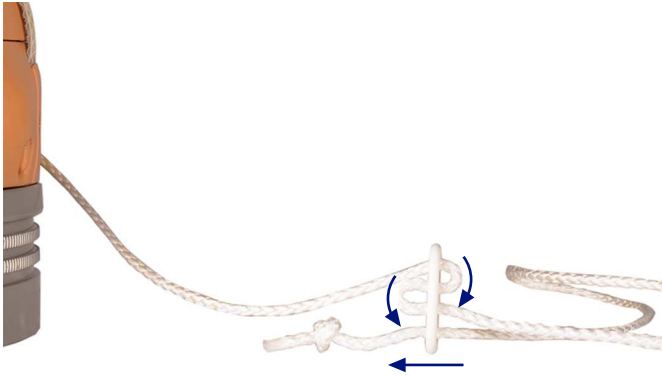

### 6.11.1 Shortening the switch cable (only for 12K50)

#### ⚠ CAUTION

##### Removing the switch cable

Injuries due to loss of product functionality.

- ▶ For safety reasons, the switch cable must not be removed entirely.

Complete the following steps to shorten the switch cable:

- 1) Trim the switch cable to the desired length.
- 2) Cure the end of the switch cable with the cord lock.

## 6.12 Installing the elbow joint

#### NOTICE

##### Use of the product with a damaged flat cable

Malfunction due to incorrect or interrupted data transfer.

- ▶ Avoid bending or pressing the flat cable multiple times in the same position.

Complete the following steps to install the elbow joint:

- 1) Push the elbow joint into the flat toothed disc on the forearm.

**INFORMATION: Do not pinch the existing cables (12K44, 12K50). The cables are exposed and may be damaged if they do not have cover caps.**

- 2) Apply Loctite 243 to the two elbow joint mounting screws and tighten them to 2 Nm.
- 3) **12K44/12K50:** Lubricate the wiring connectors with silicone grease.
- 4) **12K44/12K50:** Reconnect both cables.
- 5) Extend the arm.

- 6) Apply "Pattex Repair Extreme" to the flaps to secure the cover caps.
- 7) Put the cover caps on the flaps.  
**INFORMATION: Do not pinch the existing cables (12K44, 12K50).**

**Additional steps: installing the elbow joint for the 12K42, 12K44, 12K50 products**

- 1) Slide the lifter band under the axle.  
**INFORMATION: Make sure the lamination ring is correctly positioned.**  
**INFORMATION: Ensure the band is straight and not twisted.**  
**INFORMATION: There needs to be space for the band clamp between the band clamp guide and the lamination ring.**

- 2) Slide the band clamp under the lamination ring.
- 3) Secure the band clamp with the screw.

### 6.13 Setting the preflexion

**NOTICE**

**Damage to the arm housing due to a lifter band that is too long**

When extended, the edge of the forearm sleeve must not touch the stop of the joint ball. When adjusting the length of the lifter band, make sure that it is not too long and that the arm is stopped by the lifter band.

Complete the following steps to set the preflexion:

- 1) Loosen the screw on the band clamp.
- 2) Shorten or lengthen the lifter band.  
**INFORMATION: Lengthening the band decreases the preflexion. Shortening the band increases the preflexion.**  
**INFORMATION: A 3 mm change in the length of the band changes the flexion angle by about 5°. The free swing range is changed by the same amount as the flexion angle.**
- 3) Tighten the screw on the band clamp.
- 4) Repeat the process until the desired preflexion is reached.

### 6.14 Temporarily secure the switch cable

Complete the following steps to secure the switch cable:

- ▶ Secure the switch cable within the marked area with adhesive tape.  
**INFORMATION: The end of the switch cable must not be drawn into the socket when the forearm is released.**

### 6.15 Installing the elbow joint in the upper arm socket

Complete the following steps for installation in the upper arm socket: see page

- 1) Grease the exterior thread of the elbow joint with silicone grease.
- 2) Screw the elbow joint into the upper arm.

## 7 Handling

### 7.1 Adjusting the flexion assist


**CAUTION**

**Sudden flexion of the product with high flexion assist**


Risk of injury due to sudden flexion of the prosthesis.

- ▶ Observe flexion assist when taking off heavy clothing.
- ▶ Observe the flexion assist setting when the prosthesis is not used.
- ▶ The prosthesis must always be flexed for donning and doffing.

Complete the following steps to set the flexion assist: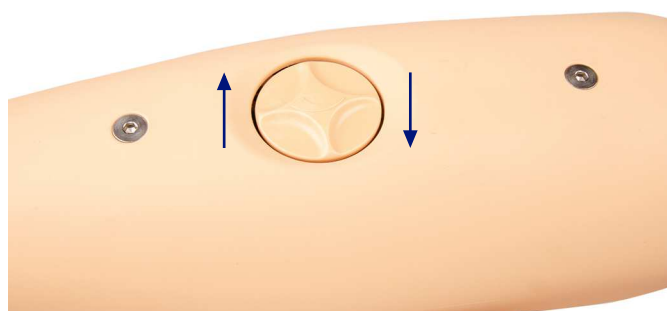

- 1) Flex the arm.
- 2) Use the handwheel to set the desired flexion assist.  
**INFORMATION: The support level can be adjusted to the weight of the clothing and the hand weight.**

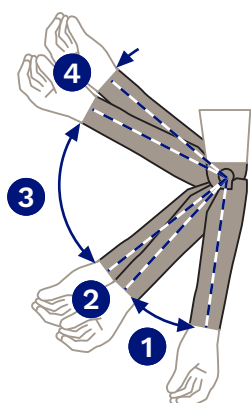**Position 1**

Low support to allow the arm to swing freely while walking.

**Position 2**

Support increases progressively when flexing the arm and decreases during extension.

**Position 3**

Support remains constant. If set correctly, the weight of the forearm is balanced by the flexion assist and the forearm "floats".

**Position 4**

Low support before flexion stop.

## 7.2 Adjusting the humeral rotation feature

The humeral rotation feature has a bilateral stop of  $\pm 80^\circ$ . The resistance of the humeral rotation feature can be adjusted using the external adjusting screw.

- 1) Increase the resistance by turning it clockwise.
- 2) Reduce the resistance by turning it counter-clockwise.

## 7.3 Adjusting the electrodes (only for 12K50)

The electrode setting depends on the chosen electric hand and the components that are used (e. g. 12E205 MyoRotronic). Observe the supplied instructions for use for the electrodes, electric hand and other components when making settings.

## 7.4 Program settings (only for 12K50)

### Program settings with the MyoSelect

Complete the following steps to make program settings with the MyoSelect:

- 1) Remove the rechargeable battery or turn off the MyoEnergy Integral.
- 2) Remove the electric wrist rotator and MyoRotronic if applicable.

- 3) Connect the 9E169 coaxial plug to the electronics housing.
- 4) Connect MyoSelect with the 9E169 coaxial plug and 12K50.
- 5) Select the appropriate program in the MyoSelect.
- 6) Pull the 9E169 coaxial plug from the electronics housing.
- 7) Reinstall the electric wrist rotator and MyoRotronic if applicable.
- 8) Install the rechargeable battery or switch on the MyoEnergy Integral.

**Configuring the control program using the programming plugs**

Complete the following steps to configure the control program using the programming plug:

- ▶ Connect the appropriate program switch (see page 36) to the electronics housing.

**7.5 Program switching (only for 12K50)**

The following switching methods can be selected:

- Switch
- Electrodes (generating a co-contraction)

**7.5.1 Switching modes**

The following control programs and programming plugs can be activated:

Control program	Programming plug	Electronic elbow lock	Slip-stop function	Switching between hand and rotation	
1	White	Press and release switch: - Lock - Unlock	No	All MyoRotronic programs	
2	Red	Co-contraction	No	Program 1 and program 4 of the MyoRotronic	
3	Green	Press and hold switch (elbow mode) - CLOSE electrode = lock - OPEN electrode = unlock Release switch (hand mode)	Yes	All MyoRotronic programs	
4	Blue	Press and release switch (elbow mode) - CLOSE electrode = lock - OPEN electrode = unlock Press and release switch (hand mode)			
5	Yellow	Press and release switch (elbow mode) - CLOSE electrode = lock - OPEN electrode = unlock No electrode signal for 10 seconds (hand mode) Press and release switch (hand mode)			
6	Violet	Co-contraction (elbow mode) - CLOSE electrode = lock - OPEN electrode = unlock Co-contraction (hand mode)			Program 1 and program 4 of the MyoRotronic
7	Orange	Co-contraction (elbow mode) - CLOSE electrode = lock - OPEN electrode = unlock No electrode signal for 10 seconds (hand mode) Co-contraction (hand mode)			

## 8 Charging the battery

### INFORMATION

Please observe the corresponding instructions for use of the charging equipment.

## 9 Cleaning

- 1) Clean the product with a damp cloth and mild soap (e.g. 453H10=1-N Ottobock Derma Clean) when needed. Make sure that no liquids get into the product and product components.
- 2) Dry the product with a lint-free cloth and allow it to air dry fully.

## 10 Maintenance

Performing regular maintenance (service inspections) every 24 months is recommended to prevent injuries and maintain the quality of the product.

The product is required for maintenance 4 weeks before, up to a maximum of 3 months after the due date.

In general, all products are subject to compliance with the maintenance intervals during the warranty period. This is the only way to maintain full warranty cover.

Additional services such as repairs may be provided in the course of maintenance. These additional services may be provided free of charge or can be billable according to an advance cost estimate, depending on the extent and validity of the warranty.

Components required for maintenance or repair:

The product, battery charger and power supply unit.

Use the service packaging to send the components requiring inspection.

## 11 Legal information

### 11.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

### 11.2 Trademarks

All product names mentioned in this document are subject without restriction to the respective applicable trademark laws and are the property of the respective owners.

All brands, trade names or company names may be registered trademarks and are the property of the respective owners.

Should trademarks used in this document fail to be explicitly identified as such, this does not justify the conclusion that the denotation in question is free of third-party rights.

### 11.3 CE conformity

Otto Bock Healthcare Products GmbH hereby declares that the product is in compliance with applicable European requirements for medical devices.

The product meets the requirements of the RoHS Directive 2011/65/EU on the restriction of the use of certain hazardous substances in electrical and electronic devices.

The full text of the regulations and requirements is available at the following Internet address: <http://www.ottobock.com/conformity>

## 12 Technical data

Environmental conditions	
Storage (with and without packaging)	+5 °C/+41 °F to +40 °C/+104 °F Max. 85 % relative humidity, non-condensing
Transport (with and without packaging)	-20 °C/-4 °F to +60 °C/+140 °F Max. 90 % relative humidity, non-condensing
Operation	-5 °C/+23 °F to +45 °C/+113 °F Max. 95 % relative humidity, non-condensing

Reference number	12K41=*	12K42=*	12K44=*	12K50=*
Current draw (in mA)	-	-	-	3.5
Power supply	-	-	-	757B20 757B21 757B35=*
Weight (g)	%Approx. 539	%Approx. 672	%Approx. 695	%Approx. 721
Length (mm)	%Approx. 350			
Max. load in locked state (N)	177			
Flexion angle (in °)	15° – 145°			
Product behaviour during the charging process	The product is non-functional			
Expected lifetime given compliance with the recommended maintenance intervals	6 years			

### 13 Symbols Used

Declaration of conformity according to the applicable European directives

Serial number (YYYY WW NNN)  
 YYYY – year of manufacture  
 WW – week of manufacture  
 NNN – sequential number

In some jurisdictions it is not permissible to dispose of these products with unsorted household waste. Disposal that is not in accordance with the regulations of your country may have a detrimental impact on health and the environment. Please observe the instructions of your national authority pertaining to return and collection.

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