8E70=*, 8E71=*, 8E72=*

 Gebrauchsanweisung [kurz] (Fachpersonal)	3
 Instructions for use [short] (qualified personnel)	9

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INFORMATION

Date of last update: 2024-05-13

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.

- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

Note the instructions for use

This document contains the sections "Intended use" and "Safety", and does not replace the instructions for use. The instructions for use are available in digital form (PDF format) (see page 2 of this document). To view PDF files, you need an Internet browser or other program capable of opening this file format.

You may request a printed copy of the instructions for use at the e-mail address on page 2. This will be delivered to the specified address within 7 days after the order is received.

The products "8E70=* bebionic hand EQD, 8E71=* bebionic hand Short Wrist and 8E72=* bebionic hand Flex" are referred to as the product/terminal device/hand in the following.

8E70=* Basic UDI-DI: 40644110000000008E550LE

8E71=* Basic UDI-DI: 406441100000000008E71HM

8E72=* Basic UDI-DI: 406441100000000008E72HP

2 Intended use

2.1 Indications for use

The product is intended **exclusively** for upper limb exoprosthetic fittings.

2.2 Conditions of use

The product is intended **exclusively** for use on **one** user. Use of the product by another person is not approved by the manufacturer.

The product was developed for everyday use and must not be used for unusual activities. These unusual activities include, for example, sports which involve excessive strain and/or shocks to the wrist joint (pushups, downhill racing, mountain biking, ...) or extreme sports (free climbing, paragliding, etc.). Furthermore, the product should not be used to operate motor vehicles, heavy equipment (e.g. construction machines), industrial machines or motor-driven equipment.

Permissible ambient conditions are described in the technical data (Technical data).

2.3 Indications

- Amputation level below-elbow, above-elbow and shoulder disarticulation
- For unilateral or bilateral amputation
- Dysmelia of the forearm or upper arm
- The patient must be able to understand usage and safety messages and put them into practice.
- The patient must fulfil the physical and mental requirements for perceiving visual/acoustic signals and/or mechanical vibrations.

INFORMATION

Increased maintenance requirements for bilateral users

In the day-to-day use of a bilateral user, the load on the prosthetic hands tends to be higher, as the user is dependent on the function of the prosthesis during all activities.

Therefore, please note there may be increased mechanical wear and tear and need for repairs when treating a bilateral amputation.

2.4 Contraindications

- All conditions which contradict or go beyond the specifications listed in the section on "Safety" and "Indications for use".

2.5 Qualification

The fitting of a patient with the product may only be carried out by O&P professionals who have been authorised with the corresponding Ottobock training.

3 Safety

3.1 Explanation of warning symbols

 **WARNING** Warning regarding possible serious risks of accident or injury.

 **CAUTION** Warning regarding possible risks of accident or injury.

3.2 Structure of the safety instructions

 **WARNING**

The heading describes the source and/or the type of hazard

The introduction describes the consequences in case of failure to observe the safety instructions. Consequences are presented as follows if more than one consequence is possible:

- > E.g.: Consequence 1 in the event of failure to observe the hazard
- > E.g.: Consequence 2 in the event of failure to observe the hazard
- ▶ This symbol identifies activities/actions that must be observed/carried out in order to avert the hazard.

3.3 General safety instructions

 **WARNING**

Non-observance of safety notices

Personal injury/damage to the product due to using the product in certain situations.

- ▶ Observe the safety notices and the stated precautions in this accompanying document.

 **WARNING**

Operating a vehicle with the prosthesis

Accidents due to unexpected actions of the prosthesis.

- ▶ The prosthesis should not be used for the operation of motor vehicles and heavy equipment (e.g. construction machines).

 **WARNING**

Using the prosthesis while operating machines

Injury due to unexpected actions of the prosthesis.

- ▶ The prosthesis should not be used to operate industrial machines or motor-driven equipment.

 **WARNING**

Use of damaged power supply unit, adapter plug or battery charger

Risk of electric shock due to contact with exposed, live components.

- ▶ Do not open the power supply unit, adapter plug or battery charger.
- ▶ Do not expose the power supply unit, adapter plug or battery charger to extreme loading conditions.
- ▶ Immediately replace damaged power supply units, adapter plugs or battery chargers.

⚠ WARNING

Use of the product while handling a firearm.

Injury due to unexpected product behaviour.

- ▶ The product may not be used for handling firearms.

⚠ WARNING

Skin contact with leaking lubricants due to mechanical defects

Injury due to skin irritation.

- ▶ Avoid contact between leaking lubricants and the mouth, nose and eyes.
- ▶ The product must be inspected by an authorised Ottobock Service Center.

⚠ CAUTION

Signs of wear on the product

Injury due to faulty control or malfunction of the product

- ▶ In the interest of patient safety and in order to maintain operating reliability, the terminal device has to be inspected by an authorised Ottobock Service Centre in case of noticeable restrictions of functionality.
- ▶ Note that the functionality of the terminal device may be limited if the battery charge level gets too low.

⚠ CAUTION

Use of a damaged product

Injury due to loss of product functionality.

- ▶ Prior to use, conduct an external visual inspection to verify that all parts of the product are undamaged.
- ▶ In case of damage, have the product repaired promptly.

⚠ CAUTION

Penetration of product with dirt and humidity

Injury due to unexpected product behaviour or malfunction.

- ▶ Ensure that neither solid particles nor liquids can penetrate into the product.

⚠ CAUTION

Independent manipulation of the product or product components

Injury due to faulty control or malfunction of the product due to manipulation

- ▶ Manipulations to the product other than the tasks described in these instructions for use are not permitted.
- ▶ Opening and repairing the product and any damaged components, and removing the back of the hand, is reserved exclusively for authorised, qualified Ottobock personnel.

⚠ CAUTION

Insufficient skin contact of the electrodes

Injury due to faulty control or malfunction of the product.

- ▶ The electrodes are to be placed on intact skin only and with as much electrode-skin contact as possible.
- ▶ In the case of strong interference from electronic devices, the position of the electrodes should be checked and changed if necessary.

- ▶ If the interference cannot be eliminated or if you do not achieve the expected results by adjustment or selection of the appropriate control programme, please contact the Ottobock branch responsible for your country.

CAUTION

Prosthesis use when battery charge level is too low

Injury due to unexpected prosthesis behaviour

- ▶ Check the current charge level before use and charge the prosthesis if required.
- ▶ Note that the operating time of the prosthesis may be reduced at low ambient temperatures or due to ageing of the battery.
- ▶ Note that the actions/reactions of the terminal device become slower when the battery voltage gets very low.
- ▶ Note that only a few grips or actions will still be possible with the terminal device when the battery voltage gets very low.
- ▶ A small opening width may indicate a low battery voltage.

3.4 Information on Alignment/Adjustment

CAUTION

Operator errors during the adjustment process with the adjustment software

Injury due to unexpected product behaviour.

- ▶ Participation in an Ottobock product training course is required prior to initial use. Additional product training courses may become necessary to qualify for software updates.
- ▶ Transfer the setting changes to the terminal device before you check the settings on the patient.
- ▶ Use the online help which is integrated into the software.

CAUTION

Use of unapproved accessories

> Injury due to product malfunction as a result of reduced resistance to interference.

- > Interference of other electronic devices due to increased emissions.
- ▶ Only use the product in combination with the accessories, signal converters and cables listed in the sections "Combination possibilities" (Combination possibilities), "Scope of delivery" (Scope of delivery) and "Accessories" (Accessories).

CAUTION

Use of unsuitable prosthetic components

Injury due to unexpected product behaviour.

- ▶ Use the product only in combination with components listed in the section "Combination possibilities" (Combination possibilities).

CAUTION

Failure to observe the instructions for use of all prosthesis components used

Injury due to unexpected product behaviour.

- ▶ Observe the instructions for use of the prosthesis components used.

CAUTION

Incorrect electrode settings/electrode assignment

Injury due to unexpected product behaviour.

- ▶ The electrodes are to be placed on intact skin only and with as much electrode-skin contact as possible. In the case of strong interference from electronic devices, the position of the electrodes should be checked and changed if necessary. If the interference cannot be eliminated or if you do not achieve the expected results by adjustment or selection of the appropriate control programme, please contact the Ottobock branch responsible for your country.
- ▶ Set the electrode sensitivity as low as possible in order to reduce interference from powerful electromagnetic radiation (e.g. visible or concealed theft prevention systems at the entrance/exit of stores), metal detectors/body scanners for persons (e.g. in airports) or other sources of strong electromagnetic interference (e.g. high-voltage lines, transmitters, transformer stations, computer tomographs, magnetic resonance tomographs, etc.).
- ▶ Make sure that the electrode connection positions correspond to physiological opening and closing for the corresponding muscle groups.

3.5 Information on Proximity to Certain Areas

CAUTION

Insufficient distance to HF communication devices (e.g. mobile phones, Bluetooth devices, WiFi devices)

Injury due to unexpected behaviour of the product caused by interference with internal data communication.

- ▶ Therefore, keeping a minimum distance of 30 cm from HF communication devices is recommended.

CAUTION

Operating the product in very close proximity to other electronic devices

Injury due to unexpected behaviour of the product caused by interference with internal data communication.

- ▶ Do not operate the product in the immediate vicinity of other electronic devices.
- ▶ Do not stack the product with other electronic devices during operation.
- ▶ If simultaneous operation cannot be avoided, monitor the product and verify proper use in the existing setup.

CAUTION

Proximity to sources of strong magnetic or electrical interference (e.g. theft prevention systems, metal detectors)

Injury due to unexpected behaviour of the product caused by interference with internal data communication.

- ▶ Avoid remaining in the vicinity of visible or concealed theft prevention systems at the entrance/exit of stores, metal detectors/body scanners for persons (e.g. in airports) or other sources of strong magnetic and electrical interference (e.g. high-voltage lines, transmitters, transformer stations, computer tomographs, magnetic resonance tomographs, etc.).
- ▶ When walking through theft prevention systems, body scanners or metal detectors, watch for unexpected behaviour of the product.

3.6 Information on Use

CAUTION

Mechanical stress on the product

Injury due to faulty control or malfunction of the product.

- ▶ Do not subject the product to mechanical vibrations or impacts.

- ▶ Check the product for visible damage before each use.

CAUTION

Improper use

Injury due to faulty operation or malfunction of the product.

- ▶ Instruct the patient in the proper use of the product.

CAUTION

Improper product care

- > Injuries due to faulty control/malfunction of the product or damage to the mechanical components
- > Damage or breakage due to embrittlement of plastics caused by the use of acetone, petrol or similar solvents.
- ▶ Clean the product only as described in the section "Cleaning and care" (Cleaning and care).
- ▶ Do not clean the product under running water.
- ▶ When using a prosthetic glove, also note the instructions for use of the glove.

CAUTION

Grasping objects with incorrect gripping forces

Injury due to unexpected product behaviour.

- ▶ Note that the gripping has to be controlled manually depending on the consistency (soft/hard) of the object being grasped.

CAUTION

Overloading due to unusual activities

Injury due to unexpected product behaviour as a result of a malfunction.

- ▶ The product was developed for everyday use and must not be used for unusual activities. These unusual activities include, for example, sports with excessive strain and/or shocks to the wrist joint (pushups, downhill mountain biking, ...) or extreme sports (free climbing, paragliding, etc.).
- ▶ Careful handling of the product and its components not only increases their service life but, above all, ensures the patient's personal safety!
- ▶ If the product and its components have been subjected to extreme loads (e.g. due to a fall, etc.), then the product must be inspected for damage immediately. If necessary, forward the product to an authorised Ottobock Service Centre.

CAUTION

Risk of pinching between the fingertips

Injury due to pinching of body parts.

- ▶ Ensure that no body parts are between the fingertips when using the product.
- ▶ Ensure that no body parts are between the fingertips when closing the hand.
- ▶ When closing the hand, ensure that fingers and other body parts are not in the area of the finger joints.
- ▶ Make sure the product is switched off for cleaning.

CAUTION

Insufficient distance from sources of high heat

Inflammation of the product.

- ▶ Do not expose the product to sources of high heat (fire, stovetop, space heater, radiator etc.).
- ▶ Do not grasp or hold any red-hot objects with the product.

 CAUTION

Unintentional unlocking of the terminal device

Injury due to releasing the terminal device from the forearm (e.g. while carrying objects).

- ▶ When connecting the hand to the prosthetic socket or components, ensure that the connection is carried out correctly.

This image shows a single sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

A series of horizontal lines for writing, spanning the width of the page.

This image shows a single sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

Otto Bock Healthcare Products GmbH
Brehmstraße 16 · 1110 Wien · Austria
T +43-1 523 37 86 · F +43-1 523 22 64
info.austria@ottobock.com · www.ottobock.com