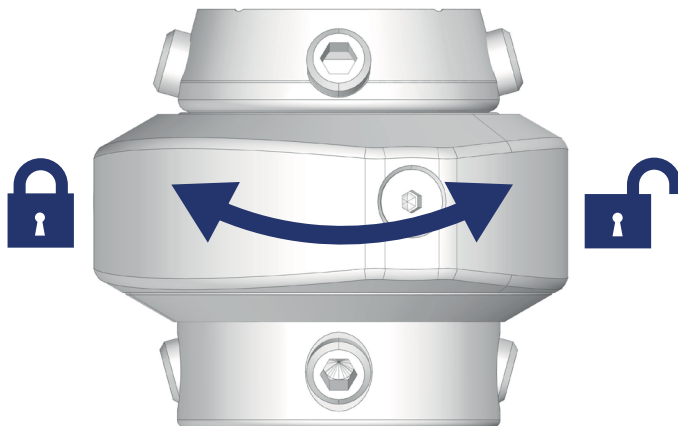
4R11 Quickchange Adapter

DE	Gebrauchsanweisung (Anwender)	3
EN	Instructions for use (user)	5
FR	Instructions d'utilisation (utilisateur)	8
IT	Istruzioni per l'uso (utente)	11
ES	Instrucciones de uso (usuario)	13
PT	Instruções de utilização (utilizador)	16
NL	Gebruiksaanwijzing (gebruiker)	18
SV	Bruksanvisning (användare)	21
DA	Brugsanvisning (bruger)	23
NO	Bruksanvisning (bruker)	26
FI	Käyttöohje (käyttäjä)	28
PL	Instrukcja używania (użytkownik)	31
HU	Használati útmutató (felhasználó)	33
CS	Návod k použití (uživatel)	36

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INFORMATION

Date of last update: 2026-03-31

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Obtain instruction from the qualified personnel in the safe use of the product.
- ▶ Please contact the qualified personnel if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

The 4R11 Quickchange adapter makes it possible to quickly change prosthetic components without tools. The adapter consists of a base and various connectors that are plugged together and locked into place.

2 Safety-related information

CAUTION!

Risk of injury and risk of product damage

- ▶ To prevent mechanical damage, treat the product with care.
- ▶ Observe the maximum lifetime of the product (see page 6).
- ▶ Observe the product's operating conditions (see page 6).
- ▶ Do not expose the product to restricted environmental conditions (see page 6).
- ▶ Check the product for damage and readiness for use prior to each use.
- ▶ Do not use the product if it is damaged or if you are not sure whether it is working properly. Have the product inspected, repaired or replaced by the manufacturer or a specialist workshop.
- ▶ Keep the product away from devices and items that are sensitive to magnetic fields. (e.g. pacemakers, electronic devices, swipe cards). The magnets in the product may affect or damage other devices and objects.
- ▶ Pay attention to the manufacturer's information regarding magnetic fields (e.g. for a medical implant).

Signs of changes in or loss of functionality during use

Among other factors, changes in functionality can be indicated by an altered gait pattern, a change in the positioning of the prosthetic components relative to each other and by the development of noises.

3 Intended use

3.1 Indications for use

The product is to be used **solely** for lower limb exoprosthetic fittings.

3.2 Operating conditions

You can obtain information about the operating conditions of your prosthesis from your O&P professional.

3.3 Environmental conditions

Storage and transport

Storage temperature: -20 °C to +60 °C, relative humidity: 20 % to 90 %, no vibrations or impacts

Allowable environmental conditions

Temperature during use: -10 °C to +45 °C

Chemicals/liquids: Fresh water, salt water, perspiration, urine, soapsuds, chlorine water

Moisture: Submersion: max. 1 h in 3 m depth, relative humidity: no restrictions

Solids: Dust, occasional contact with sand

Clean the product after contact with humidity/chemicals/solids, in order to avoid increased wear and tear and damage (see page 7).

Prohibited environmental conditions

Solids: Highly liquid-binding particles (e.g. talcum), continuous contact with sand

Chemicals/liquids: Acids, continuous use in liquid media

3.4 Lifetime

This product was tested by the manufacturer with 3 million load cycles. Depending on the user's activity level, this corresponds to a maximum lifetime of 5 years.

Recreational sports: The maximum lifetime is 3 years.

Competitive sports: The lifetime is 1 year.

4 Use

Both parts of the adapter can be pulled apart after they are unlocked. After the prosthetic components are changed, the adapter is locked once more.

Open. Clean. Close.

Watch the videos on YouTube for practical examples. Scan the QR codes or use the links

Loosening the canted adapter

Open. Clean. Close:

<https://www.youtube.com/watch?v=K1G025W5M8w>

Loosen the angled adapter:

<https://www.youtube.com/watch?v=6sILNVZeTHE>

INFORMATION: Help children to lock the adapter.

- > **Prerequisite:** Sit down before unlocking the adapter.
 - > **Prerequisite:** Make sure all parts of the adapter are clean and free of dust or sand.
- 1) Rotate the locking ring to unlock the adapter (see fig. 1).
 - 2) Pull apart both parts of the adapter.
 - 3) Use a dry cloth to remove any soiling.
 - 4) Change the prosthetic components and plug the adapter together.
 - 5) Rotate the locking ring to lock the adapter (see fig. 1).
 - 6) Place weight on the prosthesis, for example by standing up, and tighten the locking ring slightly further.

If the adapter cannot be pulled apart after unlocking

Sometimes the two parts of the adapter cannot be easily pulled apart after unlocking. This can happen, for example, if you have not unlocked the adapter for a long time. This is easy to fix:

- 1) Stand in a place where you can hold on tightly.

- 2) Unlock the adapter. Lift the prosthetic leg slightly to the side and then tap the floor with the inside of the prosthetic foot until the adapter loosens.

5 Cleaning

CAUTION

Use of unsuitable cleaning agents or disinfectants

Impairment of functionality and damage due to incorrect cleaning agents or disinfectants

- Clean the product only according to the instructions given in this section.
- For the prosthesis, observe the cleaning instructions for all prosthetic components.

Solids such as dust or sand can prevent the two parts of the adapter from connecting and locking together. As a result, special attention must be paid when removing dust and sand during cleaning.

Cleaning with water

- 1) Unlock the adapter and pull it apart.
- 2) Either rinse the two parts of the adapter with clean water or use a damp cloth.
- 3) Dry the parts with a soft cloth and allow them to air dry.

6 Maintenance

NOTICE! The individual components of the Quickchange adapter show signs of wear differently depending on frequency and intensity of use. Always bring all of the components that can be connected to each other for inspection.

7 Disposal

Do not dispose of the product in household waste. Improper disposal is harmful to health and the environment. Observe the instructions of the local authority regarding return, collection and disposal.

8 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

8.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

8.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

8.3 Local Legal Information

Legal information that applies **exclusively** to specific countries is written in the official language of the respective country of use after this chapter.

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