micelangelo 8E5=*, wrist.flex 10V61=1, wrist.eqd
10V62=1, wrist.short 10V63=1, wrist.transcarpal
10V64=1

DE Gebrauchsanweisung [kurz] (Fachpersonal)	3
EN Instructions for use [short] (qualified personnel)	5

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INFORMATION

Date of last update: 2025-12-18

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report any serious incident related to the product, in particular any deterioration of health, to the manufacturer and the competent authority in your country.

Note the instructions for use

This document contains the sections "Intended use" and "Safety", and does not replace the instructions for use. The instructions for use are available in digital form (PDF format) (see page 2 of this document). To view PDF files, you need an Internet browser or other program capable of opening this file format.

You may request a printed copy of the instructions for use at the e-mail address on page 2. This will be delivered to the specified address within 7 days after the order is received.

The "8E5=* michelangelo" terminal device is referred to as the product/terminal device/hand in the following document.

Basic UDI-DI:406441100000000008E5FG

10V61=1 Basic UDI-DI: 406441100000000010V61JS

10V62=1 Basic UDI-DI: 406441100000000010V62JU

10V63=1 Basic UDI-DI: 406441100000000010V63JW

10V64=1 Basic UDI-DI: 406441100000000010V64JY

2 Intended use

2.1 Indications for use

The product is intended **exclusively** for upper limb exoprosthetic fittings.

2.2 Conditions of use

The product was developed for everyday use and must not be used for unusual activities. These unusual activities include, for example, sports which involve excessive strain and/or shocks to the wrist (pushups, downhill racing, mountain biking, etc.) or extreme sports (free climbing, paragliding, etc.). Furthermore, the product should not be used to operate motor vehicles, heavy equipment (e.g. construction machines), industrial machines or motor-driven equipment.

The product is intended **exclusively** for use on **one** user. Use of the product by another person is not approved by the manufacturer.

2.3 Indications

- For unilateral or bilateral amputation
- Transcarpal/carpal amputation
- Wrist disarticulation
- Transradial amputation (below-elbow amputation)
- Elbow disarticulation
- Transhumeral amputation (above-elbow amputation)
- Shoulder disarticulation
- Interthoracascapular amputation
- Dismelia with characteristics similar to that of an amputation mentioned
- The user must be able to understand usage instructions and safety notices and put them into practice.
- The user must fulfil the physical and mental requirements for perceiving optical/acoustic signals and/or mechanical vibrations

2.4 Contraindications

- Finger/thumb amputation or equivalent dysmelia
- Partial hand amputation and equivalent dysmelia
- Diseased or unhealed condition of skin areas that come into contact with the system in the area of the socket while the prosthesis is being used.

3 Safety-related information

3.1 Meaning of warning levels

WARNING! Failure to follow instructions labelled in this way can lead to serious accidents and injuries.

CAUTION! Failure to follow instructions labelled in this way can lead to accidents and injuries.

3.2 Safety instructions

WARNING!

Risk of injury due to improper use

- ▶ Observe the areas of application and operating conditions of your product in combination with osseointegrated implant systems according to the information of the manufacturer.
- ▶ Note the instructions of the clinical personnel that indicated the use of the osseointegrated implant system.
- ▶ Observe the operating conditions and safety notes stipulated by the manufacturer of the implant.
- ▶ The prosthesis and implant must not be electrically connected.
- ▶ Do not make any changes to the product that have not been approved by the manufacturer.

WARNING!

Risk of injury due to malfunction of the product or interference with other electronic devices

- ▶ Only combine the product with the accessories, signal converters and cables listed in the chapters "Product combinations", "Scope of delivery" and "Accessories". This prevents a malfunction of the product (reduced electromagnetic interference immunity) or interference with other electronic devices (increased emissions).
- ▶ Do not operate the product in the immediate vicinity of other electronic devices.

- ▶ Do not stack the product with other electronic devices during operation.
- ▶ If simultaneous operation with other electronic devices cannot be avoided, monitor the product and verify its intended use in the existing setup.

WARNING!

Risk of injury due to skin contact with allergenic substances

- ▶ Wear suitable gloves during the work to avoid contact with lubricants, production-related residues and components containing nickel.

CAUTION!

Risk of injury due to product malfunction

- ▶ Adjust the prosthesis so that the patient can disengage from the socket in case of complete hand failure.
- ▶ The prosthetic glove must extend beyond the wrist to prevent the penetration of dirt and liquids.
- ▶ For final fitting with the product, only use batteries listed in the section "**Product combinations**" (Product combinations).

CAUTION!

Risk of injury due to incorrect product alignment

- ▶ Do not use silicone spray when applying the glove to ensure a secure fit.

CAUTION!

Risk of injury due to electric shock

- ▶ Do not touch live electrical contacts. See the section Changing the terminal device.
-

Caution: Federal law (USA) restricts this device to sale by or on the order of a practitioner licensed by law of the State in which he/she practices to use or order the use of the device.

Otto Bock Healthcare Products GmbH
Brehmstraße 16 · 1110 Wien · Austria
T +43-1 523 37 86 · F +43-1 523 22 64
info.austria@ottobock.com · www.ottobock.com