317B20

DE	Gebrauchsanleitung (Benutzer / Techniker)	3
EN	Instructions for use (user / technician)	6
FR	Mode d'emploi (Utilisateur / Technicien)	10
IT	Istruzioni d'uso (utilizzatore / tecnici)	14
ES	Instrucciones de uso (usuario / técnico)	18
PT	Manual de instruções (Usuário / Técnico)	22
NL	Gebruiksaanwijzing (Gebruiker/Technicus)	26
SV	Bruksanvisning (användare/tekniker)	30
DA	Brugsvejledning (bruger/tekniker)	34
NO	Bruksanvisning (bruker/teknikker)	37
FI	Käyttöohje (käyttjä/teknikko)	41
HU	Használati utasítás (felhasználó / műszerész)	45
CS	Návod k použití (pro uživatele / techniky)	49
SK	Návod na použitie (používateľ / technik)	52
RU	Руководство по применению (пользователь / техник)	56
ZH	使用说明书 (用户/技术人员)	61

INFORMATION

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INFORMATION

Date of last update: 2022-10-04

- Please read this document carefully before using the product and observe the safety notices.

- ▶ Obtain instruction from the qualified personnel in the safe use of the product.
- ▶ Please contact the qualified personnel if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

These instructions for use provide you with important information on the use of the 317B20 NiMH interchangeable battery.

2 Product description

2.1 Available sizes

The 317B20 battery is available in one size and version.

2.2 Technical data

317B20 Interchangeable battery	
Capacity	800 mAh
Nominal voltage	4.8 V
Weight	120 g
Temperature range	-10 to +50 °C

3 Intended use

3.1 Indications for use

The 317B20 battery is to be used **exclusively** as a power supply for an E-MAG Active orthosis.

3.2 Lifetime

Given intended use and proper installation, the product is designed for a lifetime of **1 year**.

4 Safety

4.1 Explanation of warning symbols

CAUTION

Warning regarding possible risks of accident or injury.

4.2 Safety instructions

CAUTION

Mechanical damage to the product

Injuries due to changes in or loss of functionality

- ▶ Use caution when working with the product.
- ▶ Check the product for proper function and readiness for use.
- ▶ In case of changes in or loss of functionality, discontinue use of the product and have it checked by authorised, qualified personnel.

CAUTION

Incorrect alignment, assembly or adjustment

Risk of injury due to change in or loss of functionality

- ▶ Assembly, adjustment and maintenance operations may only be completed by qualified personnel.
- ▶ Observe the alignment, assembly and adjustment instructions.

CAUTION

Exposure of the product to unsuitable environmental conditions

Patient injury, damage, brittleness or destruction due to improper handling

- ▶ Do not expose the product to condensing ambient humidity or liquids.
- ▶ Do not expose the product to abrasive substances (e.g. sand, dust).

- ▶ Do not expose the product to temperatures below -10 °C or above +50 °C (e.g. sauna, excessive sunlight, hot air gun, drying on a radiator).

5 Preparing the product for use

Charging the battery

NOTICE

Continuous charging of the battery

Damage to the battery

- ▶ Avoid uninterrupted charging for several days

NOTICE

Charging the battery with soiled or damaged contacts

Loss of battery functionality due to insufficient charging functionality

- ▶ Ensure that the contacts are always clean and free of grease
- ▶ Clean all electrical contacts of the battery regularly with a lint-free cloth
- ▶ Take care to avoid damaging the contacts with pointed or sharp objects

INFORMATION

Battery charger malfunction

The LEDs on the charger must not be illuminated prior to inserting the battery.

- 1) Insert one or two 317B20 batteries into the charger (see fig. 1).
INFORMATION: The corresponding LED lights up for each battery. The charging time for a discharged battery is 8-10 hours.
- 2) Take the battery out of the battery charger after charging.

Inserting the battery

- ▶ Insert the battery into the orthosis receptacle and engage it (see fig. 2).

Removing the battery

- 1) Release the battery by pushing the blue lock lever away from the receptacle.
- 2) Take the battery out of the receptacle.

6 Repair

In the event of defects in the electronic components, the entire component must be replaced with a spare part.

7 Disposal

Dispose of the product in accordance with national regulations.

8 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

8.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

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