



100% Recycled

**Congratulations** on purchasing your new BigHorn Ramp. This ramp promises to provide you with years and years of satisfaction and access. BigHorn ramps are made out of 100% recycled plastic material and is made entirely in the United States.

The first thing to do before installing your ramp is measure the height of the door threshold or patio landing. If necessary, simply cut or notch the ramp using normal power tools. (WARNING: always follow manufacture's safety instructions and wear eye protection when using power tools).

If your door threshold or landing is greater than 1-1/2", we offer a "shim-johnny" kit that allows you to shim this ramp up to 2" and is also adjustable for heights less than 2". (Please contact your local distributor or locate your distributor at [www.bighornramp.com](http://www.bighornramp.com) to order your "shim-johnny" kit. Please be advised that shimming this ramp may not make this product federally compliant with model access code laws).

BigHorn ramps come with peel-and-stick tape for easy installation. Simply peel the tape backing as illustrated in (Fig 1). Before installing your ramp, remember to check that your substrate is dry and clean of dirt and debris. Also, it is always a good idea to dry-fit your ramp prior to removing the tape backing from the adhesive strip.

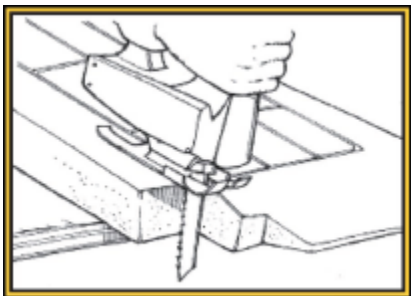


Fig 1

It's really that simple! If you have any questions or want to learn more about the BigHorn Ramp product, please visit our web site at: [www.bighornramp.com](http://www.bighornramp.com). Or e-mail us at [customer@bighornramp.com](mailto:customer@bighornramp.com).

100% Recycled

