




## EZEdge™ & Stonecap™ Threshold Ramp Installation Instructions

1. It is imperative that your substrate is CLEAN & FREE OF DEBRIS before beginning installation. A clean, dry substrate ensures maximum adhesion.
2. Dry fit the product by placing it flush against the threshold.
3. Use painter's tape to outline the ramp.
4. Remove ramp.
5. Generously apply adhesive to substrate. (We strongly recommend *SikaFiex 1a* brand adhesive). For best results, apply with a caulking gun, then use a notched cemenUgrout hand trowel to spread evenly.
6. Replace ramp onto adhesive surface.
7. Apply pressure to ramp evenly. Some glue may leak out onto tape.
8. After ramp is in place with adhesive, carefully remove painter's tape.
9. Excess adhesive can be cleaned from the substrate or product by applying acetone or paint thinner.

\*\*If not permanently adhering the ramp to the sub-floor you may use a "temporary" silicon based adhesive or commercial double stick tape.

**\*\*Note: Excess adhesive will stain clothing, shoes and carpet - immediately clean any excess.\*\***

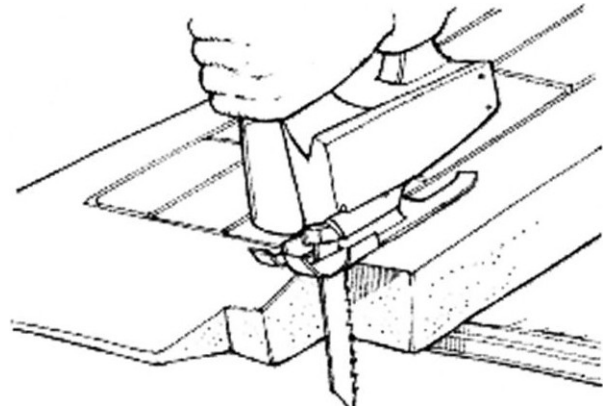

## WARNING

We DO NOT recommend the use of a power circular saw to cut ramp products. The use of a variable speed jig saw will work in some applications, with an aggressive kirfed wood blade. Cut at a slow speed being very careful of "kick back" and getting the blade too hot. Using a utility knife is the best method of trimming and cutting ramps, reducers and flat stock.