





SafeResidential™ Installation Instructions

SafeResidential™ Transitions come in various heights, yet only two configurations are used. You will have either a two or four part set to assemble (See Set 1 or 2).

SET 1

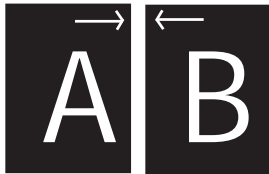

SET 2

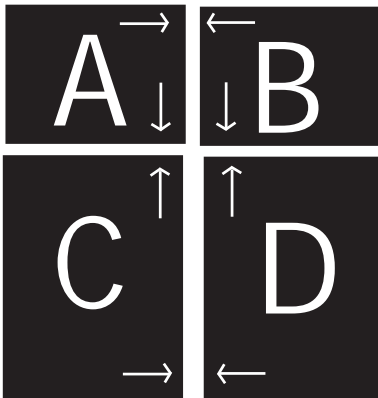

- 1) It is imperative that your substrate is clean and free of debris before beginning installation. A clean, dry substrate ensures maximum adhesion.
- 2) Dry fit the product by placing it flush against the threshold rise.
- 3) Use painter's tape to outline the ramp (see fig #1)
- 4) Remove ramp
- 5) Generously apply adhesive to substrate. (We recommend SikaFlex 1-a brand). For best results, apply with a caulking gun, then use a notched cement/grout hand trowel to spread evenly.
- 6) Place ramp sections onto freshly applied adhesive
- 7) Apply even pressure to ramp. Adhesive may leak out onto tape and substrate*
- 8) After ramp is in place with adhesive, carefully remove painter's tape.
- 9) Excess adhesive can be cleaned from the substrate or product by applying acetone or paint thinner.

*Note: excessive adhesive will stain clothing, shoes and carpet – immediately clean any excess with denatured alcohol.



Note: Sections of the SafeResidential™ Ramp has been prefitted and simply require placement as described in either set 1 or 2.

