**MGH00848**

Valid from / en cours de validité depuis le / en espera de validación desde el / in corso di validità dal / gültig seit dem

05/04/2019

**Product Sheet (EN)**

**2**

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**4**

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**6**

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**8**

**Eigenschaften (DE)**

**10**

PRODUCT SHEET				valid from 05/04/2019
IDENTIFICATION OF THE MEDICAL DEVICE	Type	Rechargeable battery		
	Commercial designation	Batterie médicale 6x AA 6S1P ST5 7.2V 2.5Ah JST		
	Reference	MGH00848		
	EAN	3660766621283		
	Brand	NX		
	Compatible / Original battery	Compatible		
Packaging	Unitary			
<b>RECOMMENDED USAGE</b>				
<i>Follow the instructions and recommendations specific to each model, using the technical instructions and document resources from the devices in which the battery is used</i>				
<b>Brands</b>	<b>Equipment</b>	<b>Models</b>	<b>PN</b>	
Kern	Batterie médicale pour balance	MBC / MPC / MPD / MPE / MTA / MWA		
Identification	GENERAL TECHNICAL CHARACTERISTICS	Chemistry	NiMH	
		Type	AA	
		IEC designation	6 HRH 15/49	
		Rated voltage	7.2V	
		Nominal capacity	2.5Ah	
		Internal resistance $\Omega$	240m $\Omega$	
<i>The voltage and the actual capacity in use can be affected by several factors, especially the temperature, the discharge current, the pack's history (ex:use, storage), etc</i>				
ELECTRICAL CHARACTERISTICS	CHARGE	Maximum charging voltage	10.2V	
		Standard charging current (15h)	250mA	
		Fast charging current (2,5h)	1250mA	
	DISCHARGE	Range of operating voltage	8V at 9.5V	
		Min tension in discharge	6V	
		Max discharge current	2000mA	
		Lifespan 80% DOD (0,5 C)	> 500 cycles	
	MAINTENANCE	Frequency of maintenance charges at 20°C	3 Months	
	CONTROL ELECTRONICS	Electrical protection	Yes	
		Low voltage power cut	No	
High voltage power cut		Yes		
Max power cut voltage		Yes		
<i>These devices not only designed to protect the pack in case of an equipment failure. They must not be used to control the discharge. The protection circuits have a response time of a few milliseconds.</i>				
MECHANICAL CHARACTERISTICS	Dimensions (+/- 2mm)	Length	30.45 mm	
		Width	153.51 mm	
		Depth	14.72 mm	
	Weight (+/- 5g)	183.8 Gr.		
	Mechanical protection	Shrink sleeve		
	Wire length (+/- 10mm)	50 mm		
Terminal	JST XHP			
CONDITIONS OF USE, STORAGE, AND TRANSPORT	CONDITIONS OF USE	Charging temperature	0 at 45 °C	
		Discharge temperature	-20 at 60 °C	
	CONDITIONS OF STORAGE	Storage temperature	-20 at 60 °C	
		Level of humidity	65.00 %	
		Max storage time	2 Years	
	TRANSPORT	UN code	3496	
		ADR/RID classification	Class 9	
		IMDG classification	Class 9	
IATA classification		Class 9		

<b>INSTRUCTIONS</b>	<b>COMMISSIONING</b>	<ul style="list-style-type: none"> <li>• Check the batteries and the connectors: wires not damaged, battery not swollen, burnt smell, oxidation of the connectors, leak...</li> <li>• Respect the polarity</li> <li>• Do a full charge with the adequate charger before the first use</li> </ul>
	<b>CHARGE</b>	<ul style="list-style-type: none"> <li>• Use an adequate charger</li> <li>• The battery is warmer during the charge: during the first charge, check that the battery's temperature stays in the temperature operating ranges.</li> <li>• In case of an abnormal heating, stop the charge by unplugging the charger within the realms of possibility, remove the battery from the equipment de l'équipement, have the equipment, the battery and the charger checked by a technician.</li> </ul>
	<b>CASE OF NON-WATERPROOF BATTERIES</b>	<ul style="list-style-type: none"> <li>• It is normal to observe a release of gas during the charge and use. Do not smoke. Place in suitable premises.</li> <li>• Open batteries need regular maintenance carried out by a qualified technician.</li> </ul>
	<b>CASE OF LITHIUM ION BATTERIES</b>	There is a fire hazard with lithium ion batteries in the following cases: overload, short circuit, charge and use outside the voltage and temperature ranges.
	<b>WARNINGS</b>	<ul style="list-style-type: none"> <li>• Read the instructions of your device.</li> <li>• Only use in compatible devices.</li> <li>• Respect the load and storage conditions.</li> <li>• Do not use if the battery is damaged, do not burn, do not pierce, do not dismantle or modify. The protection circuits protect the battery and the equipment: do not deactivate them.</li> </ul>

**EXPLANATION OF SYMBOLS**

<b>EXPLANATION OF SYMBOLS</b>	<b>LABELLING</b>	
		Catalogue reference
		Lot number
		manufacturer's address
		
		To recycle in a suitable salvage and recycling structure
		
	Read the product sheet and the instruction manual	

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