**Intended use**

The LigaFit is a portable, non-sterile, non-invasive medical device intended for long-term immobilization / support of the knee during injury to its ligaments. The device also enables the controlled (re) mobilization of the joint by setting limits for flexion and extension.

**Indications**

- ✓ Ruptures of the anterior, posterior or both cruciate ligaments with accompanying injuries;
- ✓ Isolated ruptures of the anterior or posterior cruciate ligament;
- ✓ Rupture and/or instability of the collateral ligaments;
- ✓ After reconstruction of the anterior, posterior or both cruciate ligaments;
- ✓ For conservative treatment of joint knee instability.

**Contraindications**

If undue pain, unpleasant sensation or allergy develops while you are wearing the brace, please take it off immediately and consult your healthcare professional.

**Fitting instructions**

The device acts as a brace to stabilize the knee.

It is applied by:

1. Open all the straps. Sit on the edge of a chair, pull the brace up the affected leg and flex your knee to 45°. Position the joints so that the center of the hinges (extension wedges) are on a level with the top edge of the knee cap.
  - ⚠ Make sure that the brace is not twisted on your leg.
2. First fasten the lower front calf strap, then the upper back thigh strap. Keep your knee in a flexed position and push both joints towards the back of the leg;
3. Now fasten the lower back thigh strap.
  - ⚠ Make sure that you do not have the hinge joints positioned in front of the midline of the leg.
4. Now fasten the upper front calf strap. Then the upper back calf strap, and finally the front thigh strap.
5. Bend your knee slightly a few times and refasten the straps again in order to prevent the brace from slipping.

**Setting the flexion & extension limits**

1. 10° extension wedges are pre-installed in your knee brace as standard.
2. To replace the wedges, first loosen the screws in the joint cover, then take the cover off the wedges can now be removed by being gently pulled out in a forward direction.
3. Replacement extension wedges (silver) can be found under wedge set marked R-E (right) and L-E (left). Slide the new wedge into the joint from the front and secured by tightening the screws. Insert the wedge so that the letter shows on the joint plate points upward. Now insert the screws in the thread and screw it tight.
4. To replace the black flexion wedge loosen the screws and pull the wedge out from the back of the brace. Select a suitable wedge from the wedge set marked R-F (right) and L-F (left). Insert the wedge so that letter shown on the joint plate points upwards. Now insert the screw in the thread and screw it tight.
5. Bend and flex the brace joint a couple of times and check that the wedges are positioned correctly. Finally, secure the joint cover.

⚠ The medial (inside) and lateral (outside) limit settings must always be the same.

**Care instructions**

The brace can be worn in both fresh and salt water. Rinse the brace with clean water afterwards. Leave to dry naturally.

The inner pads and the condyle pads can be washed by hand with a mild soap/detergent. Leave to dry naturally.

- ⚠ Do not wring out. Air dry. Do not iron. Remove joints before washing. Do not put the device in a washing machine or dryer.

When used normally the brace is maintenance-free. However, in exceptional circumstances the hinges can be lubricated using Teflon™ spray.

**Storage instructions**

Store in dry place and do not expose to direct sunlight and moisture.

**Warranty**

Currently valid legal conditions applicable.

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**Important notes**

- The brace is recommended for one user only.
- The brace is not intended to be applied on breached skin.
- Respect the cleaning instructions.
- Remove device before showering.

If allergic skin rashes occur, please consult your physician. Any serious incidents that occurred in relation to this device should be reported to the manufacturer and the competent authority of the member state in which the user is established.

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