Intended use

The ROM Knee Brace is a portable, non-sterile, non-invasive medical device intended for short-term immobilization / support of the knee joint. The device also enables the controlled (re)mobilization of the joint by setting limits for flexion and extension.

Indications

- ✓ Following surgery of the knee joint;
- ✓ For immobilization after knee joint injuries and subsequent controlled mobilization;
- ✓ For reliable stabilization of the collateral ligaments after injury or surgery.

Contraindications

If undue pain, unpleasant sensation or allergy develops while you are wearing the brace, please take it off immediately and consult your healthcare professional.

Failure to persevere good personal hygiene, usage of the wrong detergent can cause abrasions or other ailments associated with increased sweating and bacterial growth.

Fitting instructions

The device acts as a brace to stabilize the knee. It is applied by:

1. Open all the straps and take the joint braces out of the pads.
2. Wrap the calf and thigh pads tightly around the leg and fasten at the front.
3. First attach the lateral (outside) brace to the pad. Make sure that the brace joint is level with the patella and the midline of the knee.
4. Now attach the medial (inside) brace in the same way. Make sure that the two joints are at the same level.
5. First fasten the upper thigh strap, then the lower thigh strap. Now fasten the other straps, starting with the bottom one and working upwards.

To take the orthosis off just open the straps and pads and pull the orthosis off backwards.

Setting the flexion & extension limits

To adjust the range of movement rst turn the opening in the plastic to expose one of the pins. Now press and hold the red button, at the same time turn to set the desired degree figure. Repeat this for the second pin.

- ✓ Extension limitation: 0°, 10°, 20°, 30°
- ✓ Flexion limitation: 0°, 10°, 20°, 30°, 45°, 60°, 75°, 90°, 105°, 120°

⚠ The medial (inside) and the lateral (outside) wedge settings must always be the same.

Care instructions

Wash by hand in mild, cold soapy water. Rinse thoroughly.

⚠ Do not wring out. Air dry. Do not iron. Remove joints before washing. Do not put the device in a washing machine or dryer.

**Storage instructions**

Store in dry place and do not expose to direct sunlight and moisture.

Warranty

Currently valid legal conditions applicable.

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⚠ **Important notes**

- The brace is recommended for one user only.
- The brace is not intended to be applied on breached skin.
- Respect the cleaning instructions.
- Remove device before showering.

If allergic skin rashes occur, please consult your physician. Any serious incidents that occurred in relation to this device should be reported to the manufacturer and the competent authority of the member state in which the user is established.

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