ENGLISH

Intended use

Your doctor has prescribed the 3-panel knee splint for you. Please read through these instructions carefully to ensure that you fit your brace correctly.

Indications

- ✓ To immobilize the knee and help reduce pain
- ✓ May be used following surgery, fractures and sprains

Contraindications

If undue pain, unpleasant sensation or allergy develops while you are wearing the brace, please take it off immediately and consult your healthcare professional.

Failure to persevere good personal hygiene, usage of the wrong detergent can cause abrasions or other ailments associated with increased sweating and bacterial growth.

Not for use directly over open wounds and infected areas.

Fitting instructions

1. The front panels of the 3-panel knee splint can be repositioned to accommodate dressings and fluctuations in edema.
2. While the person is seated or lying, position the immobilizer with the wider end proximal around the leg.
3. Center the patellar opening over the knee.
4. Fasten the hook straps across the front of the leg, fastening the top strap first, then the bottom strap, and finally the middle straps.

ENGLISH

Care instructions

Remove the stays from the front panels before reapplying the immobilizer.

Wash by hand in mild, cold soapy water. Rinse thoroughly.

- ⚠ Do not wring out. Air dry. Do not iron.
- Do not put the device in a washing machine or dryer.

Be sure the immobilizer is completely dry before reapplying it.

Storage instructions

Store in dry place and do not expose to direct sunlight and moisture.

Warranty

Currently valid legal conditions applicable.

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Important notes

- The brace is recommended for one user only.
- The brace is not intended to be applied on breached skin.
- Respect the cleaning instructions.
- Remove device before showering.

If allergic skin rashes occur, please consult your physician. Any serious incidents that occurred in relation to this device should be reported to the manufacturer and the competent authority of the member state in which the user is established.

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