Intended use

The Patella Track-o-Fit is a portable, non-sterile, non-invasive medical device intended for short-term immobilization and support of the knee joint.

Indications

- ✓ Mild instabilities of the knee joint/ patella;
- ✓ Mild instabilities of the cruciate ligaments;
- ✓ Meniscal injuries;
- ✓ Strain of the collateral ligaments;
- ✓ For protection during the later stages of rehabilitation;
- ✓ Traumatic or degenerative changes of the knee;
- ✓ Patellofemoral pain syndrome;
- ✓ Patellar chondropathy;
- ✓ Retropatellar osteoarthritis.

Contraindications

If undue pain, unpleasant sensation or allergy develops while you are wearing the brace, please take it off immediately and consult your healthcare professional.

Fitting instructions

The device acts as a brace to stabilize the knee.

It is applied by:

1. Open the thigh and calf straps. Remove the straps from one of the two Velcro strips;
2. Pull the brace up over the knee with help of the loops at the top of the brace OR wrap the brace around the leg and fasten the front Velcro straps;
3. Position the polycentric joint so that the middle of the hinges are level with the upper border of the kneecap;
4. Tighten the calf strap first, then the thigh strap.

The openings for removing the joints can be found at the bottom left or at the top right.

Care instructions

Wash by hand in mild, cold soapy water. Rinse thoroughly.

- Do not wring out. Air dry. Do not iron.
- Remove joints before washing.
- Do not put the device in a washing machine or dryer.

When used normally the brace is maintenance-free. However, in exceptional circumstances the hinges can be lubricated using Teflon™ spray.

Storage instructions

Store in dry place and do not expose to direct sunlight and moisture.

Warranty

Currently valid legal conditions applicable.

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Important notes

- The brace is recommended for one user only.
- The brace is not intended to be applied on breached skin.
- Respect the cleaning instructions.
- Remove device before showering.

If allergic skin rashes occur, please consult your physician. Any serious incidents that occurred in relation to this device should be reported to the manufacturer and the competent authority of the member state in which the user is established.

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