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REF. G1125 · G1125A · G1127 · G1127A · G1129 · G1129A · G1130 · G1130A

Knee orthosis with polycentric hinges with F-E control

DECLARATION OF CONFORMITY

As the manufacturer, ORTHOSERVICE AG states under its sole responsibility that this product is a Class I medical device that has been manufactured in accordance with the requirements of Directives 93/42/ EEC and 2007/47/EC. These instructions have been written in conformity with these directives, and their aim is to ensure the safe and proper use of this medical device.

TRADEMARKS OF MATERIALS

Velcro® is a registered trademark of Velcro Industries B.V.

PRECAUTIONS

It is advised that the pressure exerted by the product does not act on parts of the body suffering from wounds, swelling or lumps. The product should not be tightened too firmly so as to avoid the development of pressure points or compression of the nerves or blood vessels underneath. If in doubt about how to apply, contact an orthopaedic technician. Please read the composition of the product on the inside label carefully. The device should not be worn in the vicinity of open flames or strong electromagnetic fields.

WARNINGS

The product has been designed for the specific indications stated below and must be prescribed by a doctor and fitted by an orthopaedic technician. The orthopaedic technician is responsible for fitting the product and providing information about how to use it safely in accordance with individual requirements. The product must be fitted with the utmost care to ensure its efficacy, tolerability and correct function.

Any changes to the structure or adjustment of the device must be prescribed by a physician and performed by an orthopaedic technician. Under no circumstances should the adjustment made by the physician/orthopaedic technician be altered. The manufacturer is no longer liable if the product is used or adapted inappropriately. The orthosis is intended to be used only by a single patient; if it is used for more than one patient, the manufacturer accepts no responsibility or liability in accordance with medical device legislation. Direct contact with the skin may cause reddening or irritation in hypersensitive individuals. In the case of onset of pain, swelling, lumps or any unusual reaction, immediately contact your physician. The orthopaedic efficacy of the product may only be guaranteed when all of its components are used.

WARRANTY

This warranty is governed by the statutory provisions of the country in which the product was purchased. Where relevant that country's regulations concerning warranties and the relationship between the seller and buyer shall apply. If you believe that the terms of the warranty may be applicable, first contact the retailer where the product was purchased. Orthoservice will repair or replace all or part of the device and its accessories if defects in materials and workmanship arise during the period of six months from the date of purchase.

SELECTION/SIZE

Code	REF.G1125/G1125A/G1127/G1127A/G1129/G1129A/G1130/G1130A
Size	XS S M L XL XXL
Thigh circ. taken 15 cm above mid-patella cm	33/39 39/45 45/52 52/58 58/66 66/75
Length cm Genüfit 25/25A/29/29A	34 34 34 34 34 34
Length cm Genüfit 27/27A/30/30A	41 41 41 41 41 41
Colour	black

Fits right and left

MAINTENANCE

- ☒ Do not bleach ☒ No chemical cleaning
- ☒ Do not iron ☒ Do not tumble-dry
- ☒ Washing instructions: hand wash at max. 30° C with neutral soap, do not dry in the vicinity of heat sources.

Do not dispose of the product or any of its components into the environment.

INDICATIONS

- Functional treatment of sprain injuries grade I and II of the collateral medial/lateral ligament
- Prevention of relapsing MCL and LCL sprains during sports activities
- Chronic latero-medial instability
- Femoro-patellar arthrosis
- Arthritic instability (inoperable knee)

CONTRAINDICATIONS

Currently no known

CHARACTERISTICS AND MATERIALS

- New anatomical, tight-fit design allows flexible movement and adjusts optimally to the joint
- Breathable, skin-friendly AirX™ fabric and Velcro® fastening straps
- New, very soft and seamless popliteal insert made of Soft-X fabric for pleasant wearing comfort
- New soft elastic border
- Genüfit 25/27/29/30 only: Patella opening with foam stabiliser
- Genüfit 25A/27A/29A/30A only: New front opening for easier application: internal fastening system for prefixing and external fastening system for final fixing. Patella cut-out
- Genüfit 25/25A/27/27A only: Very flat polycentric hinge, easily adjustable using wedges: Extension: 0°, 10°, 20°, 30°, 40°. Flexion: 45°, 60°, 75°, 90°. Immobilisation: 0°, 10°, 20°, 30°
- Genüfit 29/29A/30/30A only: Polycentric hinge which can be easily adjusted externally using pins: Extension, flexion and immobilisation at: 0°, 15°, 30°, 60°, 90°
- Available in two lengths: 34 cm (Genüfit 25/25A/29/29A) and 41 cm (Genüfit 27/27A/30/30A)
- Genüfit 27/27A/30/30A only: Stretch insert in the back, at the proximal part of the thigh*

PUTTING ON THE APPLIANCE - ADAPTING FOR THE DOCTOR/TECHNICIAN

- Genüfit 25 - Genüfit 27 - Genüfit 29 - Genüfit 30**
Fully open the securing straps, pull out of the appropriate loop and release the Velcro® closure from the knee brace. Pull the knee brace over the knee so that the orange label is facing upwards (fig. A). The cut-out on the front must lie precisely over the patella.
Genüfit 25A - Genüfit 27A - Genüfit 29A - Genüfit 30A
Open the securing straps, pull the tapes out of the appropriate loop and release the Velcro® closure from the knee brace. Position the knee brace on the extended knee in such a way that the orange label faces upwards. The cut-out on the front should be located over the patella as precisely as possible. Secure the Velcro® closures (inner one first (fig. B and C), then the outer Velcro® closures (fig. D and E), always start with the topmost Velcro® fastener).
- If necessary, remove the brace, pull the joint braces out of the shaft, bend to shape, insert back into the shaft and put the knee brace back on again.
- Guide each strap through the appropriate buckle and tighten the Velcro® closure until the desired tension is achieved, then secure. First close the straps at the thigh (fig. F and G), then at the calf (fig. H). For the longer versions of the knee brace (REF. G1127, REF. G1127A, REF. G1130 and REF. G1130A) first close the distal strap at the thigh.
- Genüfit 25 - Genüfit 25A - Genüfit 27 - Genüfit 27A**
If necessary set the flexion/extension restriction using the wedges (fig. I-K-L): For this purpose, loosen the screws, change the wedges and put the screws back in place.
• Extension restriction: 0° - 10° - 20° - 30° - 40°
• Flexion restriction: 45° - 60° - 75° - 90°
• Possible joint block at 0° - 10° - 20° - 30°
- Genüfit 29 - Genüfit 29A - Genüfit 30 - Genüfit 30A**
If necessary set the flexion/extension restriction using the wedges (fig. M-N-O): Remove the cover, position the wedges at the appropriate degree and close the cover again.
• Flexion/extension restriction: 0° - 15° - 30° - 60° - 90°
• It is possible to fully block the joint

PUTTING ON THE APPLIANCE FOR THE PATIENT

- Genüfit 25 - Genüfit 27 - Genüfit 29 - Genüfit 30**
Fully open the securing straps, pull out of the appropriate loop and release the Velcro® closure from the knee brace. Pull the knee brace over the knee so that the orange label is facing upwards (fig. A). The cut-out on the front must lie precisely over the knee cap.
Genüfit 25A - Genüfit 27A - Genüfit 29A - Genüfit 30A
Open the securing straps, pull the tapes out of the appropriate loop and release the Velcro® closure from the knee brace. Position the knee brace on the extended knee in such a way that the orange label faces upwards. The cut-out on the front should be located over the knee cap as precisely as possible. Secure the Velcro® closures (inner one first (fig. B and C), then the outer Velcro® closures (fig. D and E), always start with the topmost closure).
- Guide each strap through the appropriate buckle and tighten the Velcro® closure until the desired tension is achieved, then secure. First close the straps at the thigh (fig. F and G), then at the calf (fig. H). For the longer versions of the knee brace (REF. G1127, REF. G1127A, REF. G1130 and REF. G1130A) first close the distal strap (the one furthest from the centre of the body) at the thigh.

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Velcro® è un marchio di fabbrica registrato di Velcro Industries B.V.