Intended use

The UomoFit with Zipper is a portable, non-sterile shoulder brace to be used for short-term immobilization and support of the shoulder while injured.

Indications

The UomoFit is used after:

- ✓ Bankhart repair;
- ✓ Capsular shift;
- ✓ Soft tissue repair / strain;
- ✓ Shoulder instability/injury.

The final decision concerning the usage and selection of the appliance shall be made by the healthcare professional.

Contraindications

 UomoFit is not indicated for patients with pacemaker.

Fitting instructions

- ✓ Position the forearm in a neutral position, this means parallel with your body. The fingers are facing the front.
- ✓ Make sure that the magnetic buckles are closed around the strap with the red sewing.
- ✓ Close the strap with the red sewing.
- ✓ Place the adjustable strap around the neck.
- ✓ Bring the outer magnetic lock with the blue sewing (the one that is the most far away from the body) to the buckle and it will lock automatically.
- ✓ Bring the inner lock to the buckle in order to secure the strap.
- ✓ Place the strap with the green sewing into the lock.
- ✓ Use the single strap to fasten your forearm.

Care instructions

Used materials: Knitted fabrics out of cotton and/or polyamide and/or polyester

Cleaning: The UomoFit may be cleaned by hand using a mild soap and lukewarm water. Hang to dry.

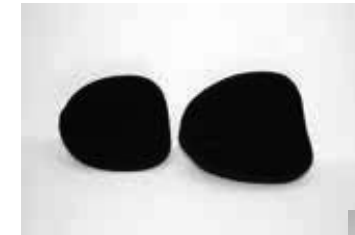
 Do not wring out. Do not iron.
Do not put the UomoFit in a washing machine or dryer.

Warranty

Currently valid legal conditions applicable.

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Important notes

- The brace is recommended for one user only.
- The brace is not intended to be applied on breached skin.
- Respect the cleaning instructions.
- Remove device before showering.

If allergic skin rashes occur, please consult your physician. Any serious incidents that occurred in relation to this device should be reported to the manufacturer and the competent authority of the member state in which the user is established.

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