ENGLISH

Intended use

The FitLine ROM Elbow brace is a portable, non-sterile, non-invasive medical device intended for long-term immobilization and support of the elbow joint while injured. It enables controlled mobilization of the joint by setting limits for flexion and extension.

Indications

- ✓ Conservative treatment for dislocation of the elbow with associated injury;
- ✓ Orthotic care for fractures of the medial or lateral epicondyles;
- ✓ Post-operative treatment following implantation of prostheses;
- ✓ Functional aftercare following stable fractures of the distal humerus;
- ✓ Temporary immobilization after arthrolysis of the elbow joint.

Contraindications

If undue pain, unpleasant sensation or allergy develops while you are wearing the brace, please take it off immediately and consult your healthcare professional.

Fitting instructions

1. Open all velcro straps and put the brace on so that its joint lies directly over the lateral epicondyle. The plastic casts and pads can be trimmed to size with scissors if necessary.
2. First fasten the upper forearm strap and then the lower humerus strap. Then fasten the lower forearm strap and the upper humerus strap.
3. Guide one end of the neck strap through the ring on the hand rest, and fasten the velcro.
4. The neck pad can be adjusted individually, but it should lie over the middle of the shoulder. You can adjust the length of the arm rest with the two screws on the forearm splint.

Setting the flexion/extension limits

To adjust the range of motion first turn the opening in the plastic to expose one of the pins. Now press and hold the red button, at the same time turn to set the desired degree figure. Repeat this for the second pin.

Extension and flexion limits: 0° to 120°.

-  The medial (inside) and lateral (outside) limit settings must always be the same.

ENGLISH

Care instructions

The pads can be washed by hand with a mild soap/detergent. Rinse thoroughly. Leave to dry naturally.

-  Do not wring out.
Do not iron.
-    

When used normally the brace is maintenancefree. However, after using the brace in water, the hinges can be lubricated using Teflon™ spray. Check regularly that the joint screws are secure, tighten them if necessary.

Storage instructions

Store in dry place and do not expose to direct sunlight and moisture.

Warranty

Currently valid legal conditions applicable.

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CAUTION!

Important notes

- The brace is recommended for one user only.
- The brace is not intended to be applied on breached skin.
- Respect the cleaning instructions.
- Remove device before showering.

If allergic skin rashes occur, please consult your physician. Any serious incidents that occurred in relation to this device should be reported to the manufacturer and the competent authority of the member state in which the user is established.

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MD: medical device | Other: single patient multiple use