REDUCE PAIN & SWELLING

WITH THE LATEST COLD COMPRESSION TECHNOLOGY

IMPORTANT NOTES

This product is recommended for one patient use only. If it is used for treating more than one patient, the manufacturer's product liability according to the Medical Devices Act will become invalid.

The final decision concerning the usage and selection of the appliance shall be made by the doctor or medical specialist.

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Manual CCT Knee 05/03/2018

dr. **aktive**
LIVE LIFE

Knee Support
Cold Compression Therapy

Handleiding
Instructions for use
Mode d'emploi
Gebrauchsanweisung

Easy Use

Durable

Lightweight

INSTRUCTIONS FOR USE

ENGLISH

Your doctor has prescribed the **CC Knee Pad** for you. Please read through these instructions carefully to ensure that you fit your brace correctly.

IMPORTANT

This product is recommended for one patient use only. If it is used for treating more than one patient, the manufacturer's product liability according to the Medical Devices Act will become invalid.

The final decision concerning the usage and selection of the appliance shall be made by the doctor or medical specialist.

INDICATIONS

- Sprains
- Sport injuries
- Rehabilitation
- Arthritis
- Muscle tiredness

CONTRA-INDICATIONS

No contra-indications are known until today.

WASHING INSTRUCTIONS

Store the product under well ventilated and dried condition and avoid direct sunlight.

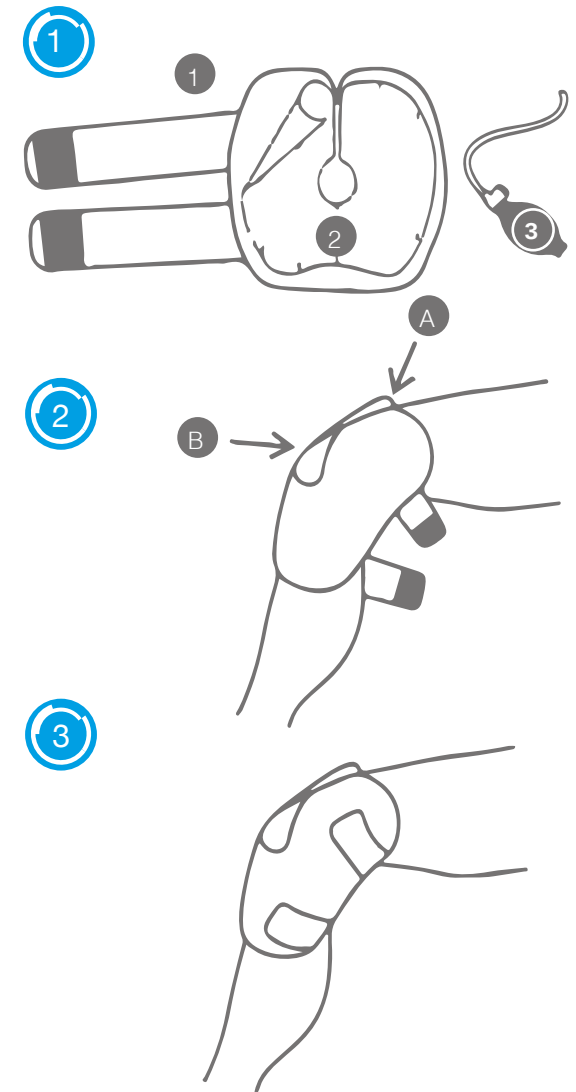
Only use mild detergents and warm water to wash the product. Do not use strong detergents!

Using the washing machine and the spin-drier to wash and dry the product is not recommended.

FITTING INSTRUCTIONS

STEP BY STEP

- 1 Check the parts first. (1) compressor (2) gel pack and (3) pump with tube.
- 2 Refrigerate (2) at least 2 hours before usage.
- 3 Put (2) back on the (1) after refrigeration.
- 4 Connect the tube to the ON/OFF switch and turn "ON" the switch.
- 5 Position (A) on the center of the thigh. The (B) hole aims at the knee joint. (Picture 2).
- 6 Stick the two tapes on the appropriate position and strap them (Picture 3).
- 7 When turning ON/OFF the switch, please fixate it with one hand and turn it with the other to ensure the switch is on the appropriate position and then pump.
- 8 To get the best effect: ON for 5-10 minutes, OFF for 5-10 minutes to let the skin rest and then repeat.
- 9 Empty it before taking off and then refrigerate (2) for next usage.

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