

Aquaplast-T™

Moderate stretch. Coated.

Characteristics

- **Versatile** material: Has a moderate stretch, flexibility and conformability.
- Retains shape without reinforcement.
- White colour **turns transparent** when heated, making pressure points and markings visible.
- **100% memory** allows for repeated reheating and reshaping for splint revisions.
- **With coating:** Create a temporary bond by pinching together heated.

Heating



- 1,6 mm: 70°C to 75°C for 35 seconds
- 2,4 mm: 70°C to 75°C for 1 minute
- 3,2 mm: 70°C to 75°C for 1 minute
- 4,8 mm: 70°C to 75°C for 1-2 minutes

Handling time



- 1,6 mm: 1 to 2 minutes
- 2,4 mm: 2 to 3 minutes
- 3,2 mm: 4 to 6 minutes
- 4,8 mm: 4 to 7 minutes

Reference	Perf.	Colour	Dimensions	Quantity
-----------	-------	--------	------------	----------

1.6 mm thickness

A96240	Solid	White	46 x 61 cm	1 sheet
A96240C4	Solid	White	46 x 61 cm	4 sheets
A962541	13 %	White	46 x 61 cm	1 sheet
A96254	19 %	White	46 x 61 cm	1 sheet

2.4 mm thickness

A96270	Solid	White	46 x 61 cm	1 sheet
A96270C4	Solid	White	46 x 61 cm	4 sheets
A962090	2 %	White	46 x 61 cm	1 sheet
A96255	19 %	White	46 x 61 cm	1 sheet
A96271	38 %	White	46 x 61 cm	1 sheet

3.2 mm thickness

A96210	Solid	White	46 x 61 cm	1 sheet
A9621	Solid	White	46 x 61 cm	4 sheets
A96272	Solid	White	61 x 91 cm	1 sheet
A96230	1 %	White	46 x 61 cm	1 sheet
A96230C4	1 %	White	46 x 61 cm	4 sheets
A96273	1 %	White	61 x 91 cm	1 sheet
A96256	19 %	White	46 x 61 cm	1 sheet
A96220	42 %	White	46 x 61 cm	1 sheet

4.8 mm thickness

A96274C4	Solid	White	46 x 61 cm	4 sheets
A96275C2	Solid	White	61 x 91 cm	2 sheets

Special features

- **Combination of strong conformability and elasticity**
- 13% UltraPerf®: ideal for light finger or hand splints made of breathable material
- 19% OptiPerf™: excellent ventilation while maintaining material strength
- Easy to cut edges

Clinical recommendations

- Serial splints
- Hand, wrist, finger and thumb splints
- Dynamic outrigger bases

Bonding capacity

- Temporary bonding:
pinch warm surfaces together
- Permanent bonding:
take off the coating with spiritus or by scraping with a sharp instrument

100% memory,
with coating.

