Intended use

The (ROM) Walker is portable, non-sterile, non-invasive medical device intended for short-term stabilization and support of the ankle joint while injured. The ROM device enables the controlled (re)mobilization of the joint by setting plantar- and dorsiflexion.

Indications

- ✓ Post ligament, soft tissue and tendon repairs;
- ✓ Conservative or post-operative treatment of fractures (malleoli, calcaneus, talus,...).

Contraindications

If undue pain, unpleasant sensation or allergy develops while you are wearing the brace, please take it off immediately and consult your healthcare professional.

Fitting instructions

1. Open all the velcro fasteners and the leg/foot padding. Place the foot in the padding so that the heel stands firmly in the pre-formed heel bed and close the padding. Patients with very slim calves can trim the pad to the right length using scissors if there is a large overlap of the padding.
2. Place your foot in the rigid shell of the walker. The two guide supports should lie exactly along the middle of the inside and outside of the calf. then close both the Velcro fasteners of the foot section and the back heel strap.
3. Finally fasten the three leg straps from below upwards. Make sure that the straps lie flat against the pad and are tightly closed at the front.

- ⚠ Be careful not to impair the blood circulations.
For additional stabilisation of the lower leg the enclosed plastic shells can be fixed to the pad. The broader one is intended for the calf area. The padding can be used for additional cushioning (e.g. across the instep, at the heel strap, between the ankle bone and joint).

Adjustment of plantar- and dorsiflexion

To adjust the range of movement, turn the opening in the plastic cover to the current position of the pins. Press the pins in and slide to the desired degree figure.

- ✓ Plantarflexion: 0°, 10°, 20°, 30°, 40°
- ✓ Dorsiflexion: 200, 10°, 0°

- ⚠ The medial (inside) and the lateral (outside) wedge settings must always be the same.

Information for the technician

Where the joint is fixed at an angle other than 0° (e.g. immobilised at 30° plantarexion), the patient will require additional walking supports. If the patient is intended to be able to put weight on the foot, a plantarflexion can also be achieved by elevating the heel using the heel wedge set which is available separately, whereby the joint is then fixed at 0°.

Care instructions

Wash by hand in mild, cold soapy water. Rinse thoroughly.

- ⚠ Do not wring out. Do not iron.
Remove the joints before washing.

When used normally the brace is maintenance free. However, in exceptional circumstances the hinges can be lubricated using Teflon™ spray.

Warranty

Currently valid legal conditions applicable.

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Important notes

- The brace is recommended for one user only.
- The brace is not intended to be applied on breached skin.
- Respect the cleaning instructions.
- Remove device before showering.

If allergic skin rashes occur, please consult your physician. Any serious incidents that occurred in relation to this device should be reported to the manufacturer and the competent authority of the member state in which the user is established.

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