INSTRUCTIONS FOR USE ENGLISH

- 1 Check the parts first. (1) compressor (2) gel pack and (3) pump with tube.
- 2 Refrigerate (2) at least 2 hours before usage.
- 3 Put (2) back on the (1) after refrigeration.
- 4 Connect the tube to the ON/OFF switch and turn "ON" the switch.
- 5 Position (A) around the left or right thumb, depending on which hand requires therapy (picture 1).
- 6 While winding the support around the wrist, the velcro tape sticks together adequately (picture 2).
- 7 When turning ON/OFF the switch, please fixate it with one hand and turn it with the other to ensure the switch is on the appropriate position and then pump.
- 8 To get the best effect: ON for 5-10 minutes, OFF for 5-10 minutes to let the skin rest and then repeat.
- 9 Empty it before taking off and then refrigerate (2) for next usage.

INDICATIONS

sprains / sport injuries / rehabilitation / arthritis / muscle tiredness / overload

CONTRA-INDICATIONS

No contra-indications are known until today.

WASHING INSTRUCTIONS

Store the product under well ventilated and dried condition and avoid direct sunlight.
Only use mild detergents and warm water to wash the product. Do not use strong detergents!
Using the washing machine and the spin-drier to wash and dry the product is not recommended.

IMPORTANT NOTES

IMPORTANT NOTES

This product is recommended for one patient use only. If it is used for treating more than one patient, the manufacturer's product liability according to the Medical Devices Act will become invalid.

The final decision concerning the usage and selection of the appliance shall be made by the doctor or medical specialist.

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dr. **aktive**
LIVE LIFE

Wrist Support
Cold Compression Therapy

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