

PHILIPS

Core M2

IVUS workflow

1. Press start a new case and enter patient information
2. Prep catheter and connect to PIM
3. Adjust image settings to your preference
 - Optional: press **Ringdown** once the catheter is out of the sheath and into the vessel
4. Select **ChromaFlo** if desired
5. Advance catheter distal to lesion:
 - Press **Record** key on console and begin pullback
 - Use bookmark to identify a frame of interest
6. Press **Stop** on console when pullback is complete
7. Review recording and take measurements using diameter, area or dots
8. Repeat steps 5-7 to acquire additional images
9. Once case is completed, end case and archive to DVD and/or DICOM/PACS





© 2017 Koninklijke Philips N.V. All rights reserved.
Trademarks are the property of Koninklijke
Philips N.V. or their respective owners.

D000179467/A

Philips
3721 Valley Centre Drive, Suite 500
San Diego, CA 92130 USA

www.philips.com/IGTdevices